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declares on his own responsibility that the following device:

Product Name and Product Code	ROLLER 20	SIR20-PN
Intended Purpose	ROLLER 20 is an in vitro diagnostic automatic analyzer for the quantitative determination of Erythrocyte Sedimentation Rate (ESR) in human blood samples with EDTA from adult and pediatric patients with suspected inflammation. ROLLER 20 provides results to inform clinical management for serious and non-serious conditions that require further diagnostic investigations and evaluation of the clinical status. The physician performs his/her expert evaluation on the information provided by the analyzer thanks to his/ her professional knowledge, competence and skills as per the local law.	
Basic UDI-DI	805604014SIR20.195.804DA	
Risk Class	Class A, according to IVDR 746/2017.	

complies with the General Safety and Performance Requirements of the Regulation 746/2017 related to the In Vitro Diagnostic Medical Devices (IVDR).

We declare also that the device has been designed and manufactured in conformity to the below:

Harmonised Standards	EN ISO 13485:2016 A11:2021: Medical devices - Quality management systems - Requirements for regulatory purposes EN ISO 14971:2019: Medical devices - Application of risk management to medical devices EN ISO 15223-1:2021: Medical devices — Symbols to be used with medical device labels, labelling and information to be supplied EN ISO 18113 -1:2011: In vitro diagnostic medical devices - Information supplied by the manufacturer (labelling) EN ISO 18113 -3:2011: Information supplied by the manufacturer (labelling) - Part 3: In vitro diagnostic instruments for professional use EN 62304:2006+A1:2015: Medical device software - Software life cycle processes
Technical Standards	IEC 62366-1:2015: Application of usability engineering to medical devices IEC TR 62366-2:2016: Guidance on the application of usability engineering to medical device IEC 61326-1:2012; IEC 61326-2-6:2012: Electrical equipment for measurement, control and laboratory use - EMC requirements IEC 61010-1:2001; IEC 61010-2-101:2002: Safety requirements for electrical equipment for measurement, control, and laboratory use
Other	Directive 2015/863/EU (RoHS3)

The device has been CE Marked as IVD Medical Device according to Article 48 (10) of Reg. 746/2017 (IVDR).

DPGT0130 - Ed. 1 Rev. 1 - 2022-03-30

Pag. 1 di 2

Entered in Padova companies register at the n. 04337640280. Company stock 10.000.000 entirely deposited.

Company with single partner and subjected to direction and coordination of Alifax Holding S.p.A.

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©Copyright 2022. This document is exclusive property of ALIFAX S.r.l. confidential and not freely disclosable. All rights reserved.

INSTALLATION REPORT for ROLLER 20 – SI R20-PN

Rev. 2.0 Effective since 2022 May 23

Dear Distributor,

You are kindly requested to fill-in this form during the installation and return a copy within one month since the installation to:

export.tech@alifax.com

Alifax S.r.l.
Via Merano 30 - 33045 Nimis (UD) - Italy
Tel. +39-0432-547454
Fax: +39-0432-547378

IMPORTANT NOTES:

- Warranty expires 12 months after the installation date written on the present Installation Report duly filled in all its parts and returned to Alifax. If a copy of the report is not received back within one month, warranty will be invalidated.
- It is Your responsibility to verify that the device is properly installed and operates correctly and safely by filling this installation report and to keep such installation reports together with the related tests performed.

DISTRIBUTOR:	COUNTRY:
INSTALLATION SITE:	INSTALLATION DATE:
INSTRUMENT: ☐ ROLLER 20 – SI R20-PN	SERIAL NUMBER:
SOFTWARE VERSION (please see the print out):	

This document is divided in two parts: the installation report module (part one) containing **installation qualification** and **operative qualification** checklist and the installation procedure; the first one must be returned to manufacturer.

- Installation module as well as check list MUST be sent back to manufacturer by email

Please register at <http://www.alifax.com/> in order to obtain the latest version of the USER MANUAL and SERVICE MANUAL related to the software version installed on the instrument.

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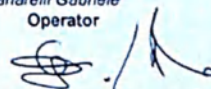
ALIFAX S.r.l. - Manufacturing Plant - via Merano 30 - 33045 NIMIS (UD) Italy

ALI
FAXFASCICOLO APPARECCHIO
DEVICE HISTORY RECORD

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DATA/Date 03/03/2022 FIRMA/Signature 07/09/2022
16:48:38
ROLLER 20 Plus Needle
R10-05.00C.00R10_U1-05.00C
Apr 27 2021 11:44:22
SN: 2278
DEBUG: OFF
MAINTENANCE
WARNING: 30000
MAINTENANCE COUNTER: 0
washing time: 45
PRINT DURING ANALYSIS: ON
PRINT EXPANDED TEXT: ON
2 CPS On Board: ON
MEC on Board: OFF
Pediatric samples: OFF
Test Mode: OFF# ESR = 200
Global count ESR: 0R10MOT-05.00B
Dec 01 2020 13:00:56
Mixing cycles: 200
Mixing speed: medium
Automatic sample ul: 175
Manual sample ul: 100
Steps UP: 3500
Steps up wash: 4500
Tube offset: -14
Syr fast: OFF
Centrifugation cycles: 30INTERNAL CIRCUIT
R10ANA-03.100
May 31 2013 17:52:25
REFERENCE: EDTA
BY EDTA: 1.4456
MF1: 1.4476
MF2: 1.4435
OFFSET1: -92.32
OFFSET2: -75.59
37.0 (37)
Wash not ok: 2
T.100 SENS1: 3815
T.100 SENS2: 3834
T.100 STOP: 3802
DAC SENS1: 509
DAC SENS2: 476
DAC STOP: 509EXTERNAL CIRCUIT
R10ANA-03.100
May 31 2013 17:52:25
REFERENCE: EDTA
BY EDTA: 1.4335
MF1: 1.4362
MF2: 1.4308
OFFSET1: -94.15
OFFSET2: -71.34
37.1 (37)
Wash not ok: 1
T.100 SENS1: 3795
T.100 SENS2: 3796
T.100 STOP: 3806
DAC SENS1: 506
DAC SENS2: 491
DAC STOP: 412R10COM-04.02B
Aug 08 2018 08:43:53
RS232: NO HUST
Instrument: 1
missingID: ON
ACK: ON
BAYER
PROTOCOL: OFF
TIMEOUT UART: 2
MAX ATTEMPTS: 3
Do on timeout: ON
6 parameters: OFF
Curve parameters: OFF
Send kinetic: OFF000 Alifax Russia
(10000153)
Old code (0x99)
Exp old 31/12/19
SM To Disable: OFF
Credit from Old Card: 200000
Version U12-D100©Copyright 2021. Documento segreto non divulgabile a terzi, di proprietà esclusiva di ALIFAX S.
©Copyright 2021. This document is exclusive property of ALIFAX S.r.l. confidential and not freely dis

FASCICOLO APPARECCHIO
DEVICE HISTORY RECORD

D205.05.03 Rev2.4

Pag. 10 di 10

s.n. N2227842 Commessa (Job Order) 2235 Data PRD (Production Date) 17/04/22

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☒ SIR20 PN

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☒ N° 2 - SI205800 Thermal paper rolls (one inside instruments and one spare)

☒ N° 2 - SI205801 (Waste) Tank (one inside instruments and one spare)

☒ N° 2 - SI195145 (Wash) Tank (one inside instruments and one spare)

☒ N° 1 - Touch Screen Pen

☒ N° 1 - CD Multi-language with Operative Manual (Version and languages)

ESR_UM_SIR20_PN_ROLLER20PN_SW S-OOC-L-Q-EN-ES-IT-PT-FR-DE ESR-CD-SIR20-
ROLLER20PN-SW-S-OOC-2-

☒ N° 2 - Spare Mains fuses T2A

☒ N° 1 - Mains Cable

☐ Personalized main cable _____

☒ ~~N° 1 - Needle~~ 13/7/22

☒ N° 1 - DPRD0105: Electrical routine tests EN61010-2-101:2002 Annex F

☒ N° 1 - EC declaration of conformity

☒ N° 1 - ESR_IQOQ_SIR20PN_ROLLER20PN: Installation Report

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©Copyright 2022. Documento segreto non divulgabile a terzi. di proprietà esclusiva di ALIFAX S.r.l. Ogni diritto riservato.