State of Minnesota

SECRETARY OF STATE

APOSTILLE

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Country: United States of America

This Public Document: Instructions for Use

has been signed by Elisabeth Ruth Marker

acting in the capacity of Notary Public, State of Minnesota

bears the seal/stamp of Elisabeth Ruth Marker, Notary Public, State of Minnesota

CERTIFIED

at: St. Paul, Minnesota

6. Date: April 10, 2015

by: Secretary of State, State of Minnesota

8. File No 8219628-25

Seal/Stamp:

10. Signature:


Steve Simon
Steve Simon
Secretary of State

3M Health Care Business

3M Center
2510 Conway Ave, Bldg. 275-5W-06
St. Paul, MN 55144
651 733 1110

To Whom It May Concern:

The attached is a copy of the Instructions for Use for 3M™ Kind Removal Silicone Tape, provided by 3M Health Care, 3M Center, 2510 Conway Ave, Bldg 275-5W-06, St. Paul, MN 55144 USA.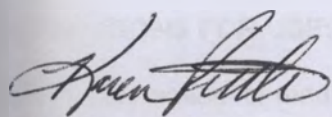


Karen Rittle
Division Regulatory Manager
Critical and Chronic Care Solutions Division
3M Health Care


10 APRIL 2015
Date

State of Minnesota)
County of Hennepin) SS

Subscribed and sworn to before me this 10th day of April, 2015.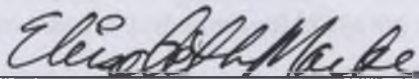

Elisabeth Ruth Marker, Notary Public
My Commission Expires: January 31, 2019

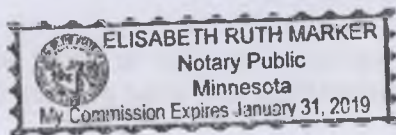3M™ Kind Removal Silicone Tape

PRODUCT DESCRIPTION:

3M™ Kind Removal Silicone Tape is a general purpose medical tape used to secure dressings, light weight tubing, and devices to skin. The 3M™ Kind Removal Silicone Tape is gentle to skin, reducing the risk of adhesive tape trauma when removed.

INDICATIONS:

A general purpose medical tape for medical application used primarily to secure dressings, light weight tubing, and devices to skin.

INSTRUCTIONS FOR USE:

General information for medical adhesive tape use includes:

Common tape application technique

1. Always apply tape to skin that is dry, clean and free from soaps, chemicals and oils. If needed, apply a skin protector and allow it to dry before using tape. Avoid using tincture of benzoin, as it can be a skin irritant.
2. Always apply the tape without tension and smooth from the center out. Allow at least one inch of tape on the skin around dressing edges.
3. Do not secure one end of the tape and apply tension while securing the other end of the tape. This may induce skin trauma in the form of blistering or skin tearing.
4. Maximize adhesion by firmly pressing or rubbing the tape into place on skin, tubing or appliances.

Tape Removal Technique:

1. Loosen all ends of the tape strips, or edges of dressings. Grasp the full width of the tape and slowly and gently pull the tape back over itself toward the wound to minimize disruption of healing tissue.
2. Tape should be removed in the direction of hair growth whenever possible, if this will not disturb the wound area. Keep the tape close to the skin surface as you pull it back, and support the skin immediately adjacent to the strip being removed.

PRECAUTIONS:

None

STORAGE CONDITIONS:

No special storage conditions for this product.

UTILIZATION/DISPOSAL:

Dispose of product in accordance with the local/regional/national/international regulations.

VALIDITY DATE:

This product has a shelf life of 2 years.

WARRANTY INFORMATION:

3M Health Care warrants this product will be free from defects in material and manufacture. 3M HEALTH CARE MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Health Care's sole obligation shall be repair or replacement of the 3M Health Care product.

For further information, please contact your local 3M representative or contact us at www.3M.com and select your country.

AUTHORISED REPRESENTATIVE OF THE MANUFACTURER:

ZAO "3M Russia", Russia 121614, Moscow, Krylatskaya str. bldg.17/3

T.+74957847474

Made in U.S.A. by
3M Health Care
2510 Conway Ave.

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3M is a trademark of 3M.
© 2014, 3M. All rights reserved.