INSTRUCTIONS FOR USE FOR A MEDICAL DEVICE

" Disposable sterile ophthalmic knife Care Edge "

Manufacture: Caregroup Sight Solution Private Limited, India.

B-6/B, GROUND FLOOR, REHAB BUILDING № 38, JK SAWANT MARG.,
DADAR(WEST), Mumbai City, Maharashtra, India, 400028

Approval Date February 2024

Purpose, scope of application, potential consumers.

Purpose:

The medical product "Safe Blades Care Edge" is intended for use during ophthalmic surgical operations.

Scope- ophthalmic surgery.

Potential consumers, conditions of use - qualified medical personnel (ophthalmic surgeon) trained to handle a medical device in the operating room of a medical institution.

Product description, design, technical and functional characteristics.

"Disposable sterile ophthalmic knife Care Edge".

Execution options:

1. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port:

1.1. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 15°- 1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

1.2. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 22.5°- 1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

1.3. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 30°- 1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

1.4. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 5° -1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2. Disposable sterile ophthalmic knife Care Edge Crescent:

2.1 Disposable sterile ophthalmic knife Care Edge Crescent, straight with a bevel up blade width 2.5 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2.2. Disposable sterile ophthalmic knife Care Edge Crescent, straight with a bevel down blade width 2.5 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2.3. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel up blade width 2.5 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2.4. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel down blade width 2.5 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2.5 Disposable sterile ophthalmic knife Care Edge Crescent, straight with a bevel up blade width 2.3 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.

2.6 Disposable sterile ophthalmic knife Care Edge Crescent, straight with a bevel down blade width 2.3 mm-1 pack.(1 pack /10 pcs.)

- Instructions for use – 1 pcs.
- 2.7 Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel up blade width 2.3 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 2.8 Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel down blade width 2.3 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 2.9 Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel up blade width 2.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.

3. Disposable sterile ophthalmic knife keratome straight/curved Care Edge Slit:

- 3.1. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.2. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.3. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.4. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.5. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.4 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.6. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.4 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.7. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.8 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.8. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.8 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.9. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.10. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.11. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.12. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.13. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.3 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.14. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,

- blade width 2.3 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.15. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.4 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.16. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.4 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.17. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.5 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.18. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.5 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.19. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.6 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.20. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.6 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.21. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.65 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.22. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.65 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.23. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.75 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.24. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.75 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.25. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 2.8 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.26. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 2.8 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.27. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 3.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.28. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 3.0 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.29. Disposable sterile ophthalmic knife keratome straight Care Edge Slit,
blade width 3.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 3.30. Disposable sterile ophthalmic knife keratome curved Care Edge Slit,
blade width 3.2 mm-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.

4. Disposable sterile ophthalmic knife microvitreoretinal straight/curved Care Edge MVR:

- 4.1. Disposable sterile ophthalmic knife microvitrectomy straight Care Edge MVR, size 19G-1 pack. (1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 4.1. Disposable sterile ophthalmic knife microvitrectomy curved Care Edge MVR, size 19G-1 pack. (1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 4.2. Disposable sterile ophthalmic knife microvitrectomy straight Care Edge MVR, size 20G-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 4.2. Disposable sterile ophthalmic knife microvitrectomy curved Care Edge MVR, size 20G-1 pack.(1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 4.3. Disposable sterile ophthalmic knife microvitrectomy straight Care Edge MVR, size 21G-1 pack. (1 pack /10 pcs.)
- Instructions for use – 1 pcs.
- 4.3. Disposable sterile ophthalmic knife microvitrectomy curved Care Edge MVR, size 21G-1 pack. (1 pack /10 pcs.)
- Instructions for use – 1 pcs.

Description of the medical device.

The medical device is used during ophthalmic surgery operations to perform precise incisions, which are one of the prerequisites for use in delicate ophthalmic surgical interventions.

The medical device consists of a blade, a handle, a cap. The blades of the knife have different sizes and shapes necessary for certain surgical manipulations.

Dimension G corresponds to the outer diameter of the tube from which the blade is made.

The cap serves as protection against cuts and injections of medical personnel.

The version of the “Ophthalmic disposable sterile sickle-shaped knife Care Edge Crescent” has a certain sharpening of the knife blade.

The diagram shows the direction of sharpening the knife blade.

In the versions “Ophthalmic disposable sterile sickle-shaped knife Care Edge Crescent”, “Disposable ophthalmic knife sterile keratome Care Edge Slit”, “Disposable ophthalmic knife sterile microvitrectomy straight/curved Care Edge MVR” the working parts of the blade can be straight or curved.

The figure shows the types of straight and curved working parts of the blade using the example of the “Disposable sterile microvitrectomy ophthalmic knife straight/curved Care Edge MVR”.

	
straight working part of the blade	curved working part of the blade

1. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port	
	For making precise puncture cuts
2. Disposable sterile ophthalmic knife Care Edge Crescent	
	To create scleral tunnel valves
3. Disposable sterile ophthalmic knife keratome straight/curved Care Edge Slit	
	For centesis incisions of sclerocornea in time for cataract surgery
4. Disposable sterile ophthalmic knife microvitreoretinal straight/curved Care Edge MVR	
	For centesis incision of the sclera during vitreous surgery. It is also used to create lateral ports during cataract surgery.

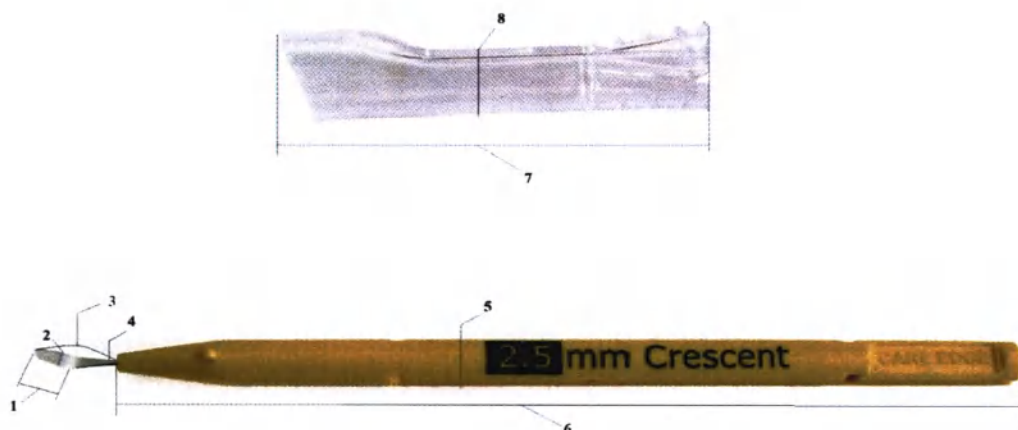
Design of a medical device.

1. Blade 2. Handle. 3.Cap.

Technical characteristics of the medical device.

General scheme of measurements of technical characteristics

1. Blade length.
2. Blade width.
3. Angle of bending of the working part of the blade.
4. Outer diameter of the blade tube.
5. Handle diameter.
6. Handle length.
7. Cap length
8. Cap diameter

Technical characteristics of the medical device.

1. Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port	Blade					Pen	
	Width, mm, ± 0.10	cutting angle, $\pm 2^\circ$	Length, mm, ± 0.10	Diameter, mm, ± 0.03	Thickness, mm, ± 0.02	Diameter, mm, ± 0.20	Length, mm, ± 0.50
Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 15°	1,5	15°	5,20	0,60	0,15	6,50	120
Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 22.5°	1,7	22.5°	5,20	0,60	0,15	6,50	120
Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 30°	1,7	30°	5,20	0,60	0,15	6,50	120

Disposable sterile ophthalmic knife with one cutting edge Care Edge Side Port, 45°	1,7	45°	5,20	0,60	0,15	6,50	120
--	-----	-----	------	------	------	------	-----

Product weight with cap, 6 g ± 0.20 g

2. Disposable sterile ophthalmic knife Care Edge Crescent:	Blade				Pen	
	Width, mm, ± 0,10	Width, mm, ± 0,10	Diameter, mm, ± 0,02	Thickness, mm, ± 0,02	Diameter, mm, ± 0,20	Length, mm, ± 0,50
						
2.1. Disposable sterile ophthalmic knife Care Edge Crescent, straight with bevel up blade width 2.5 mm	2,50	5,40	0,81	0,15	6,50	120
2.2. Disposable sterile ophthalmic knife Care Edge Crescent, straight with bevel down blade width 2.5 mm	2,50	5,40	0,81	0,15	6,50	120
2.3. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel up blade width 2.5 mm	2,50	5,40	0,81	0,15	6,50	120
2.4. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel down blade width 2.5 mm	2,50	5,40	0,81	0,15	6,50	120
2.5. Disposable sterile ophthalmic knife Care Edge Crescent, straight with bevel up blade width 2.3 mm	2,30	5,40	0,76	0,15	6,50	120
2.6. Disposable sterile ophthalmic knife Care Edge Crescent, straight with bevel down blade width 2.3 mm	2,30	5,40	0,76	0,15	6,50	120
2.7. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel up blade width 2.3 mm	2,30	5,40	0,76	0,15	6,50	120
2.8. Disposable sterile ophthalmic knife Care Edge Crescent, curved with bevel down blade width 2.3 mm	2,30	5,40	0,76	0,15	6,50	120
2.9. Disposable sterile ophthalmic knife Care Edge Crescent, curved with a bevel upwards blade width 2.0 mm	2,0	5,40	0,72	0,15	6,50	120

Angle of bending of the working part of the blade 60°, ± 2°

Blade bend length 6.75mm± 0.25mm

Product weight with cap, 6 g ± 0.20 g

3. Disposable sterile ophthalmic knife keratome straight/curved Care Edge Slit	Blade				Pen	
	Width, mm, ± 0,10	Width, mm, ± 0,10	Diameter, mm, ± 0,02	Thickness, mm, ± 0,02	Diameter, mm, ± 0,20	Length, mm, ± 0,50
						

3.1. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.0 mm	1.0	2,10	0,15	0.49	6,50	120
3.2. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.0 mm	1.0	2,10	0,15	0.49	6,50	120
3.3. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.2 mm	1.2	2,10	0,15	0.52	6,50	120
3.4. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.2 mm	1.2	2,10	0,15	0.52	6,50	120
3.5. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.4 mm	1.4	2,10	0,15	0.56	6,50	120
3.6. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.4 mm	1.4	2,10	0,15	0.56	6,50	120
3.7. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 1.8 mm	1,8	2,10	0,15	0,56	6,50	120
3.8. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 1.8 mm	1,8	2,10	0,15	0,56	6,50	120
3.9. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.0 mm	2.0	2.40	0.15	0.56	6,50	120
3.10. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.0 mm	2.0	2.40	0.15	0.56	6,50	120
3.11. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.2 mm	2,2	2,40	0,15	0,56	6.50	120
3.12. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.2 mm	2,2	2,40	0,15	0,56	6.50	120
3.13. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.3 mm	2.3	4.0	0,15	0.76	6.50	120
3.14. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.3 mm	2.3	4.0	0,15	0.76	6.50	120
3.15. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.4 mm	2.4	4.0	0,15	0.76	6.50	120
3.16. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.4 mm	2.4	4.0	0,15	0.76	6.50	120

3.17. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.5 mm	2.5	4.0	0,15	0.76	6.50	120
3.18. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.5 mm	2.5	4.0	0,15	0.76	6.50	120
3.19. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.6 mm	2,60	4,00	0,15	0,80	6.50	120
3.20. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.6 mm	2,60	4,00	0,15	0,80	6.50	120
3.21. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.65 mm	2.65	4,10	0,15	0,81	6.50	120
3.22. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.65 mm	2.65	4,10	0,15	0,81	6.50	120
3.23. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.75 mm	2.75	4,10	0,15	0,81	6.50	120
3.24. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.75 mm	2.75	4,10	0,15	0,81	6.50	120
3.25. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 2.8 mm	2,80	4,10	0,15	0,81	6.50	120
3.26. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 2.8 mm	2,80	4,10	0,15	0,81	6.50	120
3.27. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 3.0 mm	3,0	4,40	0,15	0,80	6.50	120
3.28. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 3.0 mm	3,0	4,40	0,15	0,80	6.50	120
3.29. Disposable sterile ophthalmic knife keratome straight Care Edge Slit, blade width 3.2 mm	3,20	4,70	0,15	0,92	6.50	120
3.30. Disposable sterile ophthalmic knife keratome curved Care Edge Slit, blade width 3.2 mm	3,20	4,70	0,15	0,92	6.50	120

Angle of bending of the working part of the blade $45^\circ, \pm 2^\circ$
 Blade bend length 6.50 mm, ± 0.50 mm
 Product weight with cap, 6 g ± 0.20 g

4. Disposable sterile ophthalmic knife microvitreoretinal Care Edge MVR:	Blade				Pen	
	Width, mm, ± 0.10	Width, mm, ± 0.10	Thickness, mm, $\pm$ 0.02	Size, G/ Diameter, mm, ± 0.02	Diameter, mm, ± 0.20	Length, mm, ± 0.50
						
4.1. Disposable sterile ophthalmic knife microvitreoretinal straight Care Edge MVR, size 19G	1,4	3,5	0,50	19/0,80	6,50	120
4.2. Disposable sterile ophthalmic knife microvitreoretinal curved Care Edge MVR, size 19G	1,4	3,5	0,50	19/0,80	6,50	120
4.3. Disposable sterile ophthalmic knife microvitreoretinal straight Care Edge MVR, size 20G	1,20	3,5	0,50	20/0,80	6,50	120
4.4. Disposable sterile ophthalmic knife microvitreoretinal curved Care Edge MVR, size 20G	1,20	3,5	0,50	20/0,80	6,50	120
4.5. Disposable sterile ophthalmic knife microvitreoretinal straight Care Edge MVR, size 21G	1,20	3,5	0,50	21/0,80	6,50	120
4.6. Disposable sterile ophthalmic knife microvitreoretinal curved Care Edge MVR, size 21G	1,20	3,5	0,50	21/0,80	6,50	120
Angle of bending of the working part of the blade $45^\circ, \pm 2^\circ$ Product weight with cap, 6 g ± 0.20 g						

Cap length 57.50 mm ± 0.50 mm

Cap diameter 9.00 mm ± 0.20 mm

The surface roughness of the cutting edges Ra is not more than 0.63 microns.

The roughness of the remaining surfaces Ra is not more than 1.25 μ m.

Blade hardness according to Vickers - 52-54 HRC.

The width of the cutting edge of the products should be no more than 3 microns.

Blister pack dimensions

№	Name	Technical specifications
---	------	--------------------------

1	Thermoformed tray	Length: 165.00 mm $\pm$ 2.00 mm Width: 50.00 mm $\pm$ 2.00 mm
2	Tyvek paper	Length: 168,00 mm $\pm$ 2,00 mm Width: 52,00 mm $\pm$ 2,00 mm
3	Self-adhesive label	Length: 85,00 mm $\pm$ 2,00 mm Width: 45,00 mm $\pm$ 2,00 mm

Dimensions of consumer packaging (box)

№	Name	Technical specifications
1	Length	178.0 mm to 182.0 mm
2	Width	173.0 mm to 177.0 mm
3	Height	48.0 mm to 53.0 mm

Weight of the product in a blister pack $15 \text{ g} \pm 2 \text{ g}$.

Weight of the product in consumer packaging (box) $195 \text{ g} \pm 2 \text{ g}$.

Indications, contraindications, precautions.

Indications for use.

Performing surgical operations on the organs of vision (for example, phacoemulsification of cataracts, etc.), and is also used for punctures, incisions and delamination of eye tissues during surgical ophthalmological interventions with minimal damage to the tissues of the eyeball.

Contraindications for use.

Do not use in patients with sensitivity or allergy to metals.

Precautionary measures.

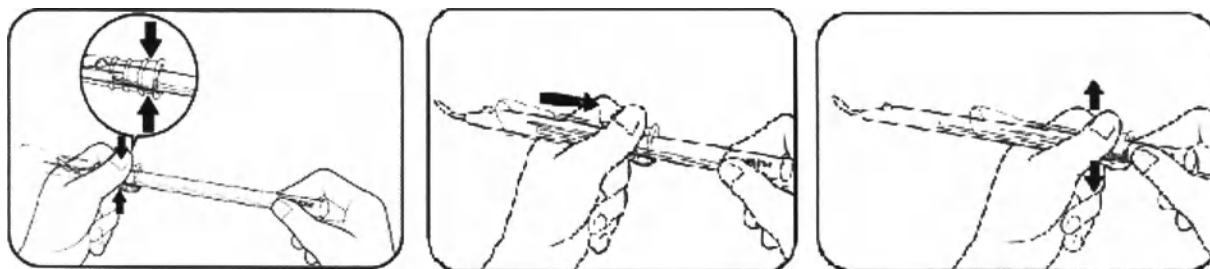
- When using a medical device, the precautions inherent in the procedure for performing an ophthalmic operation must be observed.
- Do not use if package is opened or damaged.

Risks of use, expected and predictable side effects.

When using a medical device for its intended purpose in accordance with the manufacturer's instructions for use, there are no risks of use, as well as expected and predictable side effects.

The order of work with the product.

1. Check the integrity of the blister pack.
2. Remove the medical device from the blister pack.
3. Press the back end of the protective cap to unlock it and slide the protective cap from the blade to the handle of the knife until it stops.

4. Inspect the knife blade for defects and integrity violations.
5. Perform surgical manipulations in accordance with generally accepted medical practice.
6. Slide the protective cap on the knife blade until it clicks in order to protect personnel from pricking.
7. After use, the medical device must be disposed of.

Terms of use, storage and transportation.

-Application conditions.

The product is operated at ambient temperature from 15 °C to 35 °C and humidity 50-60%.

-Storage conditions.

Store the medical product in its original packaging at a temperature of 0°C to 45°C in a dry room protected from direct sunlight at a humidity not exceeding 70%.

-Conditions of transportation.

The medical product is transported by all modes of transport in covered vehicles in accordance with the rules for the carriage of goods; applicable to every mode of transport.

Ambient temperature during transportation from 0°C to 45°C, humidity not more than 80%. Avoid damage to corrugated cardboard boxes during transportation.

Packing.

1. Individual packing:

A medical device of one of the variants is placed in an individual package (blister). Each blister is hermetically sealed.

2. Consumer packaging (cardboard pack).

The medical device is placed in consumer packaging together with the instructions for use.

Consumer packaging is sealed with shrink wrap.

3. Transport packaging (box).

A medical device in consumer packages is packed in a transport package in quantity, depending on the order.

Marking.

Labeling of the packaging of a medical device intended for circulation on the territory of the Russian Federation.

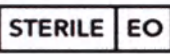
1. On the individual package (blister) it is indicated:

1.1. Information contained on the original manufacturer's labeling.

- name, address of the place of production, logo, e-mail address and website of the manufacturer;
- version, model.

Symbols and inscriptions used::

	Изготовитель		Date of manufacture		Do not re-sterilize
---	--------------	---	---------------------	---	---------------------

	Batch code		Use up to		Reuse prohibition
	Marking CE approved		Sterilization using ethylene oxide		QR-code
	Keep away from moisture		Do not expose to direct sunlight		Do not use if packaging is damaged
	Refer to instructions for use		Temperature Range		Pull over

10.1.2. Information contained on Russian-language labeling.

- information about the authorized representative in the territory of the Russian Federation (name, address, e-mail address, phone number);
- registration certificate number, date of issue;
- «H» (stainless steel)

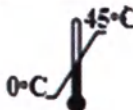
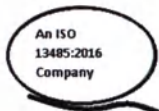
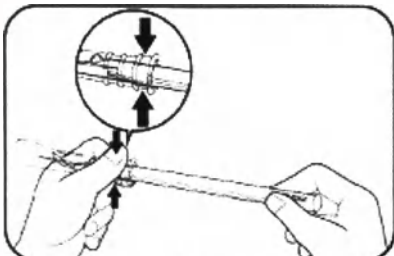
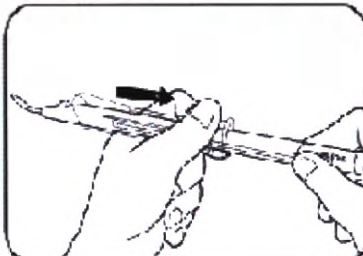
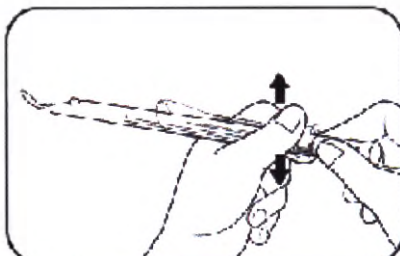
2. On consumer packaging (box) it is indicated:

2.1. Information contained on the original manufacturer's labeling.

- name, address of the place of production, logo, e-mail address and website of the manufacturer;
- registration certificate number, date of issue;
- product name;
- version, model;
- the composition of the package.

Symbols and inscriptions used:

	Manufacturer		Date of manufacture		Do not re- sterilize
	Batch code		Use up to		Reuse prohibition
	Marking CE approved		Sterilization using ethylene oxide		Representative in the EU
	QR-code		Do not expose to direct sunlight		Do not use if packaging is damaged

	Refer to instructions for use		Temperature Range		Keep away from moisture
			Compliance with ISO 13485 requirements		
Opening the knife					
					
Press the back end of the protective cap to unlock it. Hold it and slide it in the opposite direction.		Hold it and slide it in the opposite direction.		Slide the protective cap all the way and release it to lock it into place.	
How to put away a knife Press the back end of the protective cap to unlock it Slide the protective cap all the way and release it to lock it into place. (To remove the knife, follow the steps in reverse order)					

2.2. Information contained on Russian-language labeling.

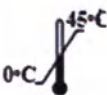
- information about the authorized representative in the territory of the Russian Federation (name, address, e-mail address, phone number);
- product name;
- version, model;
- the inscription "humidity during storage no more than 70%"
- «H» (stainless steel)

3. The transport packaging indicates:

- name, address of the place of production, logo, e-mail address and website of the manufacturer;
- information about the authorized representative in the territory of the Russian Federation (name, address, e-mail address, phone number);
- registration certificate number, date of issue;
- product name,
- version, model;
- total number of packages / number of products in the package;
- net/gross weight in kilograms;

Symbols and inscriptions used:

	Manufacturer		Date of manufacture
	Marking CE approved		Batch code

	Top		Temperature Range
	Do not expose to direct sunlight		Keep away from moisture
inscription "humidity during transportation no more than 80%"			Sterilization using ethylene oxide

List of standards.

EN ISO 13485, EN ISO 14971, EN ISO 11607-1, EN ISO 11607-2, EN ISO 10993-1
 EN ISO 10993-5, EN ISO 10993-7, ISO 10993-10, EN ISO 10993-11, EN ISO 11135-1
 EN 1041:2008, EN ISO 15223-1, EN ISO 62366, ISO 14644-1, ISO 14644-2, ISO 14644-3
 ASTM D4169 – 04, ISO 7153-1

Sterilization.

The medical device is sterile.

The method of sterilization is gas sterilization with ethylene oxide.

Disposal.

When handled on the territory of the Russian Federation, the used medical device is disposed of as medical waste of Class B in accordance with SanPiN 2.1.3684-21.

An unusable medical device (a product with broken packaging, expired shelf life) must be disposed of as Class A waste according to SanPiN 2.1.3684-21.

Maintenance and repair requirements.

Maintenance and repair requirements are not applicable to this medical device. The medical device is disposable. After use, the medical device is subject to mandatory disposal.

Manufacturer's warranty, expiration date.

The manufacturer guarantees the sterility of the product, as well as its compliance with the requirements of international and national standards throughout the shelf life of the product, subject to the rules of transportation, storage and conditions of use in accordance with the instructions for use established by the manufacturer.

The shelf life of the product is 5 years from the date of sterilization.

Manufacturer's name and address

Caregroup Sight Solution Private Limited
 B-6/B, GROUND FLOOR, REHAB BUILDING № 38, JK SAWANT MARG., DADAR(WEST), Mumbai
 City, Maharashtra, India, 400028

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www.caregroupiol.com customercare@caregroupindia.com

Address of the place of production

Block No 310/C&E, Village Dabhasa, Taluka - Padra, Dist. Vadodara -391440, Gujarat, India.

Authorized representative on the territory of the Russian Federation:

Kom Span LLC.

Russia, 115035, Moscow, Kosmodamianskaya nab., 40-42, building 3, floor 1, office XVII,
 fl. 1, of 2.

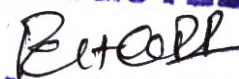
tel./fax: (495) 434-45-06.

For CAREGROUP SIGHT SOLUTION PRIVATE LIMITED

JAGRAT N. DAVE
DIRECTOR / AUTHORISED SIGNATORY

ATTESTED

PRATIMA P. PATEL
NOTARY (Govt. of India)

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B-6/B, GROUND FLOOR, REHAB BUILDING № 38, JK SAWANT MARG., DADAR(WEST),
Mumbai City, Maharashtra, India, 400028

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