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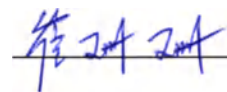
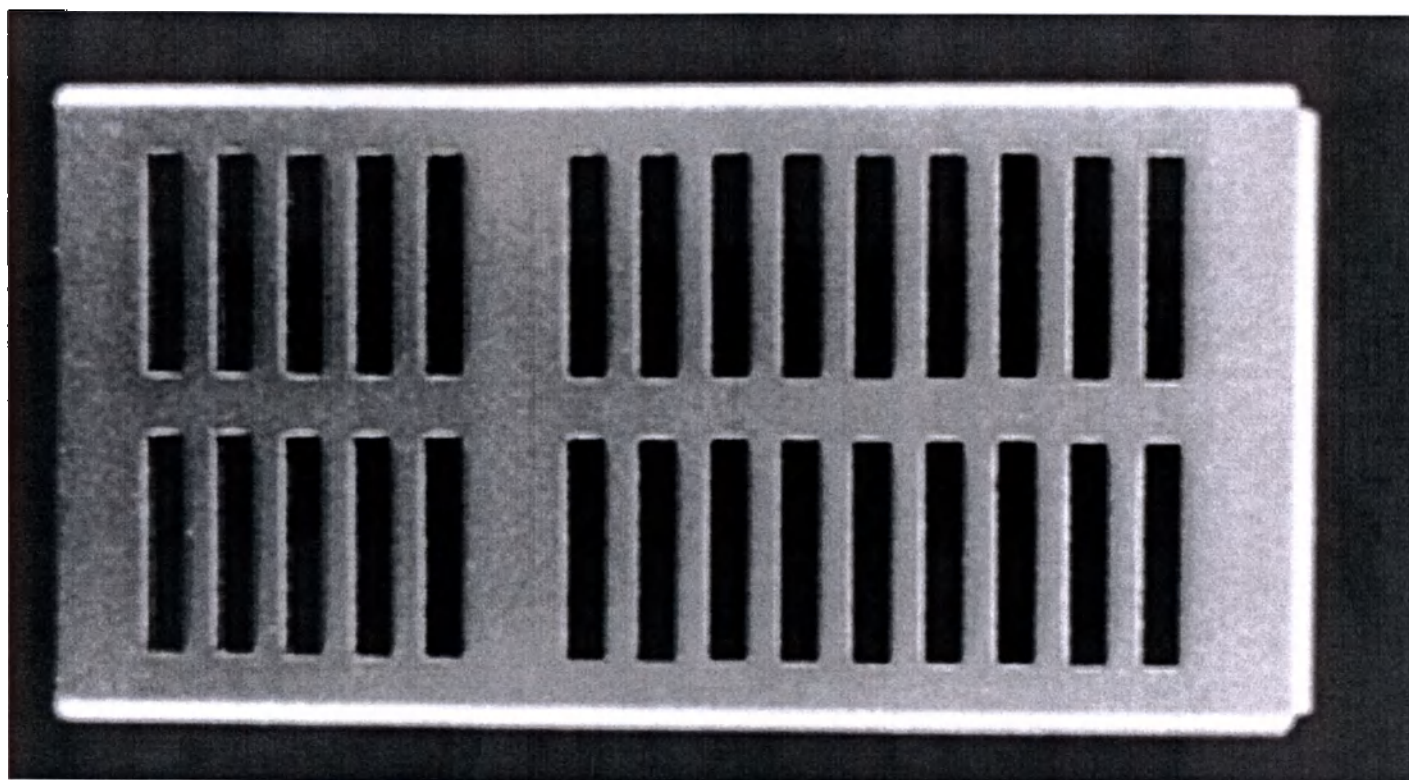
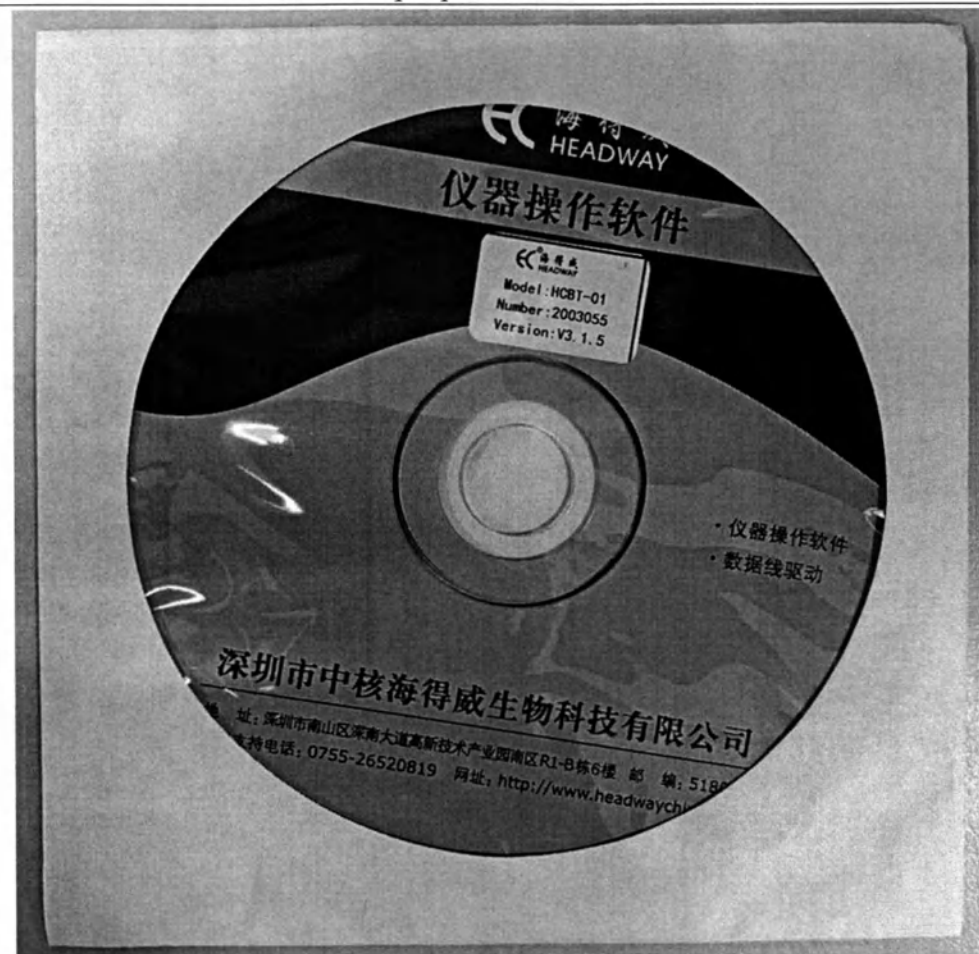
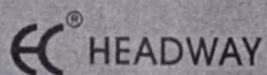
Breath Test Analyzer Systems User Manual

Model: HCBT-01 (Including Breath Bag)

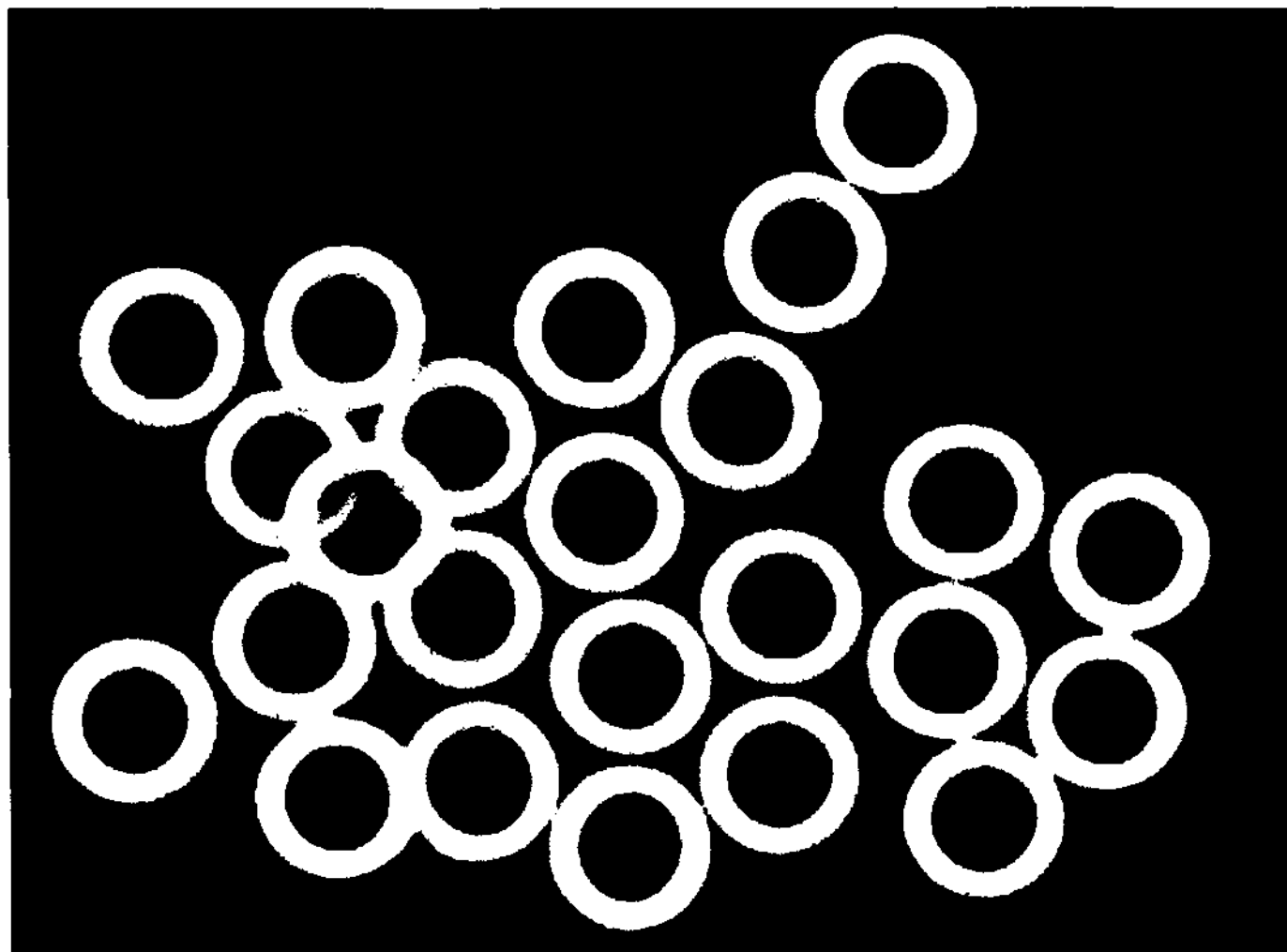
CAUTION: In order to use the product in a correct and safe way, please read this manual carefully. Should there be any problem in the course of operation, please look up this manual immediately.

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TEST REPORT

No.: 13C(01)-2023030

Product Name	Breath Test Analyzer	Model	HCBT-01
Producer	Shenzhen Zhonghe Headway Bio-Sci & Tech Co., Ltd.	Inspection Date	May 8th, 2023
Apparatus's Serial No.	202304030	Inspection Item	All items for OQC (outgoing quality control)
State of Sample	Intact and conforming to requirements for inspection		
Basis for inspection and judgment	WI-QA-16-C2 "Inspection Instruction for Breath Test Analyzer"		
Conclusion	According to the test, HCBT-01 Breath Test Analyzer complies with the requirements stipulated in WI-QA-16-C2 "Inspection Instruction for Breath Test Analyzer" and is judged as conforming.		
Remarks			

Inspected by: Lu Wei

Reviewed by: Hongyanjun Approved by: Jin Guohua

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Shenzhen Zhonghe Headway Bio-Sci & Tech Co., Ltd.

Warranty Certificate

Thank you very much for using our products. As a user of this product, you will get commitments for the after-sales service of this product as below:

1. The device Breath Test Analyzer HCBT-01 is warranted for a period of twelve (12) months from the date when it is freighted provided it is used according to the method and procedure as specified on the User's Manual.
2. When a problem occurs within the warranty period, we will provide free maintenance service.
3. When a problem occurs outside the warranty period, we will charge appropriate maintenance fees and material costs for repairing the device.
4. Please contact us at export@headwaychina.com, tel: 0086-755-26619490 or contact your local distributor to get swift and high quality service.

Product Information

Product:	Breath Test Analyzer
Type:	HCBT-01
Serial Number:	202304030
Inspection Date:	May.8th,2023
Delivery Date:	
User's Name:	
User's Address:	
User's Tel:	
Contact Person:	

Add: 6/F, Bldg R1-B, South District, High-Tech Industrial Park, Shennan Road, Shenzhen, 518057, China
Tel: +86-755-26619490 Fax: +86-755-26617554
Email: export@headwaychina.com
Web: www.headwaychina.com

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Model: HCBT-01

Instrument SN: 202304030

No.	Item	Unit	Quantity	Check
1	Breath test analyzer	unit	1	☒
2	Power cord (250V, 10A)	pc	1	☒
3	USB cable	pc	1	☒
4	T3.15A 250V Fuse	pc	2	☒
5	Y-shaped pipe	pc	1	☒
6	Breath bag (4.5L)	pc	1	☒
7	Baseline breath bag (120ml)	pc	2	☒
8	Sample breath bag (120ml)	pc	2	☒
9	CO ₂ absorber	pc	1	☒
10	CO ₂ absorber protective cap	pc	1	☒
11	User manual	pc	1	☒
12	Software CD	pc	1	☒
13	Packing list	pc	1	☒
14	Warranty certificate	pc	1	☒
15	Quality certificate	pc	1	☒
16	Test report	pc	1	☒
17	Type O circle (5×1.5mm)	pc	24	☒

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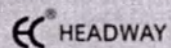
BASLINE BREATH BAG

(Model No.: BB-02)
(BEFORE TAKING THE CAPSULE)

[Name]: _____

[Sex]: _____

[Age]: _____

[No.]: _____

[Date]: _____

Lot No.:

Mfg. Date:

Exp. Date:

[European representative]

Name: Prolinx GmbH

Add: Brehmstr 56, 40239, Duesseldorf, Germany

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BASLINE BREATH BAG

(BEFORE TAKING THE CAPSULE)

INSTRUCTIONS:

1. Remove the cap.

2. Hold breath, then blow into the bag to fill it in.

3. Fasten down the cap.

KINDLY REMINDER:

1. The test should be conducted after fasting for at least two hours.
2. This bag cannot be reused.
3. This bag should be stored below 20 °C / 65% RH, kept tightly sealed.

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 HEADWAY

SAMPLE BREATH BAG
(Model No.: BB-02)
(AFTER TAKING THE CAPSULE)

[Name]: _____

[Sex]: _____

[Age]: _____

[No.]: _____

[Date]: _____

Lot No.: _____
Mfg. Date: _____
Exp. Date: _____

  [European representative]
Name: Prolinx GmbH
Add: Brehmstr 56, 40239, Duesseldorf, Germany

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SAMPLE BREATH BAG
(AFTER TAKING THE CAPSULE)

INSTRUCTIONS:

-  1. Take one Urea- ^{13}C Capsule with water.
-  2. Sit calmly for 30 minutes.
-  3. Remove the cap.
-  4. Hold breath, then blow into the bag to fill it in.
-  5. Fasten down the cap.

KINDLY REMINDER:

1. The test should be conducted after fasting for at least two hours.
2. This bag cannot be reused.
3. This bag should be stored below 20 °C / 65%RH, kept tightly sealed.

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