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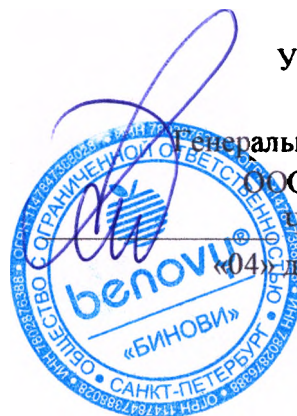
Gulin HBM Health Protections, Inc., China, No. 1-2, Shuijing East Road, Economic and Technological Development Area, 541805 Gulin, Guangxi, China.

AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER) ON THE TERRITORY OF THE RUSSIAN FEDERATION: «BENOVY» Company Limited, Russia, 194292, St. Petersburg, 1. 5-y Verkhny per., lit. J, office 205, +7 (812) 309-16-32, www.benovy-company.ru, benovy@benovy-company.ru

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SHELF LIFE OF THE MEDICAL DEVICE. The established shelf life of surgical gloves is 3 years. Do not use after the expiry date.

ATTENTION!

THE PRODUCT IS MADE OF NATURAL RUBBER LATEX, WHICH MAY CAUSE AN ALLERGIC REACTION. THE GLOVES DO NOT GUARANTEE 100% HAND PROTECTION AGAINST PUNCTURES, CUTS, AND DAMAGE CAUSED BY EXPOSURE TO CHEMICALS. STERILITY GUARANTEED WITH PACKAGE INTEGRITY.

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**ENG**

Surgical disposable sterile natural latex gloves "BENOVY" anatomical with curved fingers textured powder-free with inner polymer coating, antibacterial (beaded cuff), color: white, size: 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0.

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**AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER) ON THE TERRITORY OF THE RUSSIAN FEDERATION:** "BENOVY" Company Limited, Russia, 194292, St. Petersburg, 1, 5-y Verkhniy per., lit. J, office 205, +7 (812) 309-16-32, [www.benovy-company.ru](http://www.benovy-company.ru), [benovy@benovy-company.ru](mailto:benovy@benovy-company.ru)

**IMPORTER:** XXXXXXX

**SHELF LIFE OF THE MEDICAL DEVICE.** The established shelf life of surgical gloves is 3 years. Do not use after the expiry date.

**ATTENTION!**

**THE PRODUCT IS MADE OF NATURAL RUBBER LATEX, WHICH MAY CAUSE AN ALLERGIC REACTION. THE GLOVES DO NOT GUARANTEE 100% HAND PROTECTION AGAINST PUNCTURES, CUTS, AND DAMAGE CAUSED BY EXPOSURE TO CHEMICALS. STERILITY GUARANTEED WITH PACKAGE INTEGRITY.**

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**ENG**

Surgical disposable sterile natural latex gloves «BENOVY» anatomical with curved fingers textured powder-free with inner polymer coating, antibacterial (beaded cuff), color: white, size: 6.0, 6.5, 7.0, 7.5, 8.0, 8.5, 9.0.

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**AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER) ON THE TERRITORY OF THE RUSSIAN FEDERATION:** «BENOVY» Company Limited, Russia, 194292, St. Petersburg, 1, 5-y Verkhniy per., lit. J, office 205, +7 (812) 309-16-32, [www.benovy-company.ru](http://www.benovy-company.ru), [benovy@benovy-company.ru](mailto:benovy@benovy-company.ru)

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Gulin HBM Health Protections, Inc., China, No. 1-2, Shuang East Road, Economic and Technological Development Area, 541805 Gulin, Guangxi, China.

**AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER) ON THE TERRITORY OF THE RUSSIAN FEDERATION:** «BENOVY» Company Limited, Russia, 194292, St. Petersburg, 1, 5-y Verkhny per., lit. J, office 205, +7 (812) 309-16-32, [www.benovy-company.ru](http://www.benovy-company.ru), [benovy@benovy-company.ru](mailto:benovy@benovy-company.ru)

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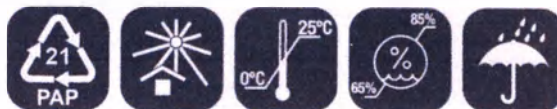
## GLOVES

ENG

**Surgical disposable sterile natural latex gloves "BENOVY" anatomical with curved fingers textured powder-free with inner polymer coating, antibacterial (beaded cuff), color: white, size: X.X.**

**DESIGNATION OF THE MEDICAL DEVICE.** The gloves are intended for single use in protecting medical personnel's hands and creating a barrier against bacterial and/or viral contamination between the health worker and the patient during surgical interventions and other sanitary actions that require compliance with sterility conditions.

**TRANSPORTATION, STORAGE AND DISPOSAL CONDITIONS OF THE MEDICAL DEVICE.** Transportation of the device can be carried out by all types of covered vehicles as per the transportation rules applied to this type of transport. During transportation the storage conditions must not be violated. The devices must be stored in a dry place, protected from direct sunlight or bright artificial light having a high ultraviolet content, at a temperature of 0°C to 25°C and relative air humidity – from 65 % to 85%. The devices should be handled with caution when they are stored at temperature below 15°C, as during careless handling, they can be stiff and sensitive to deformation. After storage at a low temperature, gloves should be warmed up to 30°C before use throughout all device volume. Storage rooms should not contain ozone-generating equipment, such as mercury lamps or high voltage electrical equipment, generating excessive sparks or inaudible electric discharges. The devices must be stored away from sources of ignition. Combustible gases and organic vapors should be excluded from the storage facilities. The used medical device belongs to the category of Infectious Wastes (Class B Wastes according to the regulatory guidelines of the Russian Federation SanPiN 2.1.3684-21). The collection, temporary storage, and removal of waste should be performed as per the medical waste management scheme adopted by the organization that carries out healthcare and/or pharmaceutical practice.

**MATERIAL:** natural rubber latex.

**ATTENTION! THE PRODUCT IS MADE OF NATURAL RUBBER LATEX, WHICH MAY CAUSE AN ALLERGIC REACTION. THE GLOVES DO NOT GUARANTEE 100% HAND PROTECTION AGAINST PUNCTURES, CUTS, AND DAMAGE CAUSED BY EXPOSURE TO CHEMICALS. STERILITY GUARANTEED WITH PACKAGE INTEGRITY.**

**INSTRUCTION FOR USE FOR THE MEDICAL DEVICE.** Before using the gloves health workers carry out hygienic treatment of their hands having performed surgeon's hand treatment technology. It is necessary to strictly follow the rules for donning gloves so as not to violate the sterility of their work surface. Open the individual outer package of sterile gloves (away from the sterile table), remove ("shake out") from it the inner wallet with gloves on a sterile surface. Then open the inner wallet with sterile forceps or aseptically treated hands. Don gloves according to the surgical donning technique. Remove used gloves carefully, avoiding splashes from the surfaces of the glove, which could cause microbial contamination of the hands and the environment. Remove gloves according to the surgical removing technique. Immerse both gloves in a container with a disinfectant solution for disinfection or in a disposable container for temporary waste storage for subsequent centralized disinfection/neutralization. After finishing the work and gloves removal, it is necessary to carry out hygienic hand treatment. To prevent dryness and adverse effects on the hand skin, it is recommended for health workers to always use professional nourishing hand cream.

**EXPLOITATION CONDITIONS OF THE MEDICAL DEVICE.** Gloves must be used as per the instruction on the medical device. The use of the device does not require special qualification or special education. Medical gloves are used in medical or healthcare facilities indoors, in a dry place, away from direct sunlight, at a temperature of 15°C to 25°C and relative humidity from 65% up to 85%. The device is for single use only. Do not use in case of size mismatch. For individual use only.

Guilin HBM Health Protections, Inc., China, No. 1-2, Shuijing East Road, Economic and Technological Development Area, 541805 Guilin, Guangxi, China.

**AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER) ON THE TERRITORY OF THE RUSSIAN FEDERATION:** "BENOVY" Company Limited, Russia, 194292, St. Petersburg, 1, 5-y Verkhny per., lit. J, office 205, +7 (812) 309-16-32, [www.benovy-company.ru](http://www.benovy-company.ru), [benovy@benovy-company.ru](mailto:benovy@benovy-company.ru)

**SHELF LIFE OF THE MEDICAL DEVICE.** The established shelf life of surgical gloves is 3 years. Do not use after the expiry date. **MANUFACTURER'S WARRANTY.** The manufacturer guarantees the preservation of all parameters and characteristics of the gloves throughout the shelf life (3 years) upon the condition of package integrity, observance of storage, transportation and operation conditions in accordance with the instruction and acceptable quality level AQL ≤ 1,5.

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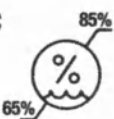
## WHITE

## LATEX

 Guilin HBM Health Protections, Inc., China, No. 1-2, Shuijing East Road, Economic and Technological Development Area, 541805 Guilin, Guangxi, China.

**AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER (PRODUCER)**  
**ON THE TERRITORY OF THE RUSSIAN FEDERATION:** "BENOVY" Company Limited,  
Russia, 194292, St. Petersburg, 1, 5-y Verkhniy per., lit. J, office 205, +7 (812) 309-16-32,  
[www.benovy-company.ru](http://www.benovy-company.ru), [benovy@benovy-company.ru](mailto:benovy@benovy-company.ru)

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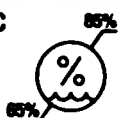
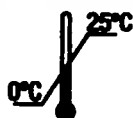
**XX pairs / XXX pairs**

**THE MEDICAL DEVICE IS STERILE. DURING TRANSPORTATION  
DO NOT VIOLATE THE PACKAGE INTEGRITY!**

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