Medtronic

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
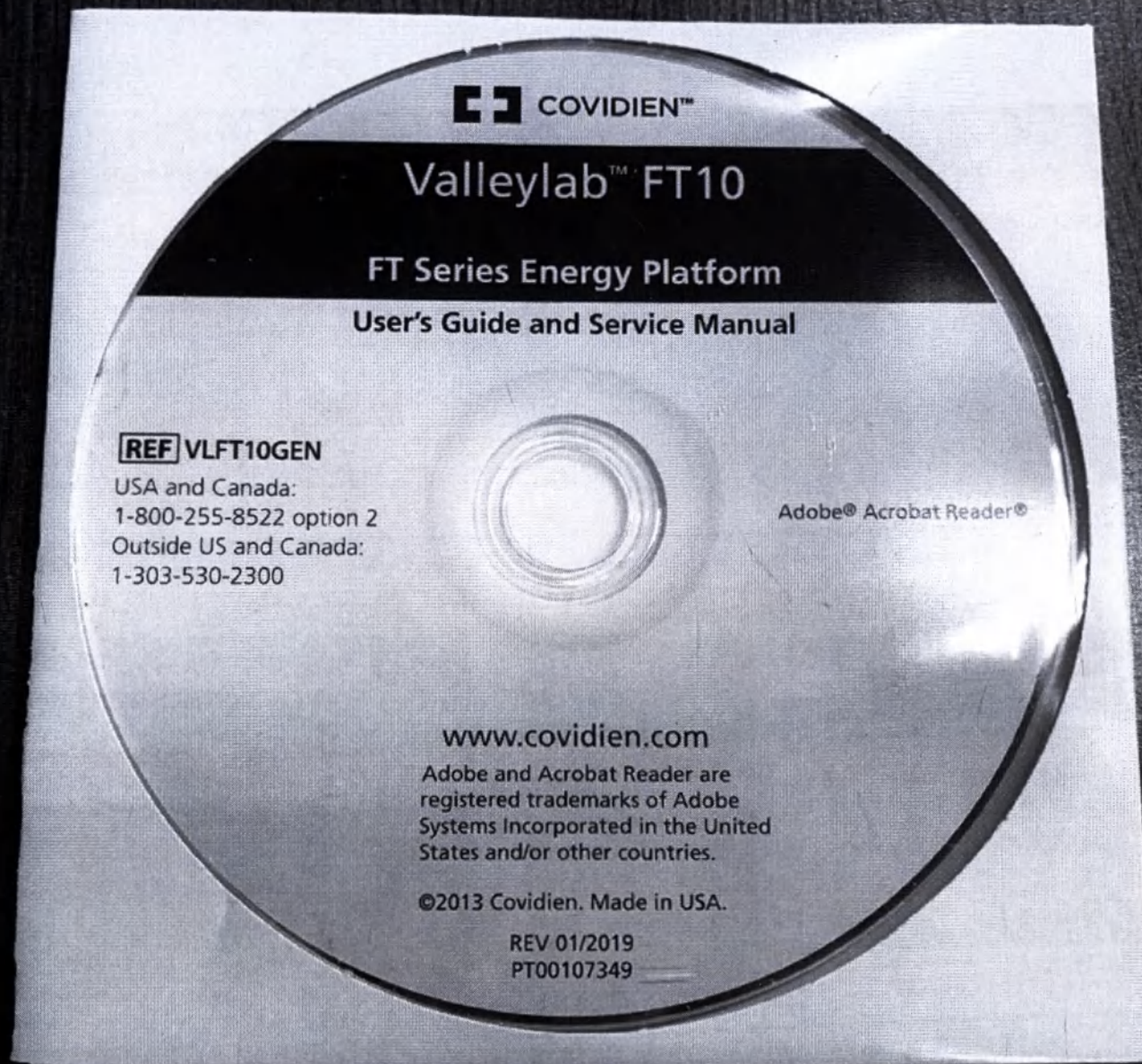
COVIDIEN

positive results for life®


Valleylab FT10 Series Energy Platform

Quick Reference Guide

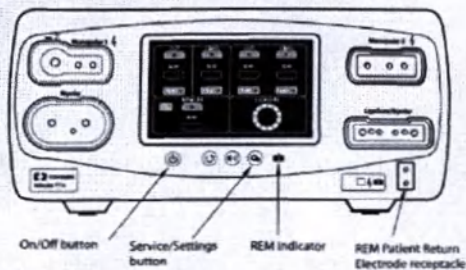Front

Back

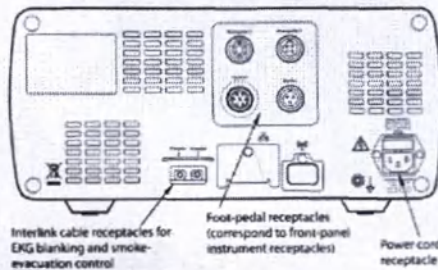Touchscreen Display

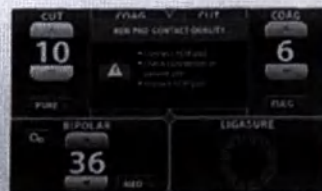
Black & Gray - System is on and no instruments are attached. Settings can be preset prior to attaching an instrument to set up for upcoming procedures.


Color on controls - Color added to each quadrant when instrument is attached. Mode and output selections have been entered.

Monopolar Setup

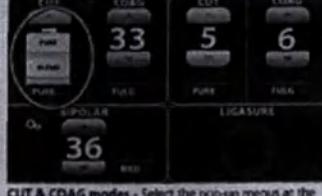REM PAD error - Attach a required patient return electrode with REM (Return Electrode Monitoring) to the generator and patient. The error is removed.

CUT & COAG modes - Select the pop-up menus at the bottom of the quadrant to view/select mode options. CUT options are PURE or BLEND; COAG options are SOFT, FULGURATE & SPRAY.

CUT & COAG - The two available monopolar modes. Select up or down arrows to increase or decrease power settings.

Pure CUT power setting - Increased to 10 W using the up arrow.

Bipolar Setup

BIPOLAR power - Select up or down arrows to increase or decrease power settings.

LOW, MEDIUM, HIGH options - Select the pop-up menus at the bottom of the quadrant to view/select mode options. The user has set to BIPOLAR HIGH at 45 W.

First-Time Setup and Software Update Guide

VALLEYLAB™ FT10 ENERGY PLATFORM

Follow the steps below to download and install the Valleylab™ Exchange Remote Software System, set up your energy platform for first-time use, or perform a software update. If you need additional help, go to medtronic.com/covidien/support/valleylab-exchange, select the Valleylab™ FT10 energy platform, and download the Valleylab™ Exchange User's Guide for the Valleylab™ FT10 Energy Platform.

IMPORTANT The steps on pages 1 and 2 must be performed with an internet connection prior to connecting to an energy platform.

Installing and Configuring Valleylab™ Exchange

NOTE Do not connect the energy platform to your computer until instructed.

Valleylab
Exchange
STEP 2

1. Go to medtronic.com/covidien/support/valleylab-exchange. Select the Valleylab™ FT10 energy platform on the website. On the First-Time Setup tab, download the Valleylab™ Exchange Installer to your desktop.
2. Locate the ValleylabExchange.exe Installer icon on your desktop. Double-click it, and follow the prompts to install the software.

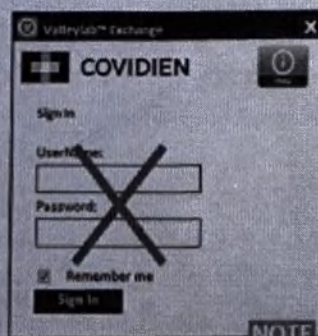
VLEXClient
STEP 3

NOTE The Valleylab™ Exchange application works with the Windows™ platform. For information on which operating systems are currently supported, please refer to the Valleylab™ Exchange User's Guide, which can be downloaded at medtronic.com/covidien/support/valleylab-exchange.

3. Locate the VLEXClient icon on your desktop and double-click it to start the application. When the User Account Control pop-up displays, click Yes.

NOTE For first-time setup, a username and password are not required.


NOTE


4. Click First-Time Setup located in the lower-left corner of the Valleylab™ Exchange Sign In screen. This will establish the initial connection to the server to configure Valleylab™ Exchange.
5. When the new window opens, verify a green indicator ● next to Server in the menu on the left, and a green indicator ● showing Online in the lower-right corner.
6. Click Logout in the upper-right corner of the screen.
7. Click the ☒ in the upper-right corner of the screen to close the Valleylab™ Exchange application. Configuration of Valleylab™ Exchange is complete.

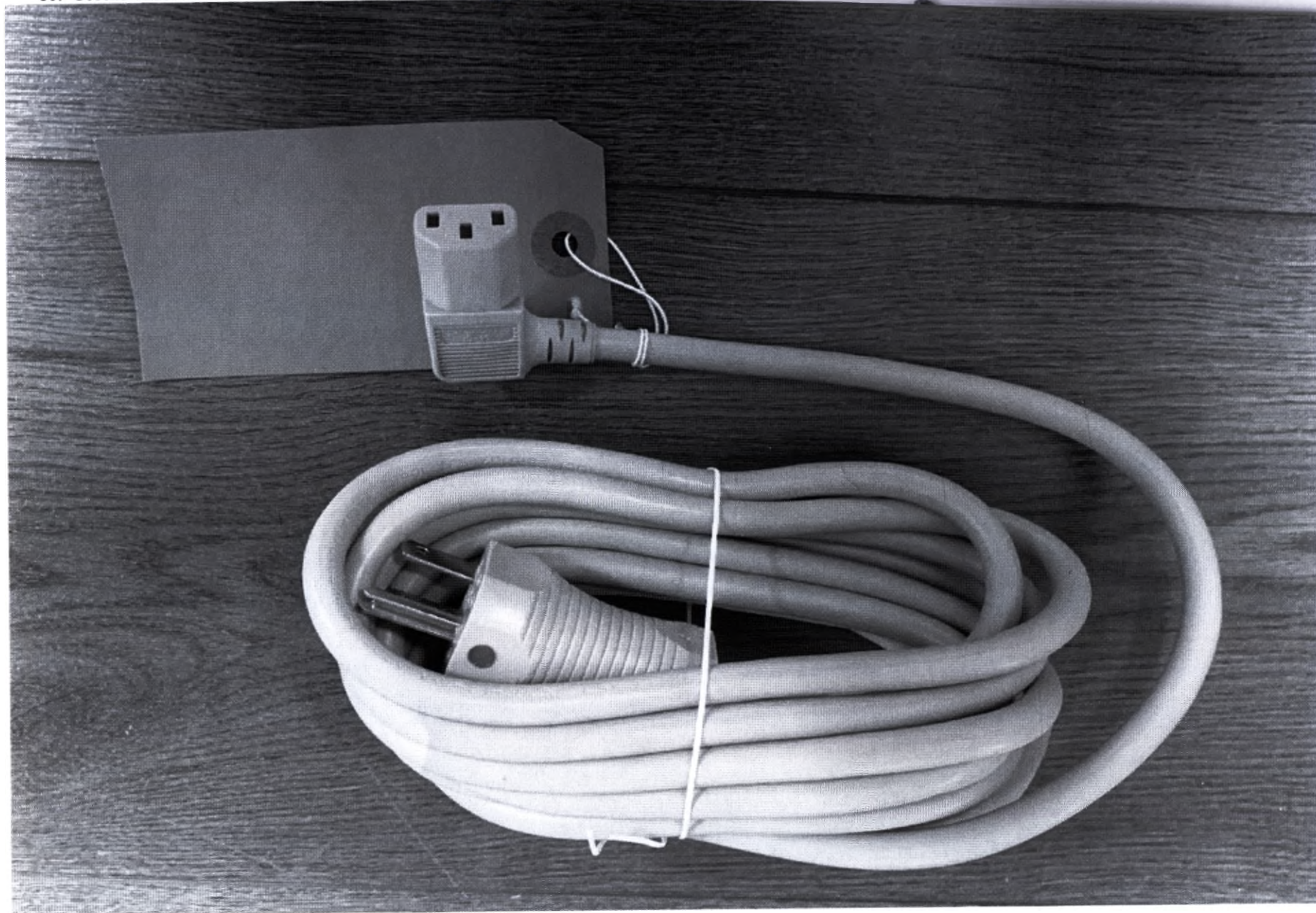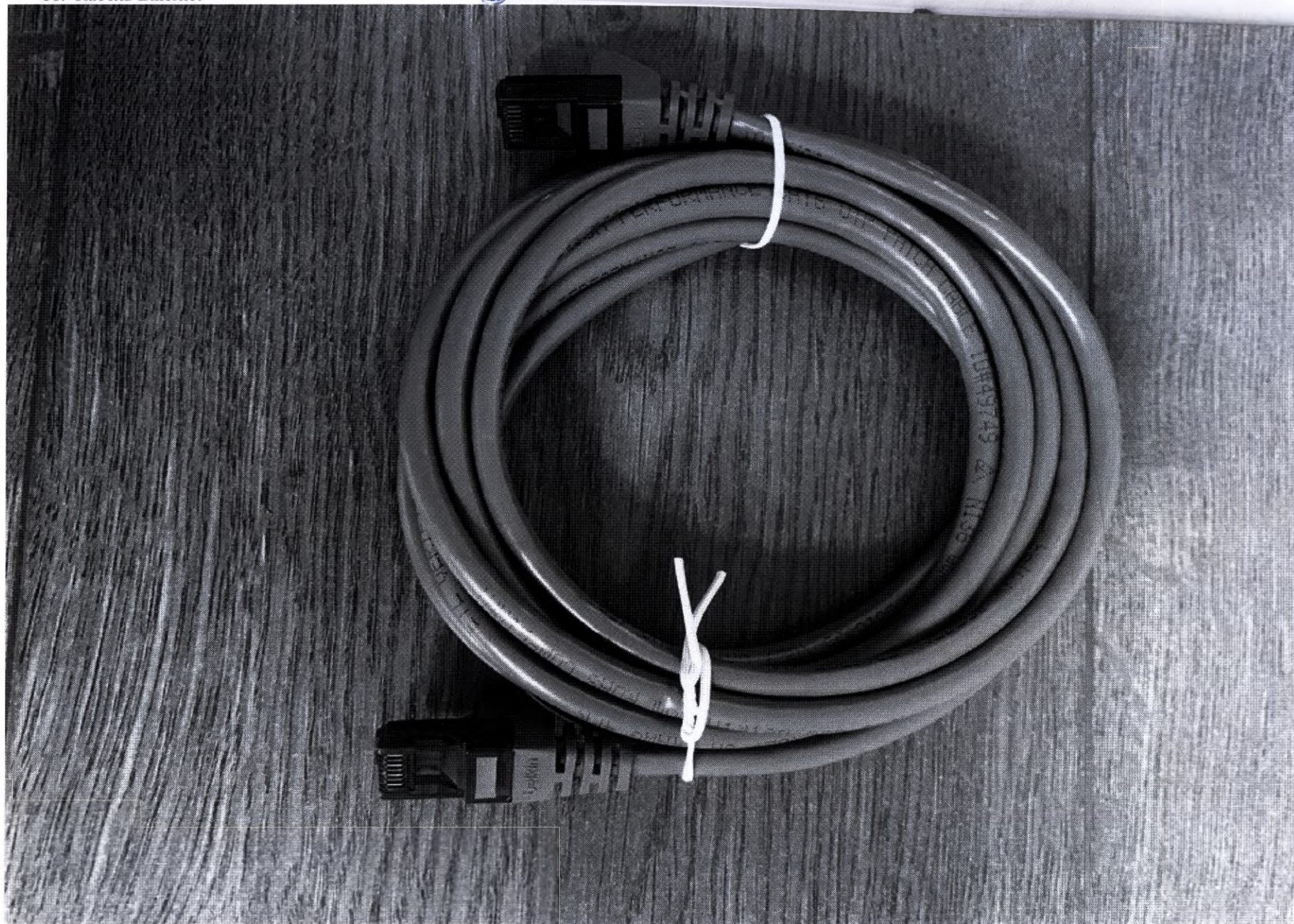IMPORTANT Before you proceed, you must have a Valleylab™ Exchange username and password. To set up an account, start the Valleylab™ Exchange application and click Create Account. It may take up to 3 business days to receive your username and password.

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COVIDIEN
positive results for life

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2021-01-23

RoHS **REACH** **CE** **0086**

Not made with natural rubber latex
Consult instructions for use
Humidity limitations
Temperature limitations

© 2013 Covidien. Made in USA.
Covidien LLC, 15 Hampshire Street, Bedford, MA 01730 USA.
KONE Covidien Ireland Limited, 25A Broomfield Technology Park, Broomfield, Dublin 24 Ireland.
0120079000

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