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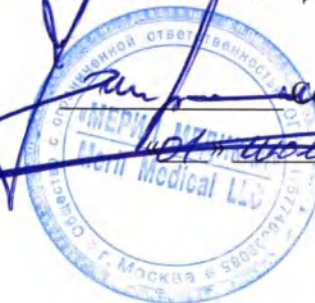
- 2). To optimize the identification conditions (aqueous phase, pH, buffer, ionic strength, organic solvent) in order to improve the efficiency of the extraction of the target compounds.

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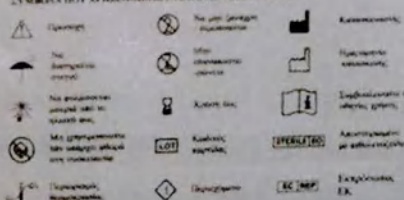
Instruction for use

MIRUS™ LINEAR CUTTER AND RELOAD

CAREFULLY READ ALL INSTRUCTIONS PRIOR TO USE.

DESCRIPTION

Mirus™ Linear Cutter stapler and the Reload have two groups of staple lines in the target area, both of which are composed of a double row of titanium staples interlaced with each other and automatically cut the tissues between the two groups of staple lines when suturing. There are three suturing lengths of 60mm, 80mm and 100mm. The cartridge/reload for Mirus™ Linear Cutter are supplied separately.

INDICATIONS

Miras™ Linear Cutter and Reload have application in gastrointestinal, gynecologic, thoracic, and pediatric surgery for transection, resection, and/or creation of anastomoses and can be used with staple line or tissue buttressing materials.

PERFORMANCE/MECHANISM OF ACTION

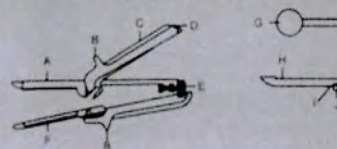
The stapler holds the staples in 2 linear rows within the staple cartridge. The linear staples, on firing (squeezing the handle), simultaneously staple and cut tissue to create an anastomosis.

CONTRAINDICATIONS

1. The product is not applicable to patients with alimentary canal mucosal edema, excessively thick muscular layer or weak healing capacity.
2. The product is not applicable for patients with suspected residual carcinoma tissues at the cutting end or severe injury of the cutting end.
3. The product is not applicable to patients who cannot tolerate traditional suture materials or suture technology.
4. MIRA™ Linear Cutter should not be used to cut or staple tissues that are necrotic, friable, or have altered integrity, e.g., ischemic or edematous tissues.

SCHEMATIC VIEW

- A) Cartridge Fork B) Shoulder C) Locking Lever Handle-cartridge
D) Release Button-cartridge E) Firing Knob F) Anvil Fork
G) Shipping Wedge H) Single Use Loading Unit (Reload)

DIRECTIONS FOR USE

- Insert to select a Relead with the appropriate staple size for the desired thickness.
Note: The stapler may be used for thin, standard and thick tissues. Overly thick (or thin) tissue may result in unacceptable staple formation.
- Open the instrument by pushing in on the release button on the cartridge lock lever handle (located at end of instrument).
- Remove the shipping wedge from Relead channel by lifting and pulling it out at the finger tabs. (Remove shipping wedge after Relead is fully loaded. Relead will "blow" on and down into final loaded position).

4. Position the tissue to be stapled between the instrument cartridge and anvil fork, or, if the tissue is to be anastomosed, place the cartridge fork in one lumen and the anvil fork into the other lumen as appropriate. Align tissue edges equally on cartridge and anvil. This may be accomplished with the instrument halves separated or hinged at the rear. Close the instrument by moving the lever handle towards the instrument body until the audible click is heard.

5. With the instrument securely closed, rotate the firing knob to either side of the instrument.

6. Place a thumb behind the firing knob and two fingers on the instrument shoulders. Fire the instrument by sliding the firing knob forward to a complete stop.
- Caution:** Do not rotate the firing knob during firing. Rotating the knob during firing may cause damage and possible instrument malfunction.

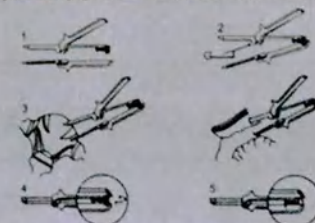
- Note:** The stapler performs only single firing. Failure to advance the handle completely may result in incomplete staple formation, which may compromise the integrity of the staple line.

7. After firing, return firing knob all the way back to its pre-fire position.


8. Push in the cartridge release button to open the instrument (refer to step 1). Remove the instrument from the surgical site.

- Caution:** After firing, always inspect the staple line for hemostasis & integrity by visual inspection for oozing or bleeding. Minor bleeding can be controlled by electrocautery or manual sutures. At the surgeon's discretion, a laparotomy or thoracotomy may be performed.

- Note:** When a fired instrument is opened, the lockout device will deploy. The safety lockout prevents clamping of the instrument with a Reload that has been fired.

RELOADING INSTRUMENT


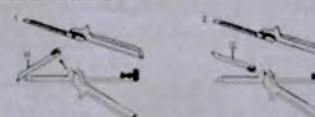
- To remove the third Reload, separate the instrument halves holding the cartridge half of the instrument, grasp the proximal end of the Reload tube and pull up and out to remove Reload from instrument. H) Reload
- To place a new Reload into the instrument, hold the Reload by the proximal end tube and insert into the cartridge lock at a 30 to 45 degree angle from the distal end down until snap into place. Removing shipping wedge after Reload is fully loaded. Reload will "float" up and down in final loaded position. G) Shipping Wedge

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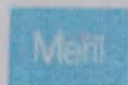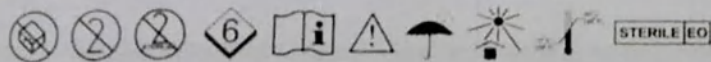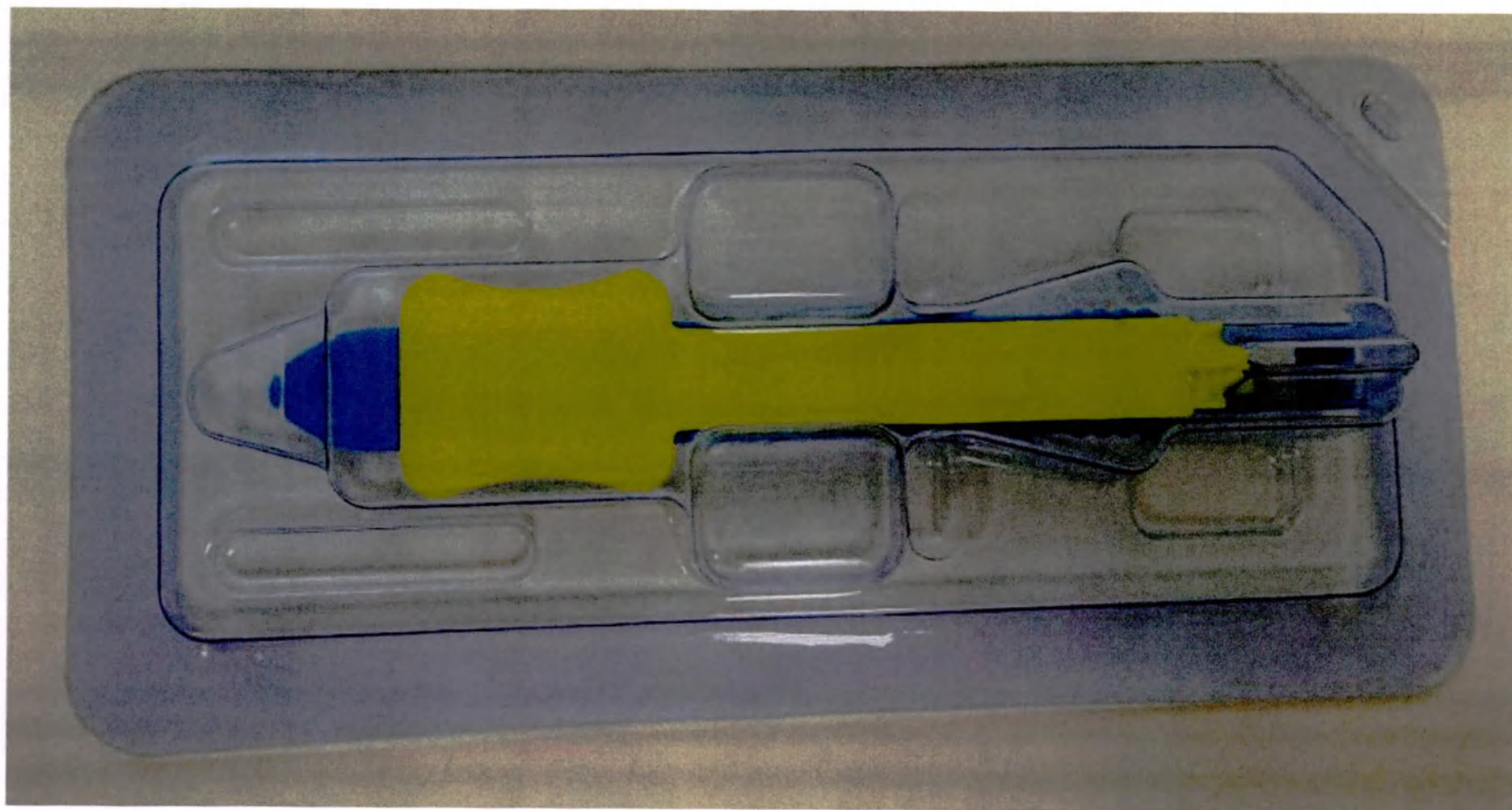
Customer Care No. for India: +91 260 2406005

Customer Care No. for other than India: +91 260 2406005
 W: www.merilife.com, E: enquiry.endosurgery@merilife.com

Mfg. Lic. No. 1

FC REP. OBELIS S.A.

Bd. Général Wakis, 53,
 1030 Brussels, Belgium.
 T: +32 2 732 59 54 F: +32 2 732 60 03
 E: mail@obelis.net W: www.obelis.net

LOT HLC8038089

DEC 2022

REF MLCR80-3.8

NOV 2025

(01)08906070909613

FOR EXPORT
 ONLY

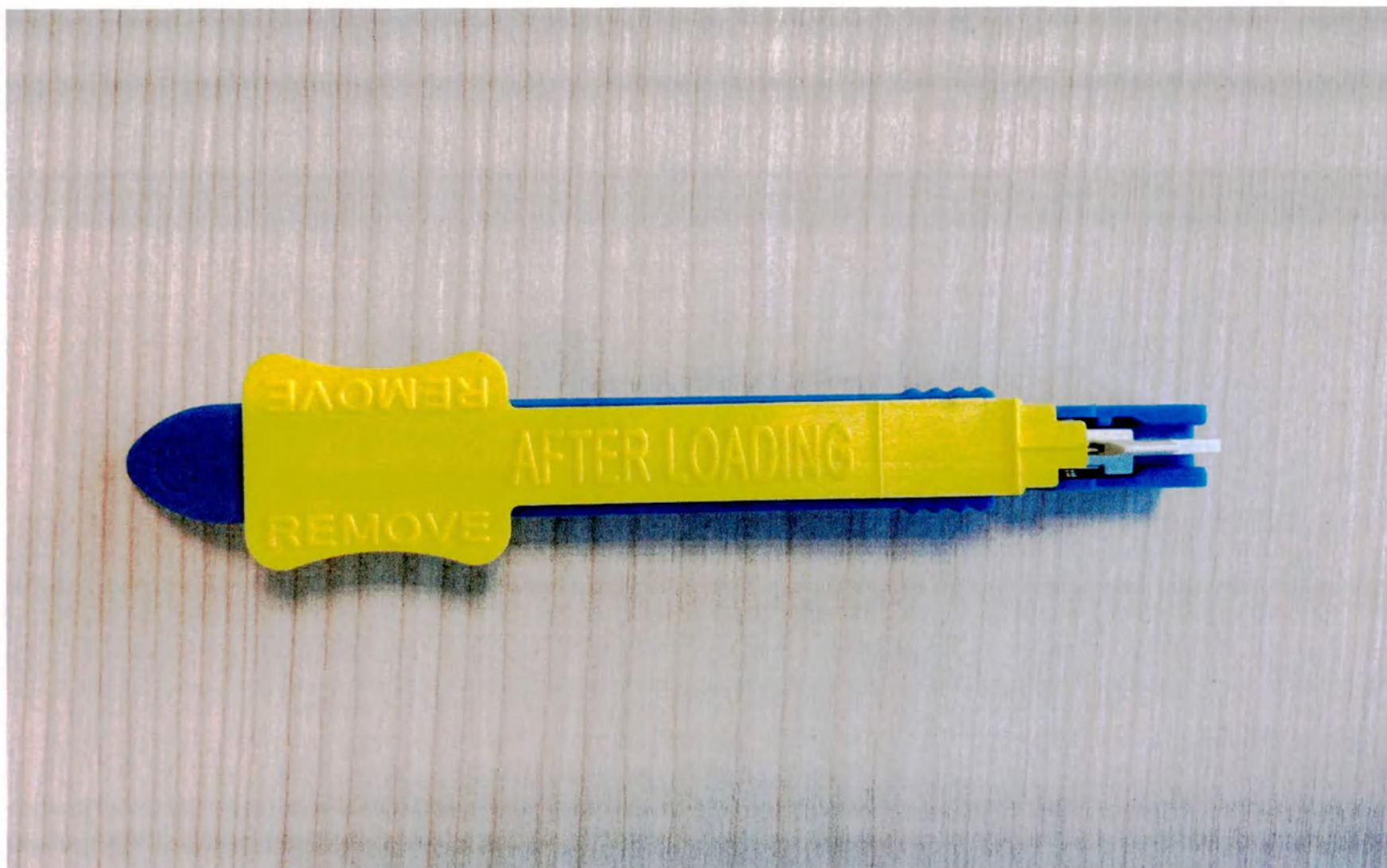
MIRUS™
 Meril
 Endo-Surgery
 Linear Cutter Reload
 (Blue)(80mm-3.8mm)
 LOT HLC8038089 DEC 2022
 REF MLCR80-3.8 NOV 2025
 STERILE EO

MIRUS™
 Meril
 Endo-Surgery
 Linear Cutter Reload
 (Blue)(80mm-3.8mm)
 LOT HLC8038089 DEC 2022
 REF MLCR80-3.8 NOV 2025
 STERILE EO


MIRUS™
 Meril
 Endo-Surgery
 Linear Cutter Reload
 (Blue)(80mm-3.8mm)
 LOT HLC8038089 DEC 2022
 REF MLCR80-3.8 NOV 2025
 STERILE EO

MIRUS™
 Meril
 Endo-Surgery
 Linear Cutter Reload
 (Blue)(80mm-3.8mm)
 LOT HLC8038089 DEC 2022
 REF MLCR80-3.8 NOV 2025
 STERILE EO

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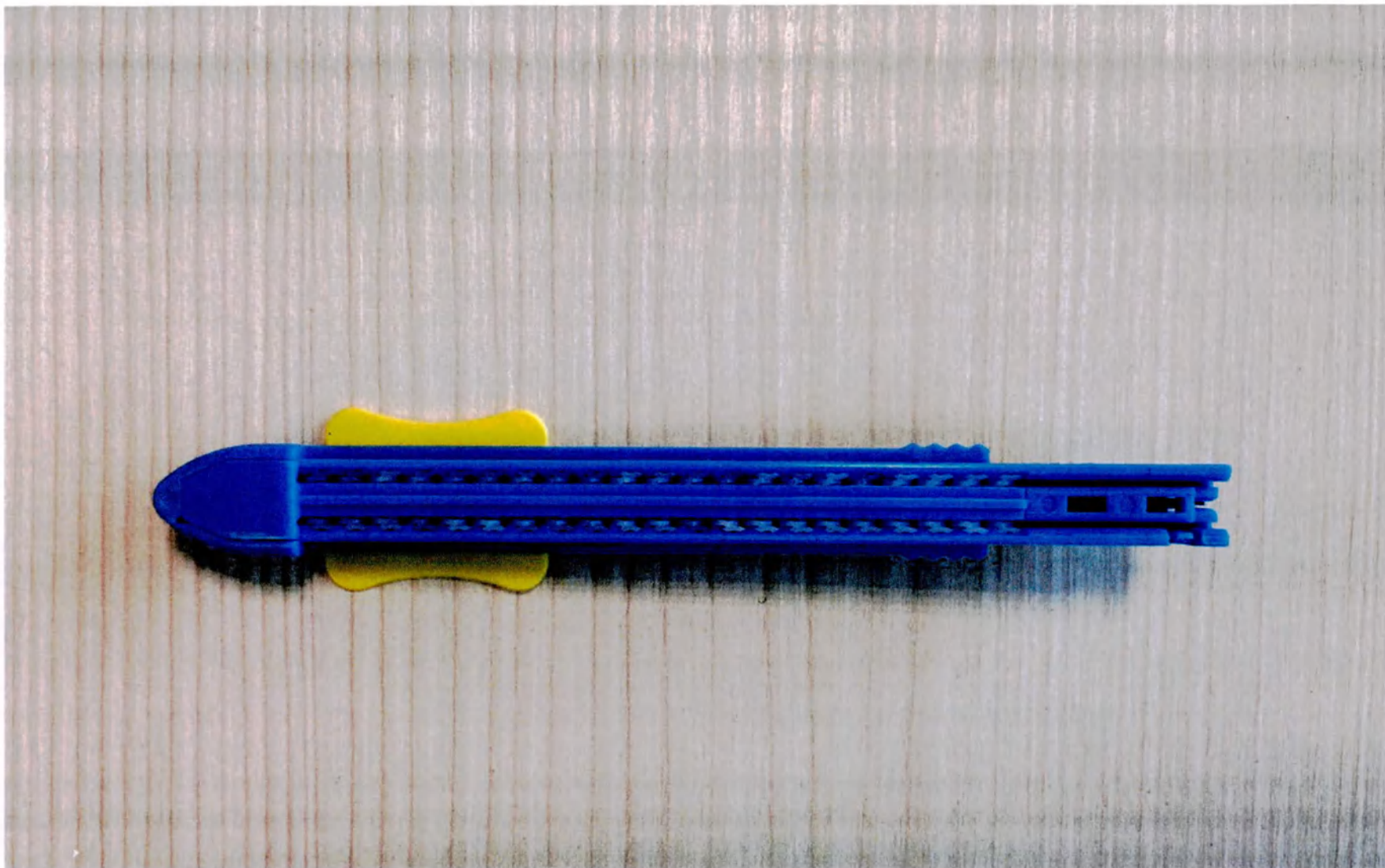
24. May be present, but not always; may be absent.

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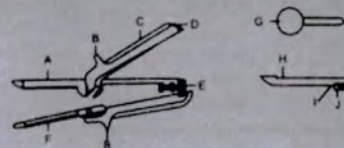
3. Remove the shipping wedge from Reload channel by lifting and pulling it out at the finger tabs. (Remove shipping wedge after Reload is fully loaded. Reload will "float" up and down in final loaded position).

7. After firing, return firing knob all the way back to its pre-fire position.

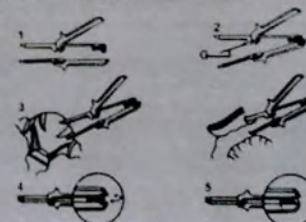- Note:** When a fired instrument is opened, the lockout device will deploy. The safety lockout prevents clamping of the instrument with a Reload that has been fired.

- To place a new Reload into the instrument, hold the Reload by the proximal end tabs and insert into the cartridge fork at a 30 to 45 degree angle from the distal end down until unit snaps into place. Removing shipping wedge after Reload is fully loaded. Reload will "float" up and down in final loaded position. (1) Shipping Wedge

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CE
2195

Manufactured by
Meril Endo Surgery Pvt. Ltd.
Third Floor, E1-E3, Meril Park, Survey No.135/2/B & 174/2,
Muktanand Marg, Chala, Vapi 396191, Gujarat, India.
Customer Care No. for India : 18004194433.
Customer Care No. for other than India : + 91-260-2408005.
W: www.merilife.com, E: enquiry.endosurgery@merilife.com
Mfg. L

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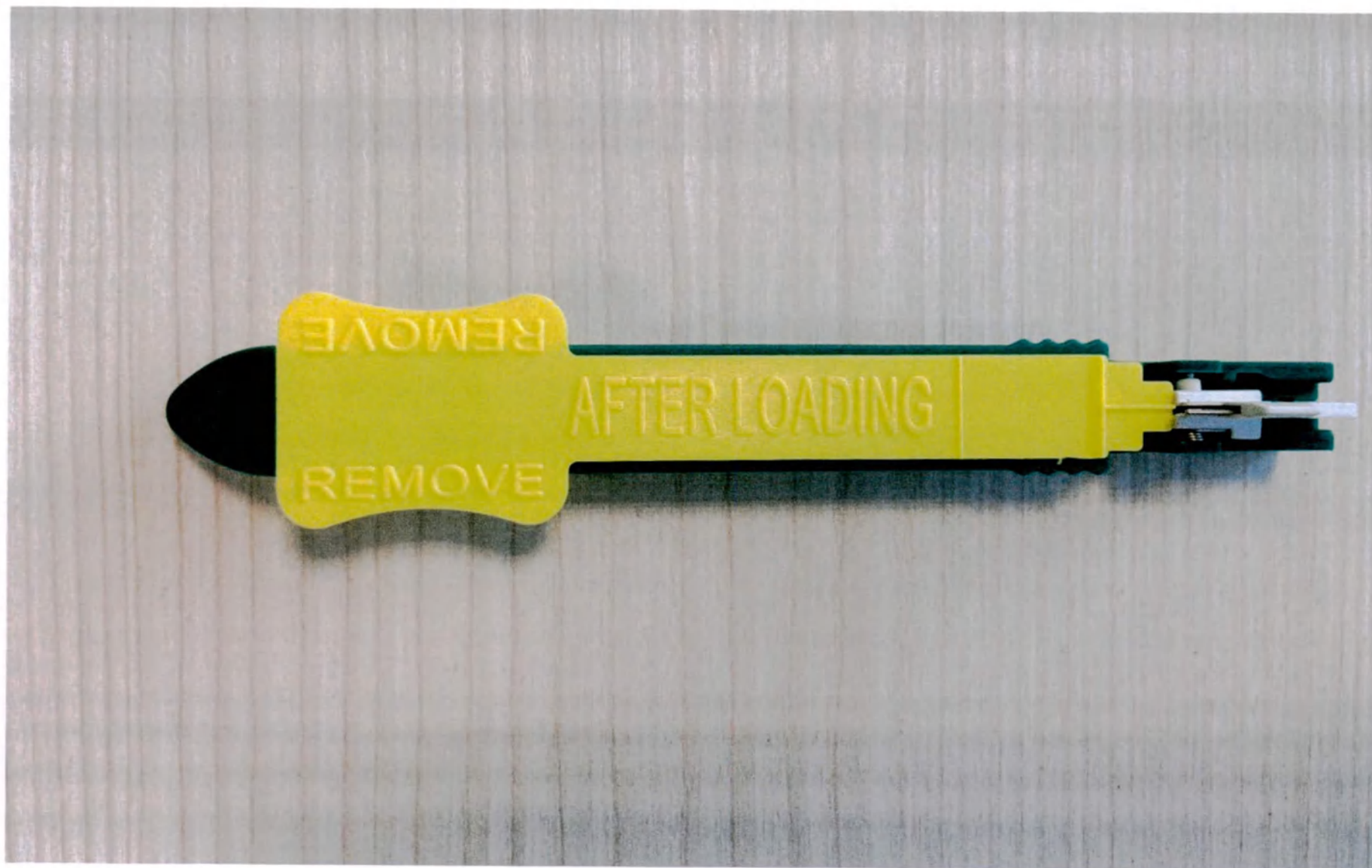
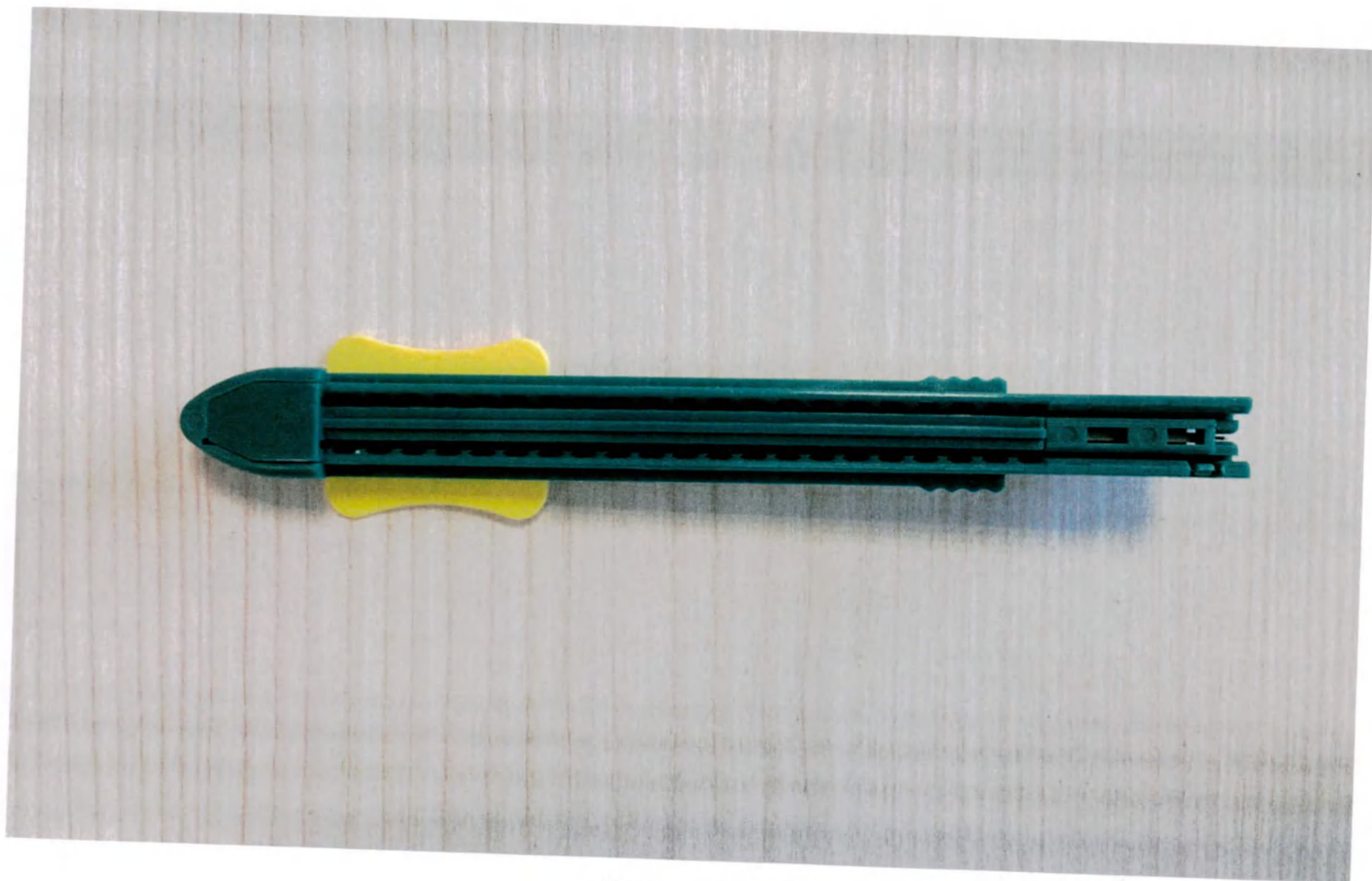
24. May to September: the only time to visit when temperatures are in the 60s

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
Species, age, sex	Nesting, egg-laying and incubation										Remarks
	Incubation period (days)	Egg-laying period (days)	Incubation period (days)	Egg-laying period (days)	Incubation period (days)	Egg-laying period (days)	Incubation period (days)	Egg-laying period (days)	Incubation period (days)	Egg-laying period (days)	
ME 1200	80	80	70	ME 1200-1.0	1.0	1.0	ME 1200	200	Incubated on 10/10/1960	20	
				ME 1200-2.0	2.0	2.0			Incubated on 10/10/1960		
				ME 1200-3.0	3.0	3.0			Incubated on 10/10/1960		
ME 1200	80	80	70	ME 1200-4.0	4.0	4.0	ME 1200	200	Incubated on 10/10/1960		
				ME 1200-5.0	5.0	5.0			Incubated on 10/10/1960		
				ME 1200-6.0	6.0	6.0			Incubated on 10/10/1960		
ME 1200	80	80	70	ME 1200-7.0	7.0	7.0	ME 1200	200	Incubated on 10/10/1960		
				ME 1200-8.0	8.0	8.0			Incubated on 10/10/1960		
				ME 1200-9.0	9.0	9.0			Incubated on 10/10/1960		

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3) Remove the shipping wedge from Reload channel by lifting and pulling it out at the finger tabs. (Remove shipping wedge after Reload is fully loaded. Reload will "float" up and down in final loaded position).

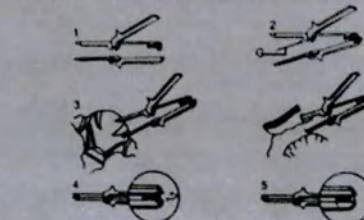-


- To place a new Reload into the instrument, hold the Reload by the proximal end tabs and insert into the cartridge fork at a 30 to 45 degree angle from the distal end down until unit snaps into place. Removing shipping wedge after Reload is fully loaded. Reload will "float" up and down in final loaded position. G) Shipping Wedge

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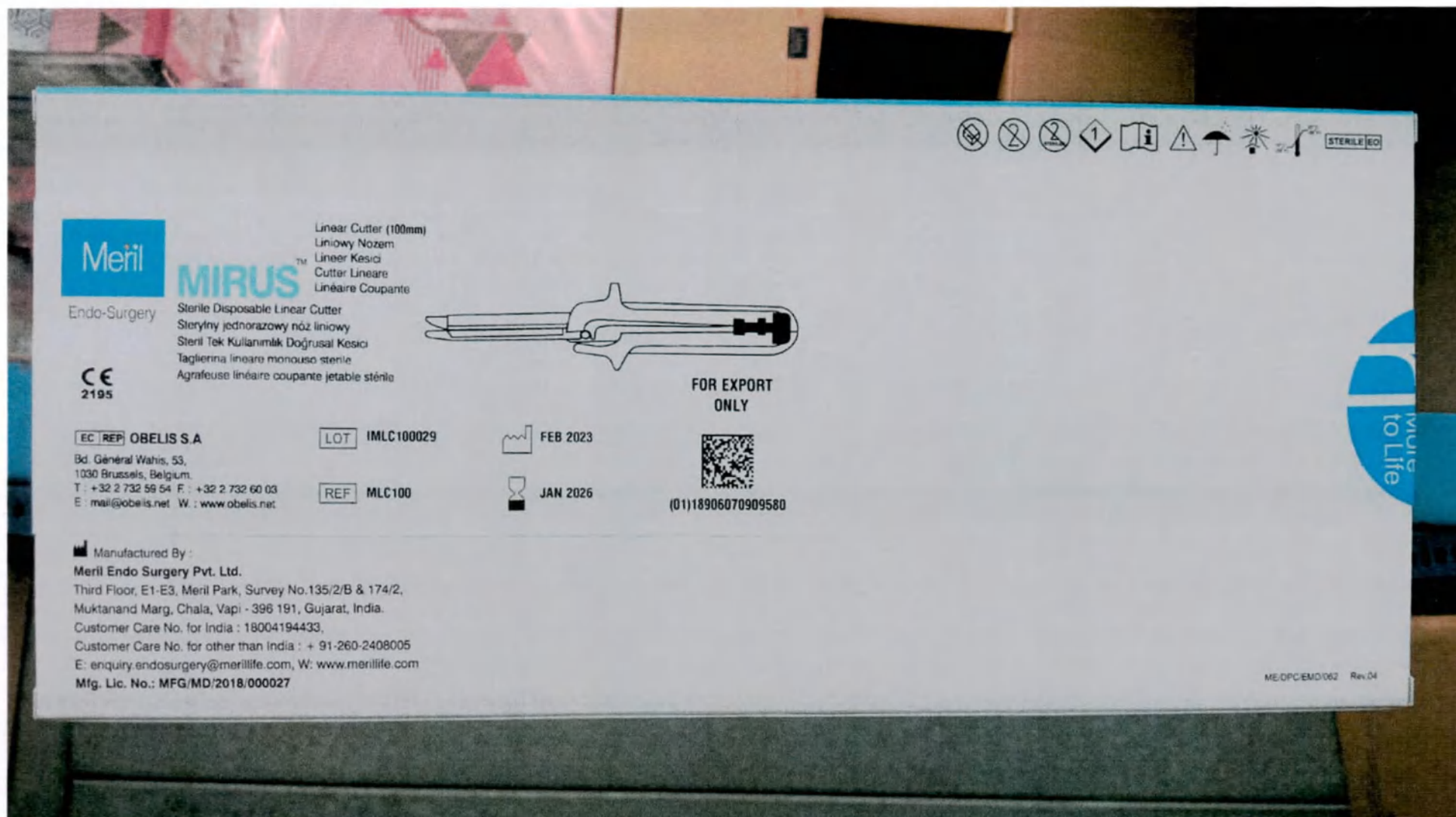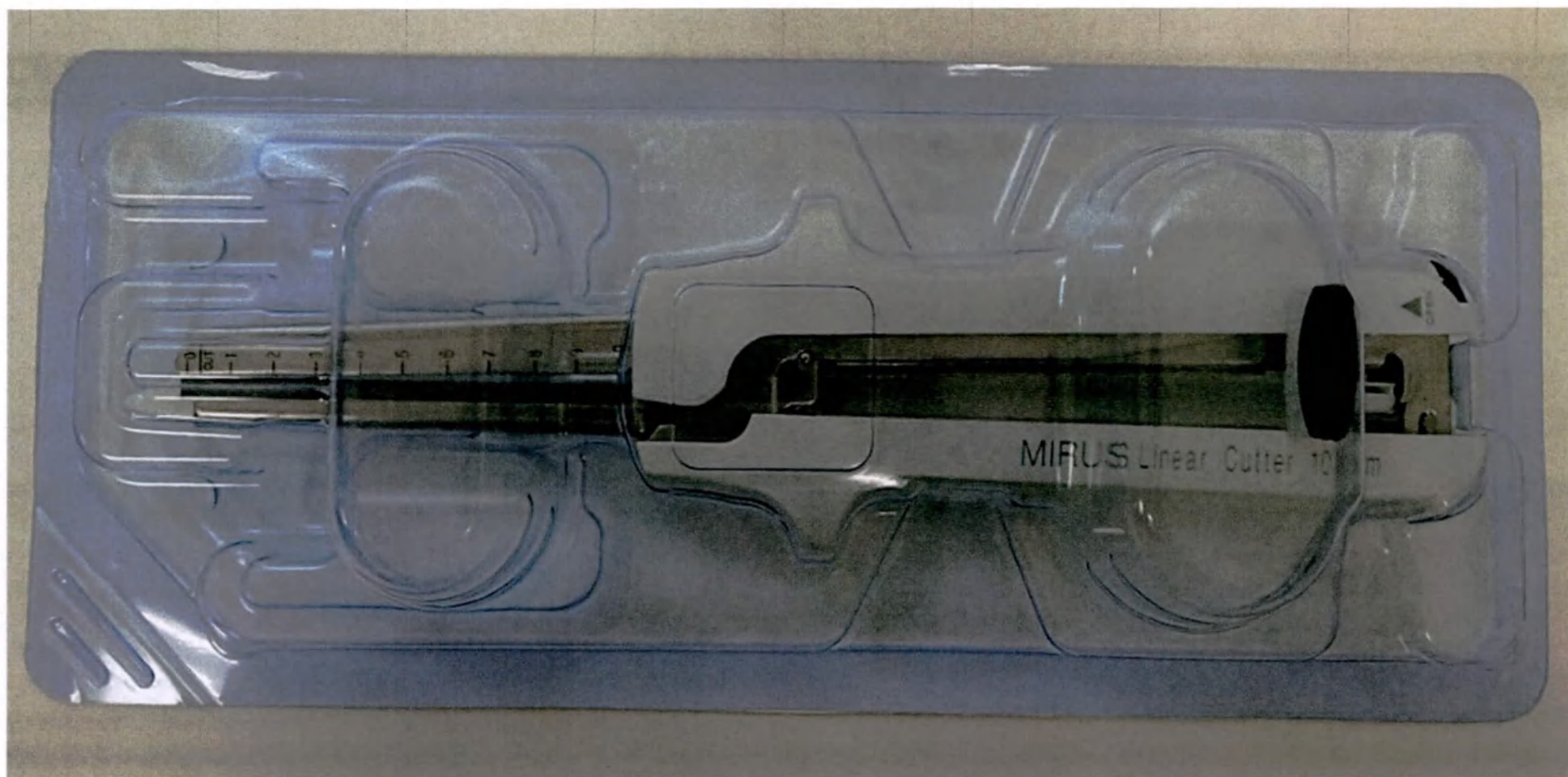
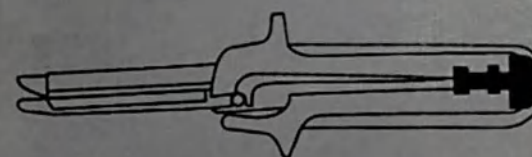
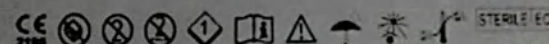
Manufactured By
Meril Endo Surgery Pvt. Ltd.
Third Floor, E1-C3, Meril Park, Survey No.126/2/B & 174/2,
Mukundan Marg, Chole, West - 386 101, Gujarat, India.
Customer Care No. for India : 18004194433,
Customer Care No. for other than India : + 91-260-3408095
E: enquiry.endosurgery@merilife.com, W: www.merilife.com
Mfg. Lic. No.: MP/180/2016/006627
MLC/L/180/2016 Rev J4

EC REP: OBELIS S.A.
Bd Général Waha, 53,
1000 Brussels, Belgium
T: +32 2 732 58 54 F: +32 2 732 60 63
E: mail@obelis.net W: www.obelis.net

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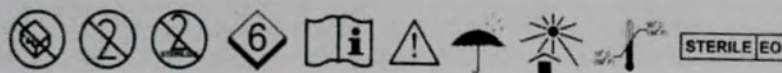
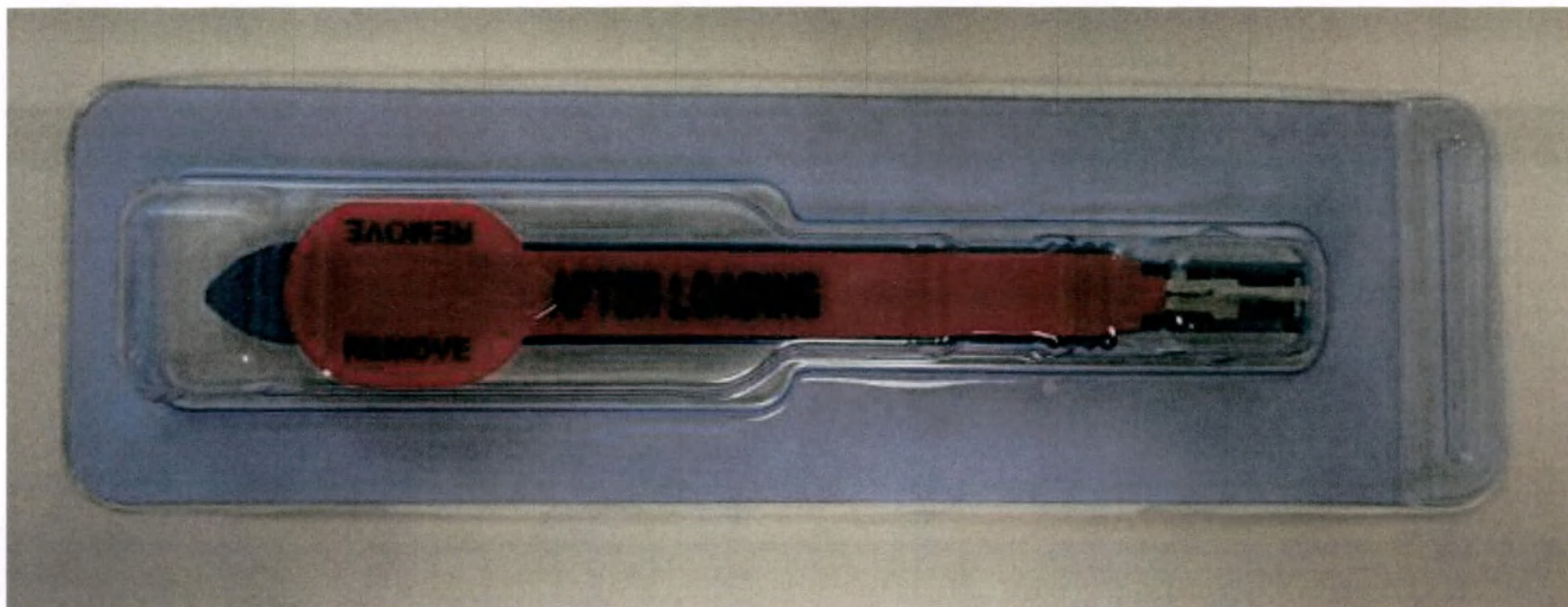
DESCRIPTION

Mirus™ Linear Cutter and Reload have two groups of single bars in the front and back of the instrument. The front bars are used for cutting and the back bars are used for reloading. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

INDICATIONS

Mirus™ Linear Cutter and Reload have applications in general surgery, orthopedics, and trauma. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

PERFORMANCE, MECHANISM OF ACTION

The instrument has two groups of single bars in the front and back of the instrument. The front bars are used for cutting and the back bars are used for reloading. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CONTRAINDICATIONS

The instrument is not recommended for use in patients with bleeding disorders, coagulopathy, or other conditions that may increase the risk of bleeding. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

STERILIZATION

The instrument should be sterilized before use. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CAUTION

The instrument should be used with care. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

RELOADING INSTRUCTIONS

DISINFECTION

The instrument should be disinfected after use. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

STORAGE

The instrument should be stored in a dry, clean place. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

WARRANTY

The instrument is covered by a warranty. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CONTACT

For more information, contact the manufacturer. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

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DESCRIPTION

Mirus™ Linear Cutter and Reload have two groups of single bars in the front and back of the instrument. The front bars are used for cutting and the back bars are used for reloading. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

INDICATIONS

Mirus™ Linear Cutter and Reload have applications in general surgery, orthopedics, and trauma. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

PERFORMANCE, MECHANISM OF ACTION

The instrument has two groups of single bars in the front and back of the instrument. The front bars are used for cutting and the back bars are used for reloading. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CONTRAINDICATIONS

The instrument is not recommended for use in patients with bleeding disorders, coagulopathy, or other conditions that may increase the risk of bleeding. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

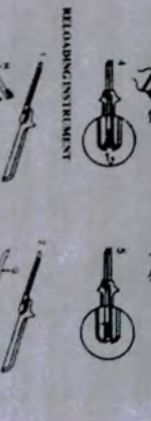
STERILIZATION

The instrument should be sterilized before use. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CAUTION

The instrument should be used with care. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

RELOADING INSTRUCTIONS

DISINFECTION

The instrument should be disinfected after use. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

STORAGE

The instrument should be stored in a dry, clean place. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

WARRANTY

The instrument is covered by a warranty. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

CONTACT

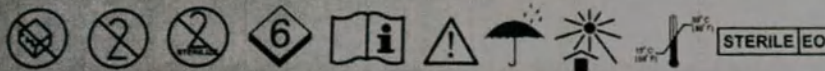
For more information, contact the manufacturer. The instrument is designed for use in the field and is easy to carry. The instrument is made of high quality materials and is very durable.

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E: mail@obelis.net W: www.obelis.net

MIRUS™

Cartridges for Mirus™ Linear Cutter (Green) (100mm – 4.8mm)

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