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CERTIFICATE

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of HYGEDENT INC on the annexed DOCUMENT is genuine.

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证明书查询网址 Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

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APPROVED

By Managing director
of
Hygedent Inc.
Peng Wang

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Alginate Impression Material HYGEDENT,
types

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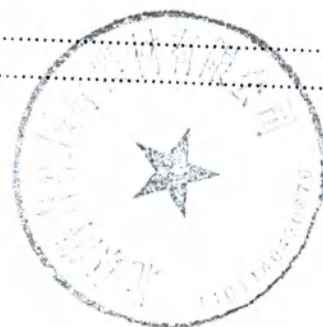
Hereafter in the text, the following designations (abbreviations) are used: Alginate impression material HYGEDENT, alginate impression material, impression material, material(s), medical device.

DISCRIPTION

Alginate impressing material HYGEDENT is a kind of medical device used to make model of teeth. With this material, the dentist could get the actual shape of the patient's teeth and remedy it. This device is a not reusable dust-free powdery material. Mix the powder with water according to the given ratio until becomes fluid paste, after the suggested times, the paste will solidify into solid. The speed of solidification depends on the temperature of the water and environment. The set time provided in the instruction is under 23°C. If the environment temperature is too differential, users can adjust the temperature of water in order to reach the desired temperature.

In normal room temperature (around 23°C), mix the material with water in a correct ratio; a fluid paste will be completed in 30 seconds. Then fill a dental tray with the paste and put it into patient's mouth quickly to make the model of teeth. After 2 ~ 3 minutes, the paste will be solidified, then take the model out of mouth. Finally, mold with gypsum, an accurate model of teeth will be completed.

Our Alginate Impression Material is made of natural Algae Hydrochloric Acid and gypsum. It consists of two parts, one is processed natural mineral, such as diatomite and talcum powder, and the other part has a little inorganic chemical material, such as phosphate. Alginate Impression Materials has been already in market for 70 years, our product has very strong reputation of high quality, advanced technology, good clinic result and great capability in Europe. We have already been in Chinese market over 18 years, according to our sales history of Alginate Impression Material HYGEDENT, we never have health and safety related problems. Because it is disposal product, it does not have infection problems. Also according to its ingredients, the product has neither incentive, explode nor radiation problems. During mixing the powder with water, the product does not release any dangers energy. This product is used for impression and cannot be used for measurement. Based on quality regulation of ISO 21563, this product is also not applicable under disinfecting and sterilizing regulation of 2007/47/EC.

All along our 18 years sale history in China, we provide reliable and high quality products, and all the Alginate Impression Materials HYGEDENT we sold fulfilled the regulation of ISO 21563-2013. The year sale is 600,000kg, besides the domestic market, we has exported the American market, and we obtain USA FDA 510(K) number K140074, K170812.

Our product has biocompatibility and does not contain any harmful cells. The test conducted by SFDA reveals that the capabilities of physical chemistry property, and the TUV

It shows that our product is nontoxic, non-potential variability and harmless product. Thus, the product can be used for clinical purpose.

Implementations and their description:

Implementation	Description
Alginate impression material Hygedent, 454 g	Alginate with high tensile strength. Red colored powder.
Alginate impression material HygePLUS, 454 g	Alginate with high elasticity and short residence time in the mouth. Yellow colored powder
Alginate impression material HygeMAX, 454 g	Alginate with long term dimensional stability and high level of detail reproduction. Orange colored powder
Alginate impression material HygeChroma, 454 g	Three phase alginate material with yellow color. chromatic alginate. Color change accompanies the different processing stages
Alginate impression material Chromatic, 454 g	Chromatic alginate with high resistance to deformation. Color change accompanies the different processing phases

INTENDED USE

Alginate for dental molds (impression)

SCOPE OF APPLICATION

The impression material is intended for use in dental and orthodontic practice.

GENERAL VIEW OF THE DEVICES AND INDICATIONS FOR USE

Apply each type according to its intended purpose.

The general view, description and indications of the use of each type are presented in Table 1.

Table 1

Type	General view	Description
Alginate Impression Material Hygedent, 454 g		Alginate with high tensile strength. Red colored powder.
Alginate Impression		Alginate with high elasticity and short

Type	General view	Description
Material HygeMAX, 454 g		residence time in the mouth. Yellow colored powder
Alginate Material Impression HygePLUS, 454 g		Alginate with long term dimensional stability and high level of detail reproduction. Orange colored powder
Alginate Material Impression HygeChroma, 454 g		Three phase alginate material with yellow color. chromatic alginate. Color change accompanies the different processing stages
Alginate Material Impression Chromatic, 454 g		Chromatic alginate with high resistance to deformation. Color change accompanies the different processing phases

Instruction for use

Hygedent

Indications

For use by dental professionals to take impressions of patients' teeth.

PACKAGING OF MEDICAL DEVICE

Package (bag) 454 g

LIMITATIONS OF USE

Use only in accordance with intended use;
Do not use if the integrity of the package is violated.

CONTRAINDICATIONS

Do not use on patients with known hypersensitivity to any of the ingredients

SIDE EFFECTS

In case of allergy to any of the components, irritation, redness may occur, it is necessary to rinse thoroughly with water the place of contact.

POTENTIAL USERS

Medical staff.

SAFETY MEASURES

Do not use after expiry date.
Use a mask, safety glasses and do not inhale dust while working.
Avoid contact with the powder in the eyes or mouth.
If the powder gets into the eyes and mouth, immediately rinse the eyes with clean water, drink plenty of water and consult a doctor.

TERMS OF USE

The medical device is used at room temperature (23 ± 2) °C and at relative humidity (50 ± 10) %, and it is recommended to use distilled or deionized water (23 ± 1) °C for mixing.

STORING AND TRANSPORTATION

Always keep the impression material in a cool, dry place at a temperature of $0 \sim 25$ °C and air humidity $0 \sim 80$ %.

Instruction for use

Hygedent

After opening the packaging store the impression material in a firmly closed storage box. Always close the box immediately after using the material.

It is recommended to use a container with a tight-fitting, moisture-proof lid designed for food or medical products (if available).

Transportation of medical device can be carried out in a tilt car or cabin transport, but it is necessary to ensure cleanliness and take into account the effects of the sun and rain.

COMPATIBILITY

The impression material meets the requirements of ISO 6873 standard and is compatible with the gypsum type 2, type 3 and type 4 gypsum of any production.

Some of compatible gypsum are given in table 2.

Table 2

Description	Manufacturer	Data on registration
Dental auxiliary materials for dental laboratory. 1.	SILADENT Dr. Böhme & Schöps GmbH, Germany	RC No. FSZ 2012/11635 dd. 05.03.2012
The material is dental, dental with accessories. Dental plaster casts, variants of execution:	Zhermack S.p.A., Italy	RC No. FSZ 2010/07890 dd. 08.10.2010

PRODUCT SPECIFICATION

The medical device is not sterile and does not require sterilization before use.

The medical product does not contain drugs, or tissues of animal or human origin.

The specifications of all versions are presented in Table 3.

Table 3

Types	Package	Weight, g	Color
Alginate impressing material Hygedent, 454 g	bag	454	Red
Alginate impressing material HygeMAX, 454 g	bag	454	Yellow
Alginate impressing material HygePLUS, 454 g	bag	454	Orange
Alginate impressing material HygeChroma, 454 g	bag	454	Yellow
Alginate impressing material Chromatic, 454 g	bag	454	White

Technical characteristics of medical device:

- 1) Attemper time: 30~45 s;
- 2) Work time: 1'30''~2'20'';
- 3) Setting time: 150~180 s;
- 4) Reappearance: 50 μ m;
- 5) Recovery from Deformation: ≥ 95 %;

- 6) Deformation under Press: 5~20 %;
- 7) Tear strength: ≥ 0.38 MPa.

COMPOSITION

Alginate impression material HYGEDENT is contains following components: diatomaceous earth, alginate, calcium sulphate, retarder (sodium bicarbonate), filler (talc), pigment (except Alginate impression material Chromatic 454 g), flavor, indicator (only types: Alginate impression material HygeChroma, 454 g and Alginate impression material 454 g)

APPLICATION

Recommendations for use

Clean your hands thoroughly immediately before use.

Use personal protective equipment (gloves, mask).

Prepare all the necessary items.

Make sure that the expiration date of the product is valid.

Stir the powder to loosen it well.

Scoop required amount of powder with a light, swift movement and then stroke the powder smooth with a measuring spoon* (in case of not having a measuring spoon, dose the powder according to the recommended weight and smooth the surface). Do not compress the powder in the scoop.

Mix water (at room temperature) and powder in the mixing cup.*

Dosing

- * for a partial impression 10g (1 scoop) + 1/3 beaker of water (15-16 ml)
- * for a full impression 20g (2 scoop) + 2/3 beaker of water (30-31 ml)
- * for an extra-large impression 30g (3 scoop) + 3/3 beaker of water (45-46 ml)

Ideal mixing ratio: 19.3g=2measuring scoops: 46ml=1full beaker.

You can make the mixture thinner or thicker by adding more or less water respectively.

*Can be used a measuring spoon, measuring cup, mixing bowl and spatula for mixing alginate masses, which are intended to use for alginate masses registered in the Russian Federation.

Examples of registered accessories (measuring cup, measuring spoon, mixing bowl, spatula for mixing alginate masses) freely used in the Russian Federation are shown in the table below.

Table 4

Accessories	Manufacturer, address	Registration Certificate
Set of measuring tools for alginates (1 measuring spoon for water и 1 measuring spoon for the powder)	«ZHERMAK S.P.A.», Italy,	№ RZN 2020/9774 dated 20.03.2020
Rubbery measuring cup	Via Bovazecchino 100,45021 Badia Polesine (RO), Italy	№ RZN 2020/9774 dated 20.03.2020
Fluorescent spatulas for mixing alginate different colores	«ZHERMAK S.P.A.», Italy,	№ RZN 2020/9774 dated 20.03.2020

Mixing

Mix both components together carefully and thoroughly for 30 seconds until a smooth and homogeneous mixture is obtained.

Fill the tray immediately and use a wettened finger to stroke it smooth.

Make sure that the patient has rinsed the mouth with warm water in the meantime.

Taking the impression

Within 1 minute after mixing, insert the filled impression tray into the patient's mouth applying gentle pressure.

Allow the impression material to set in the mouth for 1 1/2 minute.

Remove the tray from the mouth in a single rapid movement.

Rinse the tray under cold running water to remove saliva and any residues.

Remove any excess water, but leave the surface moist. Never blow-dry!

Making the plaster models

Immediately after rinsing, pour out the plaster impression and make a second plaster model if desired.

If immediate pouring is impossible: store the impression in a firmly sealed plastic bag and add a few drops of water to achieve a relative humidity of 100%. This is essential for optimum results at a later pouring.

For preference, Impression material Hygedent meets ISO 6873 standard requirement and compatibility type3 and type4 gypsum, also meets compatibility type2 gypsum.

Mixing scheme

Powder water ratio:

Alginate impression material Hygedent, 454 g	10 g : 22 ml
Alginate impression material HygeMAX, 454 g	10 g : 23 ml
Alginate impression material HygePLUS, 454 g	10 g : 23 ml
Alginate impression material HygeChroma, 454 g	10 g : 23 ml
Alginate impression material Chromatic, 454 g	10 g : 22 ml

Setting Time

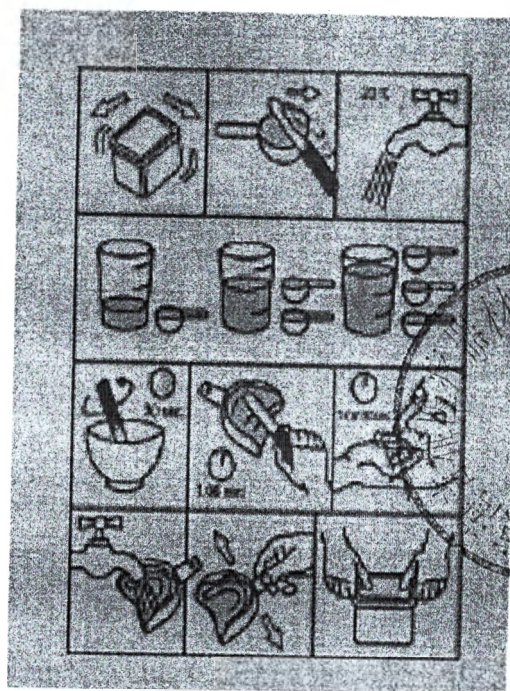
	Fast Set	Regular set	Slow set
Mixing time	30"-45"	30"-45"	30"-45"
Total working time	1'30"-1'45"	1'50"-2'15"	2'30"-2'40"
Total setting time	1'50"-2'	2'30"-3'	3'-3'30"

Note

The working/setting times increase with a lower (water) temperature. At higher temperatures the times become shorter.

The water hardness has the same effect: the harder the water used for mixing, the shorter the working/setting time.

A graphic presentation of the instructions for use is also located on the product packaging.

Disinfection

The impression can be disinfected in a 2% solution of glutaraldehyde or in a 0.5% solution of sodium hypochlorite for 10-15 minutes. Disinfection does not affect the quality of the impression.

Sterilization

Impression is a disposable product. Sterilization is not needed.

TECHNICAL SERVICE AND REPAIR

Alginate impression material HYGEDENT is not a subject to periodic technical service and current repair.

MANUFACTURER'S WARRANTIES

The shelf life of the medical device is 3 years from the date of production.

The expiration date is indicated on the package of the medical device; do not use the product after the expiration date.

In case of keeping the alginate material in a tightly closed container after opening the package, it is recommended to store the powder for up to 3 months at temperatures up to 23°C and at humidity up to 80%.

The manufacturer guarantees the quality, effectiveness and safety of Alginate impression material HYGEDENT in case of the user keeps the conditions of storage, transportation, recycling and instructions for use of the medical device during the shelf life.

ENVIRONMENTAL REQUIRMENTS

Alginate impression material HYGEDENT in case of direct contact at room temperature doesn't emit toxic substances into the environment and doesn't have a harmful effect on the human body.

RECYCLING

Waste management for the application of national standards.

MANUFACTURER

Hygedent Inc.

Legal address

E gate, Building No.1, No.97, Changping Road, Shahe Town, Changping District, Beijing City, China

Tel.: + 8613801250320

E-mail: sales@hygedent.com

AUTHORIZED MANUFACTURER'S REPRESENTATIVE IN THE RUSSIAN FEDERATION

Limited Liability Company "KARO" (KARP LLC),

Legal Address

4-ya Krasnoarmeyskaya, house 6, lit. A, building 7N, Saint Petersburg, Russia, 190005

Tel.: (812) 575-00-10

E-mail: info@karodent.ru

Explanation of symbols and designations used in the labeling of medical device:

LOT
MFD
EXP

Lot number

Date of manufacture

Expired date

CE-mark (in accordance with EC 93/42/EОC directive)

Avoid sunlight

Keep dry

The packaging has an additional marking in the form of a sticker in Russian:
The sticker contains the following information in Russian:

- 1) name of the medical product, including the design variant;
- 2) name of the manufacturer (factory) of the medical product and the address of its location;
- 3) name of the organization authorized by the manufacturer to represent its interests on the territory of the Russian Federation (authorized representative), as well as its location address;
- 4) number and date of issue of the registration certificate of the Russian Federation.

For ease of use, the package of the medical product is also equipped with brief instructions for use, which are also applied in the form of a sticker on the packaging (see figure

below). At the same time, the full text of the instructions for use in Russian is available for download on the website of the Authorized Representative of the manufacturer in the Russian Federation at www.karodent.ru, which is mentioned in the brief instructions on the packaging.

A sticker with information in Russian and brief instructions for use are applied directly to each single package (bag) with alginate material in any convenient place for their placement. In this case, the information in Russian will be considered a priority. Thus, stickers can be applied on top of the information in English and images, as they contain this information.

Stickers should be avoided over the batch number, date of manufacture, expiration date, "Do not expose to sunlight" and "Keep away from moisture" signs, as this information is not provided in Russian language stickers.

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