

Is it Chlamydia Infection?

With the ACON
Chlamydia rapid
tests, you will know
in minutes!

- *Accurate Results*
- *Immediate Diagnosis*
- *Multiple Options*



—ACON®

Global Diagnostics for Local MarketsSM

ACON Chlamydia Rapid Tests provide accurate results in minutes

Accurate Results

- High sensitivity and specificity
- Built-in procedural control

Immediate Diagnosis

- Results in 10 minutes
- Facilitates patient treatment decisions quickly

Multiple Options

- Detects male urethral, female cervical swabs, and male urine specimen
- Optional packaging available



Device Test

Male/Female Swab Specimen

- Add Reagent A and B to specimen
- Add 3 drops of extracted specimen
- Read results at 10 minutes



10 min.



Positive



Negative



Invalid

Male Urine Specimen

- Add Reagent B and A to specimen
- Add 3 drops of extracted specimen
- Read results at 10 minutes



Product Name	Specimen	Format	Catalog No.	Read Time	Sensitivity	Tests per kit	Components							
							Test Device	Reagent A & B	Female Cervical Swab	Workstation	Test Tube	Dropper Tip	Quantitative Pipet	Package Insert
Chlamydia	Swab/Urine	Device	ICH-502 ✓	10 min.	See Insert	20	*	*	*	*	*	*	*	*
			ICH-502S ✓			25	*	*		*	*	*	*	*

All Chlamydia Products Not Available for Sale in the US

Note: * - symbol indicates components are included in packaging

✓ CE Marked for Sale in the European Community



We also offer one-step strip, device, midstream and/or panel tests for:

Pregnancy (hCG), Ovulation (LH), Menopause (FSH), HIV1/2, HIV1/2/O,^{NEW} HBV, HBsAg, HBsAb, HBeAg, HBeAb, HBcAb, HCV, Strep A, *H. pylori*, Malaria, Mono,^{NEW} Syphilis, PSA, FOB, AFP,^{NEW} Troponin I,^{NEW} Drugs of Abuse and Others.

Blood Glucose Monitoring Systems Now Available!

To order, please call your local distributor or call ACON for a distributor near you.



ACON Laboratories, Inc., 4108 Sorrento Valley Boulevard, San Diego, CA 92121, U.S.A. • Tel: 1-858-535-2030 • Fax: 1-858-535-2035 • E-mail: info@aconlabs.com
Please visit our website for details: www.aconlabs.com

DN: 1150176801

© 2004 ACON Laboratories, Inc.