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
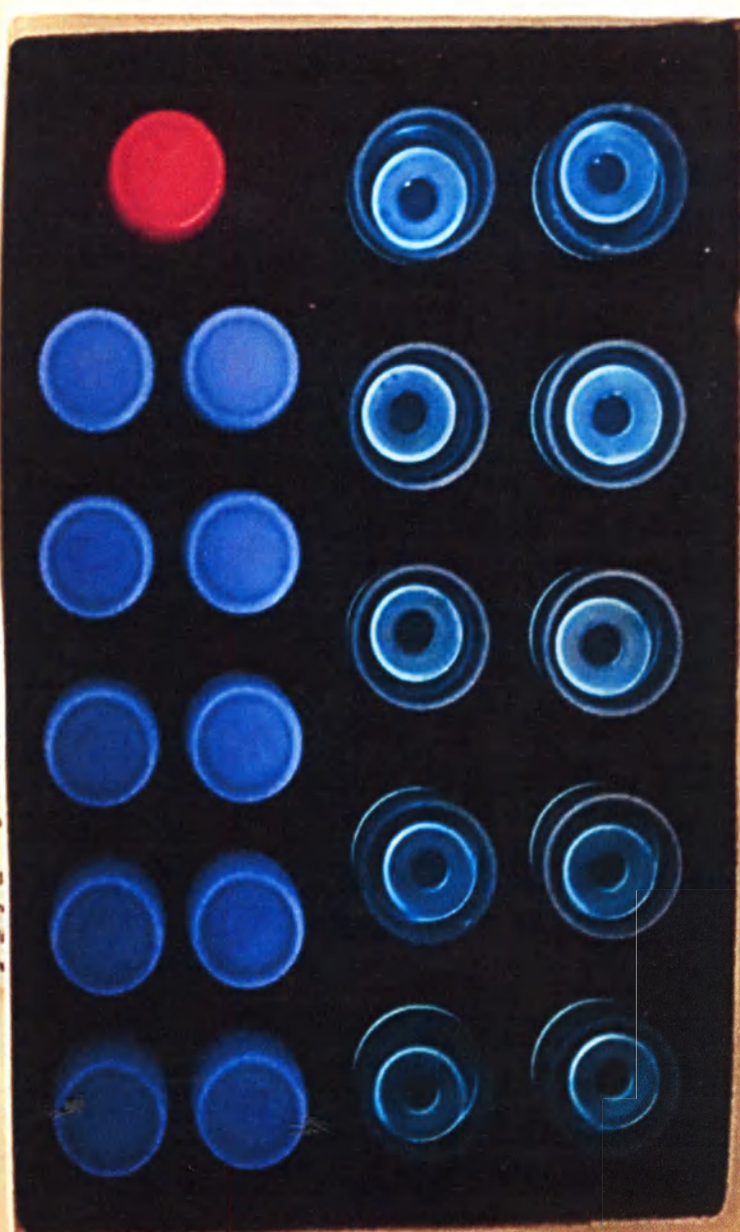
[Intended Use]

This product is intended to be used on the Haemo Analyzers for testing the blood coagulation of patients, to assess whether any heparin residue exists in the patient's blood circulation, or whether the patients become resistant to heparin treatment.

Many different forms of heparin have the anticoagulant functionality in human body, and therefore heparins are most widely prescribed anticoagulant drugs in clinical applications, such as thrombotic diseases, myocardial infarction.

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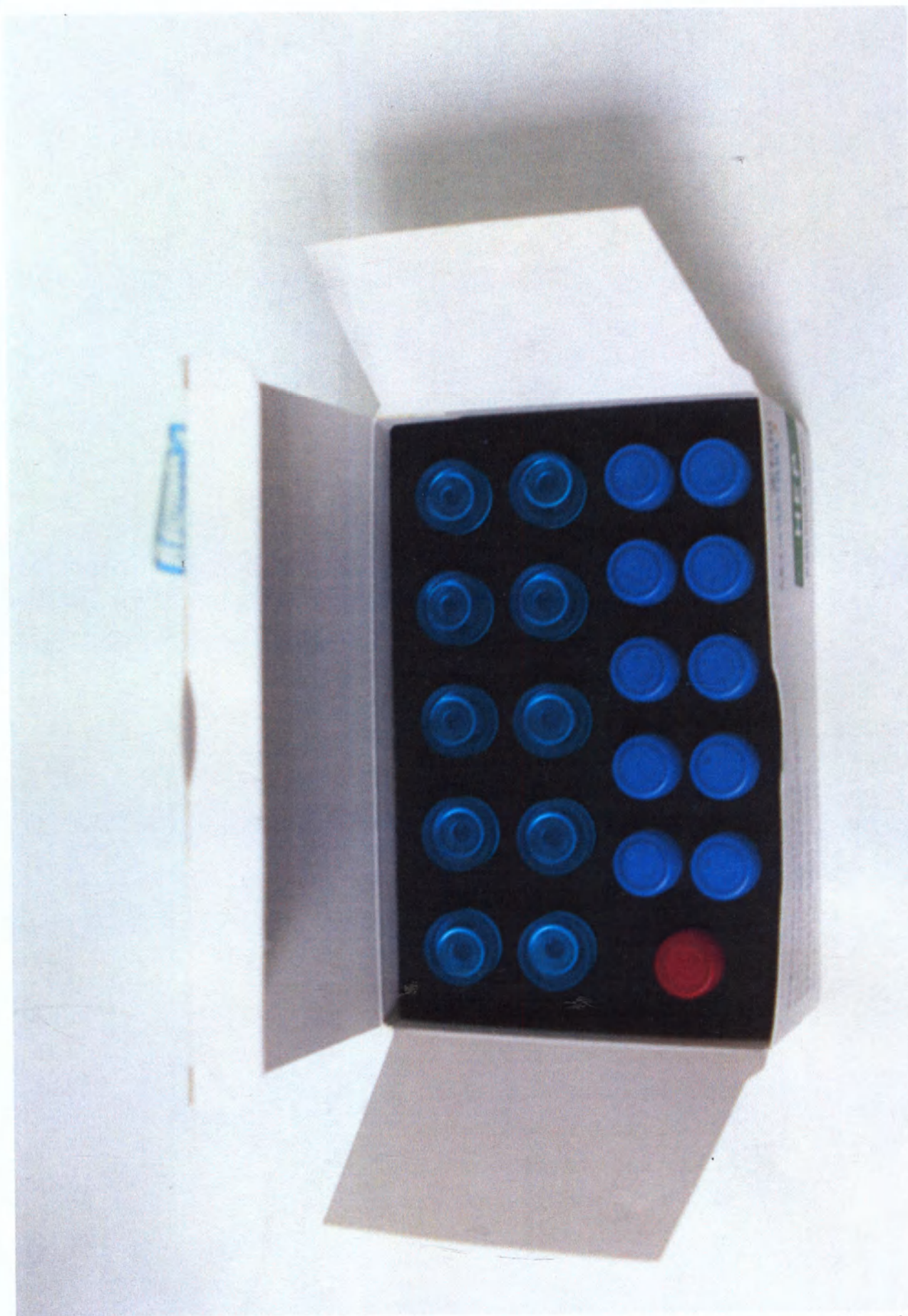HEP

Heparin Assay

MEDCAPTAIN MEDICAL TECHNOLOGY CO., LTD.

12th Floor, Baiwang Research Building, NO.5158
Shahe West Road, Xili, Nanshan, 518055 Shenzhen,
Guangdong, PEOPLE'S REPUBLIC OF CHINA
Tel: 400-809-3369 Website: www.medcaptain.com

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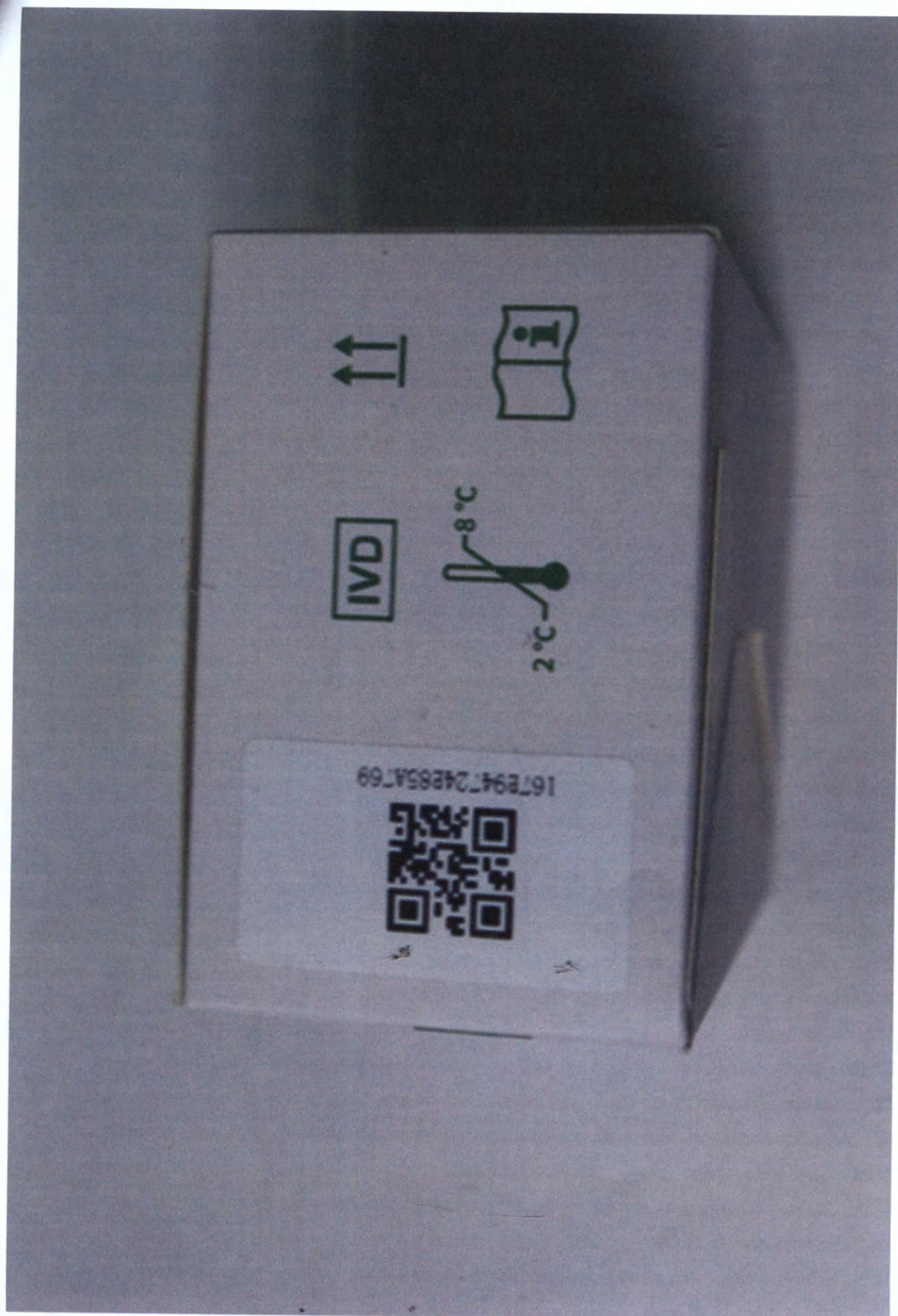
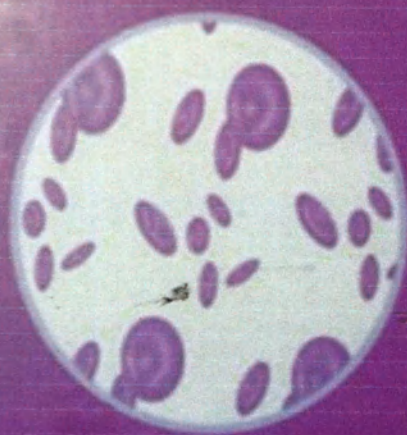
Heparin Assay

10 Tests/Pkg (M)

MEDCAPTAIN MEDICAL TECHNOLOGY CO., LTD.
Qualified

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Kaolin: 1 × 0.6 mL

Trigger: 1 × 3.0 mL

MEDCAPTAIN MEDICAL TECHNOLOGY CO., LTD.
12th Floor, Baiwang Research Building, NO.5158 Shahe
West Road, Xili, Nanshan, 518055 Shenzhen, Guangdong,
PEOPLE'S REPUBLIC OF CHINA
Tel: +86-755-26953369
Website: www.medcaptain.com

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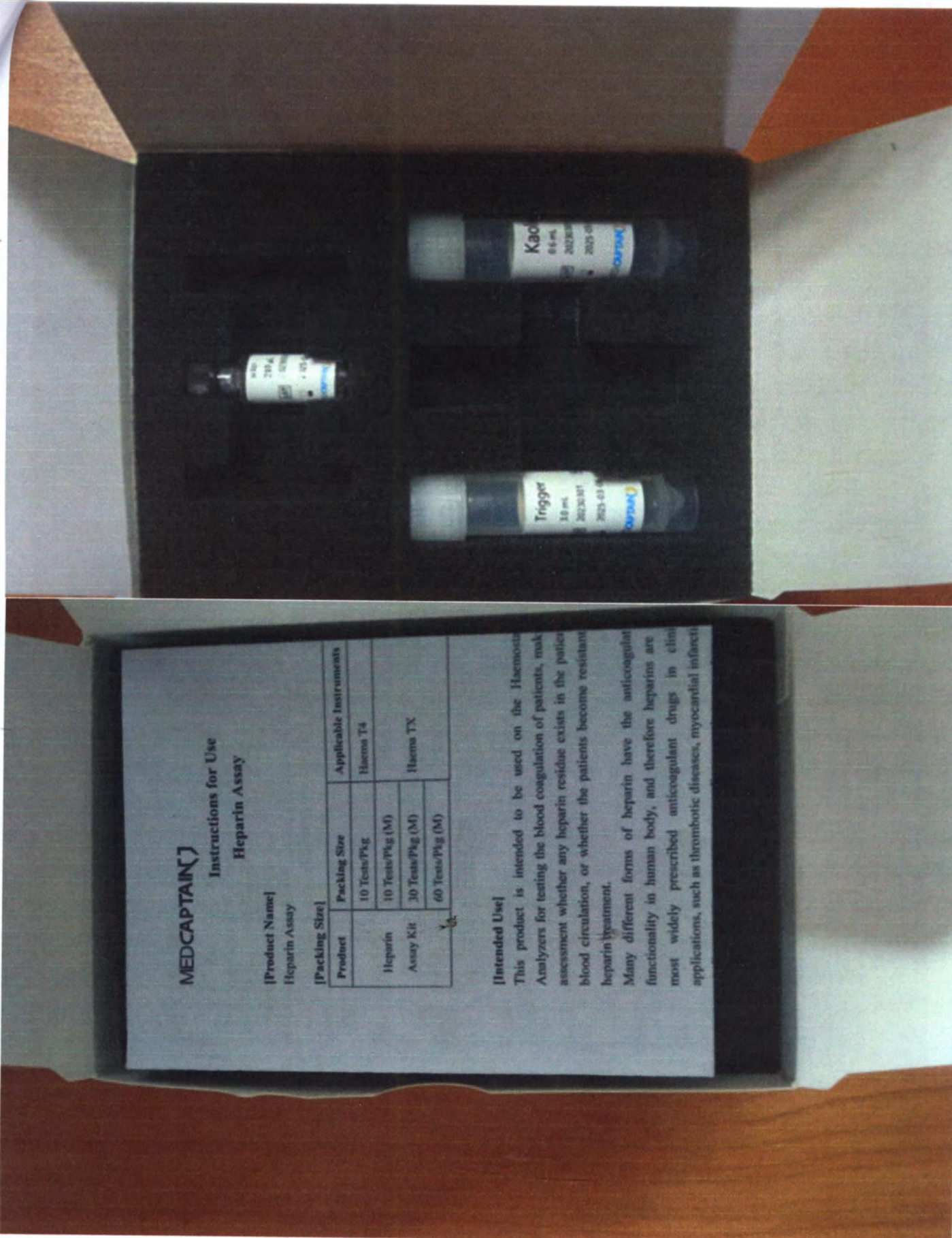
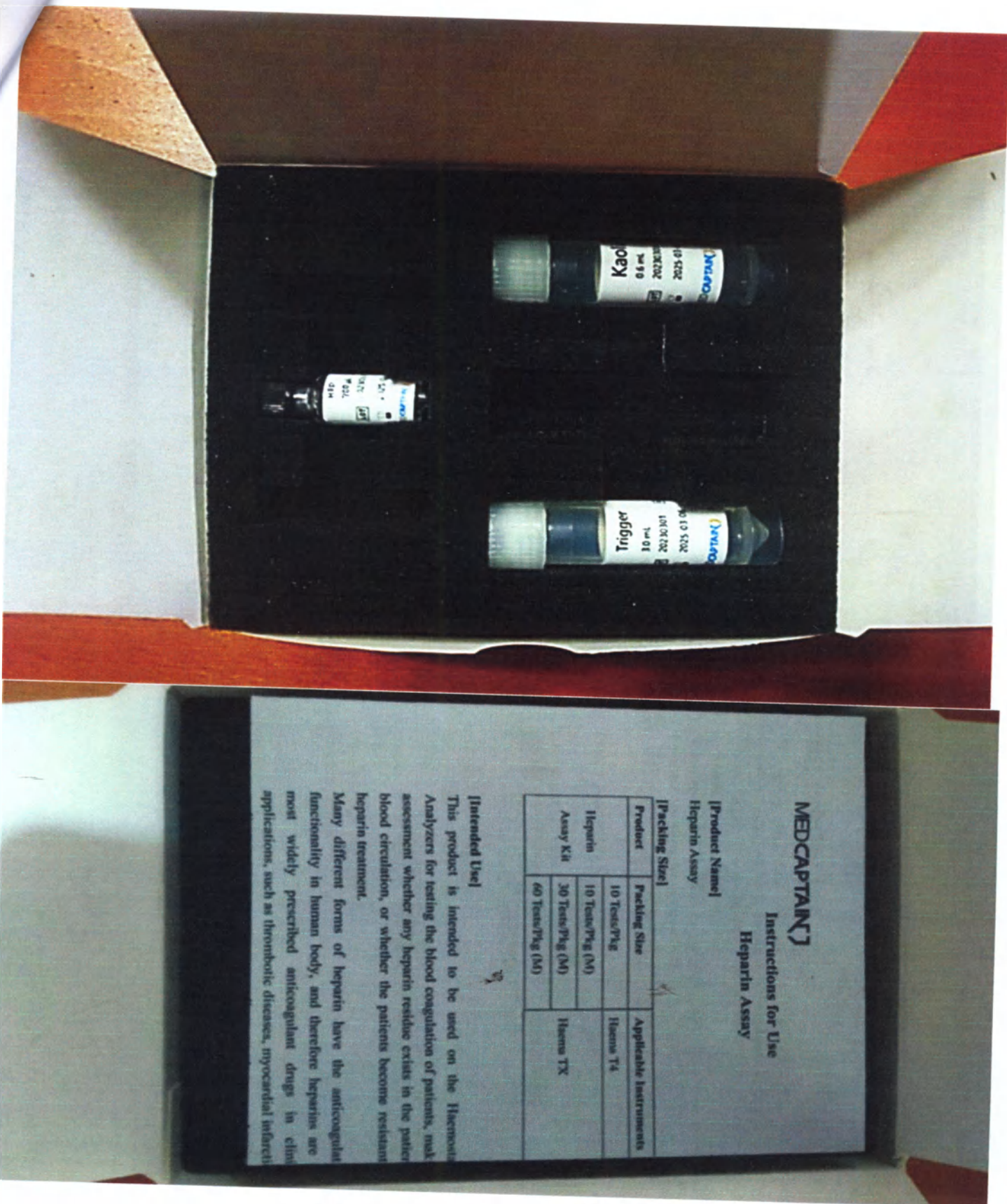
MEDCAPTAIN[®]

Instructions for Use Heparin Assay

[Product Name]
Heparin Assay

[Packing Size]			Applicable Instruments
Product	Packing Size		
Heparin Assay Kit	10 Tests/Pkg	Haema T4	
	10 Tests/Pkg (M)		
	30 Tests/Pkg (M)	Haema TX	
	60 Tests/Pkg (M)		

[Intended Use]

This product is intended to be used on the Naema[®] Analyzers for testing the blood coagulation of patients, making assessment whether any heparin residue exists in the patient blood circulation, or whether the patients become resistant heparin treatment.

Many different forms of heparin have the anticoagulant functionality in human body, and therefore heparins are most widely prescribed anticoagulant drugs in clinical applications, such as thrombotic diseases, myocardial infarction

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MEDCAPTAIN

Instructions for Use Heparin Assay

[Product Name]
Heparin Assay

[Packing Size]

Product	Packing Size	Applicable Instruments
Heparin Assay Kit	10 Tests/Pkg	Haema T4
	10 Tests/Pkg (M)	Haema TX
	30 Tests/Pkg (M)	
	60 Tests/Pkg (M)	

[Intended Use]

This product is intended to be used on the Haemostasis Analyzers for testing the blood coagulation of patients, making assessment whether any heparin residue exists in the patient's blood circulation, or whether the patients become resistant to heparin treatment.

Many different forms of heparin have the anticoagulant functionality in human body, and therefore heparins are most widely prescribed anticoagulant drugs in clinical applications, such as thrombotic diseases, myocardial infarction,

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MEDCAPTAIN

**Instructions for Use
Heparin Assay**

[Product Name]

Heparin Assay

[Packing Size]

Product	Packing Size	Applicable Instruments
Heparin Assay Kit	10 Tests/Pkg	Haema T4
	10 Tests/Pkg (M)	Haema TX
	30 Tests/Pkg (M)	
	60 Tests/Pkg (M)	

[Intended Use]

This product is intended to be used on the Haemostasis Analyzers for testing the blood coagulation of patients, make assessment whether any heparin residue exists in the patient's blood circulation, or whether the patients become resistant to heparin treatment.

Many different forms of heparin have the anticoagulant functionality in human body, and therefore heparins are most widely prescribed anticoagulant drugs in clinical applications, such as thrombotic diseases, myocardial infarction

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