1 Name of the device

Dental post STEA®

I. Versions:

1. Post Crystal Quartz Fiber Post "Simplex" SIM80Q, consisting of: 0,8-0,2-18mm – 10 pcs./pack.
2. Post Crystal Quartz Fiber Post "Simplex" SIM100Q, consisting of: 1,0-0,3-18mm – 10 pcs./pack.
3. Post Crystal Quartz Fiber Post "Simplex" SIM120Q, consisting of: 1,2-0,4-18mm – 10 pcs./pack.
4. Post Crystal Quartz Fiber Post "Simplex" SIM140Q, consisting of: 1,4-0,5-18mm – 10 pcs./pack.
5. Post Crystal Quartz Fiber Post "Simplex" SIM160Q, consisting of: 1,6-0,6-18mm – 10 pcs./pack.
6. Post Crystal Quartz Fiber Post "HS" HS50Q, consisting of: 1,25-0,5-18mm – 10 pcs./pack.
7. Post Crystal Quartz Fiber Post "HS" HS70Q, consisting of: 1,45-0,7-18mm – 10 pcs./pack.
8. Post Crystal Quartz Fiber Post "HS" HS90Q, consisting of: 1,65-0,9-18mm – 10 pcs./pack.
9. Post Crystal Quartz Fiber Post "911": 911-50Q, consisting of: 1,05-0,5-16mm – 10 pcs./pack.
10. Post Crystal Quartz Fiber Post "911": 911-70Q, consisting of: 1,25-0,7-16mm – 10 pcs./pack.
11. Post Crystal Quartz Fiber Post "911": 911-90Q, consisting of: 1,45-0,9-16mm – 10 pcs./pack.
12. Post Crystal Quartz Fiber Post "911": 911-110Q, consisting of: 1,65-1,1-16mm – 10 pcs./pack.
13. Post Crystal Quartz Fiber Post "Cobra" CBC70Q, consisting of: 1,2-0,7-16mm – 10 pcs./pack.
14. Post Crystal Quartz Fiber Post "Cobra" CBC90Q, consisting of: 1,4-0,9-16mm – 10 pcs./pack.
15. Post Crystal Quartz Fiber Post "Cobra" CBC110Q, consisting of: 1,6-1,1-16mm – 10 pcs./pack.
16. Post Red-Point Post "Simplex" SIM100QR, consisting of: 1-0,3-18mm – 10 pcs./pack.
17. Post Red-Point Post "Simplex" SIM120QR, consisting of: 1,2-0,4-18mm – 10 pcs./pack.
18. Post Red-Point Post "Simplex" SIM140QR, consisting of: 1,4-0,5-18mm – 10 pcs./pack.
19. Post Red-Point Post "Simplex" SIM160QR, consisting of: 1,6-0,6-18mm – 10 pcs./pack.
20. Post Red-Point Post "Cobra" CBC70QR, consisting of: 1,2-0,7-16mm – 10 pcs./pack.
21. Post Red-Point Post "Cobra" CBC90QR, consisting of: 1,4-0,9-16mm – 10 pcs./pack.
22. Post Red-Point Post "Cobra" CBC110QR, consisting of: 1,6-1,1-16mm – 10 pcs./pack.
23. Post Red-Point Post "HS" HS50QR, consisting of: 1,25-0,5-18mm – 10 pcs./pack.
24. Post Red-Point Post "HS" HS70QR, consisting of: 1,45-0,7-18mm – 10 pcs./pack.
25. Post Red-Point Post "HS" HS90QR, consisting of: 1,65-0,9-18mm – 10 pcs./pack.
26. Post Nano Ceramic Post "Simplex" SIM100QZ, consisting of: 1-0,3-18mm – 10 pcs./pack.
27. Post Nano Ceramic Post "Simplex" SIM120QZ, consisting of: 1,2-0,4-18mm – 10 pcs./pack.
28. Post Nano Ceramic Post "Simplex" SIM140QZ, consisting of: 1,4-0,5-18mm – 10 pcs./pack.
29. Post Nano Ceramic Post "Simplex" SIM160QZ, consisting of: 1,6-0,6-18mm – 10 pcs./pack.
30. Post Nano Ceramic Post "Cobra" CBC70QZ, consisting of: 1,2-0,7-16mm – 10 pcs./pack.
31. Post Nano Ceramic Post "Cobra" CBC90QZ, consisting of: 1,4-0,9-16mm – 10 pcs./pack.

32. Post Nano Ceramic Post "Cobra" CBC110QZ, consisting of: 1,6-1,1-16mm – 10 pcs./pack.
33. Post Nano Ceramic Post "HS" HS50QZ, consisting of: 1,25-0,5-18mm – 10 pcs./pack.
34. Post Nano Ceramic Post "HS" HS70QZ, consisting of: 1,45-0,7-18mm – 10 pcs./pack.
35. Post Nano Ceramic Post "HS" HS90QZ, consisting of: 1,65-0,9-18mm – 10 pcs./pack.
36. Post Microcore ESSE Fiberpost "Conical" MCS22, consisting of: 1,5-0,3-15,4mm – 10 pcs./pack.
37. Post Microcore ESSE Fiberpost "Conical" MCS55, consisting of: 1,7-0,9-15,4mm – 10 pcs./pack.
38. Post Microcore ESSE Fiberpost "Conical" MCS88, consisting of: 1,8-1,1-15,7mm – 10 pcs./pack.
39. Post Microcore ESSE Fiberpost "Cylindrical" MCS2, consisting of: 2,4-0,3-15,4mm – 10 pcs./pack.
40. Post Microcore ESSE Fiberpost "Cylindrical" MCS5, consisting of: 2,6-0,9-15,4mm – 10 pcs./pack.
41. Post Microcore ESSE Fiberpost "Cylindrical" MCS8, consisting of: 2,7-1,1-15,7mm – 10 pcs./pack.
42. Post Microcore Fiberpost MC1, consisting of: 1,2-0,7-13,4mm – 10 pcs./pack.
43. Post Microcore Fiberpost MC2, consisting of: 1,5-0,7-13,4mm – 10 pcs./pack.
44. Post Microcore Fiberpost MC4, consisting of: 1,4-0,9-13,4mm – 10 pcs./pack.
45. Post Microcore Fiberpost MC5, consisting of: 1,7-0,9-13,4mm – 10 pcs./pack.
46. Post Microcore Fiberpost MC7, consisting of: 1,6-1,1-13,7mm – 10 pcs./pack.
47. Post Microcore Fiberpost MC8, consisting of: 1,8-1,1-13,7mm – 10 pcs./pack.
48. Post Microcore Fiberpost MC3, consisting of: 1,2-0,7-13,4mm – 10 pcs./pack.
49. Post Microcore Fiberpost MC6, consisting of: 1,4-0,9-13,4mm – 10 pcs./pack.
50. Post Microcore Fiberpost MC9, consisting of: 1,6-1,1-13,7mm – 10 pcs./pack.
51. Post Standard Post "Simplex" SIM80V, consisting of: 0,8-0,2-18mm – 10 pcs./pack.
52. Post Standard Post "Simplex" SIM100V, consisting of: 1-0,3-18mm – 10 pcs./pack.
53. Post Standard Post "Simplex" SIM120V, consisting of: 1,2-0,4-18mm – 10 pcs./pack.
54. Post Standard Post "Simplex" SIM140V, consisting of: 1,4-0,5-18mm – 10 pcs./pack.
55. Post Standard Post "Simplex" SIM160V, consisting of: 1,6-0,6-18mm – 10 pcs./pack.
56. Post Standard Post "HS" HS50V, consisting of: 1,25-0,5-18mm – 10 pcs./pack.
57. Post Standard Post "HS" HS70V, consisting of: 1,45-0,7-18mm – 10 pcs./pack.
58. Post Standard Post "HS" HS90V, consisting of: 1,65-0,9-18mm – 10 pcs./pack.
59. Post Standard Post "mediXpost" MP50V, consisting of: 1,1-0,5-16mm – 10 pcs./pack.
60. Post Standard Post "mediXpost" MP60V, consisting of: 1,26-0,6-16mm – 10 pcs./pack.
61. Post Standard Post "mediXpost" MP70V, consisting of: 1,42-0,7-16mm – 10 pcs./pack.
62. Post Standard Post "mediXpost" MP80V, consisting of: 1,58-0,8-16mm – 10 pcs./pack.
63. Post Standard Post "911" 911-50V, consisting of: 1,05-0,5-16mm – 10 pcs./pack.
64. Post Standard Post "911" 911-70V, consisting of: 1,25-0,7-16mm – 10 pcs./pack.
65. Post Standard Post "911" 911-90V, consisting of: 1,45-0,9-16mm – 10 pcs./pack.
66. Post Standard Post "911" 911-110V, consisting of: 1,65-1,1-16mm – 10 pcs./pack.
67. Post Standard Post "Rapid" RA50V, consisting of: 1,05-0,5-16mm – 10 pcs./pack.
68. Post Standard Post "Rapid" RA70V, consisting of: 1,25-0,7-16mm – 10 pcs./pack.
69. Post Standard Post "Rapid" RA90V, consisting of: 1,45-0,9-16mm – 10 pcs./pack.
70. Post Standard Post "Rapid" RA110V, consisting of: 1,65-1,1-16mm – 10 pcs./pack.
71. Post Standard Post "Cobra" CB100V, consisting of: 1,2-1-16mm – 10 pcs./pack.
72. Post Standard Post "Cobra" CB120V, consisting of: 1,4-1,2-16mm – 10 pcs./pack.
73. Post Standard Post "Cobra" CB140V, consisting of: 1,6-1,4-16mm – 10 pcs./pack.
74. Post Standard Post "Cobra" CBC70V, consisting of: 1,2-0,7-16mm – 10 pcs./pack.
75. Post Standard Post "Cobra" CBC90V, consisting of: 1,4-0,9-16mm – 10 pcs./pack.
76. Post Standard Post "Cobra" CBC110V, consisting of: 1,6-1,1-16mm – 10 pcs./pack.
77. Post Standard Post "Hi Techno" HT25V, consisting of: 1,21-0,25-20mm – 10 pcs./pack.

Instruction for Use for the Medical Device:

Dental post STEA®

ООО ЦРПМ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

78. Post Standard Post "Hi Techno" HT30V, consisting of: 1,26-0,3-20mm – 10 pcs./pack.
79. Post Standard Post "Hi Techno" HT35V, consisting of: 1,31-0,35-20mm – 10 pcs./pack.
80. Post Standard Post "Hi Techno" HT40V, consisting of: 1,36-0,4-20mm – 10 pcs./pack.
81. Post Standard Post "Hi Techno" HT50V, consisting of: 1,41-0,5-20mm – 10 pcs./pack.
82. Post Standard Post "Hi Techno" HT60V, consisting of: 1,46-0,6-20mm – 10 pcs./pack.
83. Post Standard Post "Hi Techno" HTVSET, consisting of: 1,21-0,25-20mm (5 pcs.); 1,26-0,3-20mm (5 pcs.); 1,31-0,35-20mm (5 pcs.); 1,36-0,4-20mm (5 pcs.); 1,41-0,5-20mm (5 pcs.); 1,46-0,6-20mm (5 pcs.) – 30 pcs./pack.

II. Instruction for use.

Hereinafter referred to as the "dental posts", "fiber dental posts", "posts", "medical device", and "device".

2 Information about the manufacturer, developer and authorized representative of the manufacturer

Manufacturer

MICRO.MEDICA S.r.l., Italy
Via Robbio 18 B - 27030 Palestro (PV) - Italy
Telephone: +39 0384 672867
Fax: +39 0384 682912
www.micromedicasrl.it

Developer

MICRO.MEDICA S.r.l., Italy
Via Robbio 18 B - 27030 Palestro (PV) - Italy
Telephone: +39 0384 672867
Fax: +39 0384 682912
www.micromedicasrl.it

Authorized representative of the manufacturer in the Russian Federation

PROTECO Joint-Stock Company (PROTECO JSC)
57 Salova Strt, Bldg. 1, Lit. I, St. Petersburg 192102
Tel./fax: +7 (812) 635-88-90

3 Classification

4.1. Purpose of the device

The device is intended for introduction to the root canal of a particular patient to support and retain the crown prosthesis with a direct/indirect method of restoration.

4.2. Intended users

The devices are intended for professional use in accordance with the intended purpose and/or indications as specified by the manufacturer. They are recommended for the following users: qualified dental professionals.

4 Indications, contraindications, precautions and side effects related to application of the medical device as intended

Precautionary statements:

Instruction for Use for the Medical Device:

Dental post STEA®

- Posts are disposable devices and their reuse is prohibited.
- Attention! The use of countersinks (drills) is not indispensable for proper use of the posts.

Indications:

- Fixation of restored crown walls where remaining dentine volume is insufficient (<4 mm).
- Allergy to metals.
- Anterior teeth depulcation.
- Necessity of strengthening the tooth after endodontic treatment.
- Presence of metal structures in the mouth (to avoid galvanic current development).

Contraindications:

- The remaining dental tissue height above the gum is less than 1,5-2 mm.
- The root length is less than the restored crown height.
- Individual intolerance to the device components.
- There is no data on the use of the product in pregnant and lactating women and children under 18 years of age.

Potential side effects: not identified.

5 Device description and operating principle

Dental posts are an element of restoration of a damaged dental system. They are a lightweight and durable fiber structure, intended to be placed in a dental root canal and serving as a mechanical support for an augmented or prosthetic crown.

6 Technical and functional characteristics

Dental posts:


- The devices are manufactured in two morphological forms: cylindrical and conical.
- The devices have colour coding (Table 1) in the upper shear part of the post, having no contact with bone tissue and mucous membrane of the patient's oral cavity.

Table 1. Colour coding of posts


Post	Colour	Post	Colour	Post	Colour
SIM80Q	White	911-50Q	Yellow	MC2	Yellow
SIM100Q	Yellow	911-70Q	Orange	MC3	White
SIM120Q	Red	911-90Q	Red	MC4	Red
SIM140Q	Blue	911-110Q	Blue	MC5	Green
SIM160Q	Green	CBC70Q	Orange	MC6	Red
HS50QF	Yellow	CBC90Q	Red	MC7	Blue
HS70Q	Orange	CBC110Q	Blue	MC8	Black
HS90Q	Red	MC1	White	MC9	Blue
RA50V	Yellow	CB100V	Yellow	HT30V	Yellow
RA70V	Orange	CB120V	Red		Red
RA90V	Red	CB140V	Blue		Blue
RA110V	Blue	HT25V	White		Green

HT60V	Black	MCS22	Yellow	MCS55	Green
MCS88	Black	MCS2	Yellow	MCS5	Green
MCS8	Black	MP50V	White	MP70V	Red
MP80V	Blue	MP60V	Yellow		

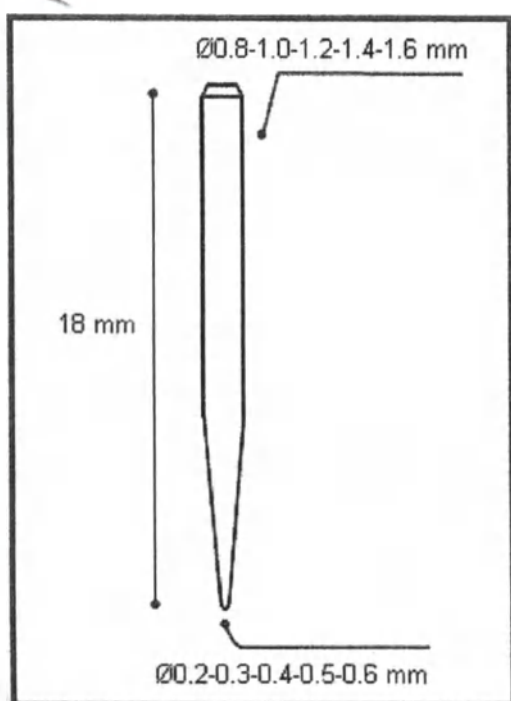
Posts forms and dimensions


"Simplex"

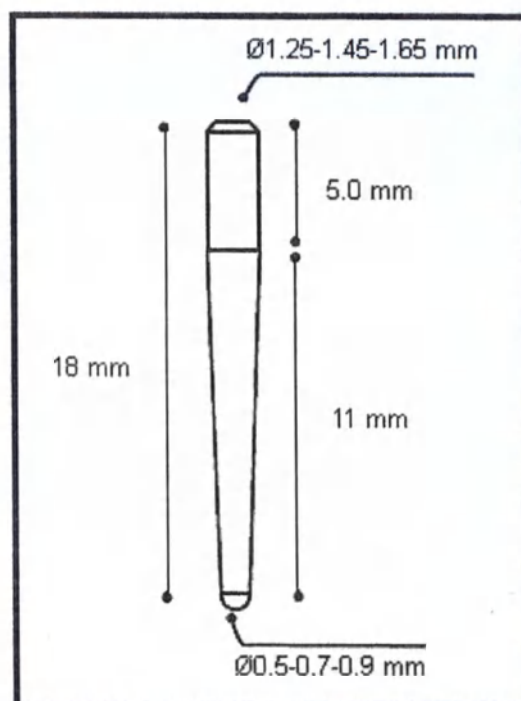
"HS"

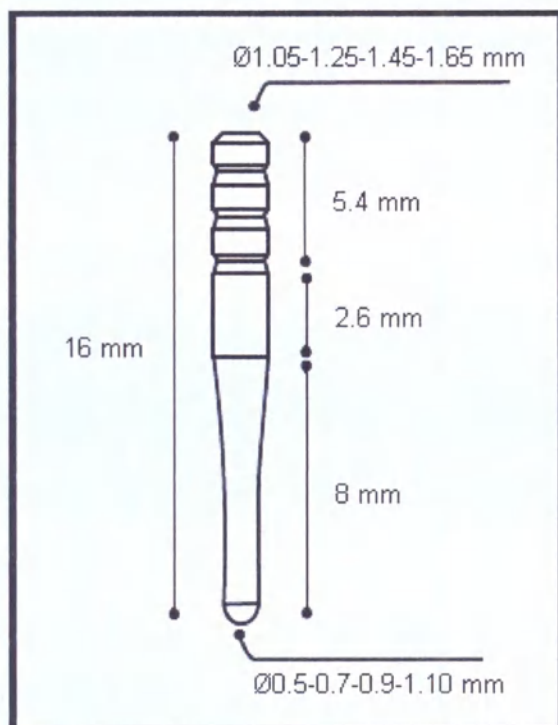
"911"

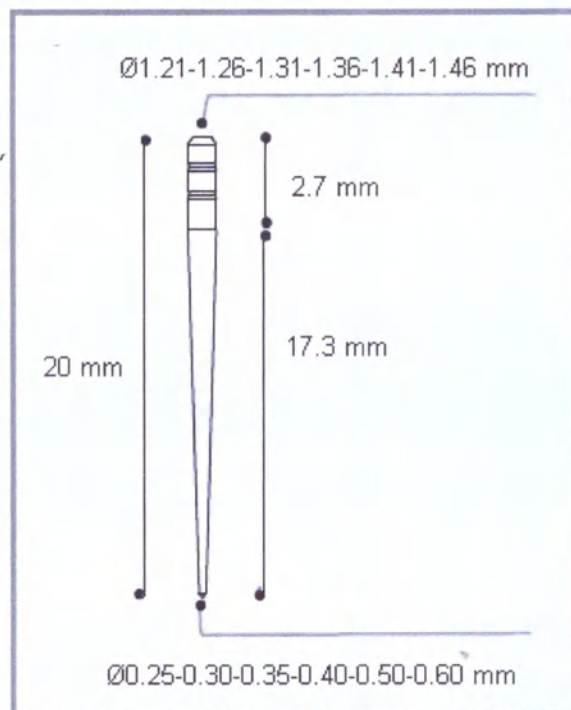"Hi Techno"

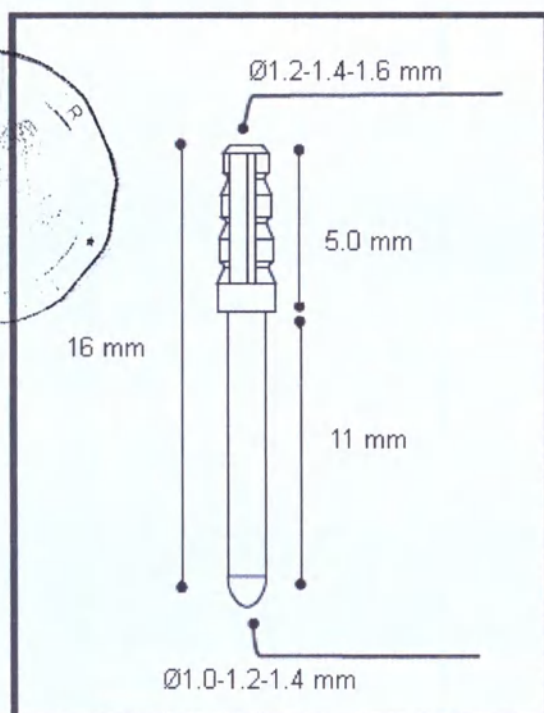
"Cobra"

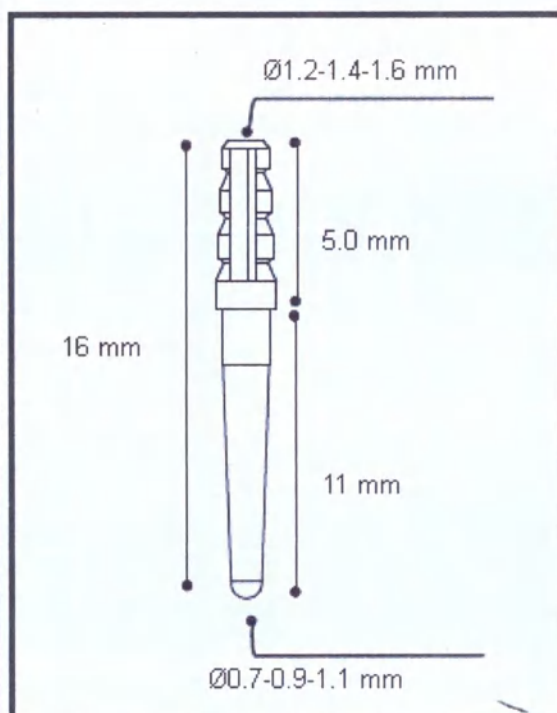
"Cobra"



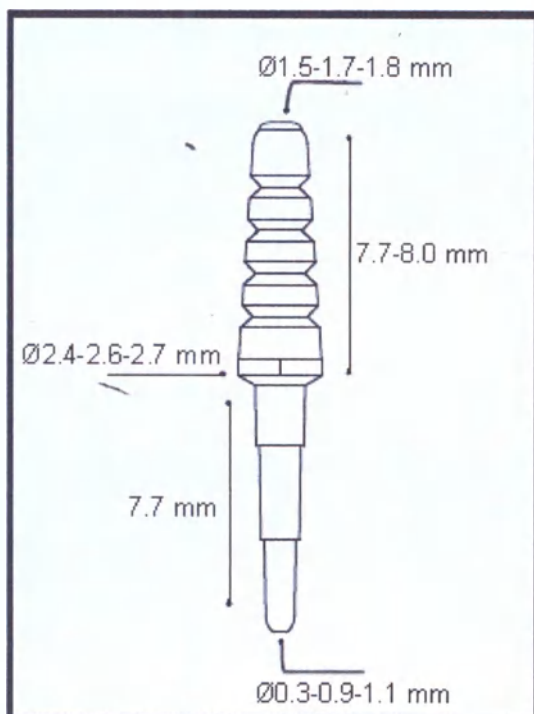
"Conical"

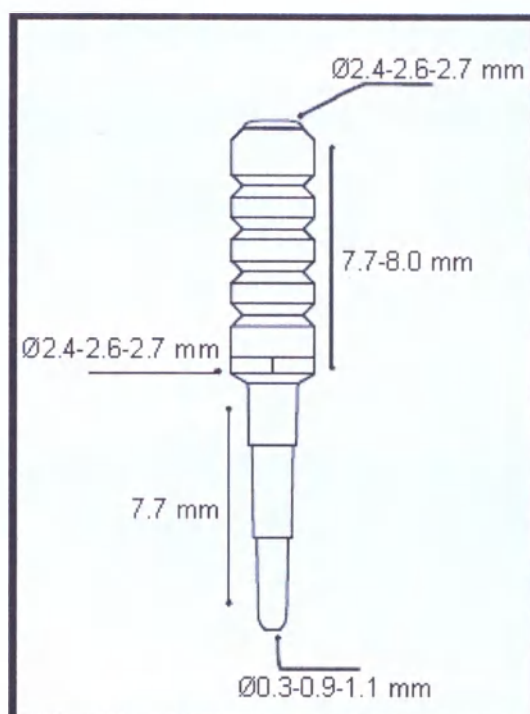
"Cylindrical"

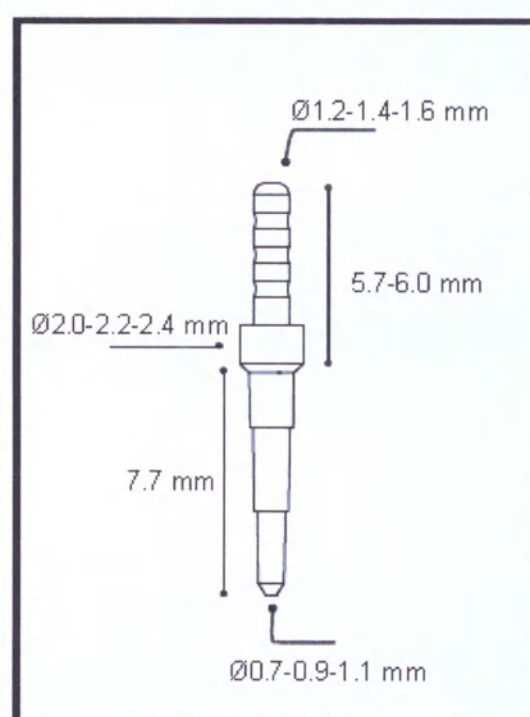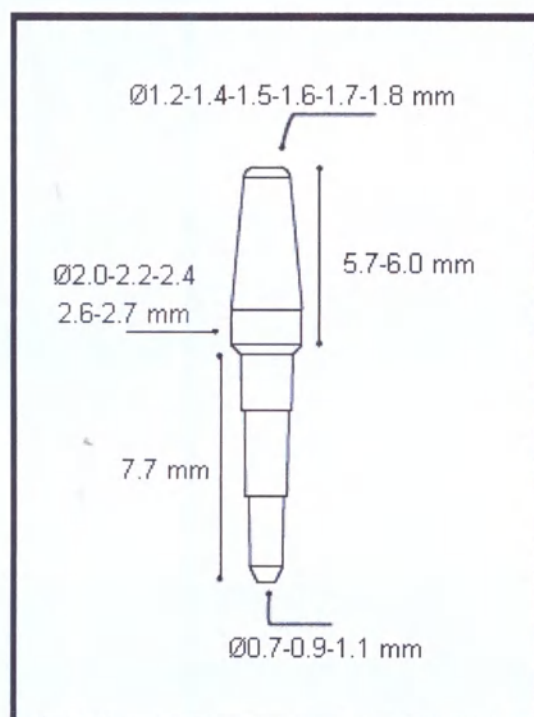Fiberpost MC

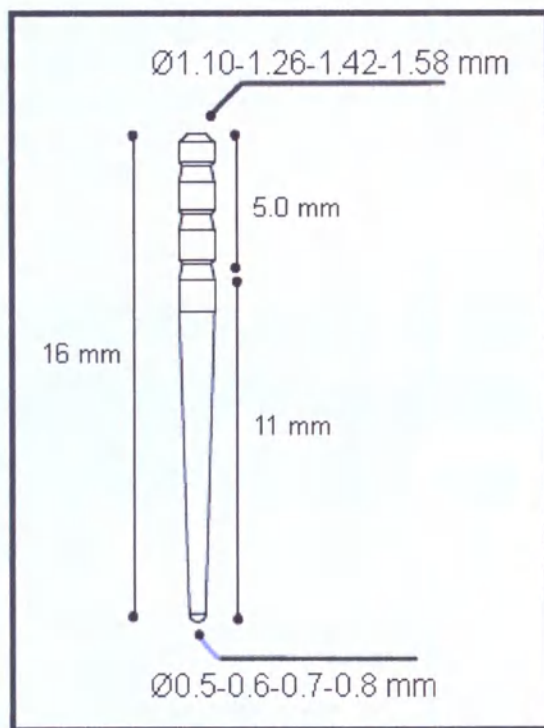"mediXpost"

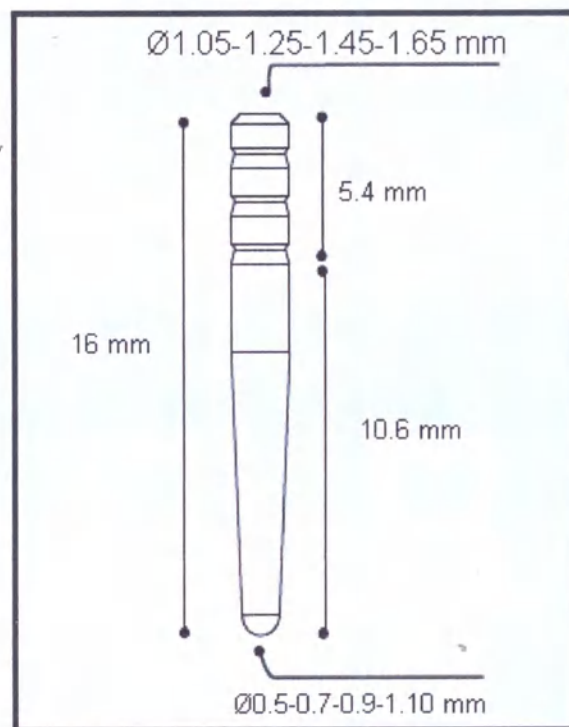
"Rapid"

7 Application procedure

1. Prepare the root canal: enlargement of the dental root and creation of sufficient space for insertion of a fiber post. It is recommended to use a cofferdam before dental treatment.
2. A preliminary X-ray examination should be performed to assess the quality of endodontic treatment to identify topography features of the root canal and to determine the tooth root inclination and curvature. Choose the appropriately sized post type based on clinical evaluation.
3. When employing root drills from the kit, it is recommended to first enlarge the root canal with smaller size tools, sequentially increasing the working tool size. Then, using drills from the kit, finalize the root canal shape that exactly matches the fiber post of one size or another.
4. When removing the root canal filling, remove it without disturbing the gutta-percha filling of the apical third (3 to 5 mm). Finish the post insertion site preparation using a bur calibrated to the selected post size (not more than 15,000 rpm).
5. During the post try-in, remember that the post must be inserted to the full depth of the root space to prevent voids and discontinuities between the post and the root filling material.
6. Make sure that the post is correctly positioned in the canal and determine its length. The root part of the post should make 2/3 and the crown part 1/3 of the post length.
7. Where necessary, cut off a part of the post outside the oral cavity with a diamond separating disc at low speed. Do not use elevators or tongs, as the pushing action will pockmark the post and damage its structure.
8. Clean the post surface with alcohol to remove any possible contamination.
9. Clean the canal in an optimal way, treat the dentine and the dentin hole according to the etchant manufacturer's instructions.
10. Flush the canal with plenty of water to remove etchant residues (preferably with an irrigation needle).

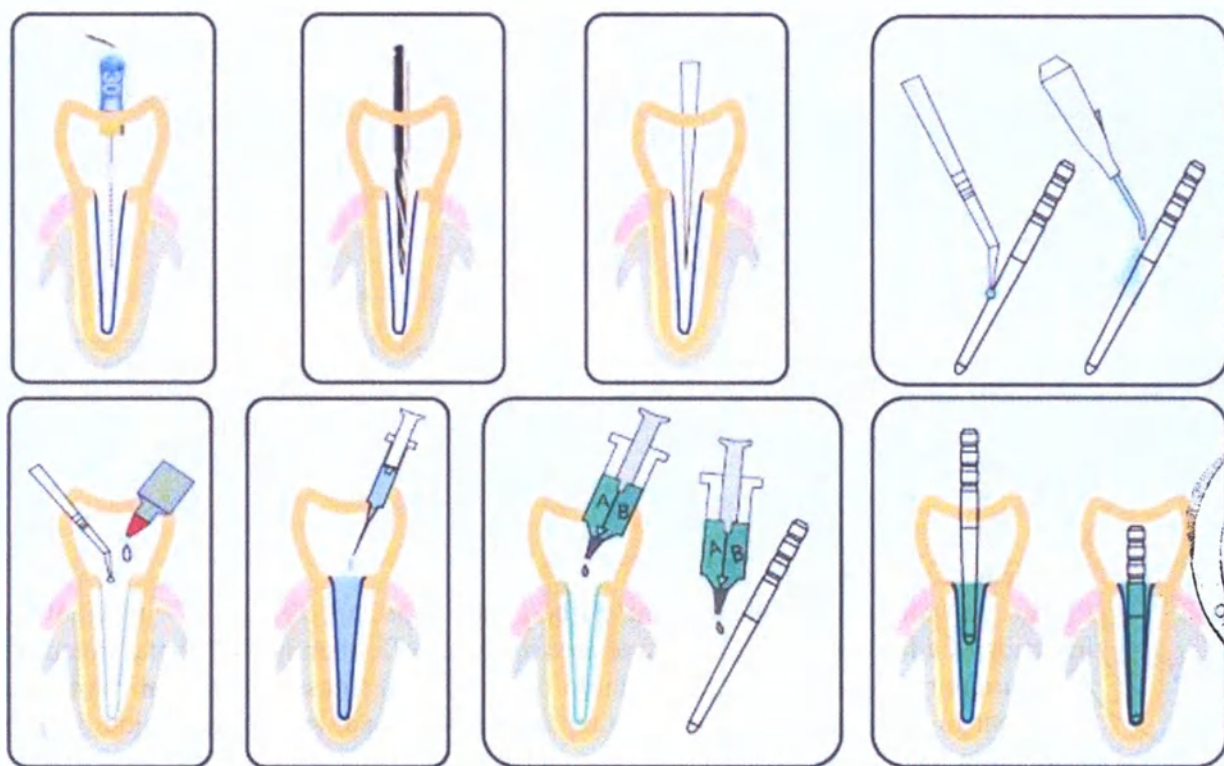11. Dry and mop the canal with paper swabs or paper points, and prepare for the subsequent cementation stages.
12. Following the manufacturer's instructions, apply adhesive agent, primer and cement to all internal surfaces of the root canal, and apply a layer of cement to the post surface.
13. Place the post into the channel and gently push it down until it stops. Spread excess cement around the post, having secured it while the cement cures.
14. Shape the restored crown part of the tooth as necessary in accordance with the instructions for use.

8 Sterilization and cleaning

Posts are disposable non-sterile medical devices; before use, treat the post surface with an alcohol solution (70 %) to remove possible contamination; reuse of the post is prohibited. Reuse significantly increases the risk of bacterial contamination and transmission of infections, and results in deformation of the post, which has already been subjected to physical and mechanical stress.

9 Components of animal origin, blood-derived products and medical preparations

Medical devices that are the subject of this Extract from the technical document do not contain components of animal origin, blood-derived products or medical preparations as constituents.

10 Phthalate content

Medical devices that are the subject of this Extract from the technical document are not intended to administer and/or remove medicinal products, biological fluids or other body substances, and are not intended to transport or store biological fluids.

In any case, both the medical devices themselves and the packaging materials used do not contain phthalates classified as carcinogenic, mutagenic or toxic to reproductive health, of Categories 1 and 2, in accordance with the Regulations (1907/2006/EC) and CLP (1272/2011/EU).

11 Use with other medical devices or medical preparations

The correct application of dental posts requires using of proper countersinks (drills); it is recommended to use MICRO.MEDICA S.r.l. dental countersinks (drills); however, use of these surgical tools is not mandatory for correct use of the posts.

Fig. 1. Cylindrical countersinks (drills)

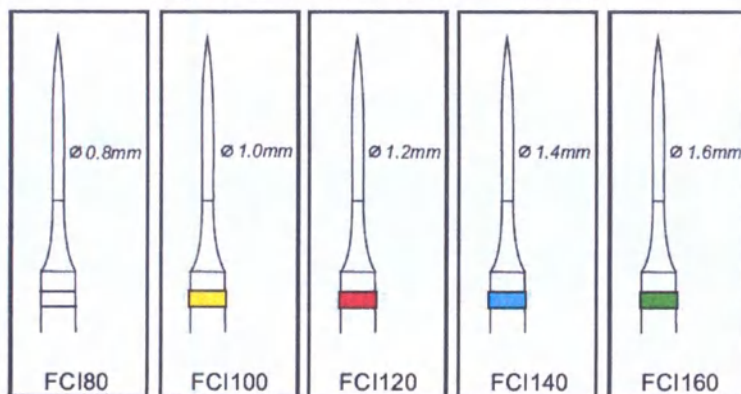
Fig. 2. Conical countersinks (drills)

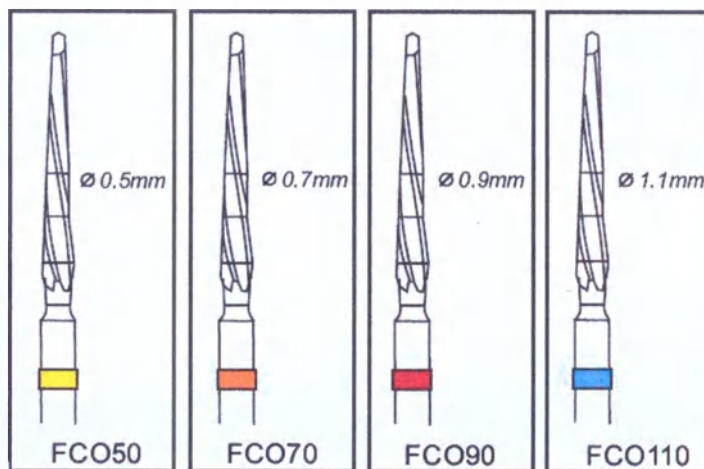
Fig. 3. Diamond countersinks (drills)

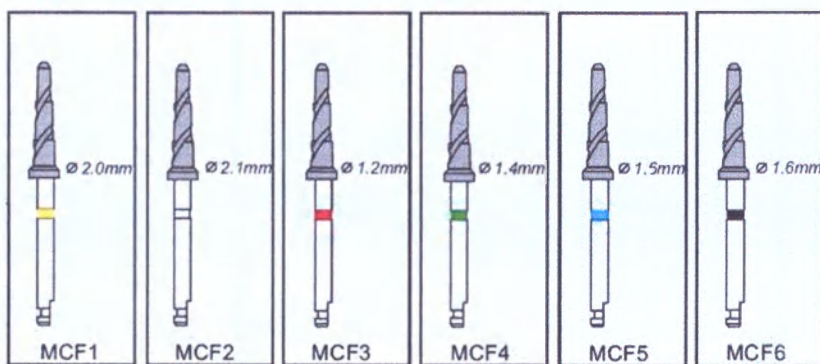
Fig. 4. Diamond countersinks (drills)

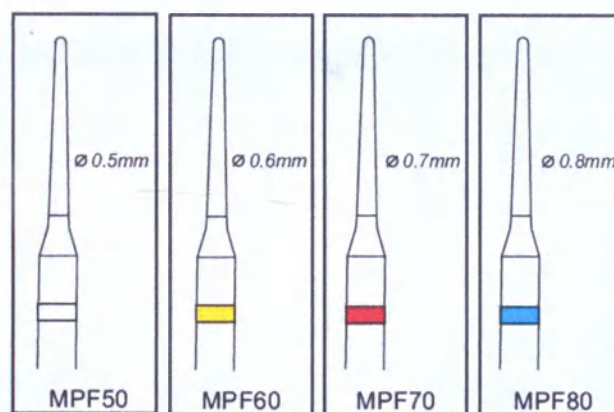
Table 14. Matching of MICRO.MEDICA S.r.l. posts and drills

	Post	Colour coding of posts	Countersink (drill)	Colour coding of drills
"Simplex":	SIM80Q, SIM80QR, SIM80QZ, SIM80V	White	FCI80	White
	SIM100Q, SIM100QR, SIM100QZ, SIM100V, CB100V	Yellow	FCI100	Yellow
	SIM120Q, SIM120QR, SIM120QZ, SIM120V, CB120V	Red	FCI120	Red
	SIM140Q, SIM140QR, SIM140QZ, SIM140V, CB140V	Blue	FCI140	Blue
	SIM160Q, SIM160QR, SIM160QZ, SIM160V	Green	FCI160	Green
"HS"	HS50Q, HS50QR, HS50QZ, HS50V	Yellow	FCO50	Yellow
	HS70Q, HS70QR, HS70QZ, HS70V	Orange	FCO70	Orange
	HS90Q, HS90QR, HS90QZ, HS90V	Red	FCO90	Red
"911"	911-50Q, 911-50V	Yellow	FCO50	Yellow
	911-70Q, 911-70V	Orange	FCO70	Orange
	911-90Q, 911-90V	Red	FCO90	Red
	911-110Q, 911-110V	Blue	FCO110	Blue
"Cobra"	CBC70Q, CBC70QR, CBC70QZ, CB70V	Orange	FCO70	Orange
	CBC90Q, CBC90QR, CBC90QZ, CB90V	Red	FCO90	Red
	CBC110Q, CBC110QR, CBC110QZ, CB110V	Blue	FCO110	Blue
	CB100V	Yellow	FCI100	Yellow
	CB120V	Red	FCI120	Red
	CB140V	Blue	FCI140	Blue
"Conical"	MCS22	Yellow	MCF2	Yellow
	MCS55	Green	MCF4	Green
	MCS88	Black	MCF6	Black
"Cylindrical"	MCS2	Yellow	MCF2	Yellow
	MCS5	Green	MCF4	Green
	MCS8	Black	MCF6	Black
Fiberpost	MC1	White	MCF1	White
	MC2	Yellow	MCF2	Yellow
	MC4	Red	MCF3	Red
	MC5	Green	MCF4	Green
	MC7	Blue	MCF5	Blue
	MC8	Black	MCF6	Black
	MC3	White	MCF1	White
	MC6	Red	MCF3	Red

	MC9	Blue	MCF5	Blue
"mediXpost"	MP50V	White	MPF50	White
	MP60V	Yellow	MPF60	Yellow
	MP70V	Red	MPF70	Red
	MP80V	Blue	MPF80	Blue
"Rapid"	RA50V	Yellow	FCO50	Yellow
	RA70V	Orange	FCO70	Orange
	RA90V	Red	FCO90	Red
	RA110V	Blue	FCO110	Blue
"Hi Techno"	HT25V	White	N/A**	-
	HT30V	Yellow	N/A**	-
	HT35V	Red	N/A**	-
	HT40V	Blue	N/A**	-
	HT50V	Green	N/A**	-
	HT60V	Black	N/A**	-

** - HT Techno does not have drills; they are used to treat canals after reamers, files or other tools

12 Requirements to maintenance and repair of the medical device

The devices are easy to use. They do not need maintenance and repair.

13 Explanation of symbols used in devices labeling:

Symbol	Description	Symbol	Description
	Manufacturer		Manufacturing date
	Lot number		Do not reuse (for posts)
	Non-sterile		Do not use if packaging is damaged
	Consult instruction for use		Catalogue number (part number)
	CE compliance mark		Warning (general risk)
	Temperature range		Use before

14 Conditions of storage, transportation and operation

Storage. No special storage procedures are required for the medical devices under consideration; you just need to ensure that the packages are not damaged.

Storage conditions:

- Temperature +5 °C to +30 °C;
- There are no special requirements to air humidity and atmospheric pressure.

Transportation. Devices packed into shipping containers should be transported by all types of covered vehicles considering the precautions indicated on the shipping container and in accordance with the guidelines for goods transportation by the specific type of transport. No special transportation procedures are required for the medical devices under consideration; you just need to ensure that the packages are not damaged.

Transportation conditions:

- Ambient temperature: +5 °C to +30 °C;
- There are no special requirements to air humidity and atmospheric pressure.

Operation. The devices are used in medical facilities, in vehicles, in home and field conditions, and in conditions requiring protection of medical worker's hands when the device is used as intended.

Indoor operating conditions:

- Ambient temperature: +15 °C to +25 °C;
- There are no special requirements to air humidity and atmospheric pressure.

15 Disposal

The medical device "Dental post STEA®" after its intended use is classified as epidemiologically hazardous waste (devices contaminated with blood and/or other body fluids of patients, as well as devices contacted with patients having infectious diseases).

In this case, a medical organization only collects, disinfects and temporarily stores the above specified waste. Disinfection should be carried out in accordance with the requirements of sanitary rules and other regulatory legal acts of a particular country in all medical and preventive institutions, regardless of their profile. The devices are collected into disposable bags or containers with color labeling adopted for hazardous medical waste in a room or in places for temporary storage of medical waste before subsequent transportation to the destruction or disposal site.

Transportation of medical waste (used disposable posts or non-reusable drills) from medical facilities, and their disposal is provided by specialized organizations licensed to treat epidemiologically hazardous medical waste.

Consumer packing, rejected and expired unused devices are classified as epidemiologically safe waste (non-hazardous waste not contacting body fluids of patients, patients having infectious diseases), which is close by its composition to municipal solid wastes (MSW). However, it is recommended to dispose of them separately as industrial waste by specialized organizations licensed to carry out this kind of activity.

Non-hazardous waste is collected in reusable containers or disposable bags (excluding bags of color used for hazardous waste). Disposable bags are placed inside reusable containers. Waste receptacle should be labelled and placed on the special site (indoors). After emptying reusable containers should be cleaned and disinfected. The procedure for collection and disinfection of reusable containers is determined in accordance with the waste management scheme in each specific organization.

Transportation and disposal are performed considering requirements adopted for a given region in accordance with the sanitary legislation of a particular country for the maintenance of populated areas and treatment of production and consumer waste.

16 Shelf life

The shelf life of posts is 5 years from the date of packaging; they are marketed non-sterile only.

17 List of international standards that the medical device complies with

l types of
ecordance
o special
st need to

Directive 93/42/EC as amended and supplemented, UNI CEI EN ISO 13485, UNI CEI EN ISO 14971, ISO 10993-1, UNI CEI EN ISO 15223-1.

The latest versions of reference standards should be used: the organization should maintain and update the list of reference standards as a record of its own management system in compliance with ISO 13485.

and field
device is

ified as
fluids of
e above
sanitary
reventive
ontainers
emporary

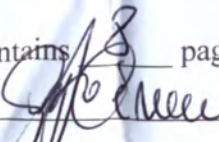
medical
to treat

ally safe
nfectious
ver, it is
nizations

g bags of
s. Waste
reusable
ection of
in each

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