UVZ Nr. 335 /2024G

We, **DeguDent GmbH**, confirm the accuracy of the provided information.

« 13 » 08 2024

Position Managing Director

Name Norbert Rabenstein

Signature

Stamp

**Dentsply
Sirona**
DeguDent GmbH
Rodenbacher Chaussee 4
D-63457 Hanau-Wolfgang

Authenticity of the signature must be confirmed notariially

Instruction for Use

For professional use by dentists only

Kiss Artist Dental Ceramic Material

manufactured by:

**«DeguDent GmbH, Rodenbacher Chaussee 4, 63457 Hanau-Wolfgang,
Tel: +49-6181 59-5826; +49-6181 59-5879**

Content

1. Description	3
1.1. Medical product.....	3
1.2. Manufacturer/Developer of the Medical Product	5
1.3. Production Facilities	5
1.4. Intended Use	5
1.5. Scope of application	5
1.6. User Profile	5
1.7. Medical Device Class.....	5
1.8. Description	5
1.9. Scope of Delivery	8
1.10. Materials.....	Ошибка! Закладка не определена.
2. Technical specification	Ошибка! Закладка не определена.
2.1. Duceram® Kiss Artist Add-on Kit	Ошибка! Закладка не определена.
2.2. Duceragold® Kiss Artist Add-on Kit	15
2.3. Cercon® Ceram Kiss Artist Add-on Kit.....	16
2.4. Duceram® Kiss Basic Set	19
2.5. Duceram® Kiss Aesthetic.....	22
2.6. Duceram® Kiss Starter Set	25
3. Images	29
4. Indications for Use	41
5. Contradictions.....	Ошибка! Закладка не определена.
6. Warnings	Ошибка! Закладка не определена.
7. Adverse events.....	Ошибка! Закладка не определена.
8. The procedure for working with the product: Step-by-step technique for applying layers of facing ceramics.....	Ошибка! Закладка не определена.
9. The principle of working with the product.....	46
10. General and special recommendations for firing when facing with Kiss Artist materials Kiss Artist	49
11. Sterilization	51
12. Conditions and shelf life	51
13. Transportation conditions	52
14. Requirements to Environmental Protection	52
15. Environmental Conditions for Operation	52
16. Conditions of use	52
17. Requirements to Maintenance and Repair	52
18. Labeling	52
19. List of applicable international standards	Ошибка! Закладка не определена.
20. Disposal.....	Ошибка! Закладка не определена.
21. Manufacturer's Warranties	Ошибка! Закладка не определена.
22. Authorized Representative.....	Ошибка! Закладка не определена.

1. Description

1.1. Medical Product

Kiss Artist Dental Ceramic Material:

1. Duceram® Kiss Artist Add-on Kit, including:

- Duceram® Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceram® Kiss Transpa Red Ceramic Material (20 g);
- Duceram® Kiss Gray Inside Fluorescent Dentine (20 g);
- Duceram® Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 ml);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

2. Duceragold® Kiss Artist Add-on Kit, including:

- Duceragold® Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceragold® Kiss Transpa Red Ceramic Material (20 g);
- Duceragold® Kiss Gray Inside Fluorescent Dentine (20 g);
- Duceragold® Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 ml);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

3. Cercon® Ceram Kiss Artist Add-on Kit, including:

- Cercon® Ceram Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Cercon® Ceram Kiss Transpa Red Ceramic Material (20 g);
- Cercon® Ceram Kiss Gray Inside Fluorescent Dentine (20 g);
- Cercon® Ceram Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 ml);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

4. Duceram® Kiss Basic Set, including:

- Duceram® Kiss Dentine (20 g), shades: DA 1, DA 2, DA 3, DA 3,5, DA 4, DB 1, DB 2, DB 3, DB 4, DC 1, DC 2, DC 3, DC 4, DD 2, DD 3, DD 4, Bleach
- Duceram® Kiss Schneide (20 g), shades: S 1, S 2, S 3, S 4, S 5, S 6;
- Duceram® Kiss Transpa TC mass (20 g);
- Duceram® Kiss Final Kiss Korrekturmasse mass (20 g);
- Duceram® Kiss Paste Opaque (PO) (3ml), shades: A1, A2, A3, A3,5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4, Bleach;
- Duceram® Kiss Paste Opaque (PO) Orange (3ml);
- Duceram® Kiss Paste Opaque (PO) Neutralpaste (3ml);
- Dentsply Universal NE-Bonder / NP-Bonder (4ml);
- Ducera® Liquid FORM (50 ml);

- Ducera® Liquid SD (50 ml);
- Pastenopaker Liquid (2 ml);
- Universal Malfarben u. Glasur Liquid (15ml);
- Universal Overglaze (5g);
- Shades Indikatoren: Indikator Duceram® Kiss Base, Indikator Duceram® Kiss Opaque;
- Dentsply Sirona Handtuch;
- Dentsply Sirona Schwamm;
- Probenformer ohne Verpackung
- Pinselstiel "Pinelo";
- Brush-tip: Pinselspitze "Pinelo" opaque/stain, Pinselspitze "Pinelo" No. 8;
- Instruction for Use – 1 pc.

5. Duceram® Kiss Aesthetic Set, including:

- Duceram® Kiss Power Chroma mass (20 g), ußera: PC 1, PC 2, PC 3, PC 4, PC 5, PC 6;
- Duceram® Kiss Flu Inside mass (20 g), shades: Flu 1, Flu 2;
- Duceram® Kiss Action-i Dentine (20 g), shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceram® Kiss Gray Inside mass(GI) (20 g);
- Duceram® Kiss Transpa Red mass (20 g);
- Duceram® Kiss Opalschneide (20 g), shades: OS 1, OS 2;
- Duceram® Kiss Gum Dentin (20 g), shades: Gum 1, Gum 2, Gum 3, Gum 4;
- Duceram® Kiss Opal Effekt (20 g), shades: Sunrise, Sunset, Sky, Ocean, Fog;
- Duceram® Kiss White Surface (WS)mass (20 g);
- Duceram® Kiss Stand By (S By) mass (20 g);
- Ducera Liquid FORM (15 ml);
- Ducera Liquid SD (50 ml);
- Duceram® Kiss Paste Opaque (PO) Gum (3ml);
- Shades : Indikatoren: Indikator Duceram® Kiss Power, Indikator Duceram® Kiss Opal, Indikator Duceram® Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside;
- Probenformer ohne Verpackung
- Dentsply Sirona Handtuch
- Test pill former.
- Dentsply Sirona Schwamm;
- Pinselstiel "Pinelo";
- Brush-tip: Pinselspitze "Pinelo" No. 8;
- Instruction for Use – 1 pc.

6. Duceram® Kiss Starter Set, including:

- Duceram® Kiss Paste Opaque (PO) (3ml), shades: A2, A3, A3,5, B1, B3, D3;
- Duceram® Kiss Paste Opaque (PO) Neutral (3ml);
- Dentsply Universal NE-Bonder / NP-Bonder (4ml);
- Duceram® Kiss Dentine (10 g), shades: DA 2, DA 3, DA 3,5, DB 1, DB 3, DD 3, Bleach;
- Duceram® Kiss Gum Dentin (10 g), shades: Gum 1;
- Duceram® Kiss Power Chroma mass (10 g), shades: PC 1, PC 2, PC 3, PC 4, PC 5, PC 6;
- Duceram® Kiss Schneide (10 g), shades: S 1, S 2, S 3, S 4;
- Duceram® Kiss Opalschneide (10 g), shades: OS 1, OS 2;
- Duceram® Kiss Opal Effekt mass (10 g), shades: Sunset, Ocean;
- Duceram® Kiss Stand By (S By)mass (10 g);
- Duceram® Kiss Transpa Red mass (10 g);
- Duceram® Kiss Final Kiss Korrekturmasse mass (10 g);

- Universal Overglaze (5g);
- UNIV INCISAL STAIN i1 (5g);
- UNIVERSAL BODY STAIN S2 (5g);
- Universal Malfarben u. Glasur Liquid (15ml);
- Pastenopaker Liquid (2 ml);
- Ducera Liquid FORM (15 ml);
- Ducera Liquid SD (50 ml);
- Shades Indicators: Indikator Duceram® Kiss Starter Power, Indikator Duceram Kiss Starter Base+Opal;
- Dentsply Sirona Handtuch
- Probenformer ohne Verpackung
- Dentsply Sirona Schwamm;
- Pinselstiel "Pinelo";
- Brush-tip: "Pinelo" No. 0, Pinselspitze "Pinelo" No. 8, Pinselspitze "Pinelo"; opaque/stain
- Instruction for Use – 1 pc.

1.2. Manufacturer/Developer of the Medical Product

DeguDent GmbH, Rodenbacher Chaussee 4, 63457 Hanau-Wolfgang, Germany
Tel: +49-6181 59-5826; +49-6181 59-5879; e-mail: Info.Degudent-de@dentsply.com

1.3. Production Facilities

DeguDent GmbH, Rodenbacher Chaussee 4, 63457 Hanau-Wolfgang, Germany
Tel: +49-6181 59-5826; +49-6181 59-5879; e-mail: Info.Degudent-de@dentsply.com

1.4. Intended Use

Kiss Artist Dental Ceramic Material manufactured by *DeguDent GmbH*, Germany, is intended for veneering of crowns and bridge prosthetics.

1.5. Scope of application

Dentistry.

1.6. User Profile

For use by dentists only

1.7. Medical Device Class

Class risk is 2a.

Dental ceramic materials, type 1 Class 1 according to DIN EN ISO 6872- Dental ceramic (Ceramic materials for dental purposes)

1.8. Description:

Duceram® Kiss Artist is a high-fusing dental ceramic material intended for veneering of crowns and prosthetic bridges with metal frameworks made of dental alloys with a thermal expansion coefficient (CTE) within the range 13.8 to 15.4 $\mu\text{m/m} \cdot \text{K}$ (25 - 600°C). Duceram® Kiss Artist is compatible with high- and reduced-gold and non-noble dental alloys.

Duceragold® Kiss Artist is a dental ceramic material suitable for veneering of metal bases and frameworks of single crowns and bridges with a thermal expansion coefficient (CTE) within the range 16.2 to 17.6 $\mu\text{m/m} \cdot \text{K}$ (25 - 600°C), and veneering of Crego pressable glass ceramics for fabrication of frameworks. Duceragold® Kiss Artist is compatible with high-gold alloys.

Cercon® Ceram Kiss Artist is a veneering ceramic material developed specifically and exclusively for veneering of zirconium dioxide (Y-TZP) crowns and bridges with a thermal

expansion coefficient (CTE) of $10.5 \mu\text{m/m} \cdot \text{K}$ ($25 - 500^\circ\text{C}$), which should preferably be made of Cercon® base or Cercon® base ht.

Duceram® Kiss Basic Set, Duceram® Kiss Aesthetic Set, Duceram® Kiss Starter Set - these are high-fusing ceramic materials for veneering crowns and bridges with metal frames made of dental alloys, with a range of thermal expansion coefficient (CTE) $13.8 - 15.4 \mu\text{m/m} \cdot \text{K}$ ($25 - 600^\circ\text{C}$).

Duceram® Kiss is compatible with high- and reduced-gold and non-noble dental alloys.

Action-i Dentine is a veneering ceramic material used as a material for modeling of dentine bases which are shaped with an emphasis on their shade and the degree of transparency of the natural dentine.

Duceram® Kiss Dentine are dentine that match the natural shades of teeth in color to perform ceramic restoration based on the pattern of a natural tooth

Gray Inside fluorescent dentines are unique fluorescent dentines used to compensate for especially important contours within the region of the biting edges.

Duceram® Kiss Gray Inside (GI) mass is a fluorescent mass used to compensate for contours in the cutting edge area

Duceram® Kiss Power Chroma mass is a fluorescent mass with high color saturation for individual color rendering, to enhance color in the cervical, palatine and occlusal areas

Transpa Red ceramic material is a transparent effect which gives an additional reddish hue to teeth.

Duceram® Kiss Schneider cutting edge mass – cutting edge mass to form the cutting edge area
The mass of the cutting edge is opal Duceram® Kiss Opalschneide – masses of the cutting edge with an opalescence effect to reproduce the opalescence of natural enamel

UNIV INCISAL STAIN i1 helps to reproduce most of the complex and intricate volumetric effects observed in natural teeth.

Duceram® Kiss Final Kiss Korrekturmasse correction mass is a corrective mass for firing after glossy firing.

Duceram® Kiss Opal Effekt – opal effect-masses for individualization of the cutting third of the tooth

Duceram® Kiss Flu Inside mass, Transparent Transpa TC Mass, Duceram® Kiss White Surface (WS) Mass, Duceram® Kiss Stand By (S By) mass - transparent masses, provides additional transparency of tooth surfaces.

UNIVERSAL BODY STAIN S2 is a material for reproducing surface color effects

Duceram® Kiss Paste Opaque (PO) - paste opaque, corresponding to the color of the tooth, low light permeability, provide good covering ability of the material for the manufacture of crowns.

Duceram® Kiss Paste Opaque (PO) Orange is a paste opaque in orange color to create aesthetic natural crown effects

- **Indicator Duceram® Kiss Opal** is designed for individual determination of the color nuances of opal masses according to the individual characteristics of the patient.
- **Indicator Duceram® Kiss Gingiva** designed to determine the color nuances of gingival masses, dentins for individual characteristics of the patient
- **Indicator Duceram Kiss Starter Base+Opal** - for individual determination of the color nuances of dentins, opal masses of the cutting edge and effect masses on the V-Classic scale and their mixing options

Dentsply Sirona Handtuch towel is a towel for equipping the workplace of a ceramic technician.

Dentsply Sirona Schwamm sponge is a sponge for removing excess liquid from brushes

1.9. Scope of Delivery

1. Duceram® Kiss Artist Add-on Kit, including:

- Duceram® Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceram® Kiss Transpa Red Ceramic Material (20 g);
- Duceram® Kiss Gray Inside Fluorescent Dentine (20 g);
- Duceram® Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 g);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

2. Duceragold® Kiss Artist Add-on Kit, including:

- Duceragold® Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceragold® Kiss Transpa Red Ceramic Material (20 g);
- Duceragold® Kiss Gray Inside Fluorescent Dentine (20 g);
- Duceragold® Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 g);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

3. Cercon® Ceram Kiss Artist Add-on Kit, including:

- Cercon® Ceram Kiss Action-i Dentine (20 g); shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Cercon® Ceram Kiss Transpa Red Ceramic Material (20 g);
- Cercon® Ceram Kiss Gray Inside Fluorescent Dentine (20 g);
- Cercon® Ceram Kiss Gum Dentine (20 g); shades: Gum 3, Gum 4;
- Ducera Liquid SD (Modeling Liquid) (50 g);
- Magic Wizard Brush;
- Shade indicators: Artist Gum, Artist Add-on;
- Test pill former Base for Shade Samples;
- Instruction for Use – 1 pc.

4. Duceram® Kiss Basic Set, including:

- Duceram® Kiss Dentine (20 g), shades: DA 1, DA 2, DA 3, DA 3,5, DA 4, DB 1, DB 2, DB 3, DB 4, DC 1, DC 2, DC 3, DC 4, DD 2, DD 3, DD 4, Bleach
- Duceram® Kiss Schneide (20 g), shades: S 1, S 2, S 3, S 4, S 5, S 6;
- Duceram® Kiss Transpa TC mass (20 g);
- Duceram® Kiss Final Kiss Korrekturmasse mass (20 g);
- Duceram® Kiss Paste Opaque (PO) (3ml), shades: A1, A2, A3, A3,5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4, Bleach;
- Duceram® Kiss Paste Opaque (PO) Orange (3ml);
- Duceram® Kiss Paste Opaque (PO) Neutralpaste (3ml);
- Dentsply Universal NE-Bonder / NP-Bonder (4ml);
- Ducera® Liquid FORM (50 ml);
- Ducera® Liquid SD (50 ml);
- Pastenopaker Liquid (2 ml);
- Universal Malfarben u. Glasur Liquid (15ml);
- Universal Overglaze (5g);
- Shades Indikatoren: Indikator Duceram® Kiss Base, Indikator Duceram® Kiss Opaque;
- Dentsply Sirona Handtuch;
- Dentsply Sirona Schwamm;
- Probenformer ohne Verpackung
- Pinselstiel "Pinelo";
- Brush-tip: Pinselspitze "Pinelo" opaque/stain, Pinselspitze "Pinelo" No. 8;
- Instruction for Use – 1 pc.

5. Duceram® Kiss Aesthetic Set, including:

- Duceram® Kiss Power Chroma mass (20 g), ubera: PC 1, PC 2, PC 3, PC 4, PC 5, PC 6;
- Duceram® Kiss Flu Inside mass (20 g), shades: Flu 1, Flu 2;
- Duceram® Kiss Action-i Dentine (20 g), shades: Butter, Chocolate, Corn, Creme, Honey, Lavend, Marble;
- Duceram® Kiss Gray Inside mass(GI) (20 g);
- Duceram® Kiss Transpa Red mass (20 g);
- Duceram® Kiss Opalschneide (20 g), shades: OS 1, OS 2;
- Duceram® Kiss Gum Dentin (20 g), shades: Gum 1, Gum 2, Gum 3, Gum 4;
- Duceram® Kiss Opal Effekt (20 g), shades: Sunrise, Sunset, Sky, Ocean, Fog;
- Duceram® Kiss White Surface (WS)mass (20 g);
- Duceram® Kiss Stand By (S By) mass (20 g);
- Ducera Liquid FORM (15 ml);
- Ducera Liquid SD (50 ml);
- Duceram® Kiss Paste Opaque (PO) Gum (3ml);
- Shades : Indikatoren: Indikator Duceram® Kiss Power, Indikator Duceram® Kiss Opal, Indikator Duceram® Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside;
- Probenformer ohne Verpackung
- Dentsply Sirona Handtuch
- Test pill former.
- Dentsply Sirona Schwamm;
- Pinselstiel "Pinelo";
- Brush-tip: Pinselspitze "Pinelo" No. 8;
- Instruction for Use – 1 pc.

6. Duceram® Kiss Starter Set, including:

- Duceram® Kiss Paste Opaque (PO) (3ml), shades: A2, A3, A3,5, B1, B3, D3;

- Duceram® Kiss Paste Opaque (PO) Neutral (3ml);
- Dentsply Universal NE-Bonder / NP-Bonder (4ml);
- Duceram® Kiss Dentine (10 g), shades: DA 2, DA 3, DA 3,5, DB 1, DB 3, DD 3, Bleach;
- Duceram® Kiss Gum Dentin (10 g), shades: Gum 1;
- Duceram® Kiss Power Chroma mass (10 g), shades: PC 1, PC 2, PC 3, PC 4, PC 5, PC 6;
- Duceram Kiss Schneide (10 g), shades: S 1, S 2, S 3, S 4;
- Duceram® Kiss Opalschneide (10 g), shades: OS 1, OS 2;
- Duceram® Kiss Opal Effekt mass (10 g), shades: Sunset, Ocean;
- Duceram® Kiss Stand By (S By)mass (10 g);
- Duceram® Kiss Transpa Red mass (10 g);
- Duceram® Kiss Final Kiss Korrekturmasse mass (10 g);
- Universal Overglaze (5g);
- UNIV INCISAL STAIN il (5g);
- UNIVERSAL BODY STAIN S2 (5g);
- Universal Malfarben u. Glasur Liquid (15ml);
- Pastenopaker Liquid (2 ml);
- Ducera Liquid FORM (15 ml);
- Ducera Liquid SD (50 ml);
- Shades Indicators: Indikator Duceram® Kiss Starter Power, Indikator Duceram Kiss Starter Base+Opal;
- Dentsply Sirona Handtuch
- Probenformer ohne Verpackung
- Dentsply Sirona Schwamm;
- Pinselstiel "Pinelo";
- Brush-tip: "Pinelo" No. 0, Pinselspitze "Pinelo" No. 8, Pinselspitze "Pinelo"; opaque/stain
- Instruction for Use – 1 pc.

1.10. Materials

1.10.1 Duceram® Kiss Artist Add-on Kit

Composition of Duceram® Kiss Action-i Dentine, Duceram® Kiss Gray Inside Fluorescent Dentine, Duceram® Kiss Gum Dentine

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

Additives for fluorescent dentin Duceram® Kiss Gray Inside:

0,1 – 5% pigment from the mixture Y₂SiO₆, Gd₂O₃, CeO₂. In order to reduce transparency, the following pigments are used: TiO₂, SnO₂, SrCl₂ и CeO₂.

Composition of Duceram® Kiss Transpa Red

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

1.10.2. Duceragold® Kiss Artist Add-on Kit

Composition of Duceragold® Kiss Action-i Dentine, Duceragold® Kiss Gray Inside Fluorescent Dentine, Duceragold® Kiss Gum Dentine

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, CaO, F, B ₂ O ₃ , CeO ₂

Additives for fluorescent dentin Duceragold® Kiss Gray Inside:

0,1 – 5% pigment from the mixture Y_2SiO_6 , Gd_2O_3 , CeO_2 . In order to reduce transparency, the following pigments are used TiO_2 , SnO_2 , $SrCl$ u CeO_2 . For opalescence, it is additionally used ZrO_2 .

Composition of Duceragold® Kiss Transpa Red

Component
SiO_2 , Al_2O_3 , K_2O , Na_2O , CaO , F , B_2O_3 , CeO_2

1.10.3. Cercon® Ceram Kiss Artist Add-on Kit

Composition of Cercon® Ceram Kiss Action-i Dentine, Cercon® Ceram Kiss Gray Inside Fluorescent Dentine, Cercon® Ceram Kiss Gum Dentin

Component
SiO_2 , Al_2O_3 , K_2O , Na_2O , Li_2O , BaO , B_2O_3 , CaO , CeO_2

Additives for fluorescent dentin Cercon® Ceram Kiss Gray Inside:

0,1 – 5% pigment from the mixture Y_2SiO_6 , Gd_2O_3 , CeO_2 . In order to reduce transparency, the following pigments are used TiO_2 , SnO_2 , $SrCl$ u CeO_2 . For opalescence, it is additionally used ZrO_2 .

Cercon® Ceram Kiss Transpa Red

Component
SiO_2 , Al_2O_3 , K_2O , Na_2O , Li_2O , BaO , B_2O_3 , CaO , CeO_2

Liquid for modeling Ducera Liquid SD in a bottle 50 ml.

Component
Bergamot Essential Oil
Strontium chloride
Water

Magic Wizard brush.

The material of the silver clip is nickel-plated brass, matte:

Brand JH65-5,

Cu – 64,0-67,0%

Ni - 5,0-6,0%

Brush pile material (bristles): Natural Siberian weasel hair

Handle material – wood

Inscription (silver) Magic Wizard FINEST KOLINSKY

Test pill former.

Material – chrome-plated steel

Brand AISI 52100 (Cr 1,48%; S 0,007%; P 0,012%; Mn 0,43%; Si 0,20%; C 0,96%; Mo 0,052%; Ni 0,10%). AISI 52100 AISI 52100

Shade Indicators: Artist Gum, Artist Add-on.

Brand: Polycarbonate APEC 1745.

1.10.4. Duceram® Kiss Basic Set

Composition of Duceram® Kiss Dentine, Duceram® Kiss Schneide, Duceram® Kiss Transpa TC

Component

SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

Composition of Duceram® Kiss Final Kiss Korrekturmasse

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, BaO, B ₂ O ₃ , CaO, CeO ₂ , Sb ₂ O ₃ , F

Composition of Duceram® Kiss Paste Opaque (PO), Duceram® Kiss Paste Opaque (PO) Orange, Duceram® Kiss Neutral

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃ , SnO ₂ , ZrO ₂
Paste opaque liquid (mixture 1,2,6- Hexantriol and 1,4-Butanediol and Optigel

1.10.5. Duceram® Kiss Aesthetic Set

Composition of Duceram® Kiss Power Chroma, Денгун Duceram® Kiss Action-i Dentine, Duceram® Kiss Gum, Duceram® Kiss Gray Inside (GI), Duceram® Kiss Flu Inside, Duceram® Kiss White Surface (WS), Duceram® Kiss Stand By (S By)

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

Additives for Duceram® Kiss Gray Inside (GI):

0,1 – 5% pigment from the mixture Y₂SiO₆, Gd₂O₃, CeO₂. In order to reduce transparency, the following pigments are used: TiO₂, SnO₂, SrCl и CeO₂.

Composition of Macca Duceram® Kiss Transpa Red

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

Composition of Duceram® Kiss Opalschneide, Macca onax-эффект Duceram® Kiss Opal Effekt

Component
Si O ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, Ca O, Ce O ₂ , Sb ₂ O ₃ , F, P ₂ O ₃ , Sn O ₂

Composition of Duceram® Kiss Paste Opaque (PO) Gum;

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃ , SnO ₂ , ZrO ₂
Paste opaque liquid (смесь 1,2,6- Hexantriol and 1,4-Butanediol and Optigel

1.10.6. Duceram® Kiss Starter Set

Composition of Duceram® Kiss Paste Opaque (PO), Duceram® Kiss Neutral

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃ , SnO ₂ , ZrO ₂
Жидкость Paste opaque liquid (смесь 1,2,6- Hexantriol and 1,4-Butanediol and Optigel

Composition of Duceram® Kiss Dentine, Duceram® Kiss Gum Dentine, Duceram® Kiss Power Chroma, Duceram® Kiss Schneide, Duceram® Kiss Stand By (S By):

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

Composition of Duceram® Kiss Opalschneide, Duceram® Kiss Opal Effekt:

Component
Si O ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, Ba O, Ca O, Ce O ₂ , Sb ₂ O ₃ , F, P ₂ O ₅ , Sn O ₂

Composition of Duceram® Kiss Transpa Red:

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, CaO, F, Sb ₂ O ₃

UNIV INCISAL STAIN II (5z)

The mass is presented in the form of a paste. The paste consists of 70% powder and 30% gel. The composition of the powder is given below:

Component
SiO ₂ , Al ₂ O ₃ , Na ₂ O, K ₂ O, Li ₂ O, BaO, CaO, B ₂ O ₃ , CeO ₂ , Tb ₂ O ₃ , TiO ₂ , Sb ₂ O ₃

The composition of the gel used for kneading the paste is given below:
ammonium nitrate (0.1 – 1%), butylene glycol (52-60%), propylene glycol (35-38%), silicon dioxide, colloidal (2-3%), tylose H (2.5-3.5%).

* The composition varies in the specified ranges based on the desired shades.

UNIVERSAL BODY STAIN S2 (5z)

The mass is presented in the form of a paste. The paste consists of 70% powder and 30% gel. The composition of the powder is given below:

Component
SiO ₂ , Al ₂ O ₃ , Na ₂ O, K ₂ O, Li ₂ O, BaO, CaO, B ₂ O ₃ , CeO ₂ , Tb ₂ O ₃ , TiO ₂ , Sb ₂ O ₃

The composition of the gel used for kneading the paste is given below:
ammonium nitrate (0.1 – 1%), butylene glycol (52-60%), propylene glycol (35-38%), silicon dioxide, colloidal (2-3%), tylose H (2.5-3.5%).

* The composition varies in the specified ranges based on the desired shades.

Dentsply Universal NE-Bonder / NP-Bonder

Component
SiO ₂ , Al ₂ O ₃ , K ₂ O, Na ₂ O, Li ₂ O, BaO, B ₂ O ₃ , CaO, Sb ₂ O ₃ , ZrO ₂ , SnO ₂
Paste opaque liquid (смесь 1,2,6- Hexantriol and 1,4- Butandiol)

Ducera Liquid FORM(50 ml), Ducera Liquid SD(50 ml)

Bergamot Essential Oil
Strontium chloride
Water

Pastenopaker Liquid (2 ml)

1,2,6 - Hexantriol
1,4 - Butandiol

Universal Malfarben u. Glasur Liquid (15 мл)

Component
Butylene Glycol, 99% (714-501)

Universal Overglaze

The overglaze is presented in the form of a paste. The paste consists of 70% powder and 30% gel.

The composition of the powder is given below:

Component
SiO ₂ , Al ₂ O ₃ , Na ₂ O, K ₂ O, Li ₂ O, BaO, CaO, B ₂ O ₃ , CeO ₂ , Tb ₂ O ₃ , TiO ₂ , Sb ₂ O ₃

The composition of the gel used for kneading the paste is given below:

ammonium nitrate (0.1 – 1%), butylene glycol (52-60%), propylene glycol (35-38%), silicon dioxide, colloidal (2-3%), tylose H (2.5-3.5%).

1.10.7. Shade Indicators: Indikator Duceram® Kiss Base, Indikator Duceram® Kiss Opaque, Indikator Duceram® Kiss Power, Indikator Duceram® Kiss Opal, Indikator Duceram® Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside, Indikator Duceram® Kiss Starter Power, Indikator Duceram® Kiss Base+Opal

Brand: Polycarbonate brand APEC 1745, manufacturer Covestro LLC

1.10.8. Полотенце Dentsply Sirona Handtuch

Brand: 96% cotton, 4% polyester, brand DPDDFS-041, manufacturer Newstetic

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Handle material	Aluminum tube, AlMgSi05, brand EN-AW 6060, manufacturer Aluminco S.A.
The end of the handle	A magnet for placing the tip of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd.

1.10.11. Brush-tip: Pinselspitze "Pinelo" opaque/stain, Pinselspitze "Pinelo" No. 8

The material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67%, Ni 5—6,5%, Zn 26,5-31%), black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene Terephthalate (PBT), brand LUPOX, manufacturer LG Chem, Ltd.
Tip of the nozzle	A magnet for placing the brush handles, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd.

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Material – chrome-plated steel

Brand AISI 52100 (Cr 1,48%; S 0,007%; P 0,012%; Mn 0,43%; Si 0,20%; C 0,96%; Mo 0,052%; Ni 0,10%), manufacturer RGPBALLS S.R.L.

1.10.13. Test pill former

Material – chrome-plated steel

Brand AISI 52100 (Cr 1,48%; S 0,007%; P 0,012%; Mn 0,43%; Si 0,20%; C 0,96%; Mo 0,052%; Ni 0,10%), manufacturer RGPBALLS S.R.L.

2. Technical Specifications

2.1. Duceram® Kiss Artist Add-on Kit

- Dentine thermal expansion coefficient: $13.0 \mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Ceramic material thermal expansion coefficient: $12.8 \mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Dentine density: 2.42g/cm^3
- Volume shrinkage in the process of dentine baking: 24%
- Linear shrinkage in the process of dentine baking: 13.9%

- **Baking temperature**

1 st baking of dentine	Approx. 910°C	Instructions for baking are included to the instruction manuals.
-----------------------------------	---------------	--

- **Vitrification temperature**

Dentine	520°C
Ceramic material	520°C

- Dental ceramic material, type 1, class I pursuant to DIN EN ISO 6872;

- Hardness: 490 – 550 HV 02;
- Flexural strength: 60 – 70 MPa pursuant to DIN EN ISO 9693/6872 (≥ 50 MPa);
- Chemical solubility: $< 100 \mu\text{g/cm}^2$ pursuant to DIN EN ISO 6872 and DIN EN ISO 9693;
- Use only with alloys with a solidification temperature of at least 1 030 °C.

Vials Duceram® Kiss Artist Add-on Kit for ceramic powder and vials for modeling liquid Ducera Liquid SD 50 ml

Weight / Volume	Description	Color	Materials	Dimensions	Weight
20 g / 35 ml	Vials with a wide neck	Transparent	PAT cas# 24938-04-3	Height: 50 mm $\pm$ 10% Diameter: 37 mm $\pm$ 10%	15,3 g $\pm$ 10%
	Cap	Red	carbamide resin cas# 9011-05-6	Height: 20 mm $\pm$ 10% Diameter: 37 mm $\pm$ 10%	9,3 g $\pm$ 10%
50 ml	Plastic vial	Transparent	high density polyethylene cas# 74-85-1 / 106-98-9	Height: 84 mm $\pm$ 10% Diameter: 32,8 mm $\pm$ 10%	6,6 g $\pm$ 10%
	Cap	White	carbamide resin cas# 9011-05-6	Height: 20 mm $\pm$ 10% Diameter: 31,9 mm $\pm$ 10%	7,9 g $\pm$ 10%

The safety membrane is made of LDPE/polystyrene
All primary packaging for powders is wrapped in PVC film.

2.2. Duceragold® Kiss Artist Add-on Kit

- Dentine thermal expansion coefficient: $15.1 \mu\text{m/m} \cdot \text{K}$ (25-500 °C);
- Ceramic material thermal expansion coefficient: $14.9 \mu\text{m/m} \cdot \text{K}$ (25-500 °C);
- Dentine density: 2.41g/cm^3 ;
- Volume shrinkage in the process of dentine baking: 23.9 %;
- Linear shrinkage in the process of dentine baking: 13.9 %;

- Baking temperature

1 st baking of dentine	Approx. 780°C	Instructions for baking are included to the instruction manuals.
-----------------------------------	---------------	--

- Vitrification temperature

Dentine	490°C
Ceramic material	490°C

- Dental ceramic material, type 1, class 1 pursuant to DIN EN ISO 6872;
- Hardness: 450 – 510 HV 02;
- Flexural strength: 100-120 MPa pursuant to DIN EN ISO 9693/6872;
- Chemical solubility: $< 100 \mu\text{g/cm}^2$ pursuant to DIN EN ISO 6872 and DIN EN ISO 9693;
- Use only with alloys with a solidification temperature of at least 900 °C.

Vials Duceragold® Kiss Artist Add-on Kit for ceramic powder and vials for modeling liquid Ducera Liquid SD 50 ml

Weight / Volume	Description	Color	Materials	Dimensions	Weight
20 g / 35 ml	Vials with a wide neck	Transparent	PAT cas# 24938-04-3	Height: 50 mm $\pm$ 10% Diameter: 37 mm $\pm$ 10%	15,3 g $\pm$ 10%
	Cap	Gold	carbamide resin cas# 9011-05-6	Height: 20 mm $\pm$ 10% Diameter: 37 mm $\pm$ 10%	9,3 g $\pm$ 10%
50 ml	Plastic vial	Transparent	high density polyethylene cas# 74-85-1 / 106-98-9	Height: 84 mm $\pm$ 10% Diameter: 32,8 mm $\pm$ 10%	6,6 g $\pm$ 10%
	Cap	White	carbamide resin cas# 9011-05-6	Height: 20 mm $\pm$ 10% Diameter: 31,9 mm $\pm$ 10%	7,9 g $\pm$ 10%

The safety membrane is made of LDPE/polystyrene
All primary packaging for powders is wrapped in PVC film.

2.3. Cercon® Ceram Kiss Artist Add-on Kit

- Dentine thermal expansion coefficient: $9.2 \mu\text{m/m} \cdot \text{K}$ (25-500 °C);
- Ceramic material thermal expansion coefficient: $9.2 \mu\text{m/m} \cdot \text{K}$ (25-500 °C);
- Dentine density: 2.31g/cm^3 ;
- Volume shrinkage in the process of dentine baking: 30.4%;

- Linear shrinkage in the process of dentine baking: 14.5 %;

- Baking temperature

1 st baking of dentine	Approx. 830°C	Instructions for baking are included to the instruction manuals.
-----------------------------------	---------------	--

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The safety membrane is made of LDPE/polystyrene

All primary packaging for powders is wrapped in PVC film.

Secondary packaging used for Набор Duceram® Kiss Artist Add-on Kit, Duceragold® Kiss Artis Add-on Kit, Cercon® Ceram Kiss Artis Add-on Kit

Description	Color	Materials	Dimensions	Weight
Suitcase	black	aluminum	270 x 200 x 53 mm	980 g ± 10%

Ducera Liquid SD (Modeling Liquid) in 50 ml vials

Mass of the vial filled with the liquid: 65 g ± 2;

Volume of the liquid: 50 ml

Magic Wizard brush.

Brush filament (bristle) thickness: 0.1±0.01 mm

Resistance characteristics:

- Shape and color stability when powders, pastes, and accessory materials are used;

- In the process of testing of the brush the loss of bristles shall not exceed three bristles after use.

Filament length (from the clamp to the tip): 21 ± 1 mm;

Diameter of the upper part of the silver clamp: 6 ± 0.2 mm;

Length of the silver clamp: 40 ± 1 mm;

Strength of brush filaments: 0.07 – 0.20, tolerance ± 0.05 mm;

Overall brush length: 198 ± 5 mm;

Overall brush length, including the protective cap for filaments: 210 ± 10 mm

Protective cap length: 55 ± 2 mm

Color of the handle: Blue

Handle diameter (at the thickest point): 9.1 ± 1 mm

Weight of balls on the tip: $0.30 \text{ g} \pm 0.1$; 3 balls (which help straightening brush filaments);

Mass of the brush without the protective cap: 8.3 ± 1 g

Mass of the brush with the protective cap: 8.9 ± 1 g

Test pill former Base for Shade Samples:

Dimensions: $75 \times 20 \times 3.9 \text{ mm} \pm 10\%$;

Mass: 88 ± 2 g

Shade indicators: Artist Gum , Artist Add-on, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva, Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside.

Mass: $18.5 \text{ g} \pm 2\text{g}$;

Overall dimensions: $8 \pm 0,2 \times 10.7 \pm 0.2 \text{ cm}$

2.4. Duceram® Kiss Basic Set

- Opaque thermal expansion coefficient: $13,7 \mu\text{m/m} \cdot \text{K}$ (25- 500°C)
- Dentine thermal expansion coefficient: $13,0 \mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Ceramic material thermal expansion coefficient: $12,8 \mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Dentine density $2,42\text{g/cm}^3$
- Volume shrinkage in the process of dentine baking 24 %
- Linear shrinkage in the process of dentine baking 13,9 %

• Baking temperature

Baking opaque	Approx. 930°C	Instructions for baking are included to the instruction manuals
1 st baking of dentine	Approx. 910°C	Instructions for baking are included to the instruction manuals.
Baking cutting edge masses (schneide) and Transpa masses	Approx 910°C	Instructions for baking are included to the instruction manuals
Baking Korrekturmasse	Approx 880°C	Instructions for baking are included to the instruction manuals

• Vitrification temperature

Dentine	520°C
Ceramic material	520°C
cutting edge (schneide) masses and Transpa masses,	520°C

- ## Dentaply Universal NE-Bonder / NP-Bonder

- Ducera Liquid FORM in the bottle 50 ml, Ducera Liquid SD in the bottle 50 ml.**

Liquid volume 50 ml

Liquid volume 2 ml

Liquid volume 15 ml

ООО ЦИРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

handle (including the cap)	
Color	Rod, black matt anodized Cap, anodized metallic blue
Diameter (the place with the greatest thickness, the border of the handle recess)	9,0 ± 0,02 mm
Diameter (the place with the greatest thickness, the border of the handle recess)	5,0 ± 0,02 mm
Rod Material	Aluminum tube, AlMgSi05, brand EN-AW 6060, manufacturer Aluminco S.A.
Adaptations	Magnet for placing the tip of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Brush-tip: Pinselspitze "Pinelo" opaque/stain, Pinselspitze "Pinelo" No. 8

Brush-tip Brush-tip: Pinselspitze "Pinelo" opaque/stain	
Brush fiber length	7,0 ± 1 mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	4,80 x 5,60 ± 0,2 mm
Length of the protective cover	30,0 ± 2,0 mm
Clip length	25,00 ± 1 mm
Material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene terephthalate (PBT), grade LUPOX, Manufacturere LG Chem, Ltd.
The strength of the brush fiber	0,15 / 0,07 ± 0,01 mm (according to the sample)
Adaptations	Magnet for placing the handle of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Насадка-кисть Brush-tip: Pinselspitze "Pinelo" No. 8	
Brush fiber length	19,5 ± 1 mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	5,90 x 7,40 ± 0,2 mm
Length of the protective cover	35,0 ± 2,0 mm
Clip length	25,00 ± 1 mm
Material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene terephthalate (PBT), grade LUPOX,

	производитель LG Chem, Ltd.
The strength of the brush fiber	0,15 / 0,07 ± 0,01 mm (according sample)
Adaptations	Magnet for placing the handle of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Storage box:

Overall dimensions 33,5± 0,3 cm x 26,0 ± 0,3 cm x 20,5± 0,3 cm

Vials for ceramic powder and vials for modeling liquid Ducera Liquid SD in a bottle of 50 ml, Ducera Liquid FORM 50 ml, universal liquid for dyes and glazes Universal Malfarben u. Glazur Liquid 15 ml, liquid for modeling paste paste Pastenopaker Liquid 2 ml

Capacity / Volume	Description	Color	Materials	Overall dimensions	Weight
20 g / 35 ml	bottle with a wide neck	transparent	PAT cas# 24938-04-3, manufacturer PLASTIVERD, PET Reciclado, s.a.	Height: 50 mm ± 10% Diameter: 37 mm ± 10%	15,3 g ± 10%
	cap	red	carbamide resin cas# 9011-05-6, manufacturer C HEMIPLASTIC A S.P.A.	Height: 20 mm ± 10% Diameter: 37 mm ± 10%	9,3 g ± 10%
50 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 84 mm ± 10% Diameter: 32,8 mm ± 10%	6,6 g ± 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTIC A S.P.A.	Height: 20 mm ± 10% Diameter: 31,9 mm ± 10%	7,9 g ± 10%
15 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 66 mm ± 10% Diameter: 24 mm ± 10%	5,4 g ± 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTIC A S.P.A.	Height: 18 mm ± 10% Diameter of the lid base: 18 mm ± 10%	0,8 g ± 10%
2 ml	Plastic bottle	black	low density polypropylene polyethylene brand LD PE Black Purell 1840H, manufacturer	Height: 45mm ± 10% Diameter: 13 mm ± 10%	1,86 g ± 10%

			LyondellBasell Industries Holdings, B.V		
	cap	black	polypropylene brand PP black PPH 5060, manufacturer TOTALENERG IES ONE TECH	Height: 17 mm ± 10% Diameter of the lid base: 12 mm ± 10%	0,65 g ± 10%

2.5. Duceram® Kiss Aesthetic

- Opaque thermal expansion coefficient: 13,7 $\mu\text{m/m} \cdot \text{K}$ (25- 500°C)
- Dentine thermal expansion coefficient:: 13,0 $\mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Ceramic material thermal expansion coefficient: 12,8 $\mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Dentine density 2,42g/cm³
- Volume shrinkage in the process of dentine baking 24 %
- Linear shrinkage in the process of dentine baking 13,9 %

• Baking temperature

Baking opaque	Approx. 9310°C	Instructions for baking are included to the instruction manuals
1 st baking of dentine	Approx. 910°C	Instructions for baking are included to the instruction manuals.
Baking cutting edge masses(schneide), including opal schneide; masses transparent, masses Opal Effekt; Power Chroma; masses Transpa Red	Approx 910°C	Instructions for baking are included to the instruction manuals.

• Vitrification temperature

Dentine	550°C
Ceramic material	520°C
Cutting edge (schneide) masses including opal schneide; masses transparent, masses Opal Effekt; Power Chroma; masses Transpa Red	520°C

- Dental ceramic material, type 1, class 1 pursuant to DIN EN ISO 6872;
- Hardness: 490 – 550 HV 02;
- Flexural strength: 60 – 70 MPa pursuant to DIN EN ISO 9693/6872 (≥ 50 MPa);
- Chemical solubility: < 100 $\mu\text{g/cm}^2$ pursuant to DIN EN ISO 6872 and DIN EN ISO 9693;
- Use only with alloys with a solidification temperature of at least 1 030 °C

Ducera Liquid FORM in the bottle 15 ml

Weight of the bottle with liquid 19,5 g±1,0 g.

Liquid volume 15 ml

Ducera Liquid SD in the bottle 50 ml.
Weight of the bottle with liquid 65g ±2 g
Liquid volume 50 ml.

Shade Indicators: Indikator Duceram® Kiss Power, Indikator Duceram® Kiss Opal, Indikator Duceram® Kiss Gingiva
Mass 18,5 g ± 2 g
Overall dimensions 8± 0,2 cm x 10,7 ± 0,2 cm

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Brush handle Pinselstiel "Pinelo"

Brush handle "Pinelo"	
The total length of the brush handle (including the cap)	126,6 ± 0,15 mm
Color	Rod, black matt anodized Cap, anodized metallic blue
Diameter (the place with the greatest thickness, the border of the handle recess)	9,0 ± 0,02 mm
Diameter (the place with the smallest thickness, the end of the handle)	5,0 ± 0,02 mm
Rod Material	Aluminum tube, AlMgSi05, brand EN-AW 6060, manufacturer Aluminco S.A.
Adaptations	Magnet for placing the handle of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Brush-tip: Pinselspitze "Pinelo" No. 8	
Brush fiber length	19,5 ± 1 mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	5,90 x 7,40 ± 0,2 mm
Length of the protective cover	35,0 ± 2,0 mm
Clip length	25,00 ± 1 mm
Material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-

	08-34, manufacturer DOW USA
Brush fiber material	85% нейлон прозрачный марка Supplex Nylon 15% Полибутилентерефталат (ПБТ), марка LUPOX, производитель LG Chem, Ltd.
Strength of the brush fiber	0,15 / 0,07 ± 0,01 мм (according sample)
Adaptations	Magnet for placing the handle of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Test pill former.

Dimensions 75 x 20 x 3,9 mm ± 10%

Mass 88g ± 2 g

Storage box:

Overall dimensions 33,5 ± 0,3 cm x 26,0 ± 0,3 cm x 20,5 ± 0,3 cm

Vials for ceramic powder and vials for modeling liquid Ducera Liquid SD in a bottle of 15 ml, Ducera Liquid FORM 50 ml

Capacity / Volume	Description	Color	Materials	Overall dimensions	Weight
20 g / 35 ml	bottle with a wide neck	transparent	PAT cas# 24938-04-3, manufacturer PLASTIVERD, PET Reciclado, s.a.	Height: 50 mm ± 10% Diameter: 37 mm ± 10%	15,3 g ± 10%
	cap	red	carbamide resin cas# 9011-05-6, manufacturer C HEMIPLASTIC A S.P.A.	Height: 20 mm ± 10% Diameter: 37 mm ± 10%	9,3 g ± 10%
15 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 66 mm ± 10% Diameter: 24 mm ± 10%	5,4 g ± 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTI CA S.P.A.	Height: 18 mm ± 10% Diameter of the lid base: 18 mm ± 10%	0,8 g ± 10%
50 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 84 mm ± 10% Diameter: 32,8 mm ± 10%	6,6 g ± 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTI CA S.P.A.	Height: 20 mm ± 10% Diameter: 31,9 mm ± 10%	7,9 g ± 10%

Bottle (jar) for pastes, pasten opaques 3ml

Capacity / Volume	Description	Color	Materials	Overall dimensions
5 g	Vial	transparent	hydrolytic glass of the 3rd class, brand Soda Lime Glass, manufacturer BRUNI GLASS S.p.A.	Height: 22 mm $\pm$ 10% Diameter: 30 mm $\pm$ 10%
-	Cap	white	carbamide resin case# 9011-05-6, manufacturer CHEM IPLASTICA S.P.A.	Height: 9 mm $\pm$ 10% Diameter: 29 mm $\pm$ 10%

2.6. Duceram® Kiss Starter Set

- Opaque thermal expansion coefficient: 13,7 $\mu\text{m/m} \cdot \text{K}$ (25- 500°C)
- Dentine thermal expansion coefficient:: 13,0 $\mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Ceramic material thermal expansion coefficient: 12,8 $\mu\text{m/m} \cdot \text{K}$ (25-500 °C)
- Dentine density 2,42g/cm³
- Volume shrinkage in the process of dentine baking 24 %
- Linear shrinkage in the process of dentine baking 13,9 %

• Baking temperature

Baking opaque, плечевой массы	Approx. 930°C	Instructions for baking are included to the instruction manuals
1 st baking of dentine	Approx. 910°C	Instructions for baking are included to the instruction manuals.
Baking cutting edge masses (schneide) and, including opal schneide; masses transparent, masses Opal Effekt; Power Chroma; masses Transpa Red	Approx. 910°C	Instructions for baking are included to the instruction manuals.
Baking final shoulder mass	Approx. 660°C	Instructions for baking are included to the instruction manuals.

• Vitrification temperature

Opaque	550°C
Dentine	520°C
Cutting edge masses (schneide) and, including opal schneide; masses transparent, masses Opal Effekt; Power Chroma; masses Transpa Red	520°C

- Dental ceramic material, type 1, class 1 pursuant to DIN EN ISO 6872;
- Hardness: 490 – 550 HV 02;
- Flexural strength: 60 – 70 MPa pursuant to DIN EN ISO 9693/6872 (≥ 50 MPa);
- Chemical solubility: $< 100 \mu\text{g}/\text{cm}^2$ pursuant to DIN EN ISO 6872 and DIN EN ISO 9693;
- Use only with alloys with a solidification temperature of at least 1 030 °C

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Pastenopaker Liquid in the bottle 2 ml

Weight of the bottle with liquid 3,6 g $\pm 0,5$ g.

Liquid volume 2 ml

Ducera Liquid FORM in the bottle 15 ml

Weight of the bottle with liquid 24 g $\pm 1,0$ g.

Liquid volume 15 ml

Ducera Liquid SD in the bottle 50 ml.

Weight of the bottle with liquid 65g ± 2 g.

Liquid volume 50 ml.

Shade Indicators: Indikator Duceram® Kiss Starter Power, Indikator Duceram Kiss Starter Base+Opal;

Mass 18,5 g ± 2 g

Overall dimensions 8 $\pm 0,2$ cm x 10,7 $\pm 0,2$ cm

Towel Dentsply Sirona HandtechWeight of the towel 63 ± 2 gWeight of the towel in the package 90 ± 2 gOverall dimensions of the towel $50 \pm 0,2$ cm x $30,7 \pm 0,2$ cm**Sponge Dentsply Sirona Schwamm**Mass $1 \pm 0,3$ gOverall dimensions $13 \pm 0,2$ cm x $8,5 \pm 0,2$ cm**Probenformer ohne Verpackung**Mass: $44,63 \pm 2$ gOverall dimensions : device length 16 cm $\pm 10\%$ Diameter of the 2 ends of the product 7 mm and 10mm $\pm 10\%$ **Brush handle Pinselstiel "Pinelo"**

Brush handle "Pinelo"	
The total length of the brush handle (including the blue cap)	$126,6 \pm 0,15$ mm
Color	Rod, black matt anodized Cap, anodized metallic blue
Diameter (the place with the greatest thickness, the border of the handle recess)	$9,0 \pm 0,02$ mm
Diameter (the place with the smallest thickness, the end of the handle)	$5,0 \pm 0,02$ mm
Rod Material	Aluminum tube, AlMgSi05, brand EN-AW 6060, manufacturer Aluminco S.A.
Adaptations	Magnet for placing the tip of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Brush-tip: Pinselspitze "Pinelo" № 0

Length of the brush fiber (visible at the output force)	$7,0 \pm 1$ mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	$1,50 \times 3,40 \pm 0,2$ mm
Length of the protective cover	$35,0 \pm 2,0$ mm
Clip length	$25,00 \pm 1$ mm
The material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene Terephthalate (PBT), brand LUPOX, manufacturer LG Chem, Ltd.
Strength of the brush fiber	$0,15 / 0,07 \pm 0,01$ mm (according sample)
Adaptations	Magnet for placing the tip of the brush, brand NS6130-07-

	704, manufacturer Intelligent Materials Pvt Ltd
--	---

Brush-tip: Pinselspitze "Pinelo" opaque/stain	
Length of the brush fiber	7,0 ± 1 mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	4,80 x 5,60 ± 0,2 mm
Length of the protective cover	30,0 ± 2,0 mm
Clip length	25,00 ± 1 mm
The material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene Terephthalate (PBT), brand LUPOX, manufacturer LG Chem, Ltd.
Strength of the brush fiber	0,15 / 0,07 ± 0,01 mm (according sample)
Adaptations	Magnet for placing the tip of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Brush-tip: Pinselspitze "Pinelo" No. 8	
Length of the brush fiber	19,5 ± 1 mm (slightly pointed in front)
Diameter of the part to which the force is applied (top/bottom)	5,90 x 7,40 ± 0,2 mm
Length of the protective cover	35,0 ± 2,0 mm
Clip length	25,00 ± 1 mm
The material of the part to which the force is applied	Nickel-plated brass brand CuNiZn (Cu 64—67% , Ni 5—6,5% , Zn 26,5-31%) , black matte varnish CAS #24956-08-34, manufacturer DOW USA
Brush fiber material	85% nylon transparent brand Supplex Nylon 15% Polybutylene Terephthalate (PBT), brand LUPOX, manufacturer LG Chem, Ltd.
Strength of the brush fiber	0,15 / 0,07 ± 0,01 mm (according sample)
Adaptations	Magnet for placing the tip of the brush, brand NS6130-07-704, manufacturer Intelligent Materials Pvt Ltd

Storage box:

Dimensions 33.5± 0.3cm x 26.0 ± 0.3 cm x 20.5± 0.3 cm

The ceramic powder vials match the vials from the Duceram® Kiss Basic Set and Duceram® Kiss Aesthetic Set, but are half filled with powder. Vials for modeling liquid Ducera Liquid SD in a 50ml bottle, Ducera Liquid FORM 15ml, universal liquid for dyes and glazes Universal Malfarben U.Glazar Liquid 15ml, liquid for modeling paste paste Pastenopaker Liquid 2 ml

Capacity / Volume	Description	Color	Materials	Overall dimensions	Weight
20 g / 35 ml	bottle with a wide neck	transparent	PAT cas# 24938-04-3,	Height: 50 mm ± 10% Diameter: 37 mm ± 10%	15,3 g ± 10%

			manufacturer PLASTIVERD, PET Reciclado, s.a.		
	cap	red	carbamide resin cas# 9011-05-6, manufacturer C HEMIPLASTIC A S.P.A.	Height: 20 mm $\pm$ 10% Diameter: 37 mm $\pm$ 10%	9,3 g $\pm$ 10%
50 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 84 mm $\pm$ 10% Diameter: 32,8 mm $\pm$ 10%	6,6 g $\pm$ 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTI CA S.P.A.	Height: 20 mm $\pm$ 10% Diameter: 31,9 mm $\pm$ 10%	7,9 g $\pm$ 10%
15 ml	Plastic bottle	transparent	high density polyethylene cas# 74-85-1 / 106-98-9, manufacturer MOL PETROLKÉMI A ZRT.	Height: 66 mm $\pm$ 10% Diameter: 24 mm $\pm$ 10%	5,4 g $\pm$ 10%
	cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTI CA S.P.A.	Height: 18 mm $\pm$ 10% Diameter of the lid base: 18 mm $\pm$ 10%	0,8 g $\pm$ 10%
2 ml	Plastic bottle	black	low density polypropylene polyethylene brand LD PE Black Purell 1840H, manufacturer LyondellBasell Industries Holdings, B.V	Height: 45mm $\pm$ 10% Diameter: 13 mm $\pm$ 10%	1,86 g $\pm$ 10%
	cap	black	polypropylene brand PP black PPH 5060, manufacturer TOTALENERG IES ONE TECH	Height: 17 mm $\pm$ 10% Diameter of the lid base: 12 mm $\pm$ 10%	0,65 g $\pm$ 10%

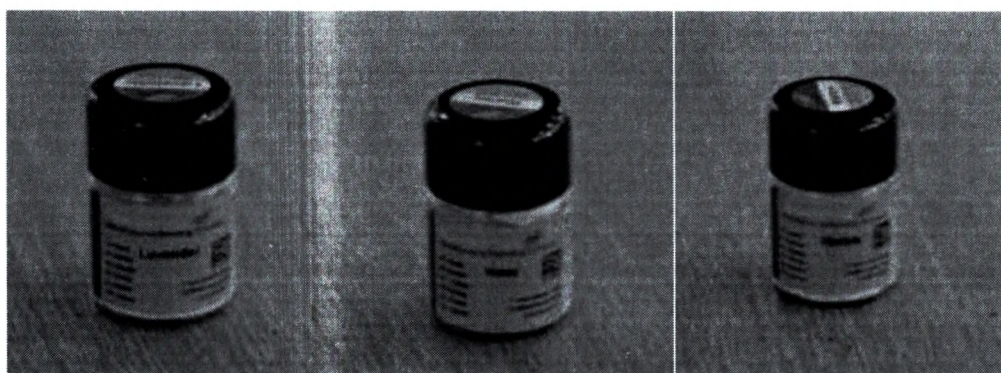
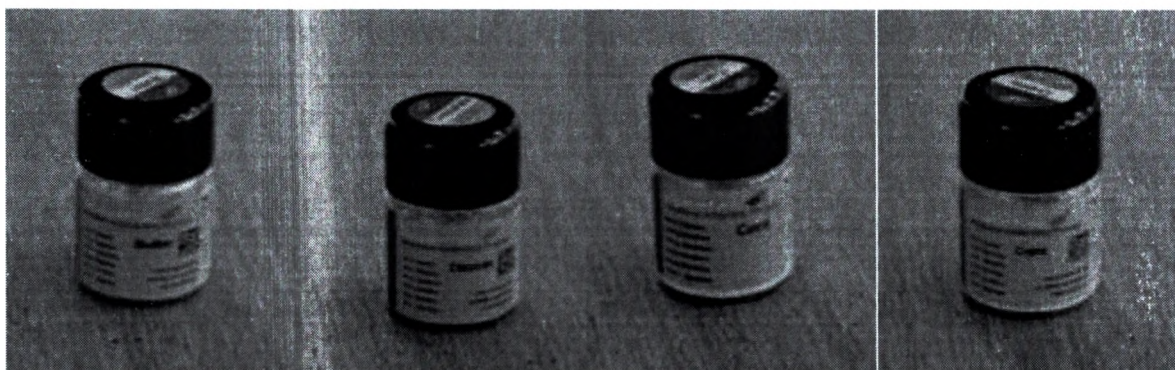
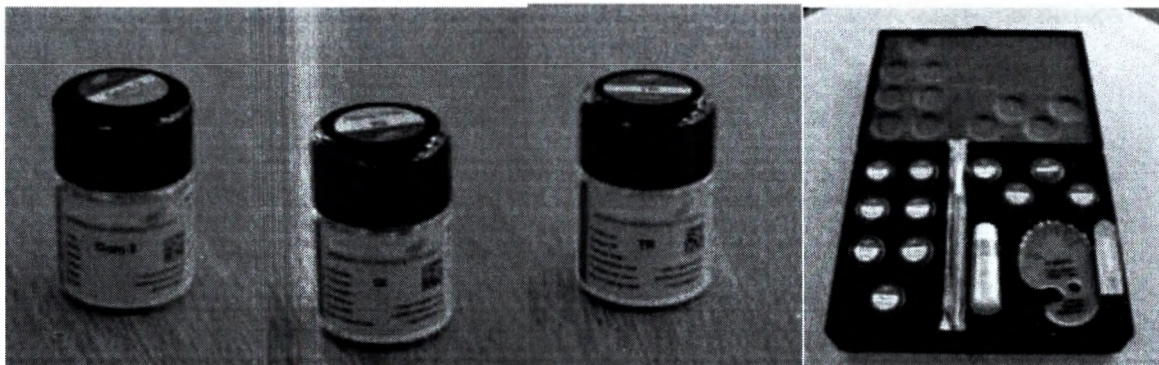
Bottle (jar) for pasten opaques 3ml, universal Dentsply Universal NE-Bonder / NP-Bonder bond 4ml, glaze Universal Overglaze 5g, dye UNIV INCISAL STAIN i1 5g, dye UNIVERSAL BODY STAIN S2 5g;

Capacity / Volume	Description	Color	Materials	Overall dimensions
5 g	Vial	transparent	hydrolytic glass of the 3rd class,	Height: 22 mm $\pm$ 10% Diameter: 30 mm $\pm$ 10%




			brand Soda Lime Glass, manufacturer BRUNI GLASS S.p.A.	
-	Cap	white	carbamide resin cas# 9011-05-6, manufacturer CHEMIPLASTI CA S.P.A.	Height: 9 mm $\pm$ 10% Diameter: 29 mm $\pm$ 10%

3. Images

3.1. Duceram® Kiss Artist Add-on Kit

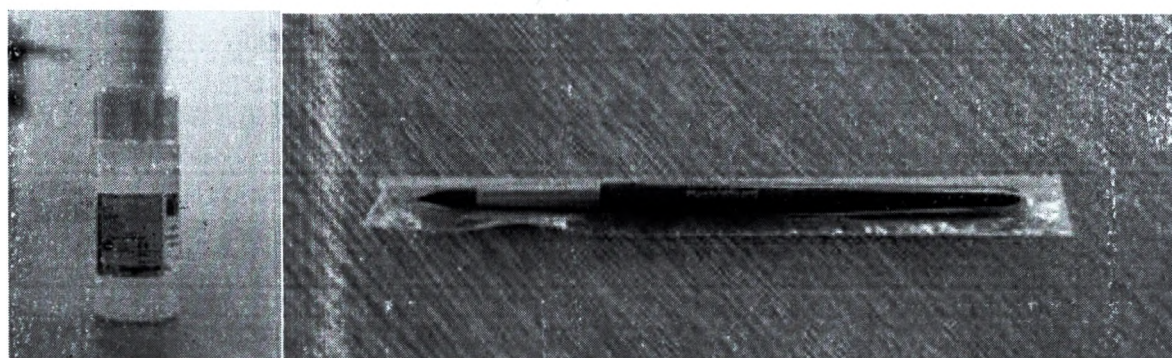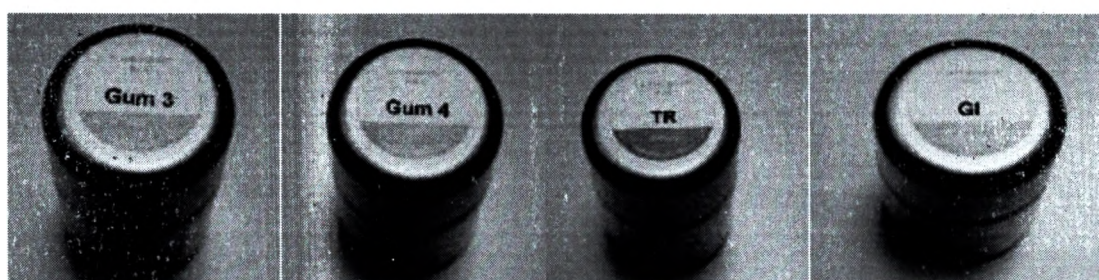
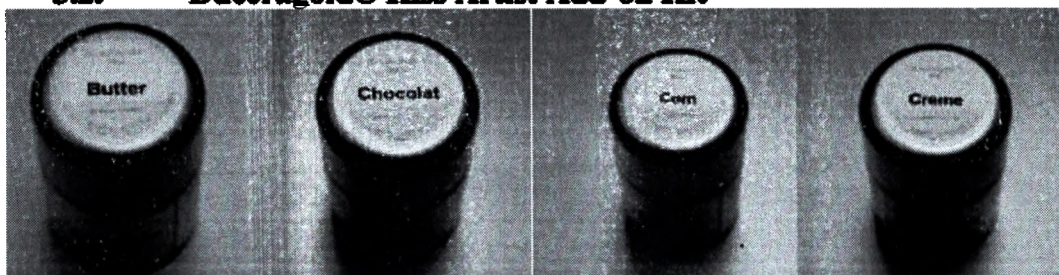
3.2. Duceragold® Kiss Artist Add-on Kit

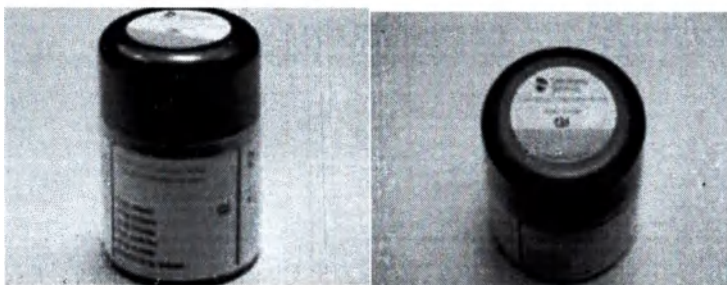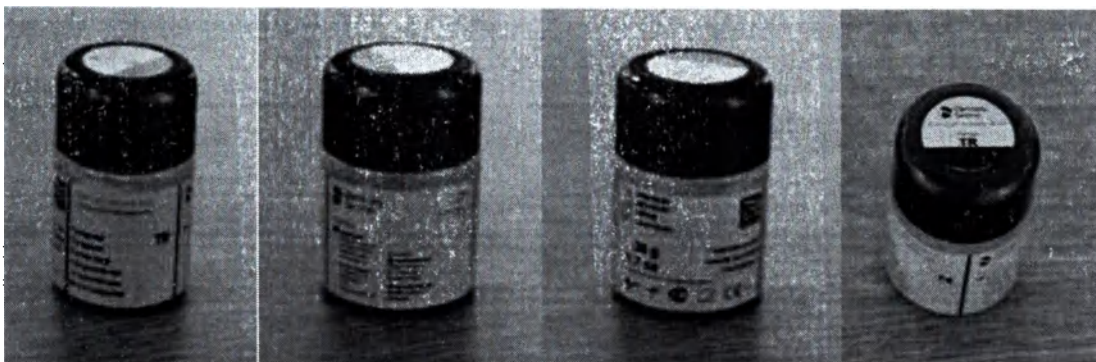
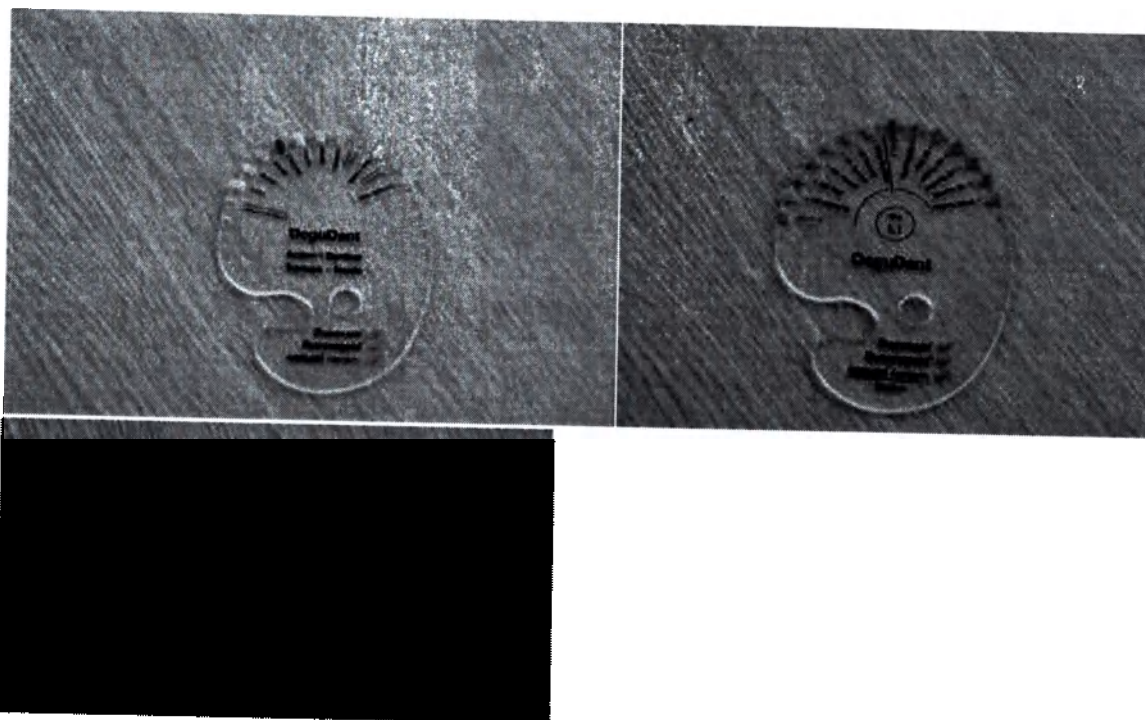
3.3. Cercon® Ceram Kiss Artist Add-on Kit

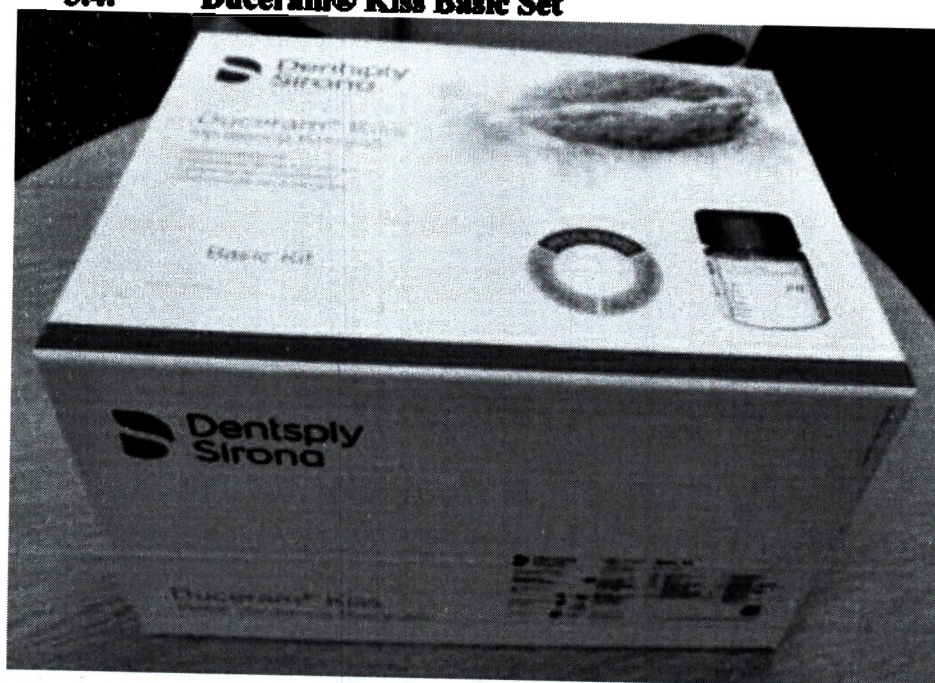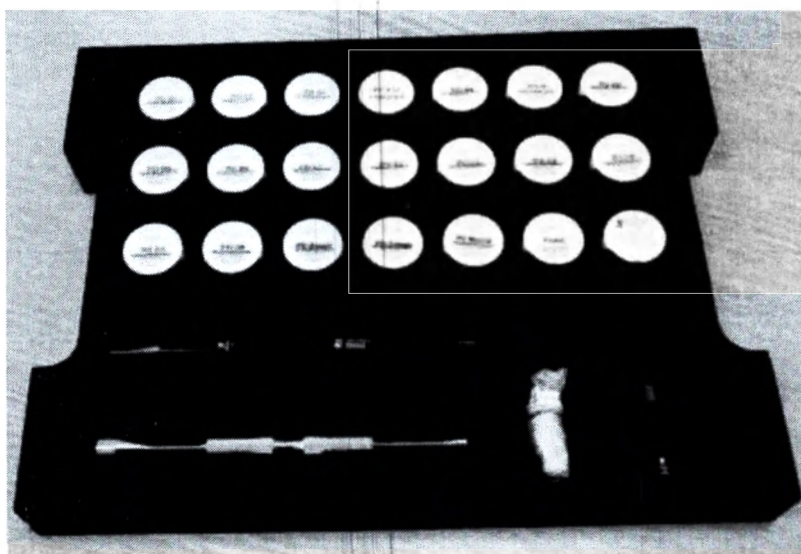
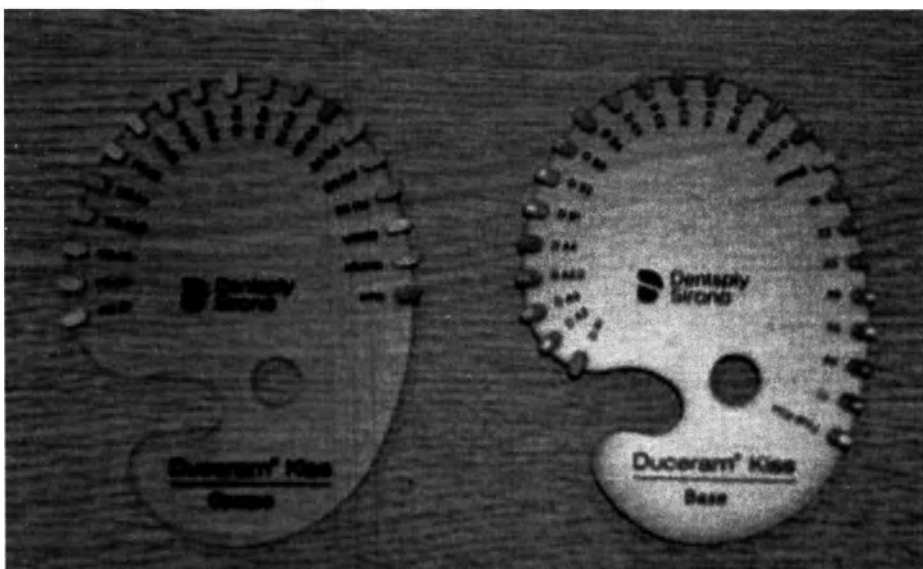
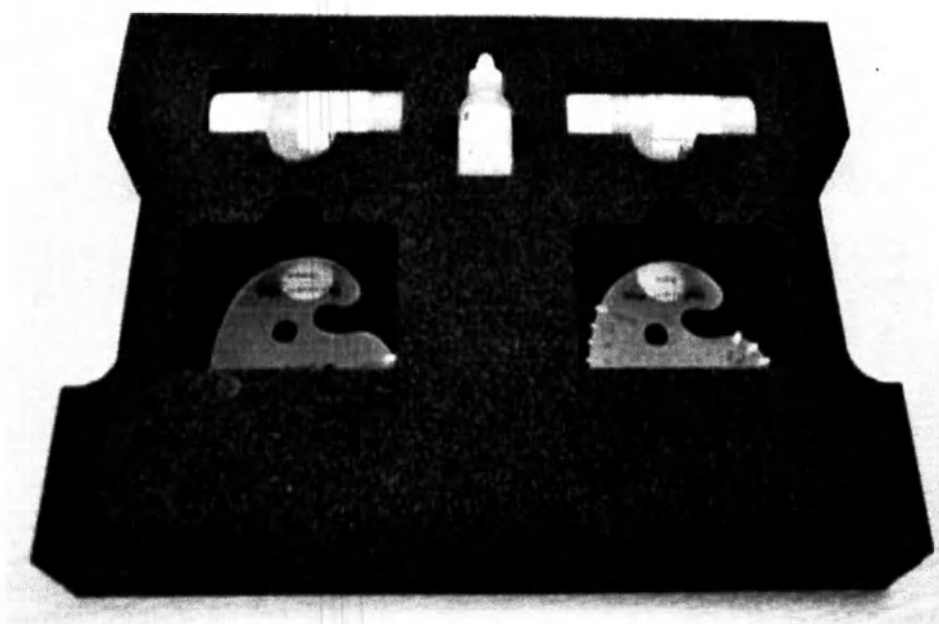
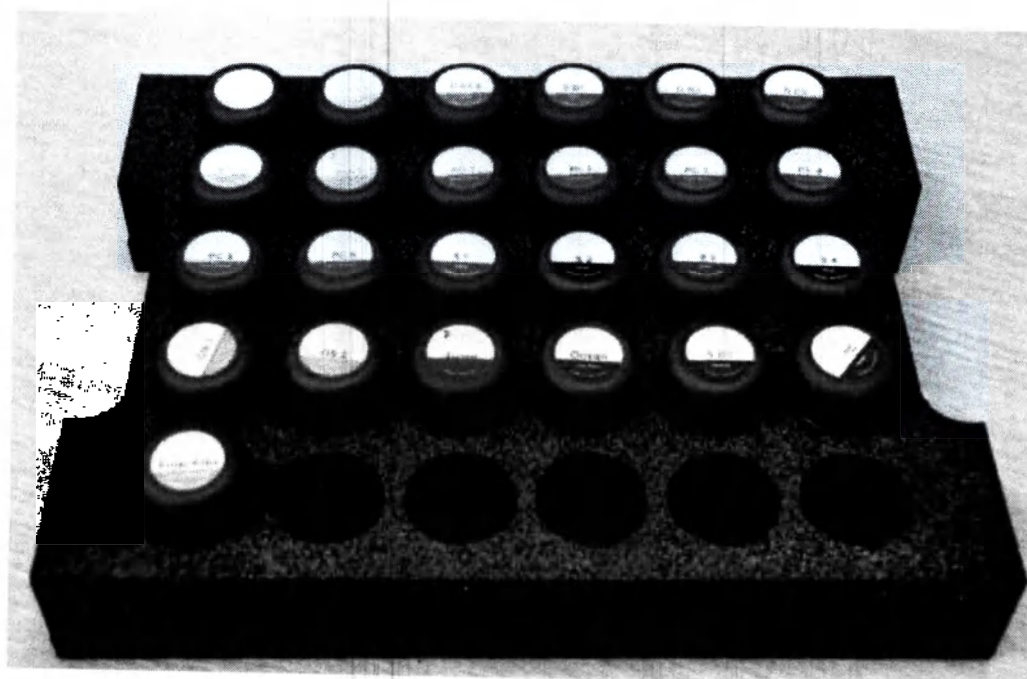
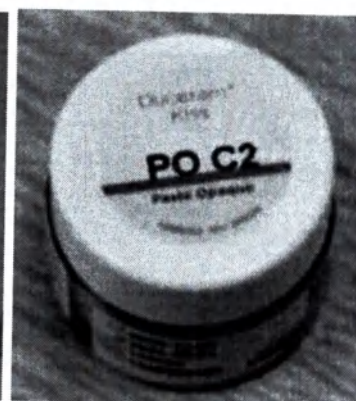
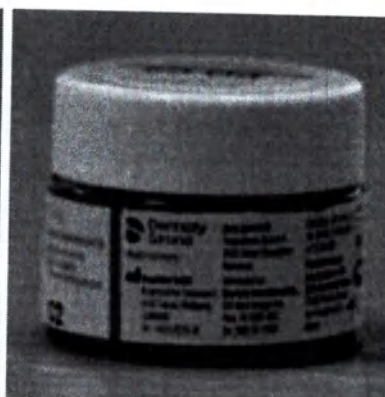
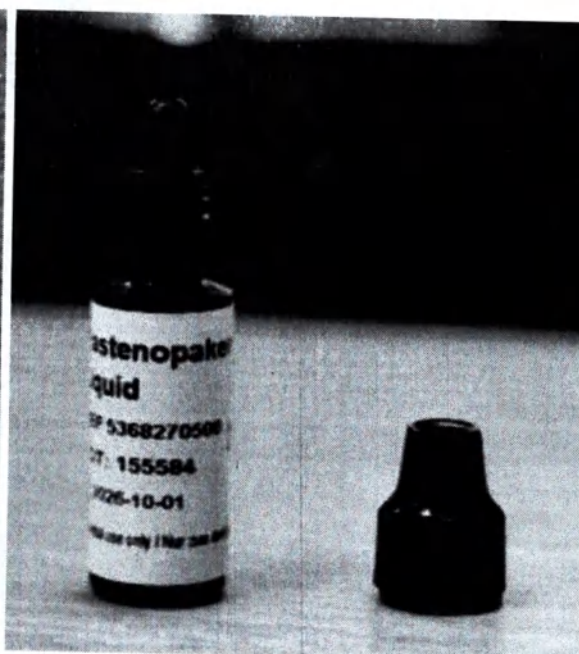
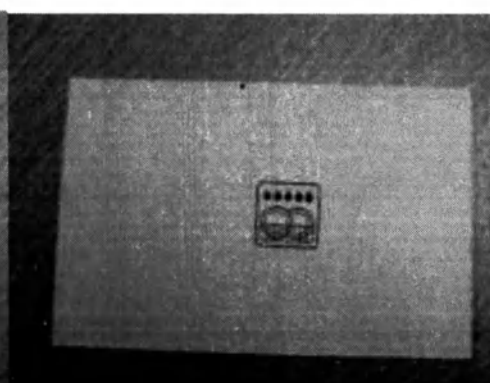
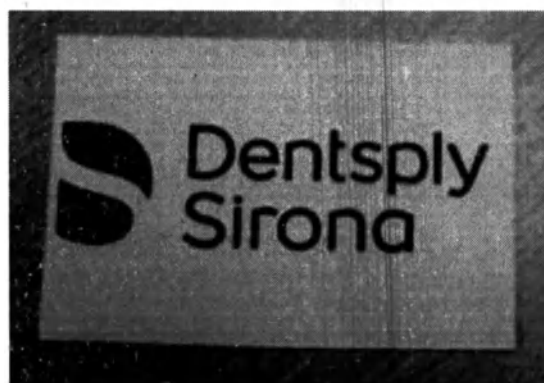
3.4. Duceram® Kiss Basic Set


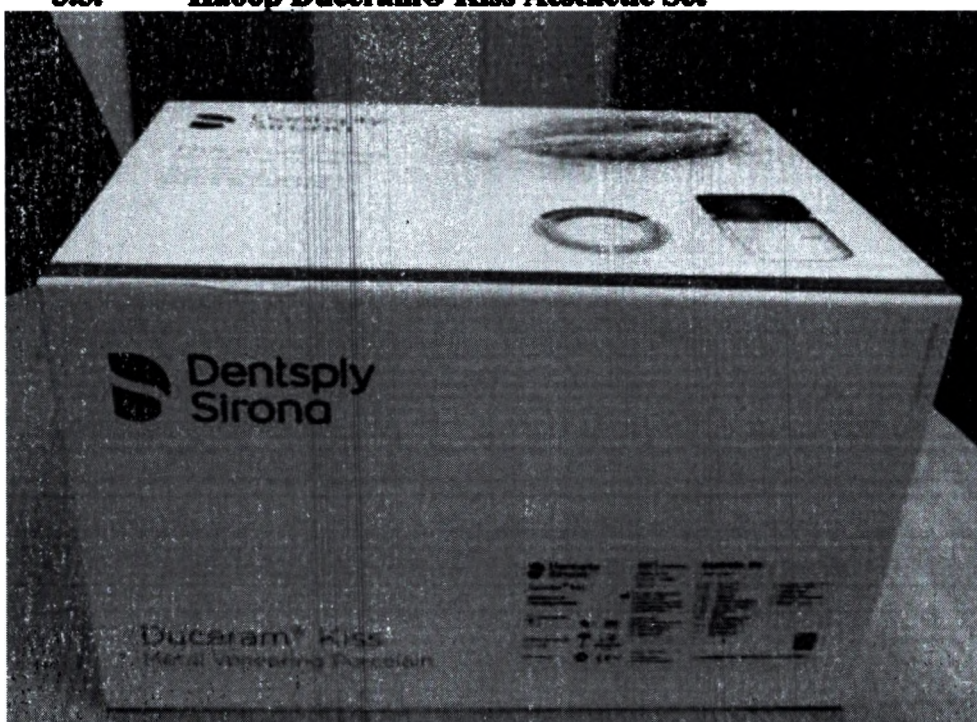
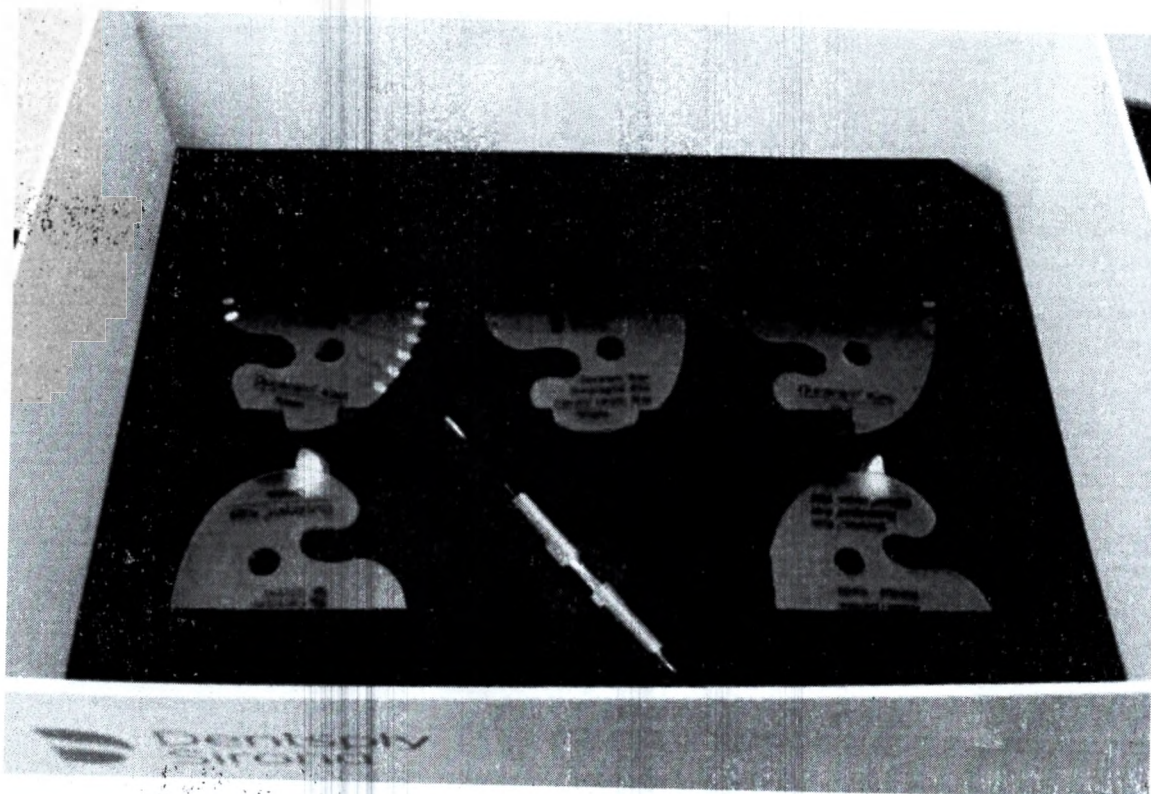
3.5. Набор Duceram® Kiss Aesthetic Set

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




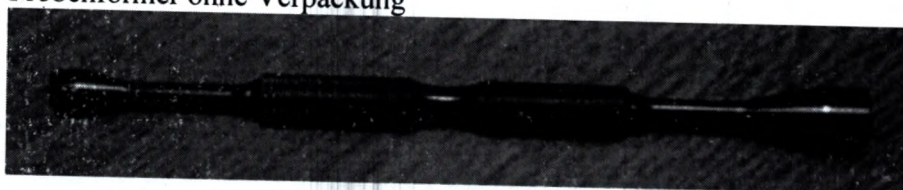
Test pill former

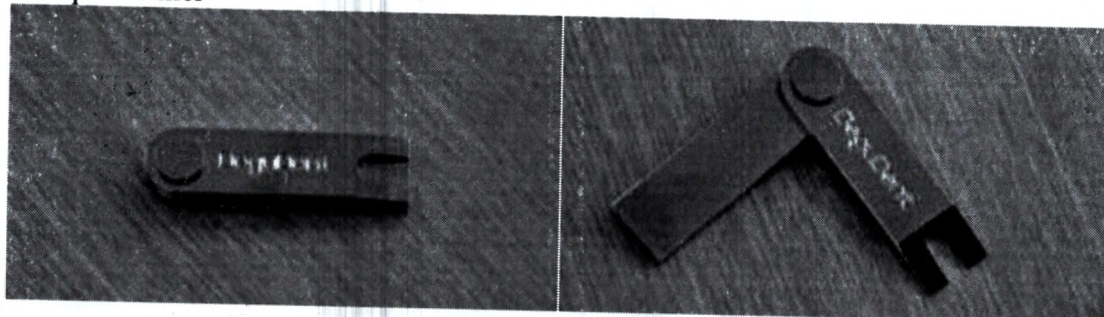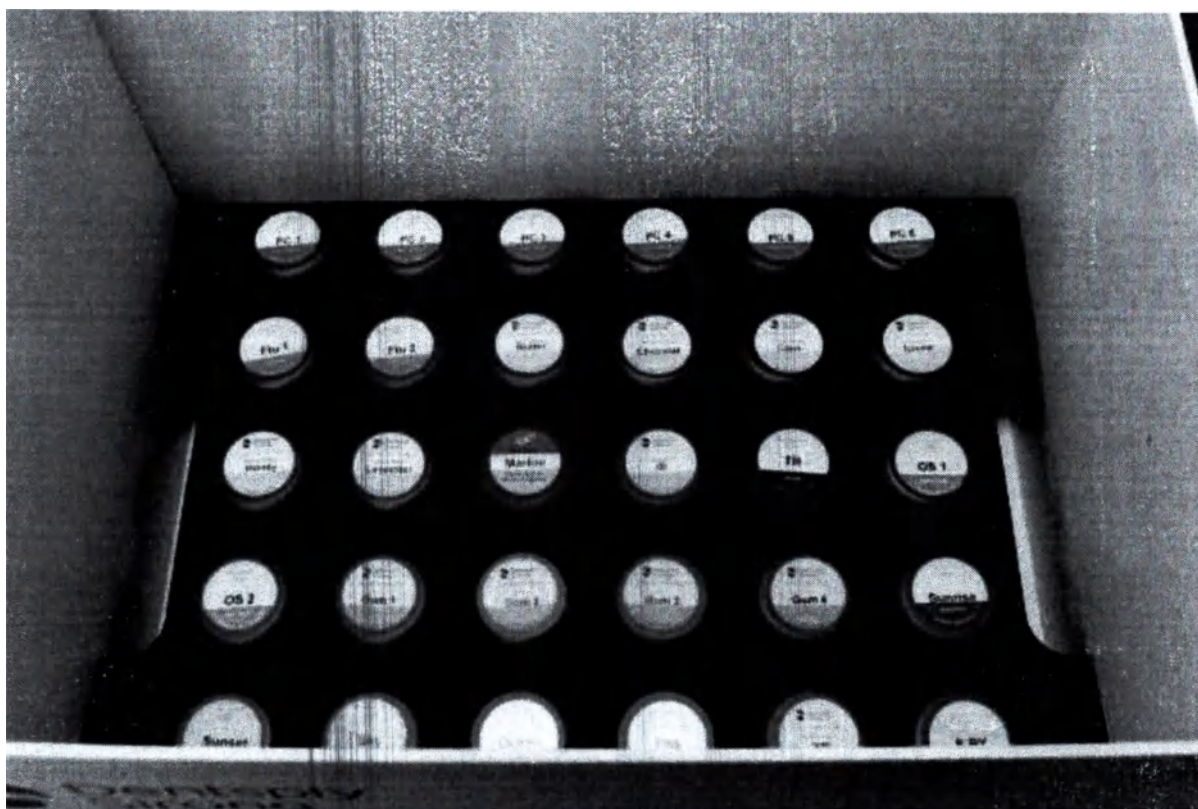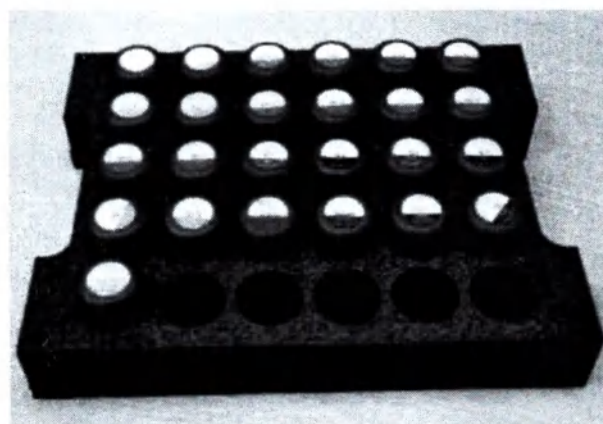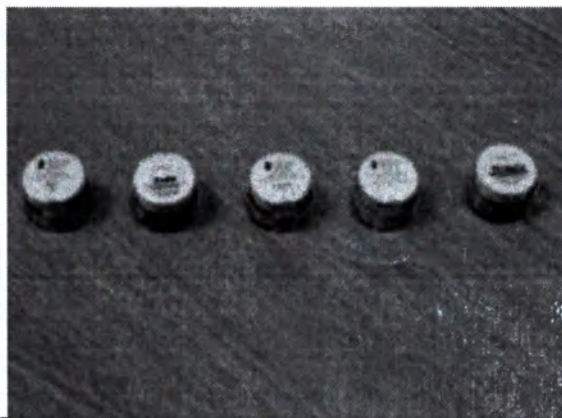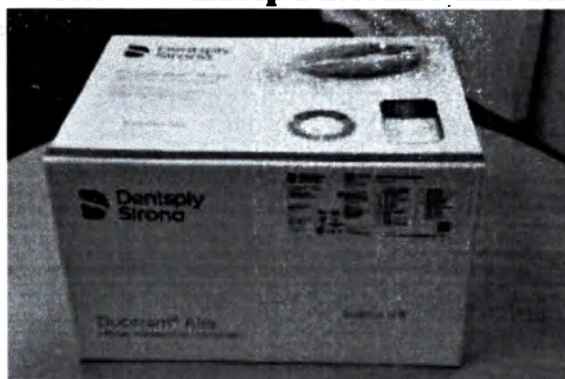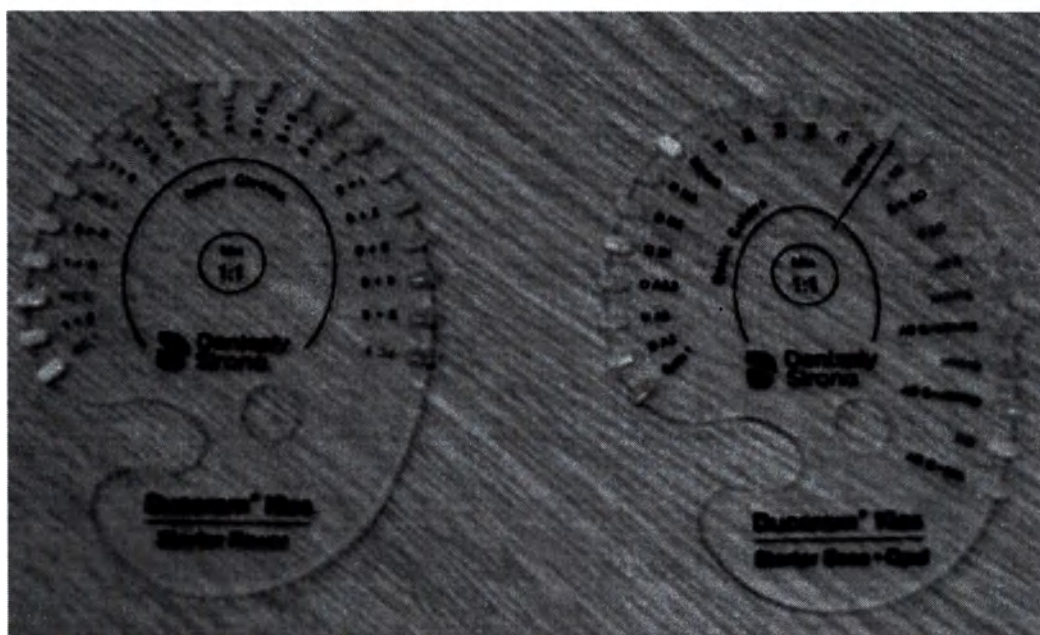3.6. Набор Duceram® Kiss Starter Set

4. Indications for Use

For dental use only.

- **Duceram® Kiss Artist:** Veneering of metal frameworks of single crowns and bridges made of dental alloys with a thermal expansion coefficient within the range 13.8 to 15.4 $\mu\text{m/m} \cdot \text{K}$ (25 - 600°C). Duceram® Kiss Artist is compatible with high- and reduced-gold alloys, non-noble alloys, and alloys containing silver.
- **DuceraGold® Kiss Artist:**
 - Veneering of metal frameworks of single crowns and bridges with a thermal expansion coefficient (CTE) within the range 16.2 to 17.6 $\mu\text{m/m} \cdot \text{K}$ (25 - 600°C). DuceraGold® Kiss Artist is compatible with high-gold alloys;

- Veneering of Cergo pressable glass ceramics for fabrication of frameworks.
- **Cercon® Ceram Kiss Artist:** Veneering of zirconium dioxide (Y-TZP) crowns and bridges, which should preferably be made of Cercon base or Cercon base ht.
- **Duceram ® Kiss Basic Set, Duceram ® Kiss Aesthetic Set, Duceram ® Kiss Starter :** Veneering of crowns and bridges with metal frames made of dental alloys with a range of thermal expansion coefficient (CTE) 13.8 - 15.4 microns/m · K (25 – 600 °C). Duceram ® Kiss is compatible with alloys with high and reduced-gold alloys, as well as with base metals
- **Action-i Dentine:** Modeling of dentine base; parameters of the shades and transparency are analogous to those of natural tooth enamel.
- **Duceram ® Kiss Dentine dentines:** modeling a dentine base that matches the natural shades of teeth in color to perform ceramic restoration modeled after a natural tooth.
- **Duceram ® Kiss Power Chroma mass:** color enhancement in the cervical, palatine and occlusal areas.
- **Transpa Red Ceramic mass:** Enhancement of an individual approach to reddish hue of teeth.
- **Duceram ® Kiss Schneide cutting edge mass:** forming the cutting edge area
- **Duceram ® Kiss Opalschneide opal mass of the cutting edge:** formation of the cutting edge area with the effect of opalescence, reproduction of opalescence of natural enamel.
- **UNIV INCISAL STAIN I1 dye:** reproduces the complex and intricate volumetric effects in the cutting edge area observed in natural teeth.
- **Duceram ® Kiss Final Kiss Korrekturmasse correction mass :** correction after glossy firing
- **Duceram ® Kiss Opal Effect mass opal effect :** individualization of the cutting third of the tooth
- **Duceram ® Kiss Flu Inside mass, Transparent Transpa TC mass, Duceram ® Kiss White Surface (WS) mass, Duceram ® Kiss Stand By (S By) mass:** providing additional transparency of tooth surfaces
- **UNIVERSAL BODY STAIN S2 dye** reproduces surface color effects.
- **Duceram ® Kiss Paste Opaque (PO) pastes:** coating of the material/ alloy of the prosthesis frame (crown, bridge), low light transmission, excellent hiding ability
- **Duceram ® Kiss Paste Opaque (PO) Orange:** creating aesthetic natural effects of a prosthesis (crown, bridge)
- **Duceram ® Kiss Neutral paste opaque :** coating of the material/ alloy of the prosthesis frame (crown, bridge), low light transmission, excellent coverage
- **Duceram ® Kiss Paste Opaque (PO) Gum opaque paste gingival:** formation of the gingival part of the crown, frame
- **Dentsply Universal NE-Bonder / NP-Bonder bond:** applied to a surface made of non-ferrous metal alloys as the first layer, improves adhesion between the alloy and facing ceramics, which is provided for processing metal frames with a coefficient of thermal expansion (CTE) of 3.8 – 15.4 microns/m · K (25-500 °C)
- **Gray Inside Fluorescent Dentines:** Compensation for highly precise light contours of the framework within the biting region; the dentine helps to reduce brightness of framework hue intensity and prevents showing-through of the framework core.
- **Duceram ® Kiss GrayInside (GI) mass -** compensation of high-precision light contours of the frame in the cutting area, a decrease in the brightness of the color tone of the frame is achieved, the transmission of the core of the frame is prevented
- **Gum Dentines:** Shaping of the gingival part of ceramic restorations.

- **Ducera Liquid SD modeling liquid:** Mixing up of modifying dentines, ceramic materials for fabrication of the biting edge, and transparent ceramic materials.
- **Ducera Liquid FORM modeling liquid:** mixing of opacs, dentins, modifiers, cutting edge masses and transparent masses
- **Pastenopaker Liquid liquid** for modeling paste opaques: mixing of paste opaques
- **Universal Malfarben u. Glazur Liquid** universal liquid for dyes and glazes: mixing dyes and glazes
- **Universal Overglaze :** the final glaze of ceramic restoration.
- **Magic Wizard brush** is designed for applying layers of facing ceramics or modeling during dental work.
- **Handle for the Pinselstiel "Pinelo" brush** is designed for planting nozzles of various "Pinelo" brushes
- **Brush tips - brush-tip Pinselspitze "Pinelo" opaque/stain and "Pinelo" No. 0, "Pinelo" No. 8 :** to complete the handle of the Pinselstiel "Pinelo" brush, then the brushes are designed for applying layers of facing ceramics
- **Probenformer ohne Verpackung** dispenser: mixing two different masses in a 1:1 ratio with a guarantee of accurate reproducibility.
- **Test pill former color sample mold** is used to create firing samples from the mass of Transpa, in order to further evaluate the results and select the optimal firing parameters for ceramics.
- **Shade Indicators:**
 - **Artist Gum:** Selection of shading of gingival dentines to match patient's individual features
 - **Artist Add-on:** Selection of shading of Action-i Dentine, Gray Inside Fluorescent Dentine, and Transpa Red Ceramic Material to match patient's individual features
 - **Indicator Duceram ® Kiss Base** - for individual determination of the color nuances of dentins and cutting edge masses on the V-Classic scale.
 - **Indicator Duceram ® Kiss Opaque:** determination of the color nuances of opaque according to the V-Classic scale for individual characteristics of the patient
 - **Indicator Duceram ® Kiss Power :** determination of color nuances of Power Chroma masses on the V-Classic scale for individual characteristics of the patient.
 - **Indicator Duceram ® Kiss Opal :** determination of color nuances of opal masses on the V-Classic scale for individual characteristics of the patient.
 - **Indicator Duceram ® Kiss Gingiva –** determination of color nuances of gingival masses, dentins on the V-Classic scale for individual characteristics of the patient
 - **Indicator Duceram Kiss Starter Base+Opal** - for individual determination of the color nuances of dentins, opal masses of the cutting edge and effect masses on the V-Classic scale and their mixing options.
- **Dentsply Sirona Handtuch** towel: equipment for the workplace of a ceramic technician.
- **Dentsply Sirona Schwamm** sponge :removing excess liquid from brushes
- **Test pill former Base for Shade Samples** is intended for shaping of samples of Transpa

5. Contraindications

- These materials shall be used as per the abovementioned indications only.

- Use of Kiss Artist materials is contraindicated if a patient suffers from bruxism or any other parafunction.
- Furthermore, use of Kiss Artist and Duceram Kiss materials is contraindicated if the interocclusal space is too small.
- Cercon ceram Kiss should not be used for surface finishing of titan and other metal frameworks.
- Cercon ceram Kiss is not compatible with any other veneering ceramics.

6. Warnings

In the process of handling of the materials, the following recommendations on their processing and use should be observed:

- Protect your airways from access of dust escaping in the process of polishing;
- Avoid ingestion of pastes and paste-like liquids which may be hazardous;
- The products are intended for professional use only.

7. Adverse Events

With proper treatment and application, undesirable side effects of these medical devices are extremely unlikely.

To the best of our knowledge, there are no risks or adverse effects associated with use of Duceram® Kiss Artist, Duceragold® Kiss Artist, and Cercon® ceram Kiss Artist veneering ceramics.

Nevertheless, in principle, it is impossible to completely exclude immunological reactions (for example, allergies) and/or local manifestations (such as irritation of the taste buds or oral mucosa).

8. The procedure for working with the product: Step-by-step technique for applying layers of facing ceramics.

8.1. Layer Application Technique for Metal Frameworks

8.1.1. Preparation of the frame

1. Metal frames, unless the alloy manufacturer recommends anything else, after processing with carbide cutters, process in a sandblaster using Al₂O₃ sand (100-150 microns under pressure of 2 bars for precious metal alloys, and up to 4 bar (250 microns) for base metal alloys). To create a ceramic shoulder, the edge of the crown is shortened using a carbide cutter so that it ends in height approximately 0.5-0.8 mm from the deepest point of the rounded transition or ledge. In addition, it is necessary to ensure that the edge of the crown ends gently in order to avoid stresses in the ceramics
2. Oxidation: the oxidation of the treated metal frames is carried out according to the recommendations of the alloy manufacturer. The oxide is then removed according to the recommendations of the alloy manufacturer by etching or blasting.

8.1.2. Application and firing of the opaque

1. To give the frame the appropriate base color, the first layer of opaque is applied with an even, semi-covering layer and then fired.
2. For base metal alloys, it is recommended to apply a thin but covering bond layer for base alloys as the first layer, which improves the adhesion between the alloy and the facing ceramics.
3. For alloys with a low solidus temperature (for example, alloys with a high gold content and "bioflavons"), the use of a neutral paste is recommended during the first firing of the opaque to reduce the firing temperature.
4. Next, a second layer of opaque is applied, covering the frame completely and fired, according to the recommendations for firing.

8.1.3. Application and firing of ceramic layers

1. Creating a crown by applying layers of dentine along the full contour of the frame.
2. Removal of excess dentin, the crown is cut off.
3. Applying the masses of the cutting edge.
4. The first firing of dentine according to the recommendations for firing.
5. Restoration of the product shape deficit (due to shrinkage during firing) using dentins and cutting edge weights.
6. The second firing of dentine according to the recommendations for firing.

8.1.4. Characterization and glossy firing

If necessary, individual features are applied with dyes and fired with or without applying glaze.

8.1.5. Correction (ceramic shoulder)

Small corrections at the end of the work can be performed after glaze firing using the final shoulder weights according to the color scheme for corrections with a low firing temperature.

General instructions on cladding of base metal alloys are available on the Internet at www.degudent.com

8.2. The technique of applying layers to zirconium oxide frames

Cercon ® Ceram Kiss material is specially designed for zirconium oxide frames.

8.2.1. Preparation of the frame

For uniform maintenance of restoration and guaranteed uniformity of the thickness of the applied ceramics, zirconium oxide frames are modeled in a reduced anatomical shape.

Minor correction, as well as a change in the design of the crown edge, are carried out with rotating diamond-coated tools.

To prevent the occurrence of stresses in Cercon ® ceram Kiss facing ceramics, all sharp corners and edges of the frame must be rounded. In order to avoid the occurrence of microcracks, the processing of the frame must be carried out by a turbine with water cooling. It is necessary to work without (or, in extreme cases, with slight) pressure of the cutting tool on the processed frame – in order to avoid overheating of the latter. Frames with visible cracks and/or perforations are not intended for further work. Sandblasting of zirconium oxide frames is carried out with aluminum oxide (grain size 110-125 µm, pressure 3-3.5 bar). Upon completion of mechanical processing, the frames are cleaned with a steam jet or in an ultrasonic bath.

8.2.2. Liners

For individual lining of the frame and the formation of gums, they can be used paste-like liners. The bleach liner in combination with whitening dentin allows you to recreate the light shades of enamel.

Liners of any colors used as the only binder are offered for the installation of opaque frames. It is possible to abandon the application of the liner and apply ceramics directly to the frame. When working without a liner, a fluorescent mass of high color saturation is first applied to the frame. For this purpose, Power Chroma masses are ideal, then dentin is applied in the usual way.

8.2.3. Application and firing of ceramic layers

1. Formation of the dentin nucleus.
2. Applying the masses of the cutting edge to the cutting edge of the tooth crown.
3. The first firing of dentine according to the recommendations for firing.

4. The resulting shrinkage deficit in the shape of the product is subject to restoration by appropriate dentins and masses of the cutting edge.
5. The second firing of dentine, as well as careful processing of the product.

8.2.4. Characterization and glossy firing

If necessary, individual features are applied with dyes and fired with or without applying glaze.

8.2.5. Correction (ceramic shoulder)

Small corrections at the end of the work can be performed after glaze firing using the final shoulder weights according to the color scheme for corrections with a low firing temperature.

9. The principle of working with the product

Kiss Artist dental ceramic materials allow you to create a clearly defined level of aesthetics when performing restorations.

- ✓ **Aesthetic Line Base (Aesthetik Line Base):**
Rational, economical application of ceramic layers..
- ✓ **Individual aesthetics line (Aesthetik Line Individuell):**
A high-tech segment. Due to clear and simple mixing dentins, masses, a lot of additional color nuances are obtained, with which you can perform restorations of a natural appearance in complex clinical cases..

Kiss Artist dental ceramic materials include a basic palette of facing ceramics, which allows you to create excellent restorations. They are a first-class foundation for both creating a basic line of aesthetics and for creating an individual line of aesthetics

Dentines

- **Action-i Dentines** are used as a mass for the formation of a dentine base, in the formation of which the main emphasis is on the color and degree of transparency of natural dentine..
- **Gray Inside Fluorescent Dentines** are used to compensate for highly precise contours of frameworks in the biting region.
- **Gum Dentines** are used for shaping of the gingival part of ceramic restorations at the stage of base line aesthetics.
- **Duceram® Kiss Dentine** are used to perform ceramic restoration of the dentine base on the model of a natural tooth, dentines correspond in color to the natural shades of teeth

Instructions for use of the dentines:

- *The dentines are mixed with a modeling liquid Ducera Liquid SD, Ducera Liquid FORM to a required concentration;*
- *The mixed up dentine is then applied to a framework in a usual way using a brush or a glass instrument.*

Opagues

- **Duceram® Kiss Paste Opaque (PO)**– opaques have low light permeability, provide good hiding ability of the material for the manufacture of crowns.
- **Duceram® Kiss Paste Opaque (PO) Orange** - opaque orange color to create aesthetic natural crown effects

- **Duceram® Kiss Neutral** - opaque, as a low light permeability, which provides a good covering ability of the material for the manufacture of crowns.
- **Duceram® Kiss Paste Opaque (PO) Gum** - opaques matching the color of the gum to form the gingival part of the crown, frame.

Instructions for opaques:

- *opaques are mixed with a modeling liquid. it is possible to use liquid for modeling paste-like paste paste Liquid, liquid for modeling Ducera Liquid FORM to the required consistency.*
- *then the mixed opaque is applied to the frame in the usual way using a brush*

Dentsply Universal NE-Bonder / NP-Bonder - paste material is used for application to the surface of non-ferrous metal alloys as the first layer and improves the adhesion between the alloy and the facing ceramics

Duceram® Kiss Transpa Red transparent effects are ceramic materials which enable additional correction of reddish hues of teeth and are applied at the stage of creation of the base line aesthetics.

Duceram® Kiss Power Chroma – fluorescent masses with high color saturation for individual color rendering, to enhance color in the cervical, palatine and occlusal areas.

Duceram® Kiss Schneide – masses for forming the cutting edge area.

Duceram® Kiss Opalschneide – cutting edge masses with opalescence effect to reproduce the opalescence of natural enamel

Duceram® Kiss Opal Effekt – opal effect masses for individualization of the cutting third of the tooth.

Duceram® Kiss Flu Inside, Transpa TC, Duceram® Kiss White Surface (WS), Duceram® Kiss Stand By (S By) - transparent masses that provide additional transparency of tooth surfaces.

Duceram® Kiss Final Kiss Korrekturmasse – corrective masses for firing after glossy firing.

Instructions for use of the masses:

- *These materials are mixed with a modeling liquid (you can use Ducera Liquid SD, Ducera Liquid FORM) to a required concentration;*
- *The mixed up materials are then applied to a framework in a usual way using a brush or a glass instrument.*

Ducera Liquid SD (Modeling Liquid) is used to mix up dentines, modifiers, materials for the biting edge, and transparent materials.

Ducera Liquid FORM: mixing of opacs, dentins, modifiers, cutting edge masses and transparent masses.

Pastenopaker Liquid is used to mix up pasten opaques.

Universal Malfarben u.Glasur Liquid : mixing of dyes and glazes

Dyes

- **UNIV INCISAL STAIN 11** help to reproduce most of the complex and intricate volumetric effects observed in natural teeth.

- **UNIVERSAL BODY STAIN S2-** material for reproducing surface lighting effects

Universal Overglaze final glazing of the ceramic restoration.

Magic Wizard brush is used to apply layers of veneering ceramics or perform modeling in the process of dental works.

Brush handle Pinselstiel "Pinelo" designed for planting nozzles of various brushes "Pinelo".

Brush-tip Pinselspitze "Pinelo" opaque/stain and "Pinelo" No. 0, No. 8: to complete the brush handle Pinselstiel "Pinelo", next, the brushes are designed for applying layers of facing ceramics.

Probenformer ohne Verpackung: mixing two different masses in a 1:1 ratio with a guarantee of accurate reproducibility.

Test pill former is used for shaping of samples made of Transpa ceramic material to be baked.

1. Ceramics mixed up with Ducera Liquid SD modeling liquid is tamped into a wedge-shaped Test pill former base.
2. The sample is taken out of the base using a paper towel. Any excess of ceramic material is removed using a spatula, and the surface is carefully smoothed out.
3. The sample is then baked with the baking parameters used for baking of veneering ceramic layers and assessment of the resulting sample is performed.
4. The resulting sample is assessed in order to select the optimal baking parameters for the veneering ceramics.

Shade Indicators: Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva и Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside:

Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Gingiva It is used to determine the color nuances of Gum gingival dentin according to the individual characteristics of the patient. The definition is made visually.

Indikator Duceram® Kiss, Duceragold® Kiss, Cercon® ceram Kiss Action-i Dentine-Transpa-Inside is used to determine the color nuances of Action-Identifier dentins, GrayInside fluorescent dentin and Transpa Red mass for individual patient characteristics. The definition is made visually

Indikator Duceram® Kiss Opaque: determining the color nuances of the packaging according to the individual characteristics of the patient.

Indikator Duceram® Kiss Power: determination of the color nuances of the Power Chroma masses according to the individual characteristics of the patient.

Indikator Duceram® Kiss Opal: determination of the color nuances of opal masses according to the individual characteristics of the patient.

Indikator Duceram® Kiss Gingiva – determination of color nuances of gingival masses, dentins for individual characteristics of the patient.

Indicator Artist Gum is used for shading Gum Dentine to match patient's individual features. Shading is performed visually.

Indicator Artist Add-on is used for shading of Action-I Dentine, Gray Inside Fluorescent Dentine, and Transpa Red Ceramic Material to match patient's individual features. Shading is performed visually.

Towel Dentsply Sirona Handtuch: equipment of the workplace of a ceramic technician.

Mischinstrument fuer Koffer Kiss: placement of sets of ceramic materials

Dentsply Sirona Schwamm: removing excess fluid from the brushes

10. General and special recommendations for firing when facing with Kiss

Artist materials

Basic Recommendations on Baking

Duceram® Kiss

Baking		Pre-heating,	Drying	Heating	Finishing	Hold time,	Vacuum,	Tempering
		°C	time, min	rate, °C/min	temperature, min	min	GPa	
Oxidation Detailed information is given in instructions for processing of respective non-noble metals.								
Bioalloys	Paste opaque	575	7:00	55	900	3:00	50	—
	Powder opaque	575	5:00	55	900	3:00	50	—
Common alloy	Paste opaque 1 + 2	575	7:00	55	930	2:00	50	—
	Powder opaque 1 + 2	575	5:00	55	930	2:00	50	—
Samples Without long-term cooling	Margin 1							
	Margin 2	575	7:00	55	920	1:00	50	—
	Dentine 1	575	6:00	55	910	1:00	50	—
	Dentine 2	575	4:00	55	900	1:00	50	—
DeguDent Kiss	Glazing 2	575	3:00	55	890	1:00	—	—
	Corrective material	575	4:00	55	880	1:00	50	—
	Final shoulder	450	4:00	55	660	1:00	50	—
Long-term cooling from	Dentine 1	575	6:00	55	910	1:00	50	3 min/850 °C
	Dentine 2	575	4:00	55	900	1:00	50	3 min/850 °C
CTE 14,6 μm/m · K	Glazing 2	575	3:00	55	890	1:00	—	3 min/850 °C

Duceram® Kiss – Non-Noble Alloys

Baking	Pre-heating,	Drying	Heating	Finishing	Hold time,	Vacuum	Tempering	Long-term
	°C	time, min	rate, °C/min	temperature, °C	min	GPa		cooling
Oxidation Detailed information is given in instructions for processing of respective non-noble metals.								

Bonder paste	575	7:00	55	980	2:00	50	—	—
Paste opaque	575	7:00	55	950	2:00	50	—	—
Powder opaque	575	5:00	55	950	2:00	50	—	—
Margin 1 + 2	575	7:00	55	930	1:00	50	—	—
Dentine 1	575	6:00	55	920	1:00	50	3 min/850 °C	Up to 600 °C
Dentine 2	575	4:00	55	910	1:00	50	3 min/850 °C	Up to 600 °C
Glazing 2	575	3:00	55	890	1:00	—	3 min/850 °C	Up to 600 °C
Corrective material	575	4:00	55	880	1:00	50	3 min/850 °C	Up to 600 °C
(Final Kiss) Final 450 shoulder		4:00	55	660	1:00	50	—	—

Stress relief by cooling to the baseline temperature should be performed regardless of the thermal temperature coefficient (CTE). Please, follow instructions provided by manufacturers of alloys. For all bridges longer than five units the baking temperature or time of baking of dentine should be increased to compensate for low thermal conductivity of non-noble alloys. The settings given above are for reference only and should be considered as the starting point. All outcomes of baking depend on power of the furnace used which, in turn, depends on the manufacturer, model, and service life of the furnace. Therefore, these reference settings should be adjusted individually at each baking. We recommend performing test baking to regulate the furnace.

DuceraGold® Kiss

Baking			Degunorm				
	Pre-heat temperature, °C	Drying time, min	Heating rate, °/min	Baking temperature °C	Hold time, min	Vacuum, GPa	Tempering
Oxidation	575	0:00	55	780	5:00	50	—
Degunorm classic Base	575	7:00	55	780	1:00	50	—
Paste opaque	575	7:00	55	780	1:00	50	—
Margin 1	450	5:00	55	780	1:00	50	—
Margin 2	450	5:00	55	780	1:00	50	—
Dentine 1	450	6:00	55	780	1:00	50	3 min/720 °C
Dentine 2	450	5:00	55	780	1:00	50	—
Glazing 2	450	3:00	55	770	1:00	—	—
Correction (Final Kiss)	450	2:00	55	720	1:00	50	50
Final shoulder	450	2:00	55	720	1:00	50	—

Tempering and cooling for each baking process starting from the first baking of dentine, including glazing, for Degunorm eco and Econolloy Au: Tempering for 3 min at 720°C, or, alternatively, cooling for 5 minutes starting from 680 °C; use the cooling level recommended. If you use Degunorm logic, no tempering or cooling is required.

The settings given above are for reference only and should be considered as the starting point. Outcomes of baking may differ. All outcomes of baking depend on power of the furnace used which, in turn, depends on the manufacturer, model, and service life of the furnace. Therefore, these reference settings should be adjusted individually at each baking. We recommend performing a test baking to regulate the furnace. The settings and information given above have

been thoroughly developed and verified; however, our Company makes no warranty with regard to the said settings and information.

Note:

Please, pay attention to the following procedure of preliminary drying of workpieces made of Duceragold Kiss ceramic materials after trial insertion of restorations:

- Workpieces should be heated to a temperature within the range 80°C to 90°C and then left to dry for 30 minutes or 1 to 1.5 hour (for larger workpieces) to remove any moisture and organic remnants accumulated during the trial insertion of the workpiece.
- Depending on the degree of its contamination, the workpiece should be treated by steam or slightly sandblasted by aluminum oxide (particle size: 50 µm) to remove any organic remnants.
- The restoration should be heated to 450°C at a heating rate from 5°C to 10°C per minute, and then held for one hour.
- All defects should be corrected to complete the process.

Cercon® Ceram Kiss

Baking	Pre-heating, °C	Drying time, min	Heating rate, °C/min	Finishing temperature, °C	Hold time, min	Vacuum, GPa	Long-term cooling, min
Paste opaque 1	575	8:00	5 5	970	1:00	50	—
Paste opaque 2	575	8:00	5 5	960	1:00	50	—
Margin 1 + 2	450	6:00	5 5	850	1:30	50	—
Dentine 1	450	5:00	5 5	830	1:30	50	—
Dentine 2	450	5:00	5 5	820	1:30	50	—
Glazing 2	450	3:00	5 5	800	1:00	—	6:00
Correction (Final Kiss)	450	5:00	5 5	680	1:00	50	6:00
Final shoulder (F-SM)	450	5:00	5 5	680	1:00	50	6:00

Long-term cooling is compulsory after the last baking; this rule also applies to correctional baking of restorations after their trial insertion.

Baking temperature should be adjusted depending on the number of items baked in one cycle. Each 5-10 additional items require increase in temperature by +5°C to +10°C; if the number of items added exceeds 10, the temperature should be increased by +10°C to +20°C.

The settings given above are for reference only and should be considered as the starting point. Outcomes of baking may differ. All outcomes of baking depend on power of the furnace used which, in turn, depends on the manufacturer, model, and service life of the furnace. Therefore, these reference settings should be adjusted individually at each baking. We recommend performing a test baking to regulate the furnace.

Current information about basic and specific recommendations on baking in the process of veneering with Kiss Artist materials is also available on our website: www.degudent.com.

11. Sterilization

Not applicable. Kiss Artist and Duceram® Kiss materials are not sterile.

12. Conditions and shelf life

Shelf life: 4 years (pastes)

6 years (liquids)

15 years (powders)

- Store in a well-ventilated room; keep away from light.
- Powders and pastes: Keep in tightly sealed containers at a temperature within the range from 2°C to 25°C.
- Liquids: Keep in tightly closed containers at a temperature of at least 10°C.
- Close containers tightly after taking some material, and keep the containers closed.
- Protect from water penetration.
- Do no freeze.
- Do not use after the expiration date.

Moisture can negatively affect properties of the materials.

If all the described conditions are met, the warranty period of storage is not limited within the shelf life of medical devices.

13. Transportation conditions:

These products should be transported pursuant to rules applicable to each type of transportation by covered vehicles at a temperature within the range 5°C to 40°C, relative humidity < 95%, and atmospheric pressure from 800 to 1 200 GPa.

14. Requirements to Environmental Protection

The products have no negative effect on the environment during their life cycle. No specific requirements are set out for these products.

15. Environmental Conditions for Operation

Temperature: 15 to 40°C; Relative humidity: <75% (without condensate)

16. Conditions of Use

These products are used in oral cavities in presence of human biological fluids at rated temperatures from 32°C to 42°C. Temperature range for accessories which are not in contact with oral cavities: +10°C to +35°C.

These products are for use by qualified medical specialists at clinical institutions only.

If all the described conditions are met, the warranty period is not limited within the shelf life of medical devices.

17. Requirements to Maintenance and Repair

These products are not subject to repair and maintenance.

18. Labeling

Symbols used in labeling:

Symbol	Interpretation
REF	Catalogue reference number

	Instruction for Use is available on our official website: www.dentsply.eu/ifu
	Serial number (batch/lot number)
	Date of manufacturing
	Best before
	Manufacturer's details
	CE certification mark
	Keep in a dry place.
	Keep away from sunlight
	"For dentist professional use only"

19. List of applicable international standards

- ISO 13485:2016 Medical devices -- Quality management systems -- Requirements for regulatory purposes.
- Directive 93/42/EEC Council Directive concerning medical devices dated June 14, 1993
- EN ISO 10993 series Biological evaluation of medical devices.
- EN 1041:2008 Information supplied by the manufacturer of medical devices
- ISO 15223-1:2023 Medical devices -- Symbols to be used with medical device labels, labelling and information to be supplied- Part 1: General requirements.
- EN ISO 9693:2019 Metal-ceramic dental restorative systems.
- EN ISO 6872:2015+A1:2018 Dentistry - Ceramic materials.
- GOST 31575-2012 Dental Prosthetic Metal Ceramics. Technical Requirements. Testing Methods.
- GOST R ISO 15223-1-2023 Medical Devices. Symbols Used in Marking and Labeling of Medical Devices and in Accompanying Documents. Part 1. Basic Requirements.
- GOST ISO 14971-2021 Medical Devices. Application of Risk Management to Medical Devices.

20. Disposal

The disposal of unused medical devices and packaging is carried out in the same way as household waste (for the Russian Federation – waste of hazard class A). The hazard classes and the method of disposal are given in accordance with SanPiN 2.1.3684-21. Packaging waste should be disposed of at local packaging collection points, in specialized containers.

21. Manufacturer's Warranties

The manufacturer shall not be held responsible and will not indemnify for any loss related to possible damage or accidents resulting from:

- Using components unrelated to the product that may affect its normal operation;
- Non-compliance with the following instructions for use.

The user is held responsible for checking usability of the products for any purposes not provided for in the said instructions!

If any questions about product quality arise, please, contact the authorized representative of the manufacturer.

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List of Documents No.: 335 / 2024 G

I herewith certify the signature acknowledged in front of me of

Mr. **Norbert Georg Rabenstein**, born on June 18th 1972,
with registered office at Rodenbacher Chaussee 4, 63457 Hanau.

Mr. **Norbert Georg Rabenstein** is acting as Managing Director with sole power of representation for the company DeguDent GmbH with registered office at Hanau.

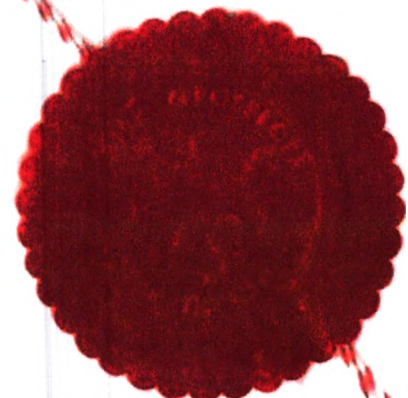
I hereby certify on the basis of today's inspection of the electronic commercial register at the local court in Hanau under HRB 7182, that Mr. **Norbert Georg Rabenstein** is entitled to represent the company DeguDent GmbH at Hanau with sole power of representation.

The interested party is personally known by the notary.

I do establish at the same time, that the interested part denied the question of a prior involvement according to § 3 Sec. No. 7 BeurkG.

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