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SECRETARY OF STATE
STATE OF INDIANA

Connie Lawson
Secretary of State


APOSTILLE

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1. Country: United States Of America
2. This public document has been signed by *Barbara Walsh Brown*
3. Acting in the capacity of notary public in & for *Hancock County*
4. And bears the seal/stamp of notary public in & for the State of Indiana

CERTIFIED

5. at Indianapolis, Indiana
6. This *Fourth* day of *August*, 2020
7. by the Secretary of State of Indiana
8. No. A2020-0804014643
9. Seal/Stamp:
10. Signature: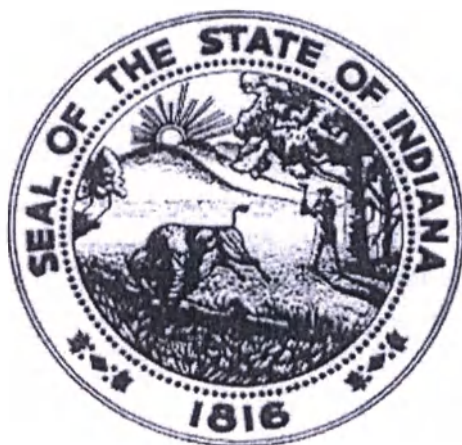

Connie Lawson

Connie Lawson
Indiana Secretary of State

*Effective May 1, 2003 all apostilles from the Indiana Secretary of State will have an electronically printed seal.
This document was revised March 17, 2012.*

HumaPen® Ergo II

INSULIN DELIVERY DEVICE

INSTRUCTIONS FOR USE

Intended use: HumaPen® Ergo II is a reusable pen device for self-medication with insulin. HumaPen® Ergo II pen can only be used with 3 mL Lilly insulin cartridges and disposable needles (sold separately).

Intended user: HumaPen® Ergo II pen is designed for patients recommended to use 3 mL Lilly insulin cartridges, namely adults, adolescents and children able to perform injections themselves or adults trained to perform injections for other persons.

Contraindications: There are no known contraindications to the use of HumaPen® Ergo II pen.


The device is designed to be used with 3 mL Lilly insulin cartridges only (100 IU/mL or 100 U/mL) under the brand names of Humulin®, Humalog®, or Basaglar®. Do not use other brands of insulin cartridges.

The color of your HumaPen® Ergo II is Two-Tone Blue.

Lilly (red handwritten font)

Pen-injector needle
(sold separately)

HumaPen® Ergo II Pen Parts

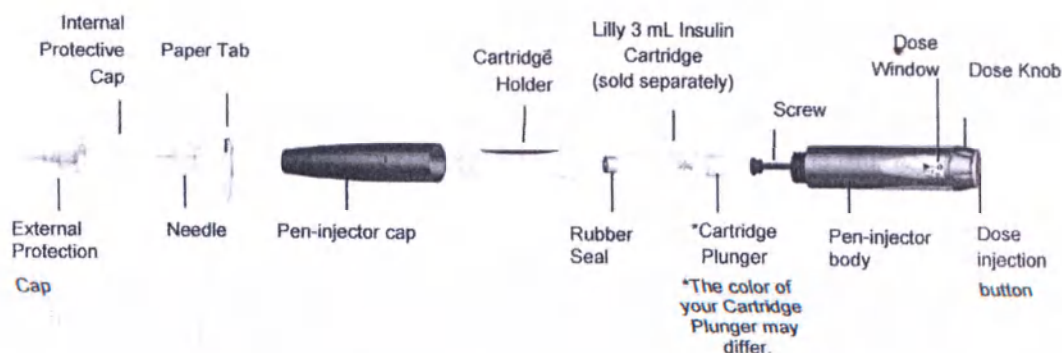HumaPen Ergo® II is designed for ease of use. One 3 mL Lilly insulin cartridge (100 IU/mL or 100 U/mL) allows you to perform several insulin injections. You can inject from 1 to 60 units of insulin from a Lilly 3 mL insulin cartridge in each injection. You can dial your dose one unit at a time. If you dial the wrong dose, you can correct the dose without wasting any insulin.

Warning!

Read these instructions completely and follow the directions carefully before using the pen. Failure to follow the instructions may result in the wrong insulin dose being delivered.

Do not share your pen or needles to others as this may cause a risk transmission of infectious agents.

If any of the parts of your Pen appear broken or damaged, DO NOT USE.

Contact your physician or buy a new HumaPen® Ergo II pen from a pharmacy. HumaPen® Ergo II is not recommended for the blind or visually impaired without the assistance of a sighted individual trained to use it. Always carry a spare insulin pen in case your pen is lost or damaged.

1 INSERT THE INSULIN CARTRIDGE

Important Notes

- Your healthcare professional has prescribed the type of insulin best for you. Any changes in insulin should be made only under medical supervision.

	HumaPen® Ergo II is designed to be used with 3 mL Lilly insulin cartridges only. Do not use other brands of insulin cartridges. - Before each injection, read the cartridge label and be sure the pen contains the correct Lilly 3 mL HumaPen® Ergo II insulin cartridge with the type of insulin that suits you.
---	---

- Read and follow the instructions provided in your Lilly 3 mL insulin cartridge Patient Information Leaflet.
- The color of the pen is not intended to indicate insulin type.
- The numbers on the cartridge holder give an estimate of the amount of insulin remaining in the cartridge. Do not use these numbers for measuring an insulin dose.

Frequently Asked Questions about Inserting the Insulin Cartridge

1. Why doesn't the screw move out when there is no cartridge in the pen?

The screw may not move out when you push the injection button unless there is a cartridge in the pen. This feature of the pen allows you to easily push the screw into the pen body when replacing a cartridge. Once a cartridge is inserted, the screw will move out when the injection button is pushed.

2. What should I do if I can't attach the cartridge holder to the pen body?

Check that the insulin cartridge is fully inserted into the cartridge holder. Then carefully line up the cartridge holder with the pen body and screw together until secure.

2 INSPECT INSULIN DELIVERY FROM THE PEN-INJECTOR (BEFORE EACH INJECTION)

Important Notes

- HumaPen® Ergo II is suitable for use with Becton, Dickinson and Company pen needles.
- The directions regarding needle handling are not change the rules of a health authorities or medical institution.

	Use a new needle for each injection. This will help ensure sterility. It will also help prevent leakage of insulin, keep out air bubbles, and reduce needle clogs.
---	--


Frequently Asked Questions about Priming

1. Why is it important to prime before every injection?

If you do **NOT** prime, you may get the wrong insulin dose. Priming helps to ensure that the pen and needle are working properly. Once the pen is properly primed, insulin will flow from the needle. You may need to prime several times before you see insulin at the tip of the needle.

2. Why can it take several attempts to prime when a new cartridge is inserted?

There may be a gap between the screw and the cartridge plunger. Repeating the priming steps will move the screw out to touch the cartridge plunger. Once the end of the screw pushes the cartridge plunger out, insulin will flow from the needle.

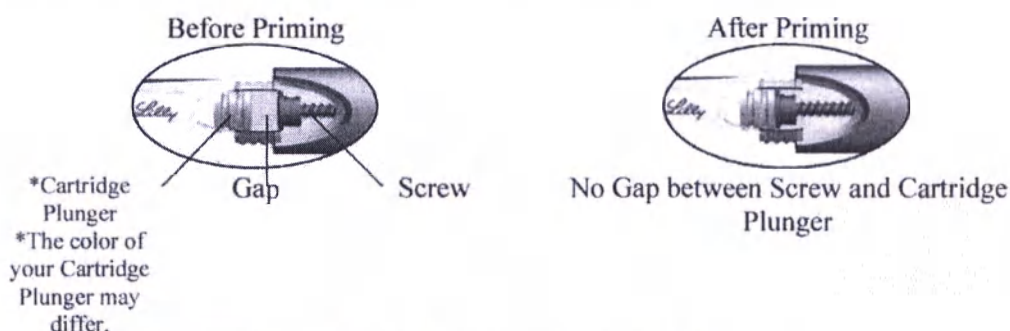- Repeat the priming steps until insulin is seen at the needle tip.
- If you are still unable to see insulin flow from the needle, go to Question 3.

3. Why should I prime until I see insulin at the tip of the needle?

Priming moves the screw into contact with the cartridge plunger and gets the air out of the cartridge.

- You may see insulin on the tip of the needle when you first attach it. This only shows that the needle is attached and not clogged. You must still prime the pen.
- You may also see no insulin at all when you are priming the pen. This may be because the screw is moving forward to close a gap between the screw and the cartridge plunger.
- Insulin will flow only when the screw is immediately adjacent to the cartridge plunger, this shows that the priming of the pen was tested correctly.
- If the injection button is hard to push, the needle may be clogged. Attach a new needle. Repeat the priming steps until insulin is seen at the needle tip.

If you are still unable to see insulin flow from the needle, **do NOT** use the pen. Use a new HumaPen® Ergo II pen you can buy in a pharmacy.

4. What should I do if I have an air bubble in the cartridge?

Priming your pen will remove air. Point the pen up, and tap the cartridge gently with your finger so any air bubbles can collect near the top. Repeat the priming steps until insulin is seen at the needle tip. A small air bubble may remain in the cartridge after completion of the priming steps. If you have properly primed the pen, this small air bubble will not affect your insulin dose.

3 INJECT THE DOSE

Important

- You must **PUSH** the injection button straight down for the dose to be delivered.
- You **will NOT** receive your insulin by turning the dose knob.

- Do not attempt to change the dose while injecting.

Frequently Asked Questions about Injecting

1. Why is it difficult to push the injection button when I try to inject?

- Your needle may be clogged. Try attaching a new needle, and then prime the pen.
- Pushing the injection button down quickly may make the button harder to push. Pushing the button more slowly may make it easier.
- Using a larger diameter needle will make it easier to push the injection button during injection. Ask your healthcare professional which needle is best for you.
- Your injection button may become harder to push if the inside of your pen gets dirty with insulin, food, drink or other materials. Following the CARE, STORAGE, AND DISPOSAL instructions below should help prevent this.

If none of the above steps resolves the problem, your pen may need to be replaced.

2. Why doesn't the dose knob go to zero when I inject my dose?

This can happen if the insulin cartridge does not have enough insulin left in it for your entire dose. The number in the dose window is the amount you did **NOT receive**. To get the rest of your dose, remove the needle, change the insulin cartridge, attach a new needle, and prime the pen. Then complete your dose by dialing the amount that you did **NOT receive** and inject only this amount.

CARE AND STORAGE

- Remove the needle after every use. Do not store the pen with the needle attached.
- Keep your HumaPen® Ergo II, Lilly 3 mL insulin cartridges, and needles out of the reach and sight of children.
- Protect the pen from dust.
-  Protect from water.
-  Protect from sunlight.
-  Temperature limits.
- A pen without insulin cartridge can be stored at – 40°C to 70°C.
- Wipe the pen cap, pen body, and case with a damp cloth to clean them.
- Do not use alcohol, hydrogen peroxide, or bleach on the pen body or dose window. Also, do not cover in liquid or apply lubrication such as oil, as this could damage the pen.
- Protect your pen with insulin cartridge from extreme high or low temperatures.
- Refer to the 3 mL insulin cartridge Patient Information Leaflet for complete insulin storage instructions.
- The device is not currently designed to undergo maintenance or repairs.
- Do not attempt to repair a faulty pen yourself.

PEN REPLACEMENT AND DISPOSAL

Do not use your HumaPen® Ergo II for more than 3 years after the first use or past the expiration date on the carton.

After removing the needle, dispose of the used pen with your household garbage.

If you need a new pen, contact your physician or buy pen in a pharmacy.

Write the date of the first use of your pen here as day/month/year: _/_/.

Write the expiration date of your pen stated on the package, as day/month/year: _/_/.

The manufacturer is not responsible for any malfunction of your HumaPen® Ergo II pen caused by violation of these Instructions.

If your HumaPen® Ergo II pen was damaged, Lilly will replace it provided you submit your HumaPen® Ergo II pen to the supplier for inspection.

If you have any questions or problems related to the use of HumaPen® Ergo II pen, please contact your physician or Lilly Pharma LLC.

Lilly (red handwritten font)

Preparation


Wash your hands and make sure you have the following items before beginning:

- HumaPen® Ergo II Pen
- Lilly 3 mL Insulin Cartridges
- New Pen Needle
- Alcohol wipe

HumaPen® Ergo II INSULIN DELIVERY DEVICE


Pen-injector needle
(sold separately)

HumaPen® Ergo II Pen Parts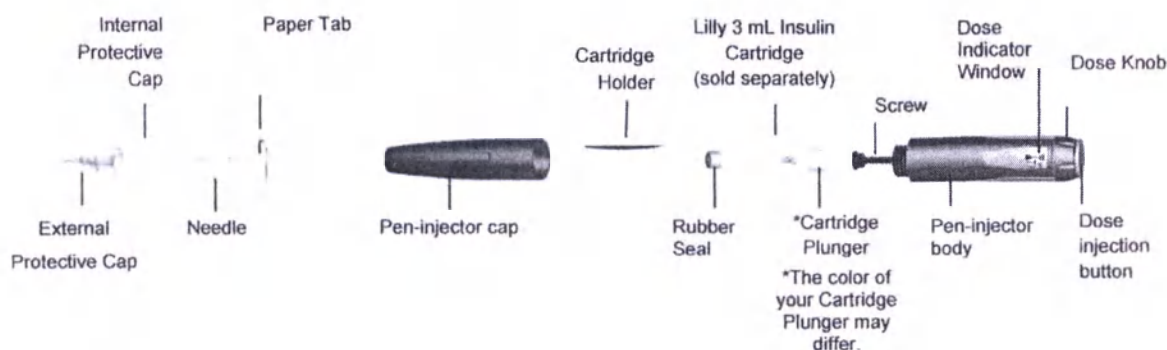

1. INSERT THE INSULIN CARTRIDGE

Pull Off Pen Cap

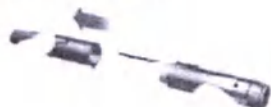Insert Cartridge

Push Screw Back

Remove Cartridge Holder

Insert small end of Lilly 3 mL insulin cartridge into the cartridge holder.

Note:

Check the insulin cartridge to make sure it is not cracked or broken and your insulin is not expired before inserting the insulin cartridge.

- The screw may be in or out when you get the pen.
- If the screw is out, use the cartridge plunger to push the screw back.

Note:


If you push the injection button without a cartridge in the cartridge holder the screw will not move forward.

A

B

C

Attach Cartridge Holder

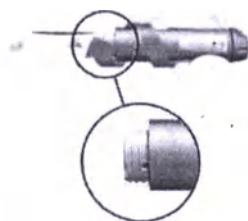Push the cartridge holder and pen body straight together. Screw the pen body onto the cartridge holder until it is secure.

Note: If the cartridge holder is not completely attached, the screw may not move and you may not get your full dose.

D

Inspect Insulin

Be sure to check your insulin for:

- Type
- Appearance

Wipe the small rubber seal on the end of the cartridge with an alcohol wipe

E

Only for insulin mixtures (suspensions)

Roll the pen gently 10 times.

and invert the pen 10 times. The insulin should look evenly mixed.

F

2. INSPECT INSULIN DELIVERY FROM THE PEN-INJECTOR (BEFORE EACH INJECTION)

Note: If you do **NOT** prime, you may get the wrong insulin dose.

Remove paper tab from the needle

Remove Needle Cap

Dial 2 units

Attach the needle

Pull off the outer cap.

Save the outer cap to remove the needle after your injection.

Pull off the small needle protector and throw away.

Tap cartridge

Point pen up.

Tap cartridge to collect air at the top for removal.

Screw needle straight onto the cartridge holder

A

B

C


Look for insulin from the pen injector.

Push injection button and hold for 5 seconds.

If no insulin is seen, repeat priming

Look for insulin at the tip of the needle.

Dial 2 units.

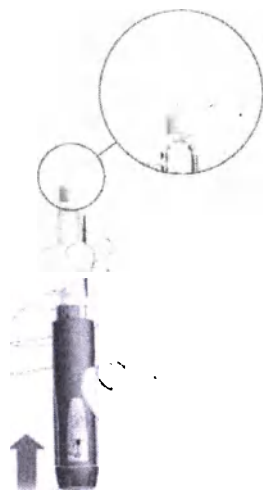Tap cartridge.

Note: Several prime attempts may be needed the first time a cartridge is used. If no insulin flows after several attempts the needle may be clogged.

Push injection button and look for insulin at the needle tip.

D

E

3. INJECT THE DOSE

Dial your dose

Turn dose knob to your desired dose.

Example: 15 units shown in the drawing above.


If you dial too many units, you can correct the dose by dialing backwards.

The dose knob clicks as you turn it. **DO NOT** dial your dose by counting clicks because you may dial the wrong dose.

A

Inject your dose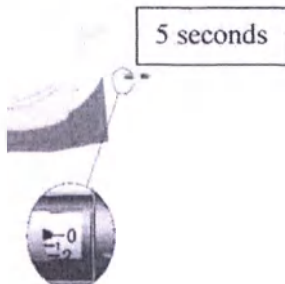

Insert the needle as directed by your healthcare professional.

Place your thumb on the injection button then slowly and firmly push the button until it stops moving.

Continue to hold the button for **5 seconds**, then remove the needle from your skin.

Check to make sure you see a **0** in the dose window to confirm you received the complete dose.

Note:

It is possible to set a dose larger than the amount of insulin left in the cartridge.

At the end of an injection, the number in the dose window should be **0**. If it is not, this is the amount of insulin you did **NOT receive**. Remember this number.

Remove the needle and the empty cartridge.

Insert a new cartridge as shown in 1, attach a new needle, and prime the pen as shown in 2.

Complete your dose by injecting only the amount you did not receive.

If you do not think you received your full dose, do not take another dose. Call your healthcare professional for assistance.

B

4. REMOVAL AND DISPOSAL OF NEEDLE

Put the large needle cover on the needle.

 Carefully put the large needle cover on the needle.

Remove the needle

- Remove the capped needle by twisting it off.
-  Put used needles in a sharps container or a hard plastic container with a secure lid. Do not throw needles directly into your household trash.
- Do not reuse the filled sharps container.
- Ask your healthcare provider about options to dispose of the sharps container properly.
- The directions regarding needle handling are not intended to replace local law, healthcare provider or medical institution.

Note: To prevent air from entering the cartridge, do not store the pen with a needle attached.

Replace Pen Cap

C

D

Excuse

If you have any questions or problems related to the use of HumaPen® Ergo II pen, please contact your physician or Lilly Pharma LLC.

Please inform Lilly Pharma LLC of any complaints or adverse events, including potentially serious adverse events (incidents), noticed during use of HumaPen® Ergo II pen.

You can also report all adverse events, including potentially serious adverse events (incidents), noticed during use of HumaPen® Ergo II pen to the Federal Service for Surveillance in Healthcare (Roszdravnadzor).

Marketing
authorization
No. FSZ 2012/12222
February 19, 2016

Authorized
representative in the
European
Community:
Eli Lilly Nederland
B.V.
Papendorpseweg 83,
3528 BJ UTRECHT
The Netherlands

 Manufacturer:
Tech Group Tempe
640 South Rockford Drive, Tempe,
AZ 85281, USA

Packager:
Lilly S.A.
Avenida de la Industria, 30,
28108, Alcobendas-Madrid,
Spain

Organization in Russia accepting
claims:
Lilly Pharma LLC, Russia,
123317 Moscow,
10, Presnenskaya Embankment
Tel: +7 (495) 258-50-01
Fax: +7 (495) 258-50-05

CE
0344

HumaPen® Ergo II meets the dose accuracy and functional requirements of ISO 11608-1 with
Lilly 3 mL cartridges.

HumaPen® and HumaPen® Ergo II are trademarks of Eli Lilly and Company.
Copyright © 2005, 2020, Eli Lilly and Company. All rights reserved.

Lilly (red handwritten font)

Date of last revision of the Instructions for Use:

If you have any questions or problems related to the use of HumaPen® Ergo II pen, please contact your physician or Lilly Pharma LLC.

Please inform Lilly Pharma LLC of any complaints or adverse events, including potentially serious adverse events (incidents), noticed during use of HumaPen® Ergo II pen.

You can also report all adverse events, including potentially serious adverse events (incidents), noticed during use of HumaPen® Ergo II pen to the Federal Service for Surveillance in Healthcare (Roszdravnadzor).

Marketing
authorization
No. FSZ 2012/12222
February 19, 2016

Authorized
representative in the
European
Community:
Eli Lilly Nederland
B.V.
Papendorpseweg 83,
3528 BJ UTRECHT
The Netherlands

 Manufacturer:
Tech Group Tempe
640 South Rockford Drive, Tempe,
AZ 85281, USA

Packager:
Lilly S.A.
Avenida de la Industria, 30,
28108, Alcobendas-Madrid,
Spain

Organization in Russia accepting
claims:
Lilly Pharma LLC, Russia,
123317 Moscow,
10, Presnenskaya Embankment
Tel: +7 (495) 258-50-01
Fax: +7 (495) 258-50-05

CE
0344

HumaPen® Ergo II meets the dose accuracy and functional requirements of ISO 11608-1 with
Lilly 3 mL cartridges.

HumaPen® and HumaPen® Ergo II are trademarks of Eli Lilly and Company.
Copyright © 2005, 2020, Eli Lilly and Company. All rights reserved.

Lilly (red handwritten font)

Date of last revision of the Instructions for Use:

Executed this 31st day of July, 2020.
ELI LILLY AND COMPANY


By Eileen McCall
Eileen McCall, Advisor
Global Regulatory Affairs-International Scientist
Eli Lilly and Company

UNITED STATES OF AMERICA)
STATE OF INDIANA) SS:
COUNTY OF MARION)

Subscribed and sworn to before me, a Notary Public in and for said County and State, this 31st day of July, 2020.

Commission Expires: March 6, 2026
Commission Number: 711188

Barbara Walsh Brown
Barbara Walsh Brown
Resident of Hancock County, Indiana, USA

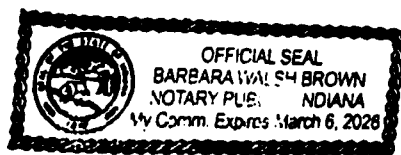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