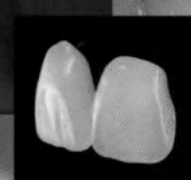
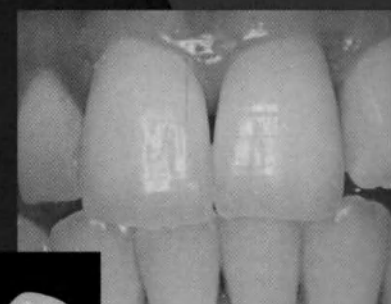
Major Dent®

MOULD
CHART

MAJOR

ISO 3336: 1993, Type 1 - Type 2
ADA 15, Type I - Type II

ANTERIOR MOULDS CLASSIFICATION ■ KLASIFIZIERUNG DER FRONTZAHNFORMEN ■ CLASSIFICATION DES FORMES ANTERIEURES ■ CLASSIFICACION DE LAS FORMAS ANTERIORES ■ CLASSIFICAZIONE DELLE FORME ANTERIORI ■ KLASIFICERING AF FORMERNE FORAN ■ ETUHAMMAMALLIEN LUOKITTELU ■ Ταξινόμηση εμπρόσθιων ■ CLASSIFICATIE VAN DE VOORVORMEN ■ CLASSIFICAÇÃO DAS FORMAS ANTERIORES ■ INDELNING AV FRÄMRE FORMER

STRUCTURE ■ GESICHTSFORM ■ STRUCTURE ■ ESTRUCTURAS ■ CONFORMAZIONE ■ UDFORMNING ■ MALLI ■ Μορφή ■ CONFORMATIE ■ CONFORMAÇÃO ■ FORM

LATERAL PROFILE ■ PROFIL LATERAL ■ PROFIL LATERAL ■ PERFIL LATERAL ■ PROFILO LATERALE ■ SIDEPROFIL ■ SIVUPROFIILI ■ Πλάγια όψη ■ ZIJPROFIEL ■ PERFIL LATERAL ■ SIDOPROFIL

MESIO-DISTAL PROFILE ■ PROFIL MESIO-DISTAL ■ PROFIL MESIO-DISTAL ■ PERFIL MESIO-DISTAL ■ PROFILO MESIO-DISTALE ■ MESIO-DISTALPROFIL ■ MESIO-DISTAALINEN PROFIILI ■ Οψη MESIO-DISTAL ■ PROFIEL MESIO-DISTAL ■ PERFIL MESIO-DISTAL ■ MESIO-DISTALPROFIL

POSTERIOR MOULDS CLASSIFICATION ■ KLASIFIZIERUNG DER BACKENZAHNFORMEN ■ CLASSIFICATION DES FORMES POSTERIEURES ■ CLASSIFICACION DE LAS FORMAS POSTERIORES ■ CLASSIFICAZIONE DELLE FORME POSTERIORI ■ KLASIFICERING AF BAGPROFILERNE ■ POSKIHAMMAMALLIEN LUOKITTELU ■ Ταξινόμηση οπίσθιων ■ CLASSIFICATIE VAN DE ACHTERVORMEN ■ CLASSIFICAÇÃO DAS FORMAS POSTERIORES ■ INDELNING AV BAKRE FORMER

Total and partial prosthesis - Total und Teilprothetik - Prothèse totale et partielle - Prótesis total y parcial - Protesi totale e parziale - Fuldstændig eller ufuldstændig protese -

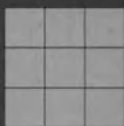
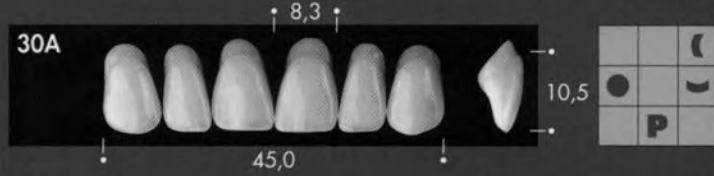
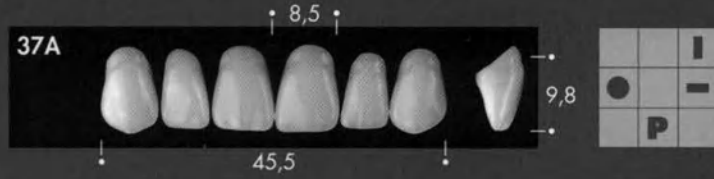
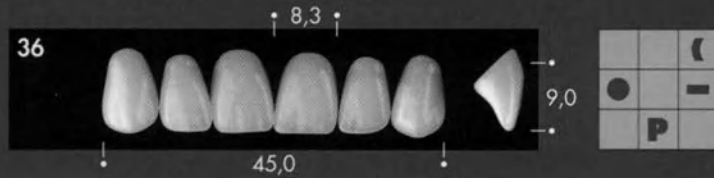
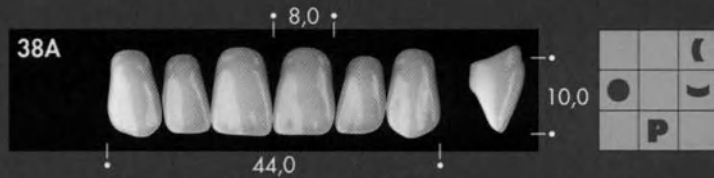
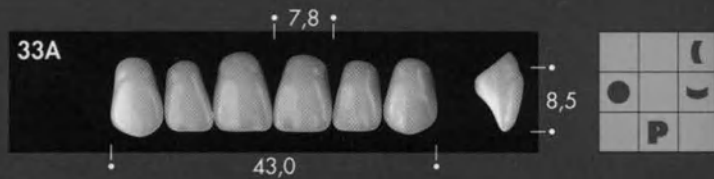
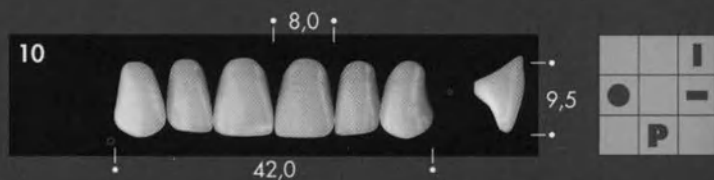
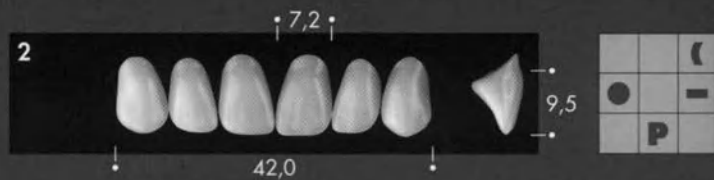
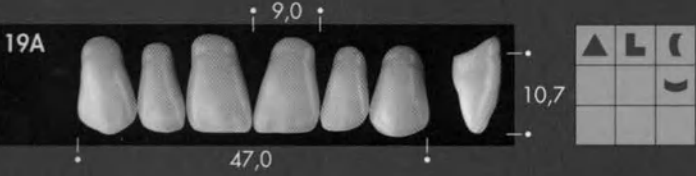
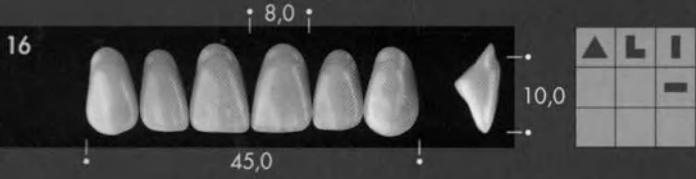
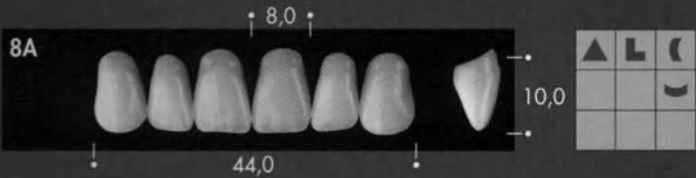
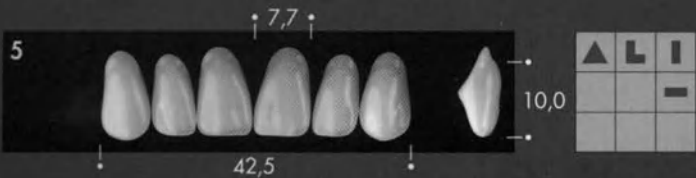
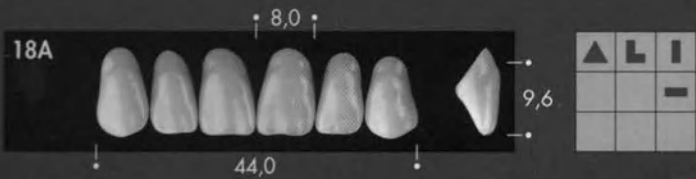
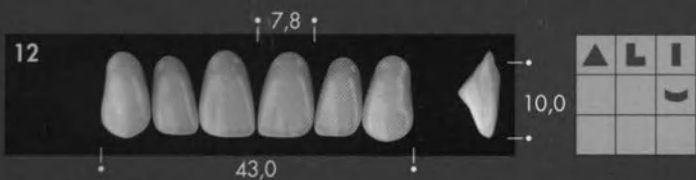
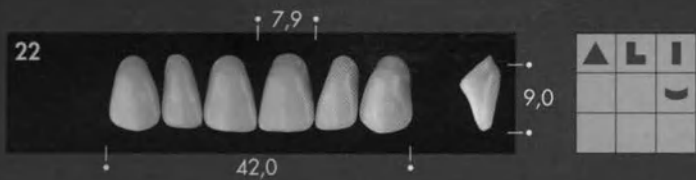
Total prosthesis with natural antagonist - Totalprothetik mit natürlichen Antagonisten - Prothèse totale avec antagoniste naturel - Prótesis total con antagonista natural - Protesi totale con antagonista naturale - Fuldstændig protese med naturligt modstykke -

Skeletal and combined prosthesis - Skelettierter- und Kombinationsprothetik - Prothèse squelettée et combinée - Prótesis esqueléticas y combinada - Protesi scheletrata e combinata - Skelet- eller kombineret protese -

Temporary prosthesis - Provisorien - Prothèse provisoire - Prótesis provisionales - Protesi provvisoria - Midlertidig protese - Tilapäinen proteesi -

Prosthesis on implant - Implantat-Versorgungen - Prothèse sur implant - Prótesis sobre implantes - Protesi su impianti - Protese på anlæg - Keinojuuriin tuettu proteesi -

Prosthesis with lingual occlusion - Prothetik nach der Zungenabstützungs-Technik - Prothèse avec occlusion linguale - Prótesis con montaje lingualizado - Protesi con montaggio lingualizzato - Protese med lingvalmontering -

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- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous conforming to Directive 67/548/EEC and subsequent amendments. Consequently, no Material Safety Data Sheet is available.
- Grinding of the teeth generates dust which is classified as "not otherwise classifiable". Do not inhale. Use appropriate protection. TLV/TWA applies.
- Dispose conforming to local law. Product and packing materials are not listed in the Hazardous Waste Inventory as per C.D. 91/689/EEC 1(4). Patient-contacted materials are at infection risk even for disposal purposes (EU Hazardous Waste Inventory No. 180103).
- Available in the following shade guides: 2C-3R REF FDA104, A1-D4 REF FDA118.

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
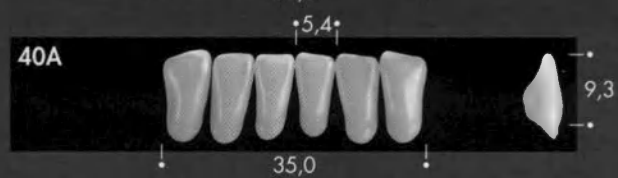
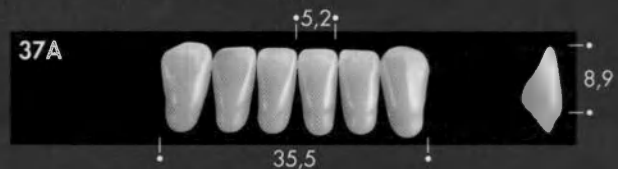
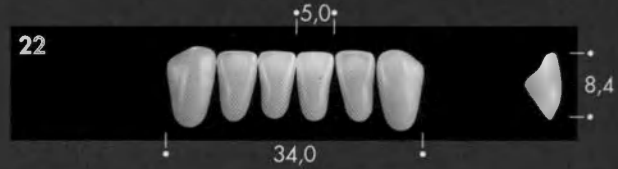
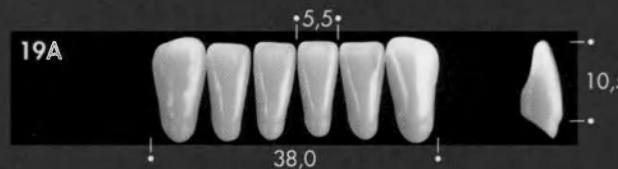
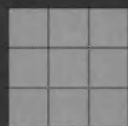
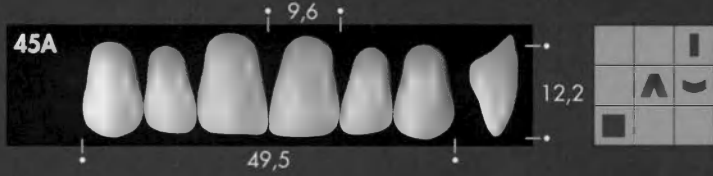
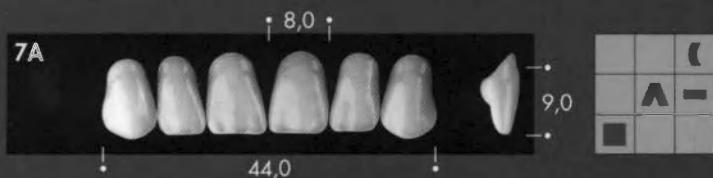
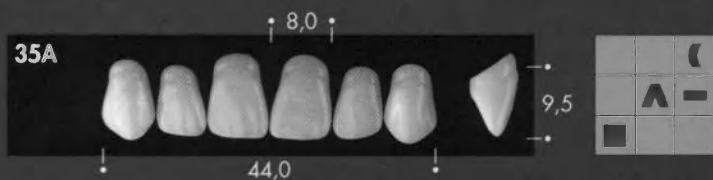
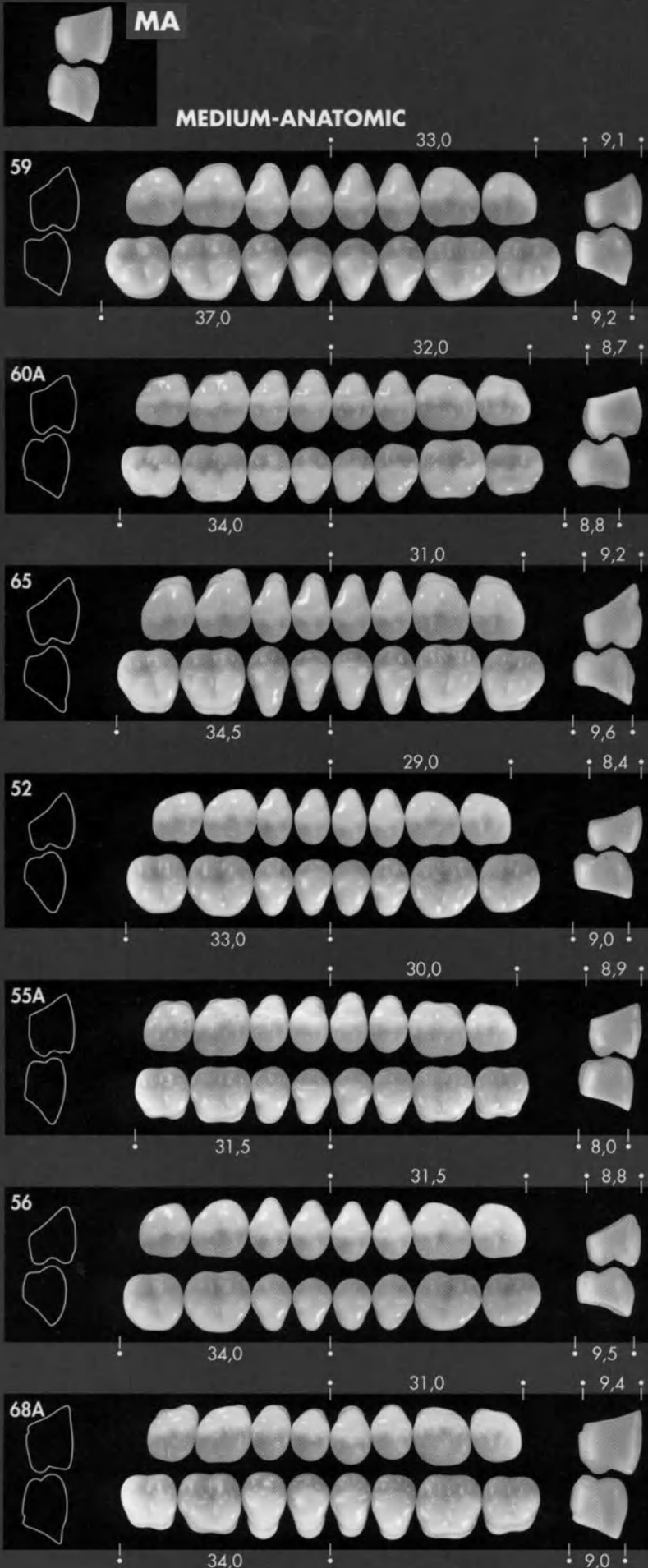
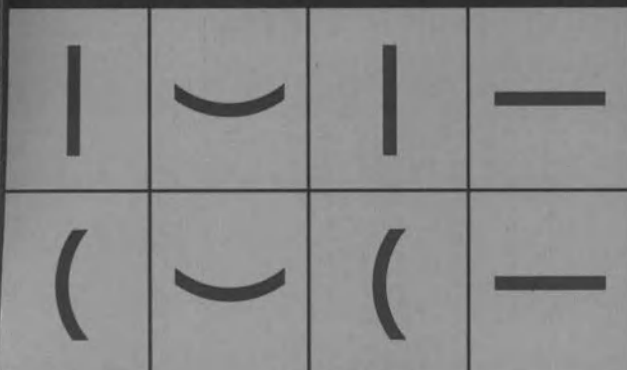
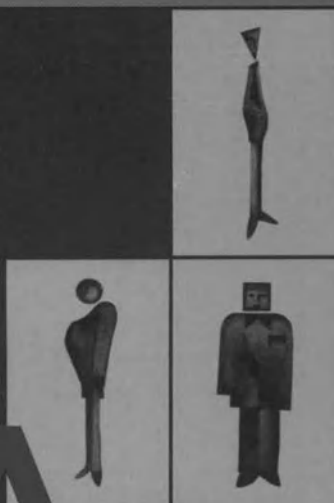
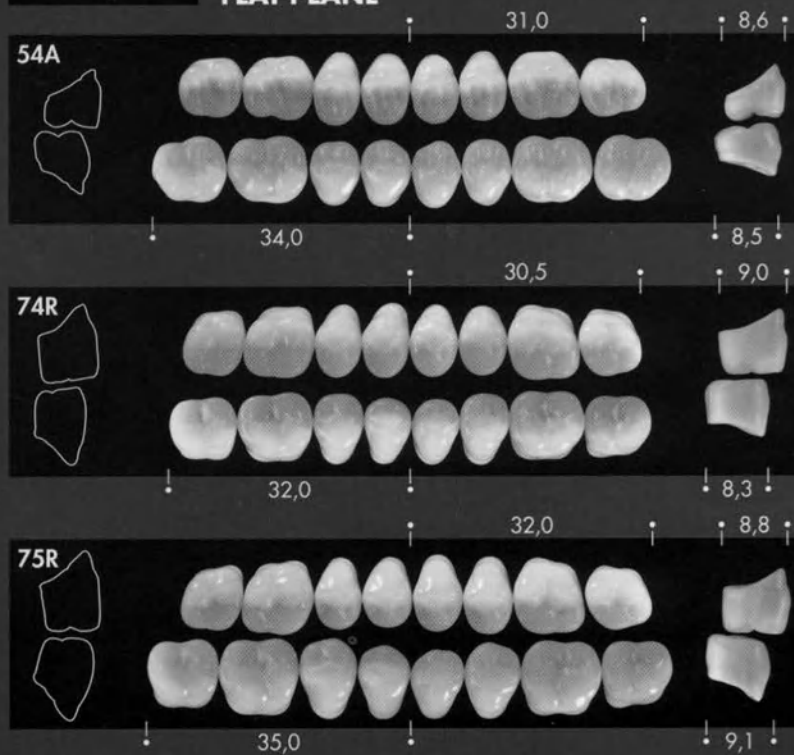
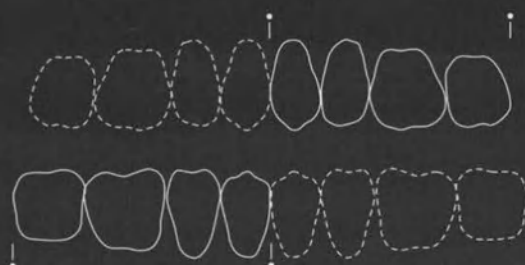
MOULD CHART

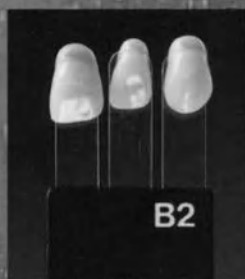MAJOR

SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESIS

GB

- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous according to Directive 67/548/EEC and subsequent amendments. Consequently, no Material Safety Data Sheet is available.
- Grinding of the teeth generates dust which is classified as "not otherwise classifiable". Do not inhale. Use appropriate protection. TLV/TWA applies.
- Dispose conforming to local law. Product and packing materials are not listed in the Hazardous Waste Inventory as per C.D. 91/689/EEC (14). Patient-contacted materials are at infection risk even for disposal purposes (EU Hazardous Waste Inventory No. 180103).
- Available in the following shade guides: A1-C3 (REF FDA106) A1-D4 (REF FDA 109) 1A-6D (REF FDA 108) 11-41 (REF FDA 107).

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