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CERTIFICATE

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of ZHEJIANG CHENGYI GLASSES CO., LTD. on the annexed DOCUMENT is genuine.

China Council for the Promotion of International Trade

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证明网站 Website for verifying the certificate: <http://www.rzcepit.com/validate.html>


"APPROVE"

Zhejiang Chenghua Glasses Co., Ltd/

Zhejiang Chengyi Glasses Co.,ltd.

General Manager

Cai Qixing/Cai Qixing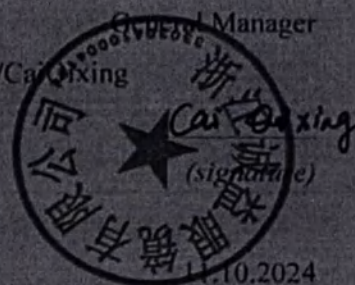

Signature date/Date

M.P. / Stamp

User manual

Frame of corrective glasseswith transport inserts

Version 2.0

2024

1. Product name

Name of the medical device -

Frames for corrective glasses with transport inserts, design options:

Frames for corrective glasses with transport inserts, design options:

1. Frame of corrective glasses with transport inserts, metal, consisting of:

-Frame size 35-60 mm with 1 mm pitch, 13-23 mm with 1 mm pitch, 95-115 mm with 2.5 mm pitch, 105-155 mm with 5 mm pitch-1 pc.;

- Transport inserts - 2 pcs.;

- Operation manual (brief) - 1 pc.

2. Frame of corrective glasses with transport inserts plastic composition:

- Frame, size 42-60 mm with a step of 1 mm, 14-20 mm with a step of 1 mm, 90-115 mm with a step of 2.5 mm, 105-155 mm with a step of 5 mm - 1 pc.;

- Transport inserts - 2 pcs.;

- Operation manual (brief) - 1 pc.

3. Frame of corrective glasses with transport inserts combined consisting of:

-Frame, size 36-60 mm with a step of 1 mm, 15-21 mm with a step of 1 mm, 110-115 mm with a step of 2.5 mm, 105-150 mm with a step of 5 mm - 1 pc.;

- Transport inserts - 2 pcs.;

- Operation manual (brief) - 1 pc.

Further in the text – Frame, spectacle frame, frame of corrective glasses.

2. Purpose of the product

The product is intended to fix corrective lenses in a given position relative to the eyes for the purpose of vision correction.

Type of contact with the body

Short-term contact (less than 24 hours) with intact skin

How to use the product

- Metal frames secure the lenses with a threaded closure.

- Plastic frames hold the lenses in place by thermal expansion and contraction as the frame cools. When the frame heats up, the light opening increases and the lens is set in place. When the frame cools down, the light opening returns to its original state and the lens is fixed in the desired position.
- Combination frames fix the lenses using thermal expansion (if the frame is plastic) or using a threaded lock (if the frame is metal).

3. Manufacturer.

Manufacturer

ZHEJIANG CHENGYI GLASSES CO., LTD. ("ZHEJIANG CHENGUA GLASSES CO., LTD"), China

Floor 11 Dormitory Building 4, Floor 8-9 Building 9, Ouhai High-Tech Glasses Industrial Park, No.32 Fuhao Road, Guoxi Street, Ouhai District Wenzhou Zhejiang China

Tel.: +86 0577 88440886; +8613587624801

E-mail: Alex@polizeneyewear.com

Place of production

ZHEJIANG CHENGYI GLASSES CO., LTD. ("ZHEJIANG CHENGUA GLASSES CO., LTD"), China

Floor 11 Dormitory Building 4, Floor 8-9 Building 9, Ouhai High-Tech Glasses Industrial Park, No.32 Fuhao Road, Guoxi Street, Ouhai District Wenzhou Zhejiang China

Authorized representative of the manufacturer in the Russian Federation:

Limited Liability Company

responsibility of "Opticus Nova"

108811, Moscow, internal territory of the city, settlement Moskovsky km, Kievskoye shosse 22-y (p. Moskovsky), dvld. 4, building 2, block G, floor 5, office 500/1G

INN 7725497583

Phone: +7 (499) 350-59-46

4. Indications and contraindications for the use of MI, possible side effects and precautions

Indications for use:

- myopia, hyperopia;
- astigmatism (loss of the ability to clearly see the surrounding world);
- strabismus;
- inflammation of the cornea of the eye (keratitis);
- non-inflammatory eye diseases (retinopathy);
- retinal detachment;
- congenital cataract;
- intraocular hemorrhage;
- amblyopia (reversible deterioration of vision associated with changes in the brain);

Frames of corrective glasses are used in ophthalmology as a device for fixing lenses in a given position.

Contraindications for use:

- individual intolerance to vision correction when wearing glasses;
- impossibility of vision correction when wearing glasses;
- allergic reactions due to individual intolerance to frame materials.

Side effects:

Allergic reactions are possible.

Warnings and precautions:

- remove and put on frames only with two hands to avoid deformation.
- the use of organic solvents (acetone, toluene, etc.) for wiping the product is not permitted.

- it is not allowed to apply significant mechanical force to the frames.
- it is not allowed to use frames in rooms with temperatures above +60°C, including in baths and saunas.
- it is not advisable to leave frames near heating devices or on the front panel of the car.

Proposed patient population

Patients in need of vision correction.

Terms of use

Frames for the production of glasses, glasses are made by specialists who have undergone the necessary training.

5. Description of the main functional elements of the MI Product design

The design of metal frames is shown in Figure 1.

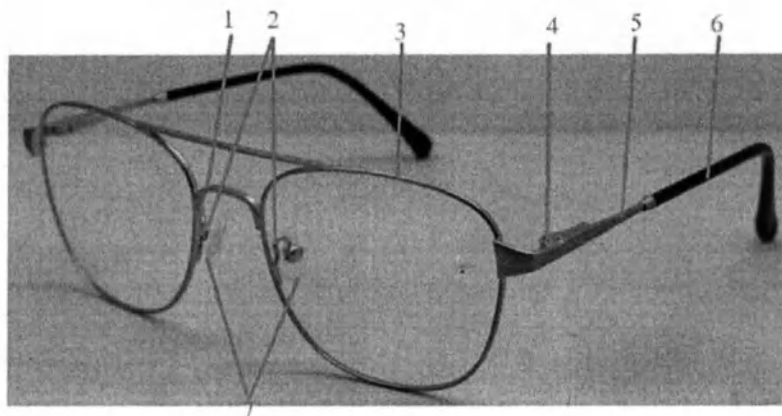
Figure 1 - Frame of corrective glasses with metal transport inserts.

1-Frame bridge, 2-Nose pad holders, 3-Frame frame, 4-Clasp or hinge, 5-Temple, 6-Temple, 7-Nose pads
The design of plastic frames is shown in Figure 2.

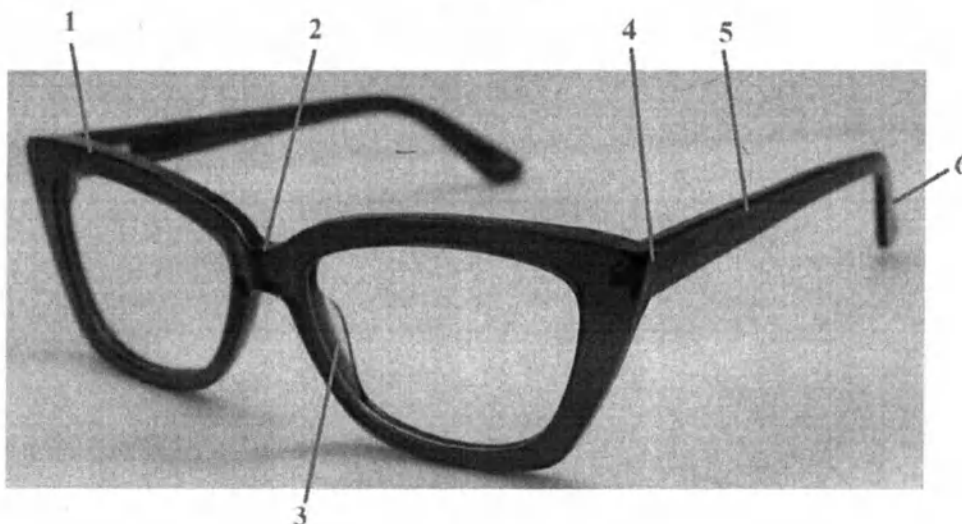
Figure 2 - Frame of corrective glasses with transport inserts, plastic

1 - Frame frame, 2 - Frame "bridge", 3 - Nose pad, 4 - Lock or hinge, 5 - Temple, 6 - Temple hook

The design of the combined frames is shown in Figure 3.

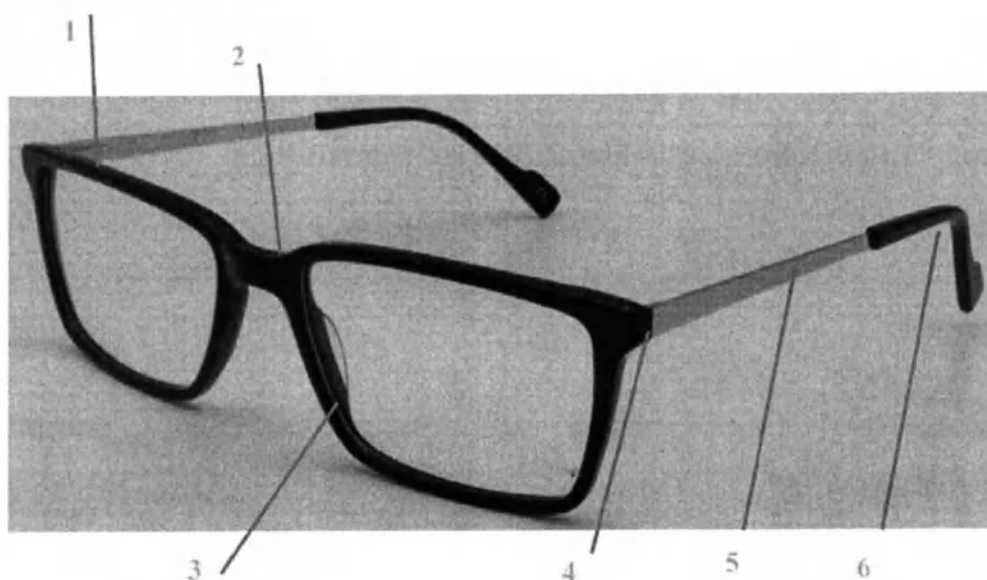
Figure 3 - Frame of corrective glasses with transport inserts, combined

1 - Frame frame, 2 - Frame "bridge", 3 - Nose pad, 4 - Lock or hinge, 5 - Temple, 6 - Temple hook

Description of the main elements:

Frame frame: the part of the frame that provides mounting and fixation of lenses in a given position.

Hinge: the part of the frame that connects the frame and the temple and allows it to move relative to the frame

Temple: the part of the frame that holds the glasses in place.

The weight of the frames is presented in table 1.

Table 1 weight of frames

Design option	Weight, g. Not more than
The frame of corrective glasses with transport inserts is made of metal.	35
The frame of corrective glasses with transport inserts is made of plastic.	29
The frame of corrective glasses with transport inserts is combined.	32

6. Technical specifications

The frames of corrective glasses are characterized by 3 basic geometric shapes parameters: width (a), bridge width (b), total temple length and temple length before bend (L1).

The dimensions of the frames and temples must correspond to those indicated in Figures 4, 5 and Table 2.

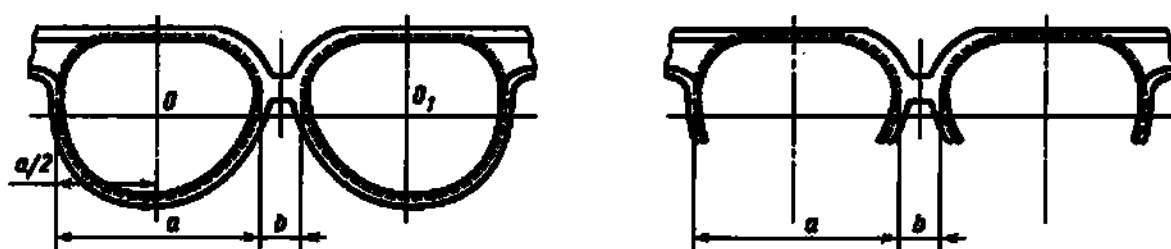
Figure 4 – Frame sizes

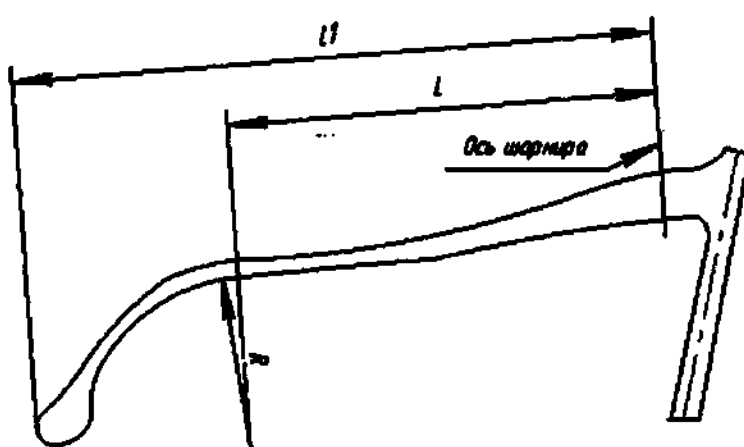
Figure 5 Temple Size

Table 2 Overall dimensions

Design option	Dimensions, mm			
	a, mm	b, mm	L ± 1	L1 ± 1
The frame of corrective glasses with transport inserts is made of metal.	35-60 in increments of 1, ±0.2	13-23 in increments of 1, ±0.2	95-115, in 2.5 increments	105-155, in increments of 5

The frame of corrective glasses with transport inserts is made of plastic.	42-60 in increments of 1, ± 0.3	14-20 in increments of 1, ± 0.3	90-115, with a step of 2.5	105-155, in increments of 5
The frame of corrective glasses with transport inserts is combined.	36-60 in increments of 1, ± 0.2	15-21 in increments of 1, ± 0.2	110-115, with a step of 2.5	105-150, in increments of 5

Dimension a: the distance between the vertical tangents to the bevel groove of the rim or to the bases of the projections in a semi-rimless frame.

Dimension b: minimum distance between vertical tangents to the base of the bevel groove of the rims or to the bases of the projections in a semi-rimless frame.

Size L: temple length from the intersection of the hinge hole axis to the center of the bend circle.

Size L1: total length of the temple from the intersection of the axis of the hinge hole to the end of the temple in the unfolded position.

The product designation when ordering and in documentation is made up of dimensions a-b-L1;

The frame of the frame must be bent in accordance with Figure 6. In this case, the OR size is selected from the range from 90 to 160 mm with maximum deviations of $\pm 10\%$ of the nominal value. It is permissible to bend each light opening and/or the entire frame along the radius of the cylinder, while the radius R_1 should be in the range from 200 to 400 mm with a maximum deviation of $\pm 10\%$ of the nominal value.

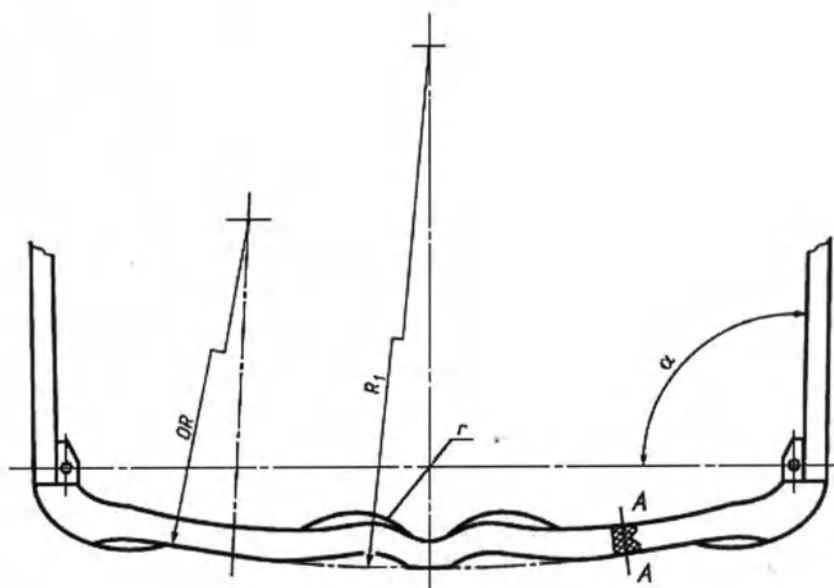
Figure 6. Frame bending.

For plastic and combined frames (Figure 7a) the angle γ and the depth S of the bevel groove should be from 80° to 120° and from 0.60 to 1.2 mm, respectively. For a metal frame (Figure 7 b), the angle γ_1 and depth S_1 The chamfer groove angle should be from 90° to 110° and from 0.30 to 1.0 mm, respectively.

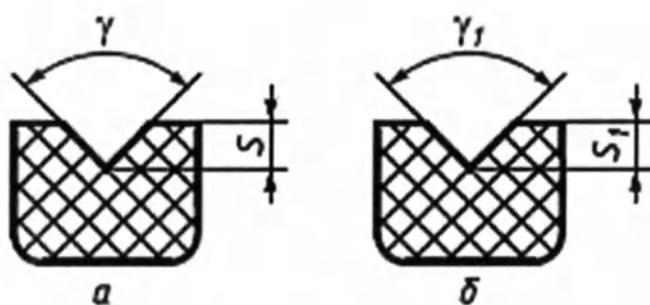
Figure 7. Angle and depth of the bevel groove

The main dimensions of the nose support surface are shown in Figure 8, and the range of numerical values for these dimensions should correspond to that given in Table 3

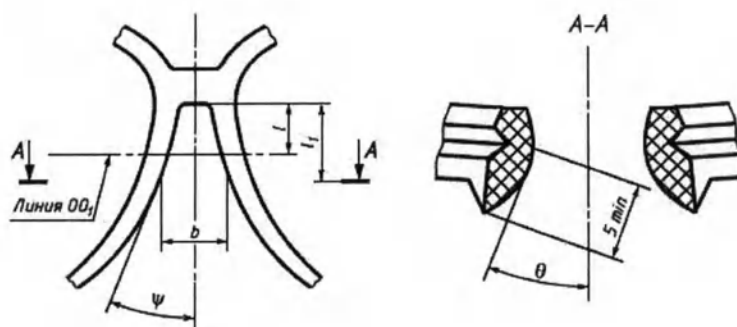
Figure 8. Main dimensions of the nose support surface

Table 3. Ranges of support value sizes

Size designation	Range of numerical values
ψ , °	From 14 to 40
θ , °	From 20 to 30
l, mm	Not less than 3
l, mm	L plus 5
b, mm	From 10 to 28

The surface of the frame, except for the facet grooves, should not have any sharp edges or burrs. The surface of the frames should not have any cavities, scratches, solder deposits, bubbles or other defects that could impair their appearance that are visible to the naked eye. The surface of the metal frame may have no more than 5 dots with a diameter of no more than 0.3 mm, and no more than 2 hairlines with a length of no more than 10 mm on the inner surface.

The connections of the frame parts must be strong and withstand the following forces:

permanent connections - (5 +/- 0.5) N (0.5 +/- 0.05) kgf;

temple tips with metal part - (2 +/- 0.2) N (0.2 +/- 0.02) kgf.

After the force is removed, the frame should not be damaged and/or deformed.

7. Data on MI labeling and packaging

Layouts of markings in Russian are shown in Figures 11-13.

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 21.11.2022

Figure 11 – Layout of individual product packaging marking in Russian

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Figure 12 – Layout of consumer packaging labeling in Russian

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Figure 13 – Layout of transport packaging marking in Russian

Marking symbols

	Store in a dry place
	European Certificate of Conformity
	Fragile
	Store upright
	Date of manufacture
	Recycling symbol
SK002	conventional designation of the coating

8. Expiry date, service life

Shelf life of frames:

plastic and combined - 5 years;

metal - 10 years;

Service life:

90% service life of frames - not less than 1.8 years with no more than 15,000 temple swings.

The total average service life of frames is at least 3 years with the number of temple swings not exceeding 30,000.

Best before date:

plastic and combined – 5 years;

metal – 10 years;

9. Cleaning/disinfection/sterilization

A. Prepare hot water of 50-60 degrees, add a little detergent to it, stir until there is more foam in the hot water;

B. Soak the frame in hot water for 10-15 minutes, then remove from hot water, wipe with a paper towel or cotton pad. Repeat the procedure.

C. Take a small amount of alcohol in a container, add the required amount of water, and then soak for 3 minutes, if there is no alcohol, you can replace it with hot water at 60 degrees. Repeat 3 times.

Disinfection should be carried out every 2 days.

Frames are supplied non-sterile, sterilization is not required.

10. Conditions of transportation, storage and operation

Transportation

The product can be transported by all types of closed transport in accordance with the transportation rules applicable to each type of transport.

Transportation conditions under the influence of climatic factors:

- ambient air temperature from -20 to +55 °C;

- relative air humidity up to 80% at a temperature of +20 °C.

During transportation and loading and unloading operations, the packaging with the product must not be subjected to sharp impacts and atmospheric precipitation. The placement and fastening of the transport container with the packaged product in vehicles must ensure a stable position and the absence of the possibility of movement during transportation.

Storage and operation

- ambient air temperature from -25 to +30 °C;
- relative air humidity from 40% to 75%.

11. Procedure and conditions for disposal of medical equipment

Dispose of the product in accordance with SanPiN 2.1.3684-21 for class A waste (epidemiologically safe waste, similar in composition to MSW).

12. Guarantee

The manufacturer guarantees that the spectacle frame meets the requirements, provided that the consumer complies with the rules of transportation, storage, and operation. The warranty does not apply if the product is used for other purposes.

13. Compliance with national standards of the Russian Federation

- GOST R ISO 15223-1-2023 "Symbols used to convey information provided by the manufacturer. Part 1. Basic requirements.";
- GOST 31589-2016 "Ophthalmological optics. Frames of corrective glasses. General technical requirements and test methods."
- GOST R 52770-2023 "Medical products. Biological action assessment system. General safety requirements."
- GOST ISO 10993-1-2021 "Medical devices. Evaluation of the biological effects of medical devices. Part 1. Evaluation and research in the risk management process";
- GOST R ISO 10993-2-2009 "Medical devices. Biological evaluation of medical devices. Part 2. Requirements for handling animals";
- GOST ISO 10993-5-2023 "Medical devices. Evaluation of the biological effect of medical devices. Part 5. Cytotoxicity tests: in vitro methods";
- GOST ISO 10993-10-2023 "Medical devices. Evaluation of the biological effect of medical devices. Part 10. Sensitization studies";
- GOST ISO 10993-12-2023 "Medical devices. Biological evaluation of medical devices. Part 12. Sampling and preparation of samples for testing"

- GOST ISO 10993-23-2023 "Medical devices. Evaluation of the biological effects of medical devices. Part 23. Irritation studies"

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