
BE IT KNOWN that I Laura Anne Pawley, Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG, a duly authorised Notary Public, duly admitted, sworn and practice throughout England and Wales.

CERTIFY ONLY that Brian Michael Howes, a United Kingdom Citizen, who is well known to me, who, is duly authorised by Jo Moore of HALEON ('the Company') to represent them in this matter, has today caused the annexed Instructions for Use for Nasal Aspirator "Otrivin Baby" with refill Nozzles for use in Russia to be produced to me and that they have represented to me on behalf of the Company that the said document is a true electronic copy of the original document.

SIGNED and sealed at Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG on the 22nd October 2024.

Protocol No: 2024-6381

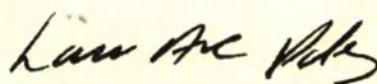Laura Anne Pawley
Notary Public

APOSTILLE (Convention de La Haye du 5 octobre 1961)	
1. Country: Pays / País:	United Kingdom of Great Britain and Northern Ireland
This public document Le présent acte public / El presente documento público	
2. Has been signed by a été signé par ha sido firmado por	Laura Anne Pawley
3. Acting in the capacity of agissant en qualité de quien actúa en calidad de	Notary Public
4. Bears the seal / stamp of est revêtu du sceau / timbre de y está revestido del sello / timbre de	The Said Notary Public
Certified Attesté / Certificado	
5. at à / en	London
6. the le / el día	24 October 2024
7. by par / por	His Majesty's Principal Secretary of State for Foreign, Commonwealth and Development Affairs
8. Number sous no / bajo el numero	APO-E5IW-GP6G-W8ZV-SJTH
9. Seal / stamp Sceau / timbre Sello / timbre	
10. Signature Signature Firma	N. Lane 

This Apostille is not to be used in the UK and only confirms the authenticity of the signature, seal or stamp on the attached UK public document. It does not confirm the authenticity of the underlying document. Apostilles attached to documents that have been photocopied and certified in the UK confirm the signature of the UK official who conducted the certification only. It does not authenticate either the signature on the original document or the contents of the original document in any way.

If this document is to be used in a country not party to the Hague Convention of the 5th of October 1961, it should be presented to the consular section of the mission representing that country.

To verify this apostille go to www.verifyapostille.service.gov.uk

HALEON

St. George's Avenue,
Weybridge, Surrey, UK
KT13 0DE
haleon.com

I, Jo Moore, of Haleon hereby certify that the attached Instruction for Use for Nasal Aspirator "Otrivin Baby" with refill Nozzles is a true electronic copy of the original document.

Name: Jo Moore

Signature:

Jo Moore

Date: 22nd October 2024

INSTRUCTION FOR USE

Nasal aspirator “Otrivin Baby” with refill nozzles

2024

A+

JMi

Name of the medical device

Nasal aspirator "Otrivin Baby" with refill nozzles

Manufacturer of the medical device

Haleon CH SARL

Route de l'Etraz

1197 Prangins

Switzerland

Site of manufacture

Stiplastics, 62 Chemin des Plantees, 38160 Saint-Marcellin, France.

Available options

1. Nasal aspirator "Otrivin Baby" with three refill nozzles (Hard Nozzles) consisting of:
 - Nasal aspirator "Otrivin Baby" – 1 pc.
 - Hard nozzle – 3 pcs.
2. Nasal aspirator "Otrivin Baby Comfort" with two refill nozzles (Soft Nozzles) consisting of:
 - Nasal aspirator "Otrivin Baby Comfort" – 1 pc.
 - Soft nozzle – 2 pcs.

Intended use of the device

Nasal aspirator "Otrivin Baby" and nasal aspirator "Otrivin Baby Comfort" are used in children (newborn babies, infants, and children) to gently remove nasal secretions.

Principle of operation

Principle of operation: Manual Aspiration/Suction

Mode of action: The compressed air generated by mouth aspiration, passing through a narrow tube, creates a vacuum which sucks the mucus from the patient's nose into the nozzle. If the mucus quantity is important, it gets absorbed in the foam present in the nozzle.

The principle of operation of aspirators is based on the removal of mucous secretions from the nasal passages of the child by blowing into air using uniform openings through the mouthpiece. Refill nozzles, which are inserted into the nasal passage of the child, contain an absorbing filter that traps mucus. Removal of nasal secretions helps to prevent further complications such as ear infections (otitis), sinus infections (sinusitis), and cough.

Nasal aspiration is recommended whenever your baby has a blocked nose, as often as necessary and especially before feeding your baby and before your baby goes to sleep.

Nasal Aspirator and Disposable Nozzle Refills clears your child's nose by gently removing nasal secretions.

The nozzle refills of Nasal Aspirator and Disposable Nozzle Refills (Hard) are designed to adjust to the child's nose, what makes it easy to install it in the child's nasal passage, eliminating injury. Each refill includes an absorbent filter to retain mucus, thus the process is completely hygienic.

Nasal Aspirator and Disposable Nozzle Refills (Soft) ergonomic shape and soft touch ensure a better handling; gentle to the baby's nose; perfect and safe adjustment to the baby's nose. Each refill includes an absorbent filter to retain mucus, thus the process is completely hygienic.

Easy and hygienic to use, Nasal Aspirator and Disposable Nozzle Refills (Hard) and Nasal Aspirator and Disposable Nozzle Refills (Soft) helps your baby to breathe, eat, and sleep normally and feel better again.

For hygienic reasons and to avoid contamination, the nozzles are for single use only. Wash your hands carefully before using "Otrivin Baby" Nasal Aspirator and ensure that the various components are properly assembled. After use, nozzle has to be discarded, the mouthpiece and your hands washed.

Indications and contraindications

Indications: stuffy and congested nose.

Contraindications: Do not use the product in case of allergy to any of the device materials.

Warnings

- Use only the manufacturer's Nasal Refills with the manufacturer's Nasal Aspirator.
- Keep out of sight and reach of children. Keep in the original pack.
- Do not use a broken or damaged product and do not use damaged nozzle or a nozzle with a missing or damaged filter.
- You should consult a doctor before using Nasal Aspirator and Disposable Nozzle Refills following an injury or recent nasal surgery.
- The product should be used on a resting subject. The mouthpiece should be held firmly between the lips.
- Do not use the product after the expiry date shown on the packaging.
- For hygienic reasons and to avoid contamination, the nozzles are for single use only.

Precautions

The aspirator should be used by the parent or caregiver an adult (parents, relatives of the child).

Nasal aspirator "Otrivin Baby" with three refill nozzles

Component parts of the aspirator include:

- Nozzle (Nasal Nozzle) (A)
- Body (Main body, Central part) (B) – allows to manipulate the device securely;
- Tube (Flexible tube) (D) – facilitates manipulating;
- Mouthpiece (Oral tip) (C).

Component parts of a refill disposable nozzle:

- an ergonomic tip allowing to penetrate into the nostril of the child;
- absorbing filter to trap mucus discharge;
- a component of transparent material – allow to control the mucus removal procedure.

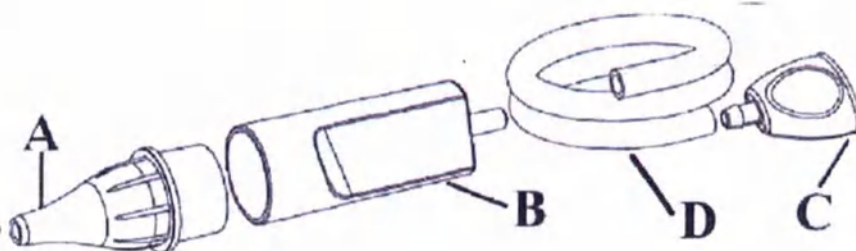
Fig. 1. Diagram of the nasal aspirator "Otrivin Baby"

Nasal aspirator "Otrivin Baby Comfort" with two refill nozzles

Component parts of the aspirator include:

- Nozzle (nasal nozzle) (A);
- Body (barrel, main body, central part) (B) – allows to manipulate the device securely;
- Stopper (E);
- Tube (flexible tube) (D) – facilitates manipulating;
- Mouthpiece (buccal device, aspiration mouthpiece) (C).

Component parts of a refill disposable nozzle:

- an ergonomic tip allowing to penetrate into the nostril of the child;
- absorbing filter to trap mucus discharge;
- a component of transparent material – allow to control the mucus removal procedure.

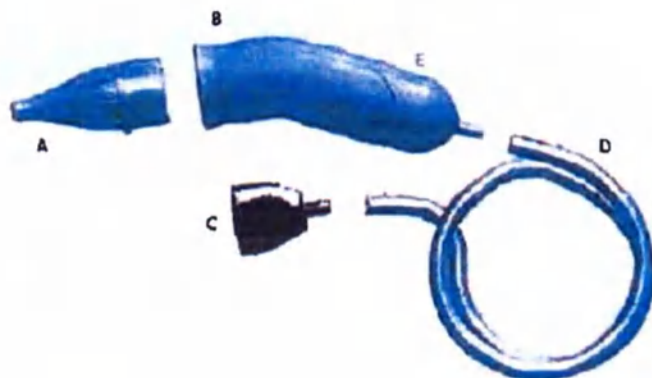
Fig. 2. Diagram of the nasal aspirator "Otrivin Baby Comfort"

Method of administration:

1. Before using Nasal Aspirator wash your hands carefully.
2. Fit a soft nozzle (A) into the central part (B) of the device. Do not assemble with a used nozzle.
3. Place the mouthpiece (C) between your lips and insert the end of the nozzle (A) to the opening of the baby's nostril.
4. Aspire gently, slowly and regularly through the mouthpiece (C).
5. If the nozzle becomes full during an aspiration, change the nozzle, do not re-use.
6. Repeat procedure with the other nostril.
7. Remove and dispose of the used nozzle (A). Wipe the outer surface of the mouthpiece (C) using a clean moistened tissue with warm water, dry it with tissue. Only in case of mucus or secretion inside the body (B), detach the silicone tubing, rinse the body with warm water and drain to dry, reattach the tube.

Device specifications

Nasal aspirator "Otrivin Baby" with three refill nozzles

Device component	Size, mm
Nozzle inlet diameter (A)	3.75 ± 0.17 mm
Nozzle outlet diameter (A)	15.55 ± 0.25 mm
Filter height in the nozzle	11 ± 1 mm
Filter diameter in the nozzle	16.00 ± 0.15 mm
Nozzle length (A)	43.77 ± 0.37 mm
Body length (B)	58 mm
Body outlet outside diameter (B)	4.30 mm
Body inlet inside diameter	18.40 ± 0.28 mm
Silicone tube length (D)	290 ± 3 mm
Silicone tube inside diameter (D)	4.10 ± 0.20 mm
Mouthpiece length (C)	$27.32 \text{ mm} \pm 0.33$ mm
Mouthpiece inside diameter (C)	2.54 ± 0.15 mm
Mouthpiece inlet outside diameter (C)	5.69 ± 0.17 mm
Weight of aspirator	$9.80 \text{ g} \pm 10\%$
Weight of refill nozzle	2.2 ± 0.5 g

Device specifications

Nasal aspirator "Otrivin Baby Comfort" with two refill nozzles

Device component	Size, mm
Nozzle inlet diameter (A)	3.75 ± 0.17 mm
Nozzle outlet diameter (A)	15.55 ± 0.25 mm
Filter height in the nozzle	11 ± 1 mm
Filter diameter in the nozzle	16.00 ± 0.15 mm
Nozzle length (A)	43.77 ± 0.37 mm
Body flexible part length (B)	64.78 ± 3.0 mm
Body flexible part outlet outside diameter (B)	22.23 ± 0.55 mm
Main body inlet inside diameter (B)	18.87 ± 0.11 mm
Stopper length (E)	27.98 mm
Stopper outlet outside diameter (E)	4.84 ± 0.50 mm
Silicone tube length (D)	290 ± 3 mm
Silicone tube inside diameter (D)	4.10 ± 0.20 mm
Mouthpiece length (C)	$27.32 \text{ mm} \pm 0.33$ mm
Mouthpiece outside diameter (C)	2.54 ± 0.15 mm
Mouthpiece inlet outside diameter (C)	5.69 ± 0.17 mm
Weight of aspirator	$18.0 \text{ g} \pm 10\%$
Weight of refill nozzle	3.5 ± 0.35 g

AF

JM

Page 6 of 10

Materials of construction

Device or part and accessory name (tested)	Materials of construction
Aspirator "Otrivin Baby" Aspirator body	Polyethylene LDPE 780E
Tube	Silicone SI 70 or R401/70S or Addisil 170E
Mouthpiece	Polypropylene MRC60MC2
Nozzle	Body: Polypropylene ISPLEN RC 590 S2M; Filter: Melamine Basotec G+
Flow pack (package for nozzles)	Polypropylene
Box	Carton
Aspirator "Otrivin Baby Comfort" Aspirator body	Polypropylene MRC60MC2; SEBS Evoprene GC5689 Light blue dye Elian SAS M145548-B32
Stopper	Polypropylene MRC60MC2
Tube	Silicone SI 70 or R401/70S or Addisil 170E
Mouthpiece	Polypropylene MRC60MC2
Nozzle	Body: Polyvinyl Chloride FV285AW2178CRY; Filter: Melamine Basotec G+
Flow pack (package for nozzles)	Polypropylene
Plastic box	Polypropylene MRC60MC2
Label	Polypropylene

The product does not connect to power sources.

Sterilization

The device is not sterile.

Cleaning

Remove and dispose of the used nozzle (A).

Wipe the outer surface of the mouthpiece (C) using a clean moistened tissue with warm water, dry it with tissue.

Only in case of mucus or secretion inside the body (B), detach the silicone tubing, rinse the body with warm water and drain to dry, reattach the tube.

Procedure and conditions for utilization/disposal of the medical device

According to SanPiN 2.1.3684-21, utilize this medical device as solid household waste. The device is intended for individual use.

Transportation

Transportation is carried out by all types of vehicles in covered vehicles with transportation rules applicable to this type of transport.

Device transportation is carried out at a temperature 0 °C to +30 °C and humidity up to 65%, non-condensing.

Storage

Store the device in its original packaging out of reach of children at a temperature between 0 °C and +30 °C and humidity of 65%.

Operation

Operate the device at a temperature between +15 °C and +42 °C.

Packaging

Nasal aspirator "Otrivin Baby" and refill nozzles are supplied in a carton box. Each nozzle is packed in a polypropylene flow pack. Carton box dimensions: 128.00 ± 0.9 mm x 39.00 ± 0.6 mm x 92.00 ± 0.9 mm. Instruction for use is provided on the carton box. Product weight with packaging: 31.16 g ± 10%.

Nasal aspirator "Otrivin Baby Comfort" and refill nozzles are supplied in a plastic box manufactured in polypropylene. Each nozzle is packed in a polypropylene flow pack. Plastic box dimensions: 128.00 ± 0.9 mm x 92.00 ± 0.6 mm x 28.60 ± 0.6 mm. Instruction for use is provided on the label of the plastic box. Product weight with packaging: 56.6 g ± 10%.

Labeling

Nasal aspirator "Otrivin Baby" with three refill nozzles

The package contains the following information:

- Name of the device (option);
- Composition of the device;
- Intended use of the device;
- Manufacturer;
- Manufacturing site;
- Details of the complaint handling organization;
- Marketing Authorization number;
- Logo of Marketing Authorization Holder;
- Storage conditions;
- Shelf life;
- Contraindications and warnings;
- Instruction for use;
- Lot number;
- Best before.

Nasal aspirator "Otrivin Baby Comfort" with two refill nozzles

The package contains the following information:

- Name of the device (option);
- Composition of the device;
- Intended use of the device;
- Manufacturer;
- Manufacturing site;
- Details of the complaint handling organization;
- Marketing Authorization number;
- Logo of Marketing Authorization Holder;
- Storage conditions;
- Shelf life;
- Contraindications and warnings;
- Instruction for use;
- Lot number;
- Best before.

Supply package

Nasal aspirator "Otrivin Baby" with three refill nozzles

Aspirator "Otrivin Baby" and three refill nozzles, packed in an individual polypropylene flow pack, are supplied in a carton box. Instruction for use is printed on the carton box.

Nasal aspirator "Otrivin Baby Comfort" with two refill nozzles

Aspirator "Otrivin Baby Comfort" and two refill nozzles, packed in an individual polypropylene flow pack, are supplied in a plastic box manufactured in polypropylene. Instruction for use is printed on the label of the plastic box.

Warranty and shelf life

The manufacturer guarantees the operability of the device throughout the expiration date, subject to the consumer observing the operating rules.

Shelf life for "Otrivin Baby" Aspirator with three refills (Hard nozzles) is 5 years.

Shelf life for "Otrivin Baby Comfort" Aspirator with two refills (Soft Nozzles) is 3 years.

Do not use after expiration date.

Complaints

In the event of complaints about the quality of the medical device or questions related to the use of the medical device, send complaints to the address:

JSC Haleon Rus

10, Presnenskaya emb., premise III, room 9, floor 6, Moscow, 123112, the Russian Federation

Tel.: +7 (495) 777-98-50

Toll-free "Hotline" telephone number

☎ 8 (800) 333-46-94

E-mail: rus.info@haleon.com

Haleon CH SARL (formerly known as GSK Consumer Healthcare SARL)

By: LM Representative
Name: Jennifer Maillard

Title: Quality Compliance Manager
Medical Devices

By: Regulatory
Name: Aurelie Farfaro

Title: Regulatory Strategy
and Innovations Lead

Date: JENNIFER MAILLARD - L'ESPEYRE AUGUSTE BLANCHARD
Signature: I am signing for device that content is aligned
with the EU Technical Document
Date: 2024-10-11 10:04:49+02:00

Date: AURELIE CLAIRE FARFARO
Signature: I am signing as a representative that this
documentation is aligned with the EU medical
device Technical file
Date: 2024-10-11 17:47:00+02:00

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