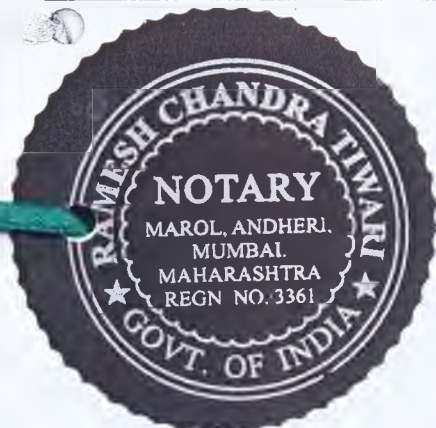
	Regulatory Submission Dossier for Russia MesireTM - Sinus Balloon Catheter	КОПИЯ Chapter 1
		Date : 18 November 2017 Page 1 of 2

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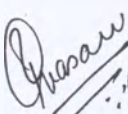
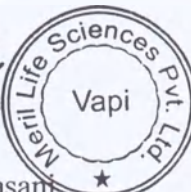
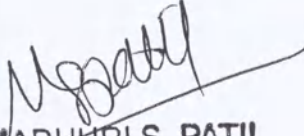
ATTESTED

Meril

Regulatory Submission Dossier for Russia
Mesire™ - Sinus Balloon Catheter

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CERTIFIED TRUE COPY

Ramesh Chandra Tiwari
RAMESHCHANDRA TIWARI
B.A. LL.B
ADVOCATE & NOTARY
GOVT. OF INDIA

Madhuri S. Patil
MADHURI S. PATIL
Deputy Secretary

22 NOV 2017

ATTESTED

 **भारत सरकार / GOVERNMENT OF INDIA**
अपोस्टिल / APOSTILLE
(Convention of La Haye du 5 octobre 1961)
INDIA

This public document of the type
COMMERCIAL DOCUMENT

is issued to **MERIL LIFE SCIENCES PVT. LTD.**

has been signed by **MADHURI S PATIL**

with the seal / stamp of **DY. SECRETARY, MAHARASHTRA
CHAMBER OF COMMERCE, INDUSTRY & AGRICULTURE**

Certified by
Section Officer (O.I.) MINISTRY OF EXTERNAL AFFAIRS
on **06 Dec 2017** at **NEW DELHI, INDIA**
with reference no. **MHMC0643611317**

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Relieva **Ultirra**TM

Sinus Balloon Catheter

- > EASY PROCEDURAL IRRIGATION
- > ENHANCED BALLOON CATHETER PERFORMANCE
- > DURABLE SINUS DILATION

Acclarent

Relieva Ultirra™ Sinus Balloon Catheter

The Relieva Ultirra Sinus Balloon Catheter combines high performance balloon dilation with intra-sinus irrigation to deliver clinically significant improvement in sinus symptoms

EASY PROCEDURAL IRRIGATION IS CONVENIENT IN EVERY CASE

- › Displace sinus contents using a combination of side and distal-tip jets
- › Increase procedural efficiency by eliminating device exchanges
- › Target irrigation by positioning the soft catheter tip deep within the sinus

ENHANCED BALLOON CATHETER PROVIDES SUPERIOR PERFORMANCE

- › Smoother passage through sinus anatomy*
- › Easier insertion and retraction through the sinus guide catheter*
- › Success through multiple sinus dilations with robust balloon catheter performance

DURABLE SINUS DILATION IN SIX BALLOON SIZES

- › Select dilation diameter and length to address specific sinus anatomy needs
- › Clinically proven to restore sinus drainage and function with minimal tissue disruption
- › Safe and effective dilation in pediatric maxillary sinuses

Catalog Number	Balloon Diameter (mm)	Balloon Length (mm)	Units (per box)
BC3512RU	3.5	12	5
BC0516RU	5.0	16	5
BC0616RU	6.0	16	5
BC0716RU	7.0	16	5
BC0524RU	5.0	24	5
BC0724RU	7.0	24	5

*As compared to Relieva Solo Pro™. Data on file.

For more information on Relieva Ultirra and Acclarent, Inc., please contact your Acclarent representative or visit www.acclarent.com.

Balloon Sinuplasty™ System is intended for use by or under the direction of a physician that is trained in the use of Balloon Sinuplasty Technology. Prior to use, it is important to read the Instructions for Use and to understand the contraindications, warnings, and precautions associated with this device.

Acclarent

1525-B O'Brien Drive
Menlo Park, CA 94025
U.S.A.

t 650.687.5888
f 650.687.5886
1 877.775.2789

© 2011 Acclarent, Inc. All Right Reserved. MKT02194 Rev. B

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