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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

02448553

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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IS TO CERTIFY THAT: the seal of TIANJIN GAPADENT LTD on the annexed DOCUMENT is genuine.

China Council for the Promotion
of International Trade

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证明书查询网址 for verifying the certificate: <http://www.rzccpit.com/validate.html>

Approved by «TIANJIN GAPADENT CO.,
LTD»

Legal Representative, President

A circular stamp with the text 'TIANJIN GAPADENT CO., LTD' and a signature across it. Below the stamp, the text 'An Ho Sang' and 'Signature and stamp' are visible.
"9" 05. 2024

INSTRUCTION FOR USE

Endocanal gutta-percha and absorbent paper points «GMG»

manufactured by TIANJIN GAPADENT CO., LTD, China

POSE

Endocanal gutta-percha and absorbent paper points «GMG» are used in dental practice for professional use in endodontic therapy for the treatment and filling of teeth, mainly for the treatment of pulpitis and diseases of the root canal tissue. Absorbent paper points are used to dry the root canals of the tooth from tissue fluid, and can also be used to apply a sealant or liquid paste (filling material) into the tooth canal to cover its walls with a thin layer during filling. Gutta-percha points are used to fill root canals when filling teeth.

The products are intended for use in dental practice in dental offices of clinics and medical institutions.

Single use.

For professional use by dentists.

2. MAIN TECHNICAL CHARACTERISTICS

Endocanal gutta-percha and absorbent paper points «GMG» are produced in the form of cones. The shape and dimensions of the points exclude the possibility of negative effects on tissues and body fluids and comply with GOST R ISO 14630-99. A wide range of precise sizes and color coding of the ends of the points are made in accordance with the ISO system. The products are supplied in plastic packages with a transparent top cover. A convenient slide cassette with a foam insert ensures safety and prevents the points from falling out. The transparent top cover of the package allows you to visually determine the color code of the products, facilitates and accelerates the identification of points by size during operation. Each package of points of the same size repeats the same color coding on the lid of the plastic tube.

I. Absorbent paper points

The products are made of cotton paper with high absorbent capacity and do not lose their shape even when completely soaked with moisture. Natural products - do not contain impurities of binders. Different size selection (see table No. 1)

Table No. 1

Sizes	Color coding and size scale	Colors	Numeric and letter size designations
Standard sizes	 15, 45, 90	White	15, 45, 90
	 20, 50, 100	Yellow	20, 50, 100
	 25, 55, 110	Red	25, 55, 110
	 30, 60, 120	Blue	30, 60, 120
	 35, 70, 130	Green	35, 70, 130
	 40, 80, 140	Black	40, 80, 140
Additional sizes	XXF	White	XX-Fine
	XF	Yellow	X-Fine
	F	Red	Fine
	M	Blue	Medium
	C	Green	Coarse
	XC	Black	X-Coarse

- The products have excellent absorption capacity
- Available in sterile packages
- The ideal density of the points ensures insertion into the channels without additional twisting and bending
- When inserted into the channel, the products do not deform
- Retain their shape when removed from the channel.

II. Gutta-percha points

The products are made of natural gutta-percha, completely biocompatible.

The size and shape of the gutta-percha points correspond to the shape and size of the tools used to clean the root canals.

The products do not have a toxic effect on the tooth tissue, do not cause irritation and can be easily removed from the canal, if necessary.

The composition of gutta-percha points (see table No. 2):

Table No. 2

Composition	% content	Conditioned load
Gutta-percha	28,4%	Creating volume, shape stability and flexibility
Zinc Oxide	59,2%	Filler, the most widely used in dental practice
Radiopaque additive (Barium sulfate) and mineral additives	12,3%	Ensuring the «insolubility» of the material in the channel under the action of apical fluid and dentine tubule fluid, as well as the possibility of mandatory monitoring in the process and upon completion of the sealing
Biological dyes and additives (coloring agent)	0,1%	Creating color coding and preventing oxidation

Gutta-percha points provide:

- ideal working properties due to its strength and flexibility
- the best plasticity for vertical and lateral condensation
- biological compatibility to the tissues of the tooth and periapical area
- high radiopacity
- high stability and smooth surface for easy insertion into the root canal of the tooth
- practicality and bio-inertia, which allow them to be a perfectly safe product
- consistency of volume
- insolubility
- bacteriostaticity and the absence of an environment for the growth of pathogenic microorganisms
- avoiding staining of hard tooth tissues
- easy extraction from the root canal if necessary
- ease of use by medical staff
- avoiding irritation of the periapical area tissues
- ease of insertion into the canal filling the canal laterally and vertically
- resistance to shrinkage
- large selection of sizes (see table No. 3)

Table No. 3

Sizes	Color coding and size scale	Colors	Numeric and letter size designations
Standard sizes	 15, 45, 90	White	15, 45, 90
	 20, 50, 100	Yellow	20, 50, 100
	 25, 55, 110	Red	25, 55, 110
	 30, 60, 120	Blue	30, 60, 120
	 35, 70, 130	Green	35, 70, 130
	 40, 80, 140	Black	40, 80, 140
Additional sizes	XF	White	X-Fine
	FF	Yellow	Fine-Fine
	MF	Red	Medium-Fine
	F	Blue	Fine
	FM	Green	Fine-Medium
	M	Black	Medium
	ML	Beige	Medium-Large
	L	Bright yellow	Large
	XL	Bright red	X-Large

3. CONTRAINDICATIONS TO USE

Missing.

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The product is intended for single use.

Do not reuse it!

For use by qualified dentists only!

4. THE ORDER OF OPERATION OF THE PRODUCT

Drill a hole in the tooth to provide direct access to the root canal.

After mechanical and medical preparation and treatment, dry the canal with paper points:

Insert paper points of the appropriate size, depending on the anatomical and topographic features of the root canal, for the entire working length, so that they absorb tissue fluid until the canal is completely dried.

After use, the products are removed and disposed of using disinfectants or recycling containers, according to the protocol adopted at the clinic.

Absorbent paper points can also be used to apply a sealant or liquid paste (filling material) into the tooth canal to cover its walls with a thin layer during sealing.

When the channel is completely dried, fill it with gutta-percha points of suitable size:

Place a gutta-percha point in the prepared root canal, corresponding to its taper and the size of the apical opening. The point should reach a narrowing and stop advancing.

The sealed channel can be filled with gutta-percha points in two ways. In the first case, the gutta-percha point is inserted after filling the channel with paste (sealant), in this case the paste is the basis, and the point ensures the reliability of filling. In the second variant, the lateral condensation method is used, in which the gutta-percha points are coated with paste and inserted into the root canals of the tooth - in this case, the basis is gutta-percha, and the filler in the form of paste provides filling density and adhesion of the root seal to the root dentin.

X-ray inspection. In the picture, the point should be positioned 1 mm short of the top of the root.
at the operation until complete obturation of the canal is achieved. To obtain a good sealing, several
ones are inserted into the root canals.

Cut off the points protruding from the mouth of the channel with a heated tool.

Use a plugger to condense the points into the mouth.

Perform a control X-ray.

For single use.

The products are intended for use in dental practice in dental offices of clinics and medical institutions.

For use by qualified dentists only!

5. RULES OF STORAGE AND USE

- Store in containers with a tightly closed lid
- Avoid moisture and direct sunlight
- Keep away from heat sources, at room temperature
- The shelf life of the product under recommended storage conditions is 4 years (the series and date of manufacture are indicated on the bottom of the container)


Disposal: together with other consumables, according to the protocol adopted at the clinic.

All products of **TIANJIN GAPADENT CO., LTD, China**, comply with international safety and quality standards, are manufactured at production facilities certified according to **ISO 13485:2016**. The product has a Free-sale Certificate, a Declaration of Conformity with the conditions of Directive 93/42/EEC on medical devices, an **EU Certificate** in accordance with Annex V of **Directive 93/42/EEC** on medical devices.

NAME AND LEGAL ADDRESS OF THE MANUFACTURER'S ORGANIZATION:

Manufacturer: TIANJIN GAPADENT CO., LTD, NO.24, FengZe Road, (ZhangJiaWo Industrial Area) Automotive Industrial Area, XiQing District, Tianjin, P. R. China
Tel.: +86 22 8799 3921 Fax: +86 22 8799 9519
www.gapadent.com, gapadent@gapadent.com

Distributor: «West» LLC, Russia,
195112, Saint Petersburg, Utkin Ave., H. 15, Liter E, Room 2-N Office 12
Tel.: +7 (812) 317-17-00
w-stom.ru, info@w-stom.ru

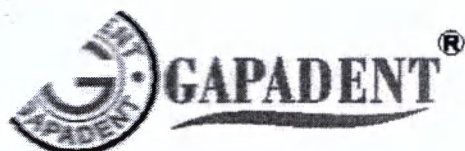www.gapadent.com

WHEN QUALITY, PRICE AND AVAILABILITY
THEY MATTER TO YOU!
GUARANTEED

GUTTA-PERCHA POINTS

Gutta Percha Points


Standardized according to ISO (ISO Type)

- Time saving, ease and speed of obturation
- High stability and smooth surface
- for easy insertion into the root canal of the tooth
- Color coding that exactly matches ISO
- High radiopacity
- Large selection of sizes

Sizes: 8, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Assortment: 15-40; 45-80; 15-80; 90-140
Taper: 02, 04, 06, 08, 10, 12
Additional sizes: XF, FF, MF, F, FM, M, ML, L, XL
Assortment: XF-M

ABSORBENT PAPER POINTS


Absorbent Paper Points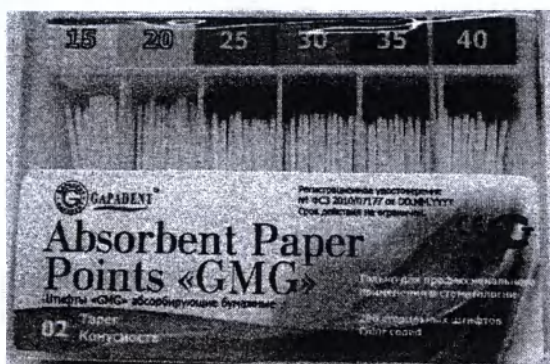

Standardized according to ISO (ISO Type)

- Excellent absorption capacity
- When inserted into the channel, they do not deform
- Ideal density for insertion into channels without additional twisting and bending
- Retain their shape when removed from the channel
- Different size selection

Sizes: 8, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Assortment: 15-40; 45-80; 15-80; 90-140
Taper: 02, 04, 06, 08, 10, 12
Additional sizes: XXF, XF, F, M, C, XC
Assortment: XXF-XC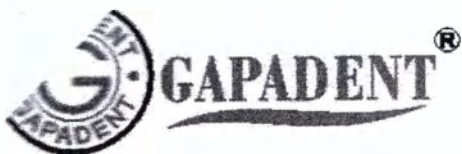

www.gapadent.com

WHEN QUALITY, PRICE AND AVAILABILITY
THEY MATTER TO YOU!
GUARANTEED


GUTTA-PERCHA POINTS Gutta Percha Points

SIZES: ISO Standardized (ISO Type):

№ 8; 10; 15; 20; 25; 30; 35; 40; 45; 50; 55; 60; 70; 80; 90; 100; 110; 120; 130; 140;

Assortment 15-40; Assortment 45-80


#8	○	#55	●
#10	●	#60	●
#15	○	#70	●
#20	●	#80	●
#25	●	#90	○
#30	●	#100	●
#35	●	#110	●
#40	●	#120	●
#45	○	#130	●
#50	●	#140	●

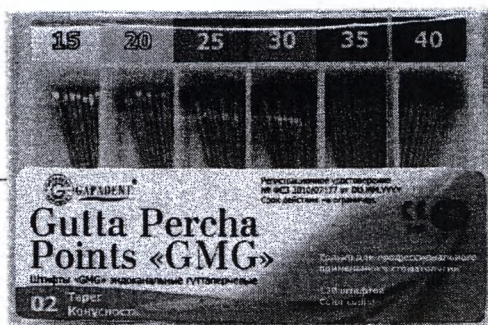Standard sizes

15, 45, 90
20, 50, 100
25, 55, 110
30, 60, 120
35, 70, 130
40, 80, 140

- High radiopacity
- High stability and smooth surface for easy insertion into the root canal of the tooth
- Color-coded exactly according to ISO
- Large selection of sizes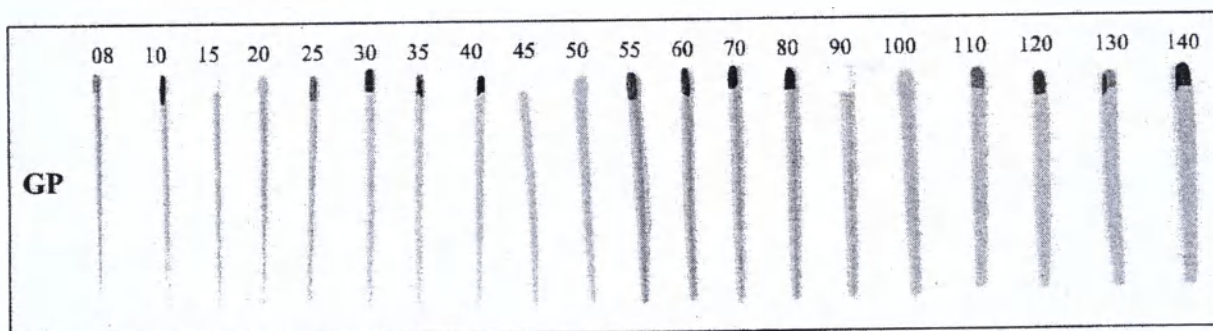

Additional sizes:


XF FF MF F FM M ML L XL

CE 0197

ISO 9001

ISO 13485

FDA Registration No. 3004580125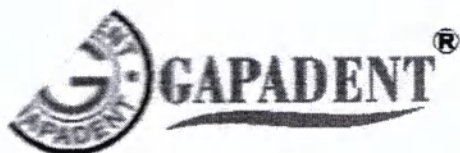

www.gapadent.com

WHEN QUALITY, PRICE AND AVAILABILITY
THEY MATTER TO YOU!
GUARANTEED

ABSORBENT PAPER POINTS

Absorbent Paper Points


Made of high absorbent paper

SIZES: ISO Standardized (ISO Type):

№ 8; 10; 15; 20; 25; 30; 35; 40; 45; 50; 55; 60; 70; 80; 90; 100; 110; 120; 130; 140;

Assortment 15-40; Assortment 45-80



#8 ----- ○	#55----- ●
#10----- ●	#60----- ●
#15----- ○	#70----- ●
#20----- ●	#80----- ●
#25----- ●	#90----- ○
#30----- ●	#100----- ●
#35----- ●	#110----- ●
#40----- ●	#120----- ●
#45----- ○	#130----- ●
#50----- ●	#140----- ●

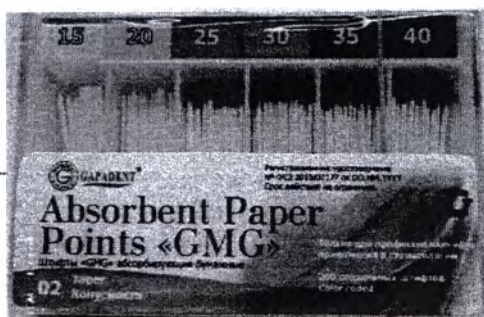Standard sizes

15, 45, 90
20, 50, 100
25, 55, 110
30, 60, 120
35, 70, 130
40, 80, 140

- When inserted into the channel, they do not deform
- Color-coded exactly according to ISO
- Large selection of sizes

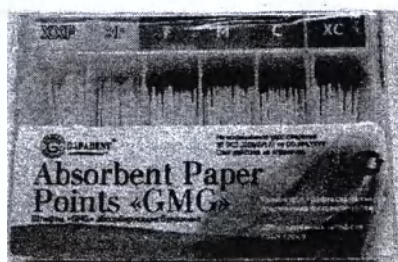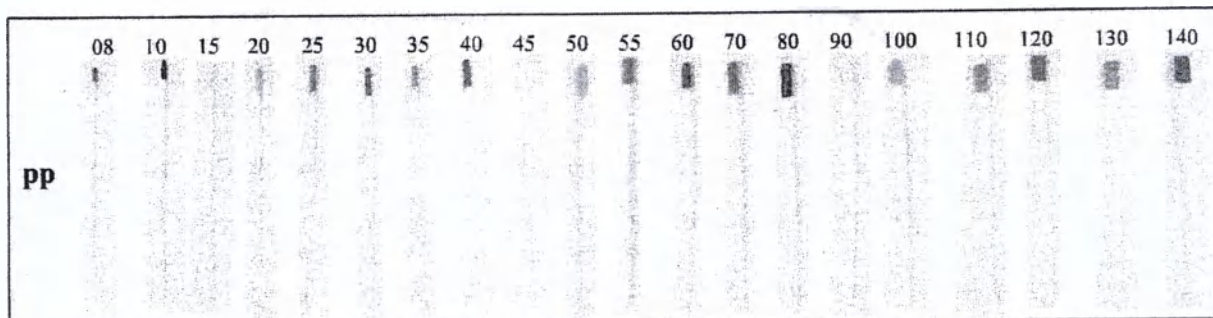Additional sizes:

XXF XF F M C XC

CE 0197

ISO 9001

ISO 13485

FDA Registration No. 3004580125

www.gapadent.com

WHEN QUALITY, PRICE AND AVAILABILITY
THEY MATTER TO YOU!
GUARANTEED

GUTTA-PERCHA POINTS Gutta Percha Points

SIZES: ISO Standardized (ISO Type):

G 20 - there are 120 gutta-percha points in this package

The size and color coding of the tips are made in accordance with the ISO system

Made of natural gutta-percha, completely biocompatible.

They do not have a toxic effect on tooth tissue, do not cause irritation and can be easily removed from the canal, if necessary

They have good plasticity for all types of condensation

Taper	02, 04, 06, 08, 10, 12
Sizes	8, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Assortment	15-40; 45-80; 15-80; 90-140
Additional sizes	XF, FF, MF, F, FM, M, ML, L, XL
Assortment	XF-M


www.gapadent.com

WHEN QUALITY, PRICE AND AVAILABILITY
THEY MATTER TO YOU!
GUARANTEED

ABSORBENT PAPER POINTS

Absorbent Paper Points

SIZES: ISO Standardized (ISO Type):

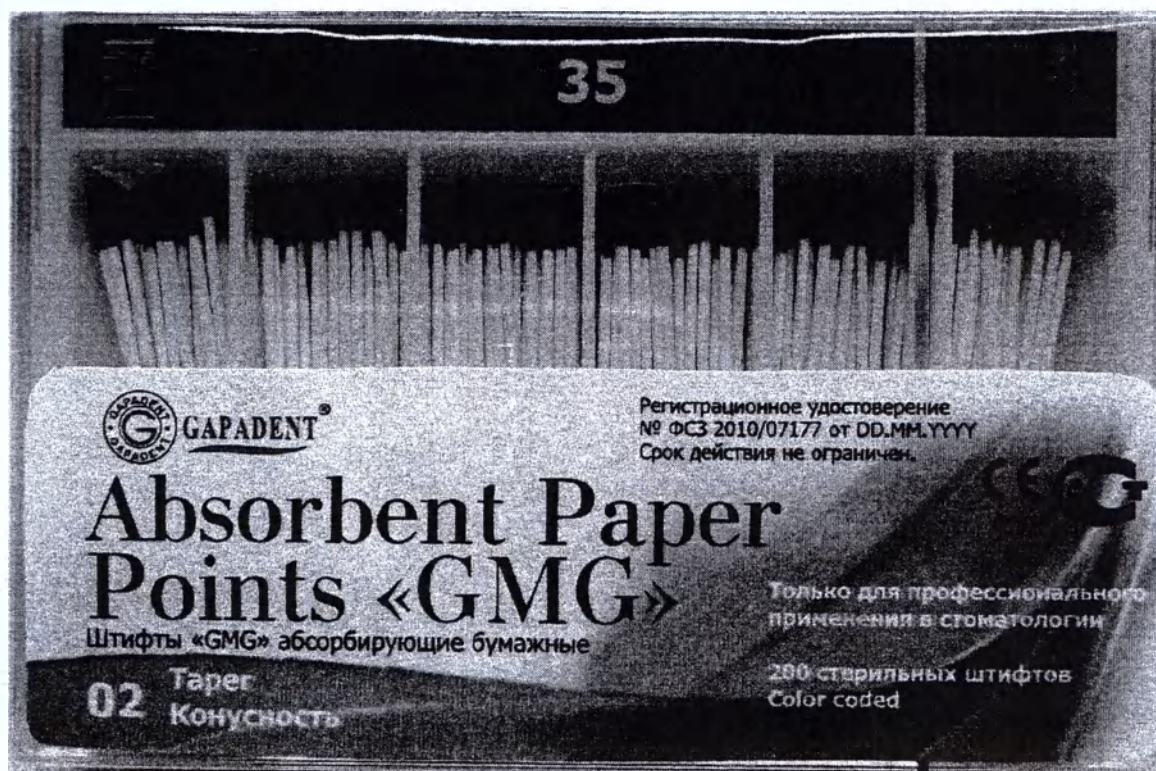G 20 - there are 200 absorbent paper points in this package

The size and color coding of the tips are made in accordance with the ISO system

Made of high absorbent paper

Natural - do not contain impurities of binders

Ideal density for insertion into channels without additional twisting and bending

They retain their shape when they are removed from the channel.

Taper	02, 04, 06, 08, 10, 12
Sizes	8, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Assortment	15-40; 45-80; 15-80; 90-140
Additional sizes	XXF, XF, F, M, C, XC
Assortment	XXF-XC

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