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Registered No. 2024-2855

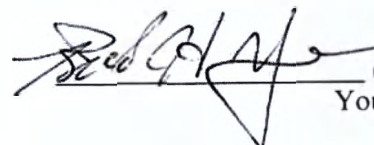
NOTARIAL CERTIFICATE

Seowon Law & Notary Office Inc.

84, Wonheong-ro, Seowon-gu, Cheongju, Korea

Date: 28th of August, 2024
«Approved»

DiaDent®
DiaDent Group International
16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu,
Cheongju-si, Chungcheongbuk-do, 28161, Korea
Tel: 82-43-266-2315 Fax: 82-43-262-8658

(signature)
You Jae Hoon
CEO

DiaDent Group International

**INSTRUCTION FOR USE
FOR THE MEDICAL DEVICE**

Root canal temporary filling dental material Healtex® in kits

The attached document is provided to DiaDent Group International. This document certifies that the contents of attached true and is confirmed by DiaDent Group International.

Version 2

1. Device name

Root canal temporary filling dental material Healtex® in kits

1. HealPaste Regular Kit Type A consisting of:
 - dental material Healtex HealPaste in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 20 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
2. HealPaste Regular Kit Type B consisting of:
 - dental material Healtex® HealPaste in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 10 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
3. HealPaste Regular Kit Type C consisting of:
 - dental material Healtex® HealPaste in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 4 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
4. HealPaste Intro Kit consisting of:
 - dental material Healtex® HealPaste in syringes 0.5 g - 1 pc
 - disposable tips for syringes HP002 - 4 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
5. HealApex Regular Kit Type A consisting of:
 - dental material Healtex® HealApex in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 20 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
6. HealApex Regular Kit Type B consisting of:
 - dental material Healtex® HealApex in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 10 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
7. HealApex Regular Kit Type C consisting of:
 - dental material Healtex® HealApex in syringes 2 g - 1 pc
 - disposable tips for syringes HP002 - 4 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc
8. HealApex Intro Kit consisting of:
 - dental material Healtex® HealApex in syringes 0.5 g - 1 pc
 - disposable tips for syringes HP002 - 4 pcs
 - silicone cap - 1 pc
 - instruction leaflet - 1 pc

Hereinafter – Root canal temporary filling dental material Healtex® in kits, dental material, kits, medical device, device.

2. Information about the manufacturer, developer and authorized representative of the manufacturer

Manufacturer

DiaDent Group International

Legal address: 16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, Republic of Korea

Tel. +82-43-266-2315

E-mail: diadent@diadent.co.kr

Developer

DiaDent Group International

Legal address: 16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, Republic of Korea

Tel. +82-43-266-2315

E-mail: diadent@diadent.co.kr

Authorized representative of the manufacturer of the medical device

“PROTECO” JSC

Legal address: 57 Salova st., build. 1, lit. I, 192102, St. Petersburg

Tel./fax: +7 (812) 779-30-90

E-mail: info@protecodent.ru

3. Intended use of the device, indicating the potential consumer and the principle of operation.

3.1 Intended use of the medical device

It is used for temporary obturation of the root canals of teeth.

3.2 Potential consumers of the medical device

Dentists

3.3 Scope

Dentistry

4. Device description

4.1 HealPaste Regular Kit Type A, HealPaste Regular Kit Type B, HealPaste Regular Kit Type C, HealPaste Intro Kit

The kits consist of one syringe (with a protective cap) with dental material Healtex® HealPaste (the weight of material in the syringe depends on the version of the kit), disposable tips HP002 (the number of tips depends on the version of the kit), a silicone cap and an instruction leaflet.

Dental material Healtex® HealPaste is a radiopaque non-hardening material, a white paste based on calcium hydroxide with barium sulfate, which is used as a root canal temporary filling dental material during resection or treatment of affected root canals.

The material does not harden and can be removed at the next doctor's visit using sodium hypochlorite.

HP002 Disposable Syringe Tips are designed to optimize material dosing and application.

The silicone cap is designed to prevent contact of the material with oxygen and moisture in the air.

4.2 HealApex Regular Kit Type A, HealApex Regular Kit Type B, HealApex Regular Kit Type C, HealApex Intro Kit

The kits consist of one syringe (with a protective cap) with dental material Healtex® HealApex (the weight of material in the syringe depends on the version of the kit), disposable tips HP002 (the number of tips depends on the version of the kit), a silicone cap and an instruction leaflet.

Dental material Healtex® HealApex is a fat-soluble non-hardening material, which is a yellow paste based on calcium hydroxide with iodoform, used as a root canal temporary filling dental material.

The addition of iodoform to calcium hydroxide improves radiopacity.

The material does not harden and can be removed at the next doctor's visit using sodium hypochlorite.

HP002 Disposable Syringe Tips are designed to optimize material dosing and application.

The silicone cap is designed to prevent contact of the material with oxygen and moisture in the air.

5. Indications, contraindications, and potential adverse reactions

Indications

Dental material Healtex® HealPaste

- Apexification and hard tissue formation
- Temporary filling of root canals
- Filling after removal of the crown pulp of the tooth
- Temporary pulp closure

Dental material Healtex® HealApex

- Apexification and hard tissue formation
- Temporary filling of root canals
- Root perforations
- Root resorption

Contraindications to the use of a medical device

Patients' intolerance to the components of raw materials of a medical device.

Potential adverse reactions

Allergic reaction to the components of the devices.

The material may cause hypersensitivity reactions very rarely. In such cases, discontinue use and consult a doctor.

6. Precautions

1. Use by non-professionals is strictly prohibited. The device is intended for dental use only.
2. Check the device for leaks, discoloration, improper packaging and expiration date.
3. Read the precautions, side effects, contraindications and instructions for use.
4. Do not use the device for purposes other than the intended use.
5. Do not use on patients with inflammation or allergy to any component of the device.
6. Avoid contact with skin and eyes, in case of contact, rinse immediately with running water.
7. The device is not intended for internal use.
8. Not suitable for permanent root canal filling.
9. Do not squeeze out the paste with excessive force, as the patient may feel a sharp pain.
10. Do not use after the expiration date.
11. Avoid contact with water.
12. Avoid removing excess material beyond the apex.
13. Do not heat the material.
14. Do not mix with other materials.

7. Application procedure**Dental materials Healtex® HealPaste and Healtex® HealApex**

1. Make sure that the syringe and the disposable tip are not damaged before use (for example, chips, cracks, deformation and/or other defects).
2. Protect the mouth with a rubber dam.
3. Rinse and dry the root canal after preparation.
4. Remove the protective cap and attach the disposable tip.
5. Insert the tip of the syringe tip, retreating from the top of the tooth root 1/5 of the length of the root canal.
6. Push the plunger slowly and gently to insert material into the canal. Withdraw the tip slowly when the paste reaches the periapical tissues. The paste should move back into the cavity of the pulp of the tooth.
7. Remove excess material from the root canal and proceed with the restoration procedure.
8. Remove the disposable tip from the syringe after use and attach a new disposable tip. Close the new disposable tip tightly with the included silicone cap.
9. Take an x-ray after the procedure to check the filling.

Removal.

1. Select an appropriately sized file to be inserted into the canal to spray the paste and secure the canal.
2. Rinse with 5.25 % sodium hypochlorite solution.

3. Irrigate the canal with an ultrasonic irrigation tip (preferably nickel-titanium alloy) for at least one minute (recommended).
4. Repeat the previous step at least 3 times to ensure complete removal of paste residue. Then dry the canal with paper points.

8. Specifications

Parameter	Value
Healtex® HealPaste	
Weight in syringe	
HealPaste Regular Kit Type A	2 g ± 5 %
HealPaste Regular Kit Type B	2 g ± 5 %
HealPaste Regular Kit Type C	2 g ± 5 %
HealPaste Intro Kit	0.5 g ± 5 %
Consistency	Homogeneous, without foreign matter
Flow property	Not less than 17 mm
Film thickness	Less than 50 µm
Radiopacity	Equivalent to a layer of aluminum with a thickness of more than 3 mm
Healtex® HealApex	
Weight in syringe	
HealApex Regular Kit Type A	2 g ± 5 %
HealApex Regular Kit Type B	2 g ± 5 %
HealApex Regular Kit Type C	2 g ± 5 %
HealApex Intro Kit	0.5 g ± 5 %
Consistency	Homogeneous, without foreign matter
Flow property	Not less than 17 mm
Film thickness	Less than 50 µm
Radiopacity	Equivalent to a layer of aluminum with a thickness of more than 3 mm

Type of the contact of the medical device - long term contact, (duration of the contact is less than 30 days), contact with a dentine

9. Device basic materials

Dental material Healtex® HealPaste

- Tween 80
- Polyether polyol
- Water
- Precipitated barium sulfate
- Zinc oxide
- Calcium hydroxide
- Titanium dioxide pigment

Dental material Healtex® HealApex

- Iodoform
- Calcium hydroxide
- Croscarmellose sodium
- Olive oil
- DOW CORNING(R) 360 MEDICAL FLUID, 350 CST.
- DOW CORNING(R) 360 MEDICAL FLUID, 1 000 CST.

10. Labeling designations and symbols

	Manufacturer		Temperature limit
	Date of manufacture		Refer to instructions for use or to electronic instructions for use
	Use before		Caution!
	Lot number		Keep away from moisture
	Catalog number		Keep away from sunlight
	Top		Limit on the number of tiers in a stack
	Fragile, handle with care		Do not use hooks/Do not pierce

11. Maintenance and repair

The device is disposable. Maintenance and repair are not required.

12. Transport and storage conditions

Storage. Keep away from direct sunlight, excessive heat and high humidity. Store in a dry, cool place tightly closed.

Transportation. Transportation of devices packed in a shipping container should be carried out by all types of covered vehicles, observing the precautions indicated on the shipping container and in accordance with the rules for the carriage of goods in force for this type of transport.

Storage, transportation and operation conditions indoors:

- Temperature: from 2 to 27 °C
- Relative humidity: no special requirements
- Atmospheric pressure: no special requirements.

13. Sterilization, disinfection

Medical devices "Root canal temporary filling dental material Healtex® in kits" are non-sterile, designed for single use. Sterilization and disinfection are not provided.

14. Disposal and/or destruction

Medical devices "Root canal temporary filling dental material Healtex® in kits " after their intended use are classified as epidemiologically hazardous waste (devices contaminated with blood and/or other biological fluids of patients).

In this case, the medical organization only collects and temporarily stores the above waste. Collection is carried out in single-use bags or containers with color coding accepted for hazardous medical waste in a room or in places for temporary storage of medical waste until further transportation to a disposal site.

Transportation of medical waste from medical institutions, their disposal are carried out by specialized organizations licensed to handle epidemiologically hazardous medical waste.

Consumer packaging, defective devices and unused devices with an expired shelf life are epidemiologically safe waste (non-hazardous waste that does not have contact with the biological fluids of patients), close in composition to municipal solid waste (MSW). For safe disposal of curable device residues,

unused material should be removed from the syringe and cured before being disposed of in accordance with local regulations.

Non-hazardous wastes are collected in reusable containers or disposable bags (except for hazardous waste bags). The procedure for collection and storage of non-hazardous waste is determined in accordance with the waste management scheme in each specific organization.

Transportation and disposal are carried out taking into account the requirements adopted for a given territory, in accordance with the sanitary legislation of a particular country for the maintenance of populated areas and the handling of manufacture and consumption waste.

15. Shelf life

HealPaste Regular Kit Type A, HealPaste Regular Kit Type B, HealPaste Regular Kit Type C, HealPaste Intro Kit, HealApex Regular Kit Type A, HealApex Regular Kit Type B, HealApex Regular Kit Type C, HealApex Intro Kit - 3 years from the date of manufacture, subject to storage conditions.

16. List of international standards that the medical device complies with

MDD 93/42/EEC, MEDDEV 2.4/1, EN ISO 13485, 21 CFR Part 820, SOR/98-282, EN ISO 14971, ISO 10993-1, EN ISO 10993-3, EN ISO 10993-5, ISO 10993-10, ISO 10993-11, ISO 7405, ISO 6876, IEC 62366-1, EN ISO 15223-1, EN 1041, EN 1641, MEDDEV 2.7/1, MEDDEV 2.12/1, MEDDEV 2.12/2.

Reference standards are used in the latest edition: the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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HA, JUNG MIN, attorney-in-fact of DIADENT GROUP INTERNATIONAL CEO JAE HOON YOU, appeared before me and admitted said principal's subscription to the attached INSTRUCTION FOR USE FOR THE MEDICAL DEVICE.

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This is hereby attested on this 26th day of September, 2024 at this office.

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Name of the office
Seowon Law & Notary Office Inc.
Belong to
Cheongju District Prosecutors' Office
Address of the office
84, Wonheong-ro, Seowon-gu, Cheongju, Korea

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PARK Jang Wook

(Signature of Notary Public)

This office has been authorized by the Minister of Justice, the Republic of Korea, to act as Notary Public since FEB, 7. 2020 under Law No. 173.

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