SEELER & DR. FELLMETH

NOTARE

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NOTARE SEELER & DR. FELLMETH · Q7, 23 · 68161 MANNHEIM

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(Instruction for use for medical device)

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Approved by
Roche Diabetes Care GmbH
Date: 28 August 2023


Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim
Germany

(Signature)

Guo-Xi Yang.

Senior Manager Regulatory Affairs

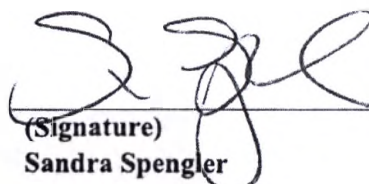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











In US distributed by:
Roche Diabetes Care Inc.
Indianapolis, IN 46256, USA
Toll Free 1-800-688-4578

 Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-check.com

CE 0123

04949048001/M Last update: 2022-03

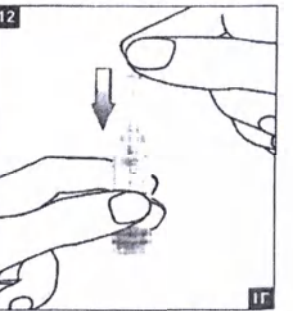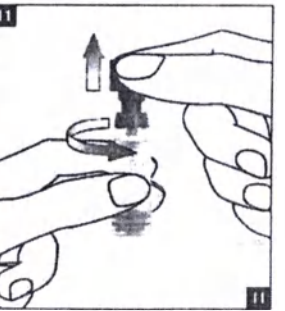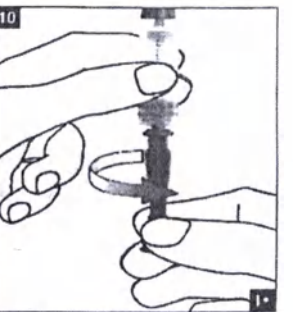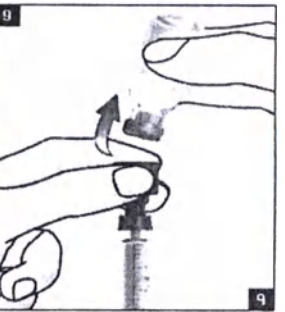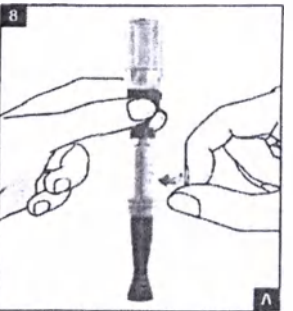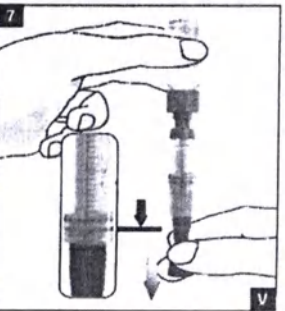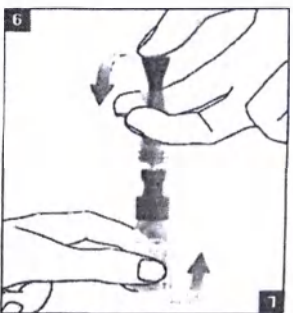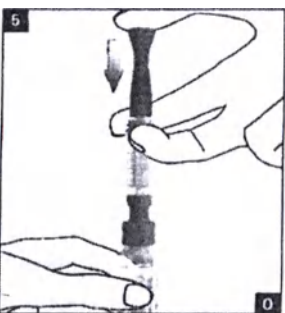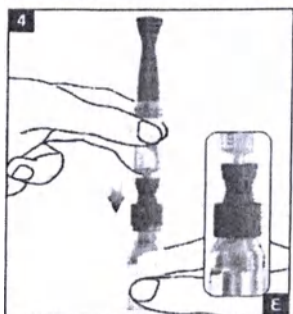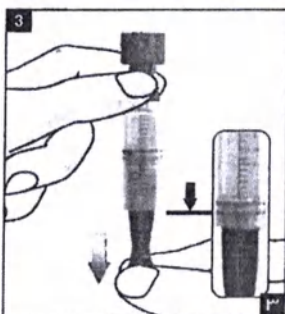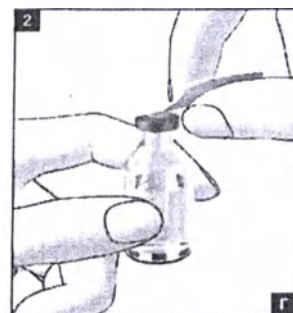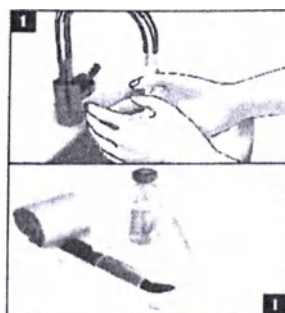EN Instructions for Use

Carefully read these instructions for use, the User's Manual of your insulin pump and the instructions for use of the infusion set before using the cartridge system.

The cartridge is intended to be used as an insulin reservoir in compatible Accu-Chek insulin pumps. Compatible insulin pumps are the Accu-Chek Spirit and the Accu-Chek Spirit Combo insulin pumps used together with their compatible infusion sets.

- For use in the therapy of diabetes mellitus.
- For use by people with diabetes and healthcare professionals or caregivers who assist people with diabetes.
- No clinical experience for the use with neonates.
- For use with insulin or insulin analogs.
- For single use only.
- Sterile product sterilized with ethylene oxide.

The cartridge system consists of a cartridge and a pre-mounted filling aid. The filling aid is used to fill insulin from commercially available 10 mL insulin vials into the cartridge.

⚠ WARNING

Risk of suffocation

This product contains small parts that can be swallowed. Keep the small parts away from small children and people who might swallow small parts.

Risk of infection

Visually inspect the sterile packaging. If the packaging is damaged or already open, do not use the product. The product can be unsterile. Do not use products that have passed their use by date. Expired products can be unsterile.

Operating conditions: 5–40 °C (41–104 °F), 20–90 % relative humidity

Storage conditions: 5–55 °C (41–131 °F), 5–85 % relative humidity, undamaged blister, dry and away from sunlight
Discard the used product according to local regulations. For information on how to discard the product correctly, contact your local council or authority.

Reporting of serious incidents: see the instructions for use of the infusion set.

In US, distributed by:

Roche Diabetes Care, Inc.
Indianapolis, IN 46256, USA
Accu-Chek Customer Care: 1-800-688-4578
www.accu-chek.com

Distributed in the United Kingdom by:

Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, United Kingdom
Accu-Chek Pump Careline ¹⁾:

UK Freephone number: 0800 731 22 91

ROI Freephone number: 1 800 88 23 51

¹⁾ calls may be recorded for training purposes
Some mobile operators may charge for calls to these numbers.

burgesshill.insulinpumps@roche.com
www.accu-chek.co.uk
www.accu-chek.ie

Australia

Roche Diabetes Care Australia Pty. Limited
Insulin Pump Support: 1800 633 457
australia.insulinpumps@roche.com
www.accu-chek.com.au

Roche Diabetes Care South Africa (Pty) Ltd.
Hertford Office Park, 90 Bekker Road
Vorna Valley, 1686

South Africa

Tel: +27 (11) 504 4600
Email: info@accu-chek.co.za
www.rochediabetescaremea.com

Canada

Accu-Chek Pump Support: 1-800-688-4578 (toll free)
www.accu-chek.ca

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© 2022 Roche Diabetes Care / © 2022 Рош Діабетес Кеа
ACCU-CHEK, ACCU-CHEK SPIRIT, ACCU-CHEK SPIRIT COMBO and АККУ-ЧЕК are trademarks of Roche.