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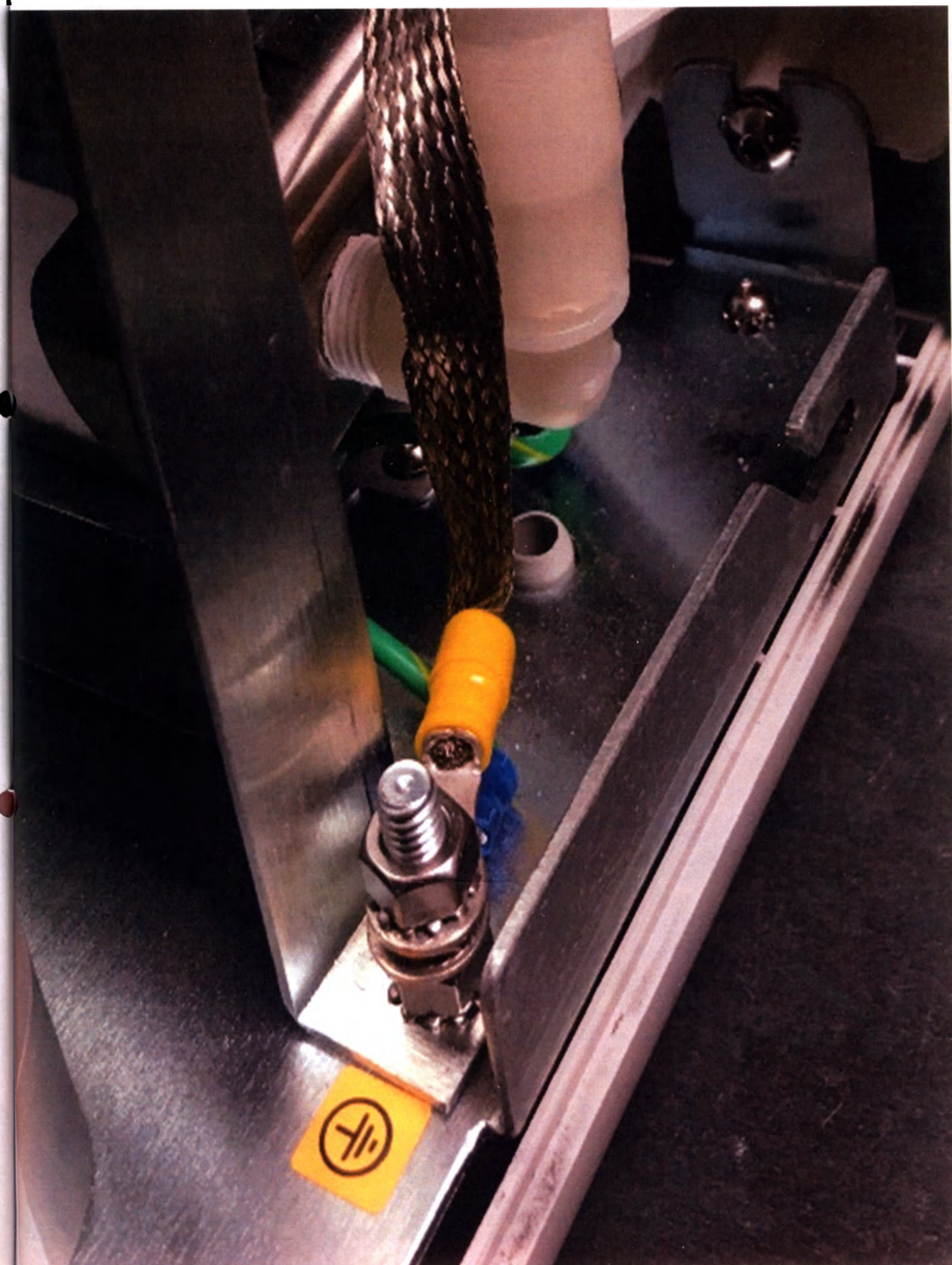
Rx ONLY

Additional information for Russia

INTEGRA

Manufacturer

Integralife Sciences (Ireland) Limited
IDA Business & Technology Park
Sragh
Tullamore
County Offaly
Ireland

USA & Canada: (800) 997-4868
Outside USA: (609) 936-5400
www.integralife.com

Assembled in: USA

Local representative in Russia

Neuroproject Ltd
127486, Moscow, Russia,
Korovinskoye shosse, 10, bld. 2,
floor 1, room 101

Tel. +7 495 6608173
e-mail: info@neurop.ru

CUSA® Clarity



Contamination Guard Instructions for Use

REF C7005

Approved by:	Sign	Date
Andrea Tommasoli Senior Director, Indirect Markets	<i>Andrea Tommasoli</i>	August 26, 2019

Integra, the Integra logo and CUSA are registered trademarks of Integra LifeSciences Corporation or its subsidiaries in the United States and/or other countries. © 2016 Integra LifeSciences Corporation. All Rights Reserved.
92905305 Revision D 11/2016

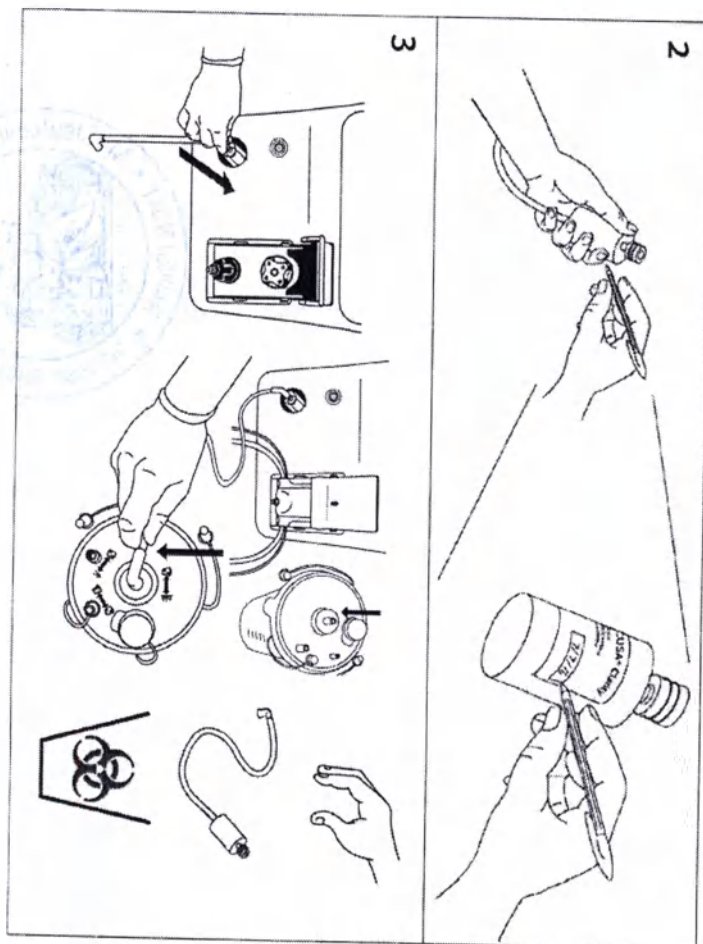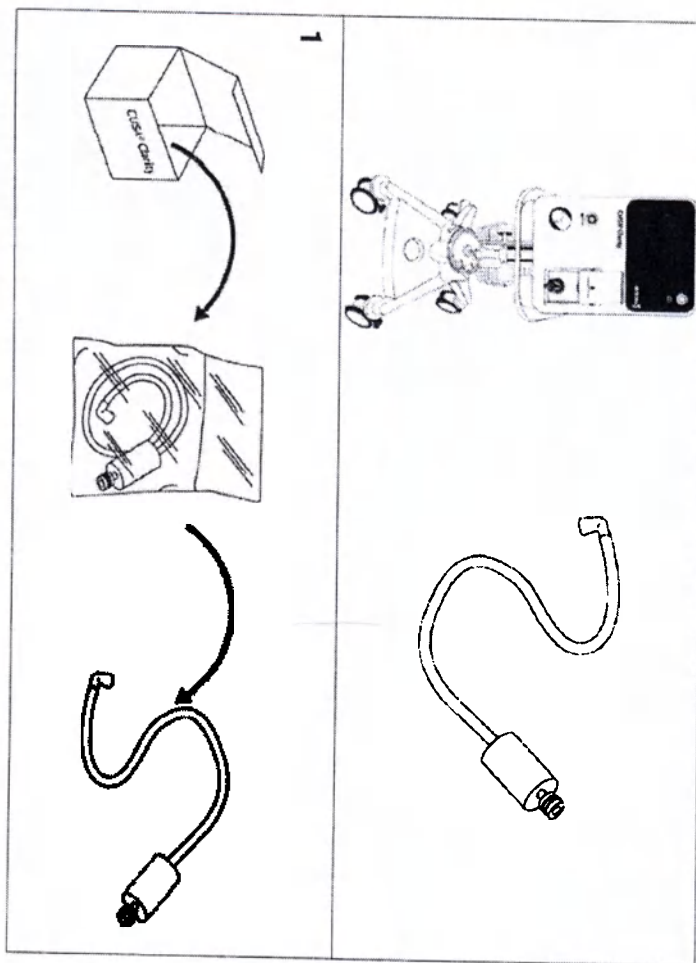INTEGRA LIFESCIENCES (France) SAS
Regulatory Affairs Department
Immeuble Séquoia 2
97 Allée Alexandre Borodine
69791 SAINT PRIEST Cedex

Introduction

The CUSA® Clarity contamination guard is the filter and tubing that connects between the canister and the CUSA® Clarity Ultrasonic Surgical Aspirator System console. This assembly filters any remaining particulate matter or moisture, preventing them from entering the vacuum pump.

NOTE:

Per the CUSA® Clarity Ultrasonic Surgical Aspirator System Operator's Manual replace the contamination guard every six months, or sooner if necessary, when used with the CUSA Clarity system.

Inspect the product for signs of damage before use. Refer to the assembly instructions overleaf. Write the installation date clearly on the Contamination Guard label.

Intended Use

The CUSA Clarity contamination guard is intended for use with the CUSA Clarity Ultrasonic Surgical Aspirator System. Refer to the CUSA® Clarity Ultrasonic Surgical Aspirator System Operator's Manual for indications, contraindications, and warnings.

Intended Users

The intended users of this Instructions for Use (IFU) and the equipment it describes are qualified medical professionals who are trained in the particular surgical technique and surgical procedure to be performed, and trained in the use of this equipment.

WARNING

No modification of this equipment is allowed.

WARNING

If the aspiration tubing and contamination guard are not connected to the correct canister ports, the system may not work.

Disposal of the Contamination Guard

The CUSA Clarity contamination guard must be disposed of after six months of use in accordance with regional regulation hospital protocols.

MR Safety Information

Refer to the CUSA® Clarity Ultrasonic Surgical Aspirator System Operator's Manual for full MR Safety information.

Symbols Used on Labeling

	Product complies with the requirements of directive 93/42/EEC
Rx ONLY	CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician
	This product is not made with natural rubber latex
	Manufacturer
	Date of Manufacture
	Follow Instructions for Use
	Operating Temperature: +10°C to +35°C Transport and Storage Temperature: -20°C to +60°C

	Catalog Number
	Lot Number
	Keep Dry
	Operating Humidity: 30% to 85% Transport and Storage Humidity: 15% to 85%
	MR unsafe

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