Kerr™

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Description of Attached Document

The preceding Certificate of Acknowledgment is attached to a document titled/for the purpose of Temp Bond NF
Directions for Use
containing 4 pages, and dated 7/25/17

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- ☐ Individual(s)
☐ Attorney-in-Fact
☐ Corporate Officer(s) _____ Title(s)

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representing: Kerr Italia S.r.l
Name(s) of Person(s) or Entity(ies) Signer is Representing

Additional Information

Method of Signer Identification

Proved to me on the basis of satisfactory evidence:
☐ form(s) of identification ☐ credible witness(es)

Notarial event is detailed in notary journal on:
Page # _____ Entry # _____

Notary contact: _____

Other

- ☐ Additional Signer(s) ☐ Signer(s) Thumbprint(s)
☐ _____

English – For dental use only

Directions for use

PRODUCT BRAND NAME

TempBond™ – Zinc Oxide/Eugenol Cement

TempBond™ NE – Zinc Oxide/Non-Eugenol Cement

DEVICE DESCRIPTION

TempBond temporary cement product series consists of TempBond and TempBond NE. Each is suitable for cementing temporary crowns, bridges, inlays, onlays and splints. Each has adequate flow to permit the restoration to be completely seated. They are designed to withstand the stresses of mastication and permit removal of the restoration when this is desired. TempBond and TempBond NE are available in traditional tubes, Unidose™ single-use foil pouches, and automix dual barrel syringes.

INDICATIONS FOR USE

TempBond and TempBond NE (non-eugenol) are temporary cements used in conjunction with temporary restorations such as crowns, bridges, inlays, onlays and splints.

CONTRAINDICATIONS

- TempBond contains eugenol and should not be used on patients with a known allergy to eugenol.
- TempBond NE may contain trace amounts of eugenol.

WARNINGS

- These products shall not be placed in contact with pulp tissue.
- Direct exposure of these products may harm the pulp.
- Avoid prolonged contact with skin, eyes, and soft tissue.
- Wash thoroughly with water after contact.
- Automix tips are intended for single patient use only to prevent cross-contamination between patients.
- Unidose is intended for single patient use only.

PRECAUTIONS

- Store at ambient temperature
- Refer to outer package for expiration date; do not use after expiry date

ADVERSE REACTIONS

None Known

INSTRUCTIONS

Work Time and Set Time: At the ideal mixing conditions of ambient temperature, the following work time and setting time from start of mixing are obtained:

Automix Syringes		
	Working Time (mm:ss)	Set Time (mm:ss)
TempBond	≥ 1:00	≤ 7:00
TempBond NE	≥ 1:00	≤ 7:00

Tubes		
	Working Time (mm:ss)	Set Time (mm:ss)
TempBond	≥ 1:30	≤ 7:00
TempBond NE	≥ 1:30	≤ 7:00

These times may vary based on storage conditions, temperature, humidity, and age of product.

Mixing:

Tube: Extrude equal lengths of Base and Accelerator onto the mixing pad provided. The length to be extruded will depend on the size and type of restoration to be cemented. **Replace caps tightly on tubes after use. Do not invert caps on tubes because this may cause premature hardening of the product.** Thoroughly mix the pastes for approximately 30 seconds.

Unidose: Unidose is for convenience and has the appropriate amount of base/catalyst for a restoration. Cut along dotted line with scissors; dispense contents of pouch onto mixing pad. Thoroughly mix the pastes for approximately 30 seconds.

Syringe: Remove cap from syringe. Always bleed syringe upon the initial use and prior to attaching an automix tip. Place automix tip onto syringe. Turn automix tip 90 degrees to lock in place. The material is now ready to be dispensed directly onto the temporary. No hand mixing is necessary.

Preparation and application

Dry the prepared teeth and the surface of the restoration. Apply a thin layer of the mixed cement to the internal surfaces of the temporary restoration. Firmly seat the restoration in the mouth. After the material has set, remove excess material with a scaler or other instrument.

NOTES

- A softer cement can be obtained by adding up to 25%, by volume, of white petrolatum to TempBond and TempBond NE.

INFORMATION REQUIRED BY ISO 3107

- Classification for TempBond and TempBond NE is Type I.
 - For TempBond and TempBond NE, the volume of product in tubes is 13 to 15ml and the volume of product in Unidose pouches is 0.5 to 0.6ml. Syringes are 5ml.
 - The mass ratio of product to use is:
 - TempBond, 3.4:1.0 (base:accelerator)
 - TempBond NE, 3.5:1.0 (base:accelerator)
- "TempBond" and "Unidose" are trademarks owned by Kerr Corporation.
- "TempBond" and "Unidose" are registered with the U.S. PTO and other countries.

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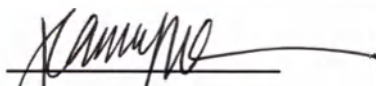
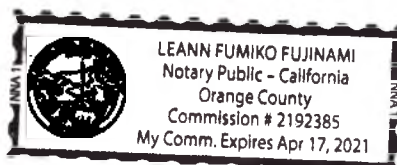
Kerr™

Maxcem Elite™

Self-Etch / Self-Adhesive Resin Cement

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State of California)

County of Orange)

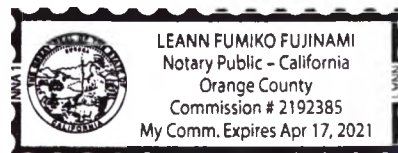
On July 25, 2017 before me, Leann Fujinami - notary public
(here insert name and title of the officer)

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Description of Attached Document

The preceding Certificate of Acknowledgment is attached to a document titled/for the purpose of Maxcom Elite
Suppl. to IFU
 containing 10 pages, and dated 7/25/17

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- ☐ Individual(s)
☐ Attorney-in-Fact
☐ Corporate Officer(s) _____
Title(s)

- ☐ Guardian/Conservator
☐ Partner - Limited/General
☐ Trustee(s)
☒ Other: Regulatory

representing: Kerr Italia S.r.l.
Name(s) of Person(s) or Entity(ies) Signer is Representing

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☐ _____

Table of content

Description	3
Technical characteristics	5
Packaging and Labelling	6
Transport conditions	8
Risk classification	9
Disposal	9
Shelf life	9
Contact information	9
Statement of relation among the facilities	10

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34057	Maxcem Elite Mini Kit · 1 syringe Maxcem Elite 5g clear shade; · 4 automix tips regular; · 4 automix tips with wide openings; · 4 intra-oral tips; · IFU;
34058	Maxcem Elite Refill Clear Shade · 2 syringes Maxcem Elite 5g clear shade; · 8 automix tips regular; · 8 automix tips with wide openings; · 8 intra-oral tips; · IFU;
34059	Maxcem Elite Refill White Shade · 2 syringes Maxcem Elite 5g white shade; · 8 automix tips regular; · 8 automix tips with wide openings; · 8 intra-oral tips; · IFU;
34060	Maxcem Elite Refill White Opaque Shade · 2 syringes Maxcem Elite 5g white opaque shade; · 8 automix tips regular; · 8 automix tips with wide openings; · 8 intra-oral tips; · IFU;
34061	Maxcem Elite Refill Yellow Shade · 2 syringes Maxcem Elite 5g yellow shade; · 8 automix tips regular; · 8 automix tips with wide openings; · 8 intra-oral tips; · IFU;
Accessories:	
34074	Automix tips 40 pcs in the packaging
34075	Intra-oral tips in the assortment in the packaging: 8 intra-oral tips, 8 automix tips with wide openings;
34076	Root canal tips in the assortment in the packaging: 8 root canal tips, 8 automix tips with wide openings;

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Technical Characteristics

Test Name	Specification
Film Thickness	≤ 50 microns
Gel/Set Time	≥ 2'00" / ≤ 5'30"
Initial set time	≤ 3'30"
Extrusion Force	≤ 85 N
Diametral Strength	≥ 35.0 MPa
Bond strength to dentin	≥ 7.5 MPa
Flexural strength	113,0 ± 12,0 MPa
Water sorption	≤ 40 ug/mm ³
Solubility	1.35 ug/mm ³
Radiopacity	≥ 238% Aluminum
Maximum applied efforts to syringe in press mode	≤ 120 N
Color stability (clear shade)	L ≥ 80.0, a ≤ 1.0, b ≤ 10.0

Weights:		
	Value	Tolerance
Weight of syringe Maxcem Elite	5 g	± 5 %
Weight of syringe TempBond NE	11,7 g	± 5 %
Weight of 1 automix tip (regular)	1,38 g	± 5 %
Weight of 1 automix tip (wide)	1,38 g	± 5 %
Weight of 1 intra-oral tip	0,25 g	± 5 %
Weight of 1 root canal tip	0,25 g	± 5 %
Weight of 1 TempBond automix tip	1,2 g	± 5 %
Dimensions:		
Poly pouch for syringe Maxcem Elite	198 x 60 mm	± 3 mm
Box of Maxcem Elite Standard Kit	51 x 181 x 182 mm	± 3 mm
Box of Maxcem Elite Value Kit	73 x 198 x 65 mm	± 3 mm
Box of Maxcem Elite Mini Kit	73 x 198 x 65 mm	± 3 mm
Box of Maxcem Elite Refill Shade	73 x 198 x 65 mm	± 3 mm
Box of Automix Tips	78 x 78 x 67.5 mm	± 3 mm
Poly bag for intra-oral tips	147 x 102 mm	± 3 mm
Poly bag for root canal tips	147 x 102 mm	± 3 mm

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Packaging & Labelling

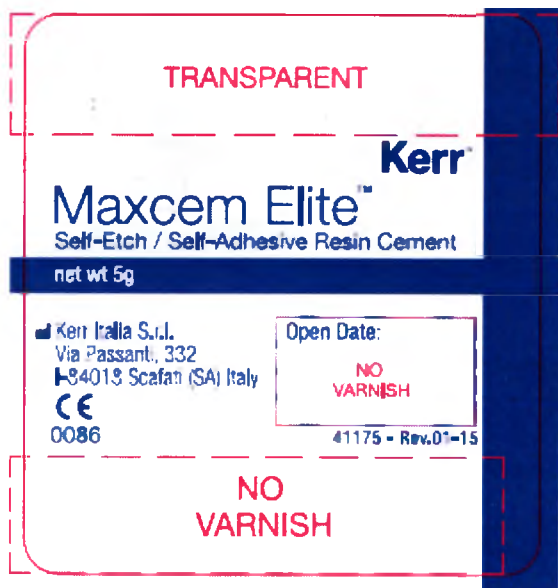
All Maxcem Elite Kits are packaged into carton boxes, sizes of which please refer to section Technical characteristics. Artworks of packaging of all items are enclosed to this technical file.

All syringes of Maxcem Elite cement are packaged into the individual poli pouch. The example of this label is below.

Poli pouch label

Representative label of syringe of Maxcem Elite

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Label of TempBond NE syringe

Automix tips for Maxcem Elite are packages into carton box, sizes of which please refer to section 9. Artwork of this box is also attached to this technical file.

Intra – oral and root canal tips are packaged into transparent poly bags. The main information of these tips are contained in white sticker on the bag. The examples of such bags are below.

Symbols are used on the packagings and labels:

	Manufacturing Date
	Lot or batch number
	Best before end

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PKG	Package date
	Manufacturer
	See instruction for use
	Compliance with EU Directives
	Symbol of compulsory certification in Russia
REF	Reference number
	Do not reuse

Transport conditions

Transport and storage of medical devices should be implemented in primary, secondary, group packaging provided by the manufacturer, which in turn must protect the packaged goods from rain, dust, sunlight, mechanical damage.

Transported by all modes of transport in covered vehicles in accordance with the requirements of GOST 50444-92 for the Russian Federation and the transportation rules, operating in each type of transport. Product set, packed in shipping containers, resistant to mechanical and environmental factors, which correspond to the requirements of GOST 50444-92 for terms of transportation.

Conditions of transportation

Parameter	Value
Temperature	from 0°C to 30°C
Relative humidity	from 10 to 95%
Pressure	From 500 hPa to 1060 hPa

Conditions of usage and storage

Usage conditions	from 32°C to 42°C (intrusive)
Environmental conditions	In the medical clinics under the temperature from 10°C to 35°C and at the relative humidity not more 85% under the 25°C
Storage conditions	from 4°C to 25°C comfort storage temperature 5-6°C

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Risk classification

Maxcem Elite™ Self-etch/Self-adhesive Resin Cement is IIa class of risk in EU.

Risk Management File is presented in a separate document, named "Risk Management (RM) Report, № Form 4D33 Rev.0" dated 11/07/2014.

Disposal

Medical devices must be disposed of in the local public entities, in accordance with SanPiN 2.1.7.2790-10 Sa-sanitary and epidemiological requirements for the handling of medical waste" and other national and regional normative legal documents.

Maxcem Elite™ Self-etch/Self-adhesive Resin Cement is not an as hazardous material (A class) and can be disposed of in general waste.

Shelf life

Shelf life of Maxcem Elite is 18 months from the date of issue. The date of best before end is indicated on the each packaging. Please be so kind to check this date before using medical device. Using Maxcem Elite after expiration date if prohibited.

Contact Information

Manufacturer

Kerr Italia S.r.l.
Via Passanti, 332
I-84018 Scafati (SA), Italy
+39-081-850-8311
kerrendental.eu

Distribution facility

KerrHawe SA
Via Strecce 4
CH-6934 Bioggio Switzerland
Tel: +41-91-610-5050
Fax: +41-91-610-0514
kerrendental.eu

Representative in Russia:

KaVo Dental Russland LLC

Legal address: 64 Malookhtinsky avenue, Building "B", premises 26H, Saint-Petersburg, 195112, Russia

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Statement of Relation among the Facilities

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CALIFORNIA CERTIFICATE OF ACKNOWLEDGMENT

A notary public or other officer completing this certificate verifies only the identity of the individual who signed the document to which this certificate is attached, and not the truthfulness, accuracy, or validity of that document.

State of California)

County of Orange)

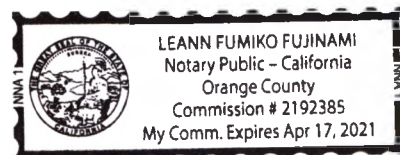
On July 25, 2017 before me, Leann Fujinami - notary public,
(here insert name and title of the officer)

personally appeared Jeannie Back

who proved to me on the basis of satisfactory evidence to be the person(s) whose name(s) is/are subscribed to the within instrument and acknowledged to me that he/she/they executed the same in his/her/their authorized capacity(ies), and that by his/her/their signature(s) on the instrument the person(s), or the entity upon behalf of which the person(s) acted, executed the instrument.

I certify under PENALTY OF PERJURY under the laws of the State of California that the foregoing paragraph is true and correct.

WITNESS my hand and official seal.

Signature

[Signature]

(Seal)

Optional Information

Although the information in this section is not required by law, it could prevent fraudulent removal and reattachment of this acknowledgment to an unauthorized document and may prove useful to persons relying on the attached document.

Description of Attached Document

The preceding Certificate of Acknowledgment is attached to a document titled/for the purpose of Maxcem Elite

IFU
containing 6 pages, and dated 7/25/17

The signer(s) capacity or authority is/are as:

- ☐ Individual(s)
☐ Attorney-in-Fact
☐ Corporate Officer(s)

Title(s)

- ☐ Guardian/Conservator
☐ Partner - Limited/General
☐ Trustee(s)

☒ Other:

representing:

Regulatory
Kerr Italia S.r.l.
Name(s) of Person(s) or Entity(ies) Signer is Representing

Additional Information

Method of Signer Identification

Proved to me on the basis of satisfactory evidence:

- ☐ form(s) of identification ☐ credible witness(es)

Notarial event is detailed in notary journal on:

Page # _____ Entry # _____

Notary contact: _____

Other

- ☐ Additional Signer(s) ☐ Signer(s) Thumbprint(s)

☐

Table of content

Description.....	3
Indications for Use	3
Contraindications	3
Warnings	3
Precautions	3
Adverse reactions	4
Instructions for use	4
Storage	6

Maxcem Elite™ Self-etch/Self-adhesive Resin Cement

Description

Maxcem Elite is a self-etch, self-adhesive resin cement indicated for indirect cementation of ceramic, resin and metal restorations. This product is radiopaque for easy identification in x-rays and dual-cure, allowing for maximum flexibility in its use. Maxcem Elite is packaged in a dual barrel syringe with single-use automix tips and optional curved dispensing tips to allow user to deliver desired volume of cement directly into restoration and/or tooth preparation.

Indications for Use

Maxcem Elite is indicated for cementation of all indirect restorations including ceramic, resin, and metal-based inlays/onlays, crowns, bridges, posts, and veneers*. Additional indications include pit and fissure sealant, and cementation of crown restoration to implants.

*Adhesive application on the preparation is required for veneer cementation using Maxcem Elite.

Contraindications

1. This product shall not be used on patients who are allergic to methacrylate resin as it may cause contact dermatitis.
2. This product shall not be placed in contact with pulp tissue. Direct exposure of this product may harm the pulp.

Warnings

Avoid contact with skin or eyes. If this product comes in contact with skin or eyes, wash thoroughly with water.

Precautions

1. Secure restoration with moderate pressure while cleaning excess cement.
2. Allow the cement to fully cure before finishing and polishing or adjusting occlusion to avoid potential debonding.
3. Do not use phosphoric acid to etch dentin when using Maxcem Elite in self-adhering mode, as this may cause post-operative sensitivity
4. Do not treat or clean the zirconia-based or metal-based restorations with phosphoric acid or hydrofluoric acid (HF) which can negatively impact adhesion.
5. Do not use hydrogen peroxide (H2O2) as its residues can adversely affect the setting reaction and adhesion of Maxcem Elite.
6. Do not use provisional cements containing eugenol, as these materials can inhibit the polymerization of resin cements including Maxcem Elite.
7. Use a rubber dam to keep the preparation or cavity free of any contamination during cementation.
8. Do not open foil package until initial use. Use the product within 6 months of opening the pouch but always ensure that it is not past the expiration date printed on the syringe label.

9. If prolonged contact with oral tissue occurs, flush with copious amount of water.

Adverse reactions

None known.

Instructions for Use

Note: Automix tips are for single patient use only to prevent cross-contamination between patients.

Maxcem Elite is compatible with adhesives such as OptiBond™ Solo Plus, OptiBond All-In-One, or OptiBond XTR self-etch universal adhesive. Refer to adhesive instructions for use for optimal procedure. The use of other dentin and enamel adhesive system is at the discretion and sole responsibility of the dental practitioner.

Restoration Surface Preparation

Prepare the internal surface of the restoration according to manufacturer's instructions. Here are some general recommendations:

Etchable Ceramic/Porcelain Restorations:

Sandblast the internal surface with 50µm alumina at a pressure of 30 psi (0.2 MPa) and etch the restoration with hydrofluoric acid. Application of the silane primer is optional.

Metal-based, Zirconia-based, and Composite Restorations:

Sandblast the internal surface with 50µm alumina at a pressure of 60 psi (0.4 MPa) for metal-based or zirconia-based restorations, or a pressure of about 15 psi (0.1 MPa) for composite restorations.

Resin bonded prostheses (Maryland bridges):

Etching of enamel surface with phosphoric acid gel etchant is recommended prior to cementation with Maxcem Elite.

Indirect Restorations (excluding Veneers) and Implant Restoration

1. Remove the provisional restoration.
2. Use the OptiClean rotary instrument or your choice of cleaning method to remove provisional cement and residue from the preparation. Rinse thoroughly.
3. Air dry the preparation.
4. Seat the restoration to ensure proper fit. Adjust if needed and thoroughly clean the bonding surfaces of the restoration after adjustment.
5. Select the desired shade of Maxcem Elite.

6. Remove the syringe from the foil pouch. Record the date the syringe was removed on the syringe label.

7. Remove the syringe cap. Before the first use, bleed the syringe by squeezing out a small quantity of the pastes onto a pad to equalize the base and catalyst pastes in the syringe. Secure the appropriate automix tip on the dual syringe cartridge by turning the tip clockwise. The wide 4:1 automix tips are intended for use with the intraoral tips and root-canal tips to dispense Maxcem Elite.

Dispensing of the material is done as follows:

Inlay/Onlay - Dispense the cement directly into the preparation covering all surfaces. Seat the restoration with moderate pressure and maintain the pressure until the cement is fully set.

Crown - Fill the crown with cement. Seat the restoration with moderate pressure and maintain the pressure until the cement is fully set.

Post - Prepare the post space. Size and fit the post. Dispense the cement onto the post or directly into the canal space using the root-canal tips. Seat and rotate the post slightly upon insertion to avoid trapping air. Allow the cement to slowly flow from canal space.

Leave the used mixing tip on the syringe as a cap until the next application.

8. When the restoration is properly seated, remove the excess cement while maintaining pressure on the restoration. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack-curing excess with a curing light for approximately 2-3 seconds, or by allowing the cement to self-cure for approximately 2-3 minutes after application or until the excess cement feels rubbery.

9. After removal of the excess cement, light cure all surfaces including margins using Demi Ultra curing light for 10 seconds. See footnote for recommended curing times for other curing light models.**

10. For metal-based and opaque restorations, before proceeding to polishing and finishing, let the cement set for at least 4 minutes after seating of the restoration.

11. If needed, use finishing strips for proximal areas. Polish margins with Gloss Plus discs, cups, or points. Use HiLuster Plus polishers for a final high luster shine.

12. Verify the occlusion.

Veneer Restorations

For cementation of veneers (ideal for 1-2 units at a time), select OptiBond™ Solo Plus Adhesive or OptiBond XTR Self-Etch Universal Adhesive. Consult adhesive product instructions for use.

After adhesive is applied and cured, follow these instructions for cementation:

1. Select the desired shade of cement.

2. Remove the syringe cap. Before the first use, bleed the syringe by squeezing out a small quantity of pastes onto a pad to equalize the base and catalyst paste in the syringe. Secure the appropriate automix tip on the dual syringe cartridge by turning the tip clockwise.

3. Dispense the cement directly onto the internal surface of the veneer. Place the veneer gently

onto the tooth, allowing the cement to slowly flow from all margins.

4. Tack cure the veneer into place on the facial surface, away from the margins, using a small diameter light guide for 2-3 seconds.

5. After removing the excess cement, light cure all surfaces using a Demi Ultra curing light for 10 seconds.**

6.

Note: Maxcem Elite minimum working time is 1 minute 20 seconds at room temperature; Set time is less than 4 minutes at oral temperature. These times may vary based on storage conditions, temperature, humidity, and age of the products.

**** Recommended Curing Times:**

Demi Ultra, Demi Plus/Demi, and L.E.Demetron II: 10 seconds.

L.E.Demetron I & Optilux 501: 20 seconds.

For all other lights, see manufacturer's recommendations.

Storage

Store the product at ambient temperature. Avoid excess heat. For optimum freshness, store product in a refrigerator. Allow the product to reach room temperature prior to use.

Keep product in foil package until initial use. Once the foil pouch is opened, use within 6 months and prior to expiration date printed on syringe label.

"OptiBond" is a trademark owned by Kerr Corporation.

"OptiBond" is registered with the U.S. PTO and other countries

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© Copyright 2007-2016 Notary Rotary, Inc. PO Box 41400, Des Moines, IA 50311-0507. Все права защищены. Номер позиции 101772. Обратитесь к своему авторизованному реселлеру за покупкой копий этой формы.
KERR ITALIA S.r.l. - Stabilimento: Via Passanti 332 - 84018 SCAFATI (Salerno)