NOTARIAL CERTIFICATE

TO ALL TO WHOM THESE PRESENTS SHALL COME, I, **TAN SEOK KETT**,
NOTARY PUBLIC officiating at Lot 333, 3rd Floor, Wisma New Asia, Jalan Raja Chulan,
50200 Kuala Lumpur, Wilayah Persekutuan, Malaysia, do hereby certify that I have sighted
the original copy of the **INSTRUCTION FOR USE OF THE MEDICAL DEVICE
VERSION 2 (LATEX DENTAL DAM)** duly certified by the Managing Director, **KOH
YEK JOO** for and on behalf of **SANCTUARY HEALTH SDN. BHD.** which is hereunto
annexed is a complete and accurate copy of that original.

(IN TESTIMONY WHEREOF
(I have hereunto subscribed my
(name and affixed my Seal of
(Office at Kuala Lumpur,
(Wilayah Persekutuan this 13th
(day of June, 2024.

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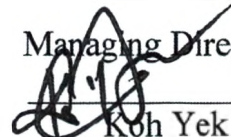
(THE NOTARY PUBLIC ASSUMES NO RESPONSIBILITY FOR THE CONTENTS OF THE DOCUMENT(S) HEREIN)

«I certify»

Managing Director

Koh Yek Joo

«04» JUNE 2024

Instruction for use of the medical device Version 2

Latex Dental Dam

*manufactured by SANCTUARY HEALTH SDN.
BHD., Malaysia*

Medical Device Name:

Latex Dental Dam in variants (hereinafter referred to as dental dams).

1. Black Latex Dental Dam, thin, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
2. Black Latex Dental Dam, medium, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
3. Black Latex Dental Dam, heavy, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
4. Silk Blue Latex Dental Dam, thin, size: 152 mm x 152 mm (6" x 6").
5. Silk Blue Latex Dental Dam, medium, size: 152 mm x 152 mm (6" x 6").
6. Pink Latex Dental Dam, heavy, size 152 mm x 152 mm (6" x 6").

Manufacturer's name and address:

SANCTUARY HEALTH SDN. BHD., No. 16, Persiaran
Perindustrian Kanthan 1, Kawasan Perindustrian Kanthan, 31200 Chemor, Perak, Malaysia.

Place of manufacture of the medical device:

SANCTUARY HEALTH SDN. BHD., No. 16, Persiaran Perindustrian Kanthan 1, Kawasan
Perindustrian Kanthan, 31200 Chemor, Perak, Malaysia.

Authorized representative of the manufacturer in Russia:

DENT-MED Ltd.

Russia, 344011, Rostovskaya obl., Rostov-Na-Donu, Gvardeysky per. 61, office 13

E-mail: dentmed2016@gmail.com

Tel.: + 7 (961) 274-89-89

Intended use

Said medical device is intended for use during dental procedures to isolate specific teeth to form a dry working field free from saliva contamination.

Indications for use

- Isolation of a specific tooth to create a dry working field free of saliva contamination.
- Achievement of optimal results on images, which is an undeniable advantage for dentists when working on a specific tooth.

Scope: Dentistry. Designed for continuous use for less than 60 minutes.

Contraindications to use:

- Patients with claustrophobia
- Patients with severe asthma
- Patients with nasal breathing problems
- Patients who are allergic to natural latex
- In cases where the patient's teeth have not fully erupted

Side Effects:

- Difficulty with the patient's speech
- Allergy to natural latex

Composition and main technical characteristics of the product.

Latex Dental Dam in the following variants

1. Black Latex Dental Dam, thin, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
2. Black Latex Dental Dam, medium, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
3. Black Latex Dental Dam, heavy, sizes: 127 mm x 127 mm (5" x 5"), 152 mm x 152 mm (6" x 6").
4. Silk Blue Latex Dental Dam, thin, size: 152 mm x 152 mm (6" x 6").
5. Silk Blue Latex Dental Dam, medium, size: 152 mm x 152 mm (6" x 6").

6. Pink Latex Dental Dam, heavy, size 152 mm x 152 mm (6" x 6").

Main characteristics of the medical device:

Description	Dental dam is a single-use device made of natural latex
Appearance of the device	Black (flavored)
	Blue silk (flavored)
	Pink (flavored)
Scent	Mint/Spearmint
Thickness	Slim/Medium/Tight
Edging	Square-shaped
Target audience	Patients of any age, both adults and children, who come to the dentist for appropriate dental procedures
Nature of a device	Universal and with a high degree of resistance against tearing, dental device
Sterilization	Not a sterile device
Frequency of use	Single-use

Variants	Size, mm	Weight, g	Thickness, mm
1. Black Latex Dental Dam, thin	152 x 152 ($\pm 5\%$)	3.74 $\pm$ 0.19	0.16 $\pm$ 0.02
2. Black Latex Dental Dam, medium	152 x 152 ($\pm 5\%$)	4.36 $\pm$ 0.22	0.20 $\pm$ 0.02
3. Black Latex Dental Dam, heavy	152 x 152 ($\pm 5\%$)	5.43 $\pm$ 0.27	0.24 $\pm$ 0.02
4. Black Latex Dental Dam, thin	127 x 127 ($\pm 5\%$)	2.48 $\pm$ 0.12	0.16 $\pm$ 0.02
5. Black Latex Dental Dam, medium	127 x 127 ($\pm 5\%$)	2.99 $\pm$ 0.15	0.20 $\pm$ 0.02
6. Black Latex Dental Dam, heavy	127 x 127 ($\pm 5\%$)	3.58 $\pm$ 0.18	0.24 $\pm$ 0.02
7. Silk Blue Latex Dental Dam, thin, size: 152 mm x 152 mm (6" x 6").	152 x 152 ($\pm 5\%$)	5.10 $\pm$ 0.22	0.22 $\pm$ 0.02
8. Silk Blue Latex Dental Dam, medium, size: 152 mm x 152 mm (6" x 6").	152 x 152 ($\pm 5\%$)	6.10 $\pm$ 0.27	0.28 $\pm$ 0.02
9. Pink Latex Dental Dam, medium, size 152 mm x 152 mm (6" x 6").	152 x 152 ($\pm 5\%$)	4.60 $\pm$ 0.20	0.20 $\pm$ 0.02

Biocompatibility: information on all materials used (grades and names), list of chemicals included in the composition.

Chemical composition	CAS No.	Percentages content, %	Function
Black Latex Dental Dam			
Hevea natural rubber latex	9006-04-6	95,51	Main active constituent

Chemical composition	CAS No.	Percentages content, %	Function
Sulfur	7704-34-9	0,53	As vulcanizing agent
Zinc oxide	1314-13-2	1,06	As a cross-linking agent
Zinc diethyl dithiocarbamate	14324-55-1	0,53	As an accelerating additive
Zinc dibutyldithiocarbamate	136-23-2	0,53	As an accelerating additive
Antioxidant Aquanox L	68610-51-5	0,92	As an antioxidant
Titanium dioxide Versanyl white Carbon black Versanyl	13463-67-7 1333-86-4	0,40 0,51	As a coloring material
Peppermint oil *	8006-90-4	0,01	As a flavoring material
Silk Blue Latex Dental Dam			
Hevea natural rubber latex	9006-04-6	95,51	Main component
Sulfur	7704-34-9	0,53	Vulcanizing agent
Zinc oxide	1314-13-2	1,06	Cross-linking agent
Zinc diethyl dithiocarbamate	14324-55-1	0,53	Accelerant
Zinc dibutyldithiocarbamate	136-23-2	0,53	Accelerant
Antioxidant Antiox-L50	68610-51-5	0,92	Antioxygen
Silicon Dioxide Aerosil W	112945-52-5	0,92	Stabilizator
Water	7732-18-5	~1	Eluent
** LUCONYL NG 6900 BLUE	55406-53-6	~1	Colouring material
** MPL2728 BLUE	147-14-8		

Chemical composition	CAS No.	Percentages content, %	Function
W70 White	13463-67-7		
Peppermint oil* FLAVOUR 9878	8008-79-5	0,01	Flavoring material
Pink Latex Dental Dam			
Hevea natural rubber latex	9006-04-6	95,51	Main component
Sulphur	7704-34-9	0,53	Vulcanizing agent
Zinc oxide	1314-13-2	1,06	Cross-linking agent
Zinc diethyl dithiocarbamate	14324-55-1	0,53	Accelerant
Zinc dibutyldithiocarbamate	136-23-2	0,53	Accelerant
Antioxidant Antiox-L50	68610-51-5	0,92	Antioxygen
Silicon dioxide Aerosil W	112945-52-5	0,92	Stabilizator
Water	7732-18-5	~1	Eluent
HVL-FBP pink	-	~1	Colouring material
W70 White	13463-67-7		
Peppermint oil* FLAVOUR 9878	8008-79-5	0,01	Flavoring material

Transport and Consumer package.

Consumer package.

Individual packaging is not available. Devices are delivered in 36 or 52 pieces in a group package.

Variants	Packaging variant	Weight of the package (without device), g, ± 2	Length x Width, mm, ± 2	Number of pieces of devices in the package	Material
Black Latex Dental Dam	127 mm x 127 mm (5" x 5")	25	143 x 143	52	HTCB lithographic paper with lamination 300 offset with matte
	152 mm x 152 mm (6" x 6")	30	165 x 165	36	

Silk Blue Latex Dental Dam	152 mm x 152 mm (6" x 6")	27,3	163 x 163 x 13 (Upper part) 160,5 x 160,5 x 13 (Bottom part)	36	Art print paper 270 g/m², single side
Pink Latex Dental Dam	152 mm x 152 mm (6" x 6")	27,3	163 x 163 x 13 (Upper part) 160,5 x 160,5 x 13 (Bottom part)	36	

Transport package

Packaging variant	Length x Width x Height, mm, ± 2	Material
127 mm x 127 mm (5" x 5")	298 x 298 x 198	Corrugated cardboard T280 / M150 / T280 BF
152 mm x 152 mm (6" x 6")	340 x 340 x 170	

40 consumer packages in a transport package.

There are no special conditions for transportation, as packaged dental dams are not dangerous cargo.

Dental dams and instructions for use of the medical device are included.

Device operation procedure

1. Make a small hole approximately 2 mm in diameter in the central area of the dental dam. Make sure that the size of the hole corresponds to the tooth to be treated.
2. Increase the size of the hole and place the metal clammer in the dental dam hole using the forceps, hold the hole open to place the wings correctly.
3. Use forceps and clamp to place the dental dam on the patient's tooth.
4. Use the dental dam frame on the outside of the mouth to retain excess dental dam material.
5. Dispose of each dental dam sheet after use.
6. Use a new dental dam sheet for each tooth you are working on.

List of national or international normative documents

International standards	
Regulation (EU) MDR 2017/745	Regulation (EU) 2017/745 of the European Parliament and of The Council of 5 April 2017 on medical devices
ISO 13485:2016	Medical Devices Management System
ISO 9001:2015	Quality management systems
EN ISO 14971:2019	Application of risk management to medical devices
ISO 15223-1:2021	Medical devices — Symbols to be used with information to be supplied by the manufacturer. Part 1: General requirements
ISO 10993-1: 2018:	Biological evaluation of medical devices. Part 1: Evaluation and testing within a risk management process

National Standards of the Russian Federation	
GOST ISO 10993-1-2021	«Medical devices. Biological evaluation of medical devices. Part 1: Evaluation and testing within a risk management process»
GOST R ISO 10993-2-2009	«Medical devices. Biological evaluation of medical devices. Part 2. Animal welfare requirements»
GOST ISO 10993-5-2011	«Medical devices. Biological evaluation of medical devices. Part 5. Tests for in vitro cytotoxicity»
GOST ISO 10993-10-2011	«Medical devices. Biological evaluation of medical devices. Part 10. Tests for irritation and delayed-type hypersensitivity»
GOST ISO 10993-12-2015	«Medical devices. Biological evaluation of medical devices. Part 12. Sample preparation and reference materials»
GOST R 52770-2023	«Medical devices. Biological impact evaluation system. General requirements for safety»
GOST 31214-2016	«Medical devices. Requirements for samples and documentation presented for toxicological tests, sanitary and chemical analyses, tests for sterility and pyrogenicity»
GOST 31870-2012	«Drinking water. Determination of elements content by atomic spectrometry methods»
GOST 31209-2003	«Containers for blood and its components. Requirements for chemical and biological safety and methods of testing»
GOST R 55227-2012	«Water. Methods for determination of formaldehyde content»
MUK 4.1.3166-14	«Determination of hexane, heptane, acetaldehyde, acetone, methyl acetate, ethyl acetate, methanol, isopropanol, acrylonitrile, n-propanol, n-propyl acetate, butyl acetate, isobutanol, n-butanol, benzene, toluene, ethyl benzene, m-, o- and p-xylenes, isopropylbenzene, styrene, alpha-methylstyrene in water and aqueous extracts from materials of different composition by gas chromatography».
GOST ISO 14971-2021	«Medical devices. Application of risk management to medical devices.»
GOST R ISO 15223-1-2023	«Medical devices. Symbols to be used with information to be supplied by the manufacturer. Part 1. General requirements»

Storage and transportation conditions

Storage conditions

- Store in a cool place at temperatures up to 40 °C and relative humidity <80%.
- Keep away from sources of heat and open flame.
- Store away from direct sunlight.

Transportation conditions

The device is classified as non-hazardous cargo, therefore there are no recommended transportation conditions. The recommended transportation temperature should be between -20 °C and +60 °C and relative humidity <80%.

Conditions of application

Operating temperature 18 to 25° C, relative humidity 80% (ambient).

For use in dental clinics. The materials also function properly inside the oral cavity in the presence of biological fluids at nominal temperature values: 32 to 42° C.

The device is used in accordance with current dental procedures, together with auxiliary devices - spike, clamps, forceps and frame.

Use medical devices registered in accordance with the established procedure in the territory of the Russian Federation.

Type of climatic design - NF 4.2

Information on recycling or disposal

The device does not belong to hazardous waste.

Use, transportation and storage of the medical device have no adverse effects on people and the environment.

To avoid health risks during the disposal of medical devices, as well as environmental risks associated with the disposal of medical devices, personnel should dispose of MD according to the protocol adopted by the individual institution.

For the Russian Federation, disposal is performed according to SanPiN 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of the territories of urban and rural settlements, for water bodies, drinking water and drinking water supply, atmospheric air, soils, residential premises, operation of industrial and public premises, organization and conduct of sanitary and anti-epidemic (preventive) measures", as for medical waste of class A ((epidemiologically safe waste, by composition close to TCS).

Disposal of packaging materials

All packaging materials are manufactured in accordance with environmental laws and can be recycled. Send used packaging to specialized waste disposal centers.

Warranty period and expiration date

- Dental dams are stored in closed packages for a maximum of 5 years from the date of manufacture.
- Devices should be stored in accordance with established guidelines to ensure compliance with the specified device expiration date.
- The expiration date of the device after opening the box is 12 months.

Maintenance and routine repairs

- The device is intended for single use only.
- Not subject to maintenance and repair. Does not have spare parts that are subject to maintenance. This means that there is no in-service repair (not applicable).

Warranty obligation

- The company warrants the fulfillment of its obligations in the event of proper use.
- The Company warrants to correct any defects in the device (whether due to faulty design, materials, components or assembly) that occur during the warranty period of use.
- The warranty does not cover defects caused by improper use, improper storage conditions, misuse or improper modification by the purchaser.

Address / phone number / website of the organization for consumer appeal (RF)

DENT-MED Ltd.

Russia, 344011, Rostovskaya obl., Rostov-Na-Donu, Gvardeysky per. 61, office 13

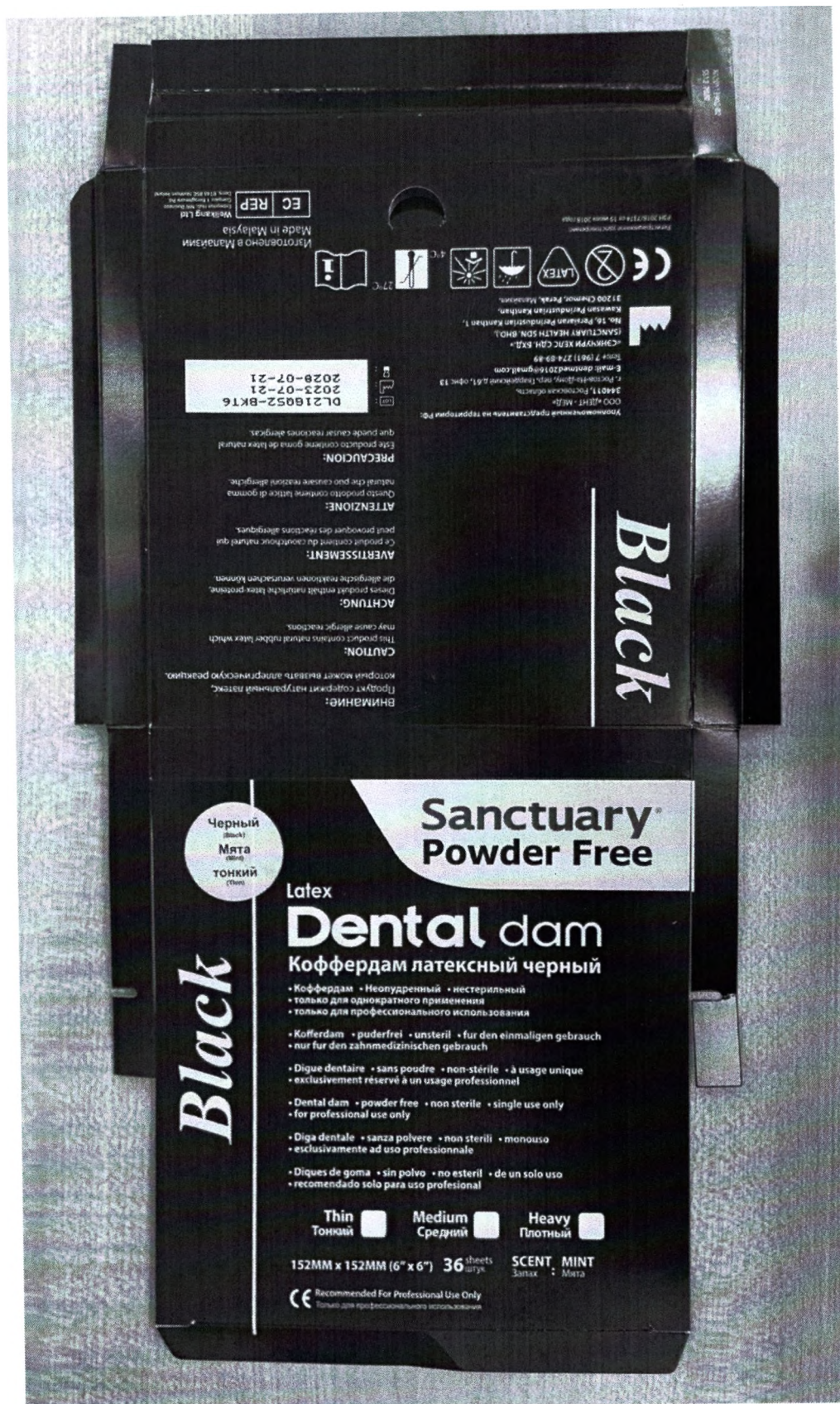
E-mail: dentmed2016@gmail.com

Tel.: + 7 (961) 274-89-89

Labeling in Russian language for Russia

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4°C

EC REP

Silk Blue Latex Dental Dam, thin, size: 152 mm x 152 mm (6" x 6")

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
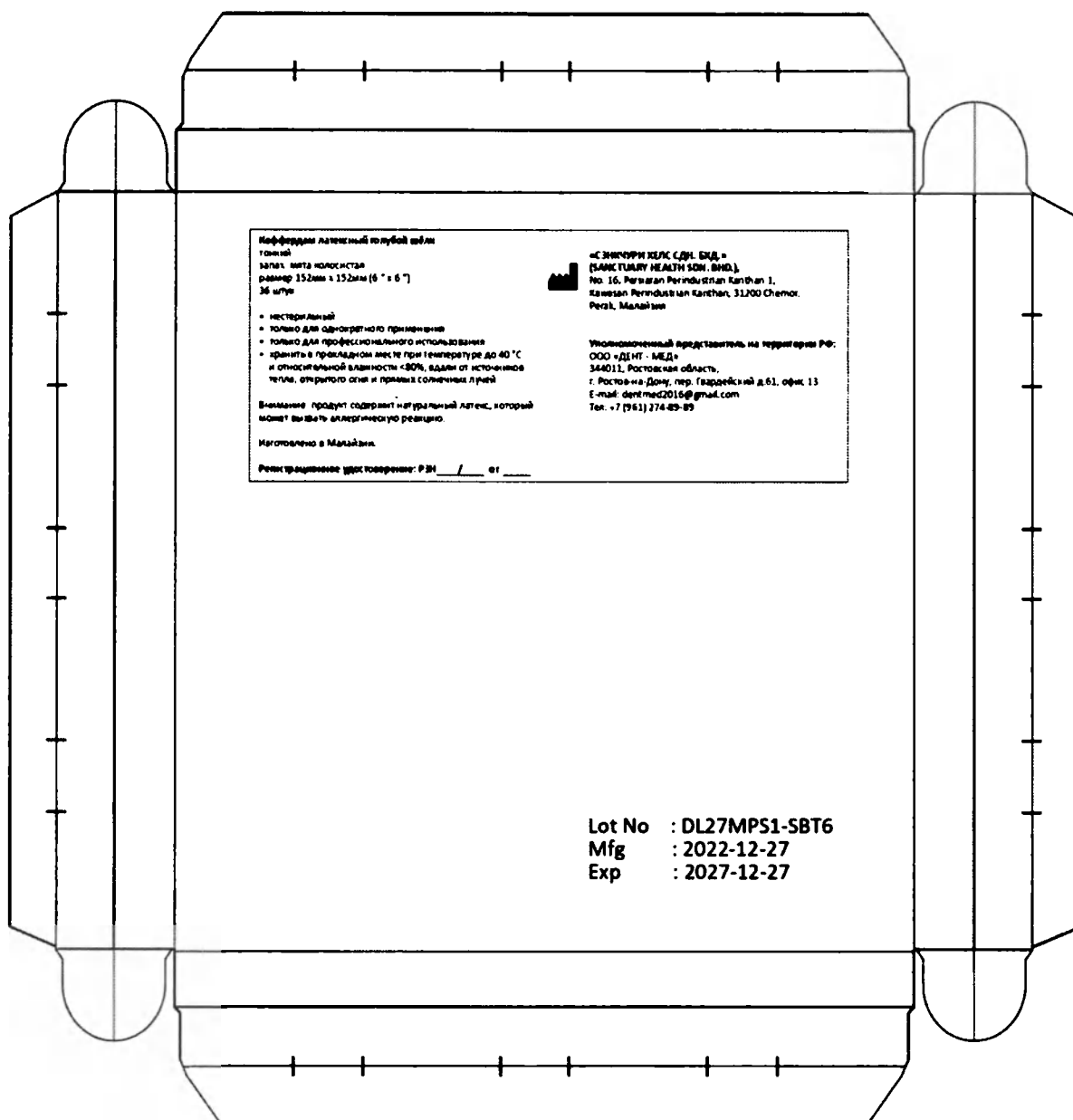
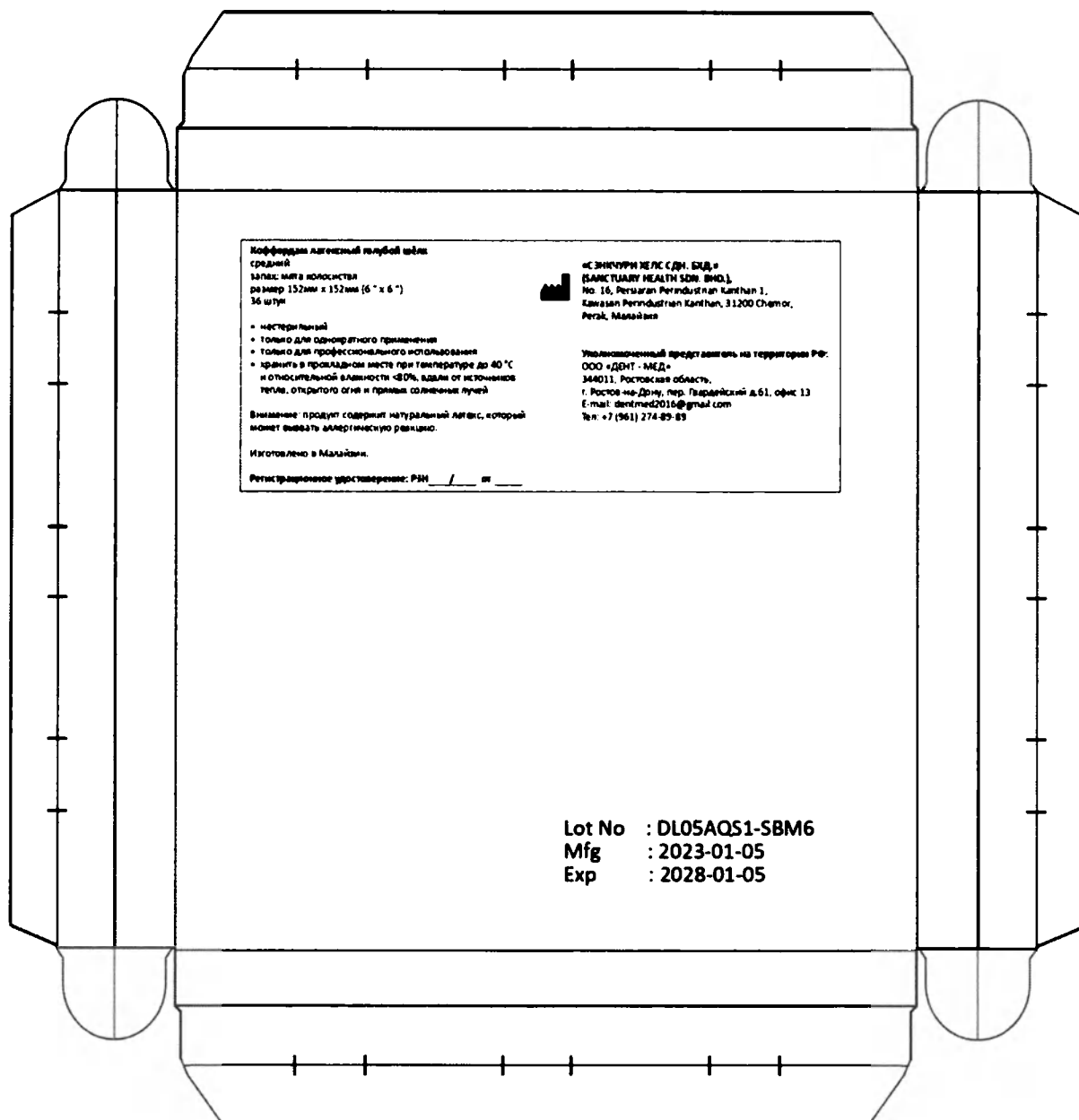
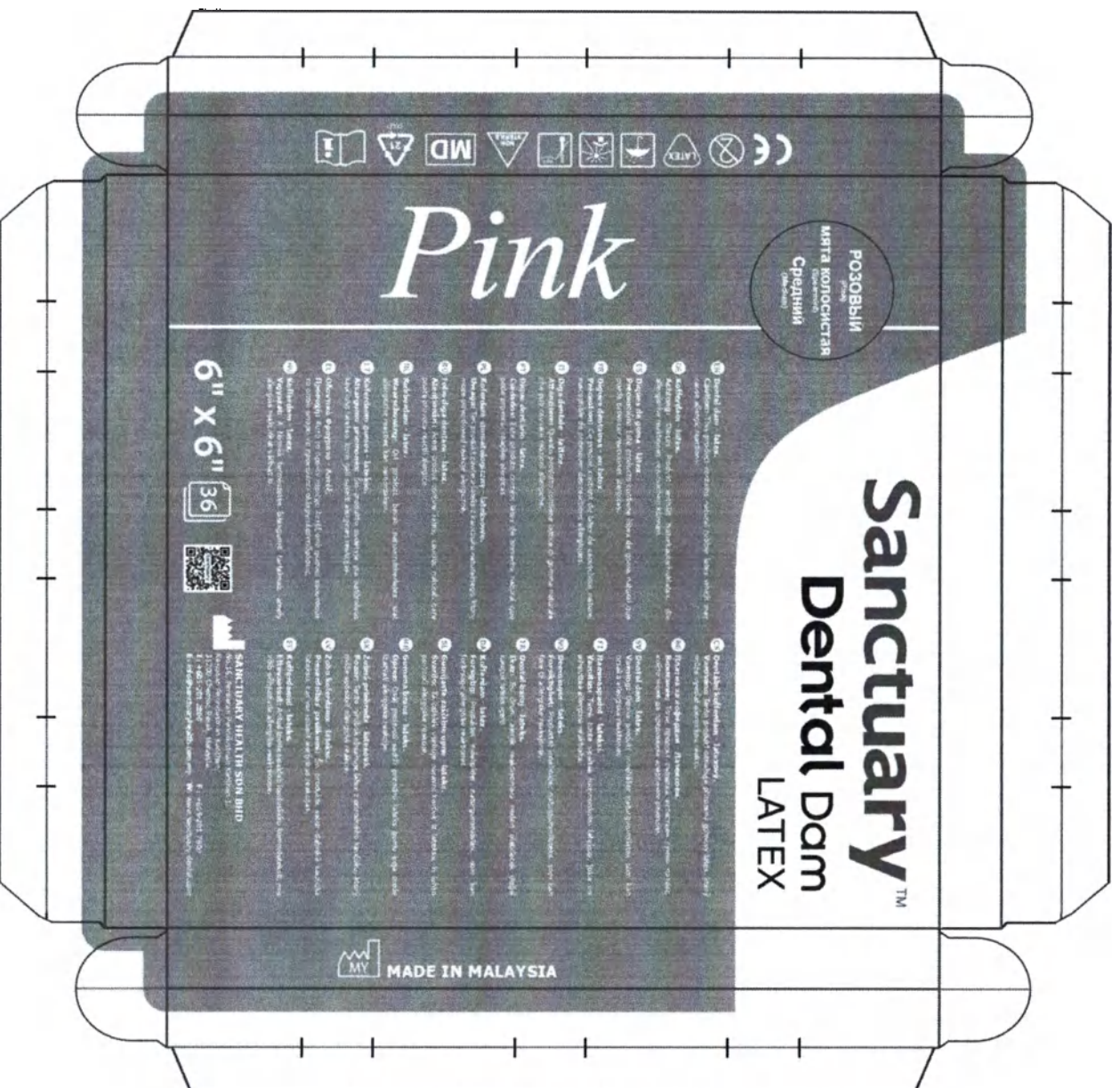
Upper part of the box

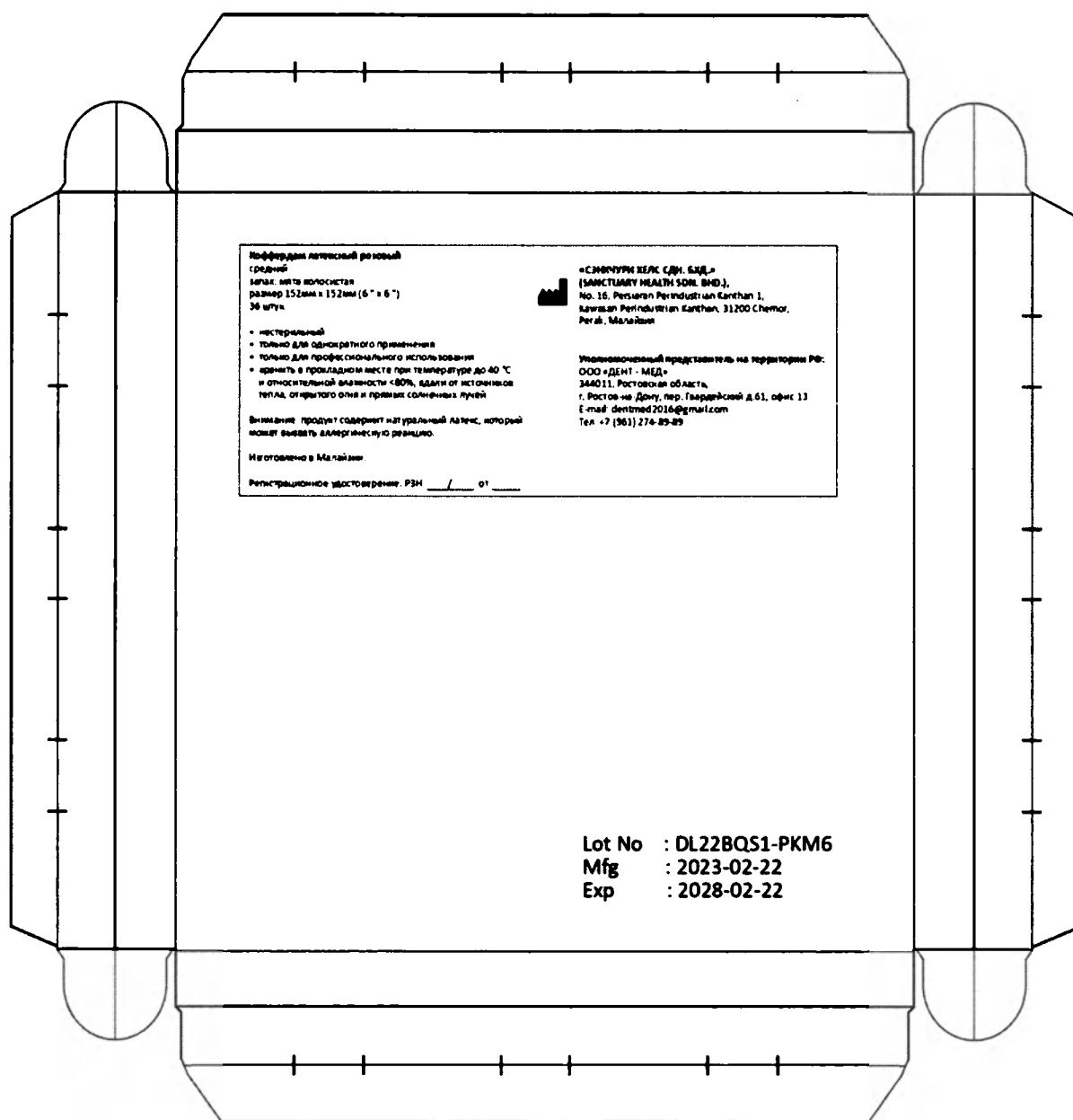Bottom part of the box

Explanation of the symbols on the label

Name and designation of the device (color, size, scent, thickness, number of sheets in the package is indicated)

Sanctuary™ Trademark of the manufacturer

 Information about the manufacturer (full company name, address).

 Lot number

 Manufacturing date

 Expiration date

CE The device complies with the basic requirements of EU directives and harmonized standards of the European Union.

-  Keep away from moisture
-  Contains natural latex
-  Do not reuse
-  Keep out of sunlight
-  Refer to the operating instructions
-  Temperature range
-  Medical device
-  Non-sterile
-  Other cardboard
-  Manufactured in Malaysia

Directions:

- Dental dam, powder-free, non-sterile, for single use only, for professional use only.
 - The device contains natural latex which may cause an allergic reaction.
 - Manufactured in Malaysia
 - Information about the registration certificate in the territory of the Russian Federation
 - Information about the authorized representative of the manufacturer on the territory of the Russian Federation
- Lot No – Lot number
 Mfg – Manufacturing date
 Exp – Expiration date

Information on the initial issue or latest revision of the instructions for use
 Version 2, 2024.

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