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Registered No. 2024-2971

NOTARIAL CERTIFICATE

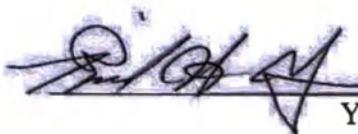
Seowon Law & Notary Office Inc.

84, Wonheong-ro, Seowon-gu, Cheongju, Korea

Date: July 28, 2022

«Approved»

  (signature)

You Jae Hoon

CEO

DiaDent Group International

DiaDent Group International

18, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu,
Cheongju-si, Chungcheongbuk-do, 28161, Korea
Tel: 82-43-266-2315 Fax: 82-43-262-8658

INSTRUCTION FOR USE FOR THE MEDICAL DEVICE

Healtex® Dental Filling Material

The attached document is provided to DiaDent Group International. This document certifies that the contents of attached true and is confirmed by DiaDent Group International.

Version 1

1. Device name

Healtex® Dental Filling Material

Versions:

Versions:

1. Healtex® HealE-Temp Dental Filling Material consisting of:
 - Healtex® HealE-Temp Dental Filling Material in a 30 g container – 1 pc.
 - instruction for use – 1 pc.
2. Healtex® HealTemp Dental Filling Material consisting of:
 - Healtex® HealTemp Dental Filling Material in syringes 3 g, color: yellow – 3 pcs.
 - instruction for use – 1 pc.
3. HealTemp Dental Filling Material consisting of:
 - Healtex® HealTemp Dental Filling Material in syringes 3 g, color: blue – 3 pcs.
 - instruction for use – 1 pc.

Hereinafter referred to as the Healtex® Dental Filling Material, filling material, medical device,, device.

2. Information about the manufacturer and the developer

Manufacturer

DiaDent Group International

16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, Republic of Korea

Tel.: +82-43-266-2315

E-mail: diadent@diadent.co.kr

Developer

DiaDent Group International

16, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, Republic of Korea

Tel.: +82-43-266-2315

E-mail: diadent@diadent.co.kr

Authorized representative of the medical device manufacturer

PROTECO Joint-Stock Company

Letter I, bld. 1, 57 Salov St., Saint Petersburg, 192102

Tel./Fax: +7 (812) 779-30-90

E-mail: info@protecodent.ru

3. Intended purpose of the device indicating the potential user and function

1. Intended purpose of the device

It is used for temporary obturation and temporary closure of the tooth cavity after root canal treatment.

2. Potential medical device users

Dental practitioners

4. Device description

Healtex® HealE-Temp Dental Filling Material in container 30 g – 1 pc.

Healtex® HealE-Temp Dental Filling Material is a temporary cured filling material used for

restoration of posterior and anterior teeth. It is used for temporary filling during treatment. It is available in a white paste packed in a container. It hardens under the influence of saliva in the mouth within less than 35 minutes and directly restores the cavity, preventing external infection until a definitive restoration is placed.

Healtex® HealTemp Dental Filling Material in syringes 3 g, color: yellow – 3 pcs.

Healtex® HealTemp Dental Filling Material in syringes 3 g, color: blue – 3 pcs.

Healtex® HealTemp Dental Filling Material is a light-cured, high-viscosity filling material that can be applied to both dental inlays and onlays. The material has an excellent adhesion, high wear resistance with an excellent mechanical strength, is quickly and easily removed all at once without any particles in the cavity. The yellow-colored material is suitable for anterior teeth (similar to the natural teeth color), the blue-colored material – for posterior teeth. The material is packed in a syringe.

5. Indications, contraindications and potential side effects

Indications

Healtex® HealE-Temp Dental Filling Material

The material is used for temporary tooth restoration, as well as for temporary restoration of posterior and anterior teeth

Healtex® HealTemp Dental Filling Material

Temporary filling of cavities during dental treatment.

Contraindications to the use of the medical product

Intolerance by patients to the components of the medical device raw materials.

Potential side effects

Allergic reaction to the device components.

In very rare cases the material may cause hypersensitivity reactions. In such cases, stop using the device and seek medical assistance.

6. Precautions

1. Check the product for leakage, discoloration, improper condition of the package and shelf life.
2. It is recommended to use a dental dam to prevent material particles entering to the oral mucosa and the patient from swallowing the device.
3. Before use read and understand the precautions, side effects, contraindications and instruction for use.
4. Do not use the device for other purposes, except for its intended purpose.
5. For professional use only according to the relevant indications.
6. Do not use in patients with inflammation or allergies to any component of the product.
7. Avoid contact with skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of contact with skin, wash off immediately with soap and water.
8. The device is not intended for internal use.
9. It is not applicable for permanent root canal filling.
10. Do not squeeze out the paste with excessive force, as the patient may feel a sharp pain.
11. Pay attention to the location of the mandibular canal under the lower molar teeth.
12. Pay attention to the relative position of the maxillary sinus and upper molar teeth.

13. Do not use after expiry date.
14. Avoid contact with water or liquids.
15. Protect from direct sunlight and excess heat.
16. Do not heat the device.
17. Do not mix with other materials.
18. Healtex® HealTemp Dental Filling Material can remain in the body for NOT MORE THAN 29 days, Healtex® HealE-Temp Dental Filling Material for not more than 7 days.

7. Application procedure.

Healtex® HealE-Temp Dental Filling Material

1. Protect the mouth with a dental dam.
2. Thoroughly rinse and dry the canal cavity prepared for filling (in some cases, when the inner surface of the tooth is not properly cleaned, the material can detach).
3. Select the required instrument size, place the required amount of the material into the cavity.
4. Carry out filling, harden the material. If the cavity is on the occlusal surface, check the filling site after correction of the occlusion.
5. Check the occlusion and remove the excess material using a suitable instrument.
6. The material should be applied and distributed in layers in small quantities until the filling material reaches the height of the tooth enamel.
7. The material cures within less than 35 minutes by absorption of an oral fluid or saliva. Avoid chewing for two hours after the filling procedure.
8. Wait for a few minutes until the material dries. Check the adhesion strength of the material to the tooth using a dental instrument (if the material has not hardened completely, the dental instrument will pass through the surface of the filled tooth).

Removal:

When removing the filling, clean the cavity with a cotton swab dipped in alcohol

Healtex® HealTemp Dental Filling Material

Rinse and dry the tooth cavity prepared for filling.

2. Select the required instrument size, place the material into the cavity and make sure that there are no pores for hermetically sealed filling.
3. After filling the cavity, check the occlusion of the tooth and remove the excess material for proper joining of the teeth.
4. Using a polymerization lamp, carry out polymerization with a light intensity of 500 mW/cm² for 20-30 seconds in a vertical direction at a distance of 2 mm. If the light intensity of the polymerization lamp is less than 500 mW/cm², the polymerization time can be increased.
5. Close the syringe with the cap and put the device aside.

Removal

1. Check the tooth cavity from which the material should be removed.
2. Remove the material using a suitable tool. The material should be easily removed in one movement.
3. Remove any material residues from the tooth cavity.

8. Technical characteristics

Characteristics of the medical device

Table 1 – Technical characteristics of the medical device

Parameter	Value
Healtex® HealTemp Dental Filling Material, colors: yellow and blue	
Weight in a syringe	3 g ± 5% from the weight indicated on the package
Ambient light sensitivity	Uniform
Polymerization depth	> 1.5 mm

Water absorption	Less than 40 $\mu\text{g}/\text{mm}^3$
Solubility	Less than 7.5 $\mu\text{g}/\text{mm}^3$
Color resistance	The color does not change
Healtex® HealE-Temp Dental Filling Material	
Weight in a container	30 g $\pm$ 5% from the weight indicated on the package
Hardening time	Less than 35 minutes
Compression strength	More than 5 MPa
Max. content of acid-soluble arsenic, mg/kg	Less than 2 mg/kg

9. Product materials

Healtex® HealE-Temp Dental Filling Material

- Zink oxide
- Gypsum (calcium sulfate)
- HEXAMOLL*[®]DINCH
- Vinyl resin

Healtex® HealTemp Dental Filling Material, color: yellow

- Aliphatic urethane acrylate
- EBECRYL® 230 Radiation Curing Resins
- Low viscosity aromatic monoacrylate
- White oil
- Silane
- (+/-)-Camphorquinone
- Ethyl-4-dimethylaminobenzoate
- 2,6-di-tert-butyl-para-cresol
- LOWILITE 22 (2-hydroxy-4-n-octoxybenzophenone)

Healtex® HealTemp Dental Filling Material, color: blue

- Aliphatic urethane acrylate
- EBECRYL® 230 Radiation Curing Resins
- Low viscosity aromatic monoacrylate
- White oil
- Silane
- (+/-)-Camphorquinone
- Ethyl-4-dimethylaminobenzoate
- 2,6-di-tert-butyl-para-cresol
- LOWILITE 22 (2-hydroxy-4-n-octoxybenzophenone)
- SunCROMA® Ultramarine Blue

10. Scope of supply

Healtex® Dental Filling Material

Versions:

1. Healtex® HealE-Temp Dental Filling Material consisting of:
 - Healtex® HealE-Temp Dental Filling Material in a 30 g container – 1 pc.
 - instruction for use – 1 pc.
2. Healtex® HealTemp Dental Filling Material consisting of:
 - Healtex® HealTemp Dental Filling Material in syringes 3 g, color: yellow – 3 pcs.
 - instruction for use – 1 pc.

3. HealTemp Dental Filling Material consisting of:

- Healtex® HealTemp Dental Filling Material in syringes 3 g, color: blue – 3 pcs.
- instruction for use – 1 pc

11. Designations and symbols

	Manufacturer		Temperature limit
	Date of manufacture		Refer to instructions for use or to electronic instructions for use
	Use by date		Caution
	Batch code		Keep dry from moisture
	Catalogue number		Fragile, handle with care
	Keep away from sunlight		Maximum stacking height
	This side up		Use no hooks/Do not puncture

12. Transportations and storage conditions

Storage. Protect from direct sunlight, excess heat and high air humidity. Keep tightly closed in a dry and cool place.

Transportation. The products packed in the shipping package should be transported by all types of covered vehicles, observing the precautions indicated on the shipping package and in accordance with the shipping rules applicable to this means of transport.

Storage, transportation and indoor use conditions:

- Temperature: from 2 to 27 °C;
- Relative humidity: no special requirements;
- Atmospheric pressure: no special requirements.

13. Sterilization, disinfection

The medical products Healtex® Dental Filling Material are non-sterile and intended for single use. Sterilization and disinfection are not provided.

14. Disposal and/or destruction

After the use of the medical products Healtex® Dental Filling Material as intended, they are classified as epidemiologically hazardous waste (products contaminated with blood and/or other body fluids of patients).

In this case, a healthcare provider only collects and temporarily stores the above waste. Collection is performed in disposable bags or containers with color codes adopted for hazardous medical waste indoors or in places for temporary storage of medical waste until further transportation to the disposal site.

Transportation of medical waste from medical institutions and its disposal are performed by specialized organizations that have a license for handling epidemiologically hazardous medical waste.

A secondary package, rejected products and unused expired products are classified as epidemiologically non-hazardous waste (non-hazardous waste that does not come into contact with body fluids of patients) similar in composition to solid domestic waste (SDW). For safe disposal of residual curing products, the unused material should be removed from the syringe or container and polymerized before disposing of it in accordance with the local regulations.

Non-hazardous waste is collected in reusable containers or disposable bags (except for bags for hazardous waste). The procedure for collecting and storing non-hazardous waste is determined in accordance with the waste management scheme in each particular organization.

Transportation and disposal are performed taking into account the requirements adopted for a particular territory, in accordance with the sanitary legislation of a particular country for the maintenance of the territories of populated areas and management of production and consumer waste.

15. Shelf life

Healtex® Heale-Temp Dental Filling Material

2 years from the date of manufacture, subject to the storage conditions.

Healtex® HealTemp Dental Filling Material

3 years from the date of manufacture, subject to the storage conditions.

16. List of the international standards to which the medical product complies

Healtex® Heale-Temp Dental Filling Material

MDD 93/42/EEC, EN ISO 13485, EN ISO 14971, EN ISO 15223-1, EN 1641, EN 1041, EN ISO 10993-1, EN ISO 10993-5, EN ISO 10993-10, EN ISO 10993-11, ISO 7405, MEDDEV 2.4/1, MEDDEV 2.7/1, MEDDEV 2.12/1, MEDDEV 2.12/2

Healtex® HealTemp Dental Filling Material, colors: yellow and blue

MDD 93/42/EEC, EN ISO 13485, EN ISO 14971, EN ISO 10993-1, EN ISO 10993-5, EN ISO 10993-10, EN ISO 10993-11, IEC 62366-1, EN ISO 15223-1, EN ISO 7405, EN 1041, EN 1641, ISO 10477, ISO 4049, MEDDEV 2.4/1, MEDDEV 2.7/1, MEDDEV 2.12/2, MEDDEV 2.12/1, ISO 11607

The reference standards are used in the latest version: the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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HA, JUNG MIN, attorney-in-fact of DIADENT GROUP INTERNATIONAL CEO JAE HOON YOU, appeared before me and admitted said principal's subscription to the attached INSTRUCTION FOR USE FOR THE MEDICAL DEVICE.

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This is hereby attested on this 17th day of October, 2024 at this office.

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Name of the office
Seowon Law & Notary Office Inc.
Belong to
Cheongju District Prosecutors' Office
Address of the office
84, Wonheong-ro, Seowon-gu, Cheongju,
Korea

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PARK JONG WOOK

(Signature of Notary Public)

This office has been authorized by the Minister of Justice, the Republic of Korea, to act as Notary Public since FEB, 7. 2020 under Law No. 173.

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