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2024-04 2034-04

(01) 0 6971824 20026 3
(11) 240426
(17) 340425
(10) 2024040013
(21) TS2J407224

Shenzhen Tisenc Medical Devices Co., Ltd.
11F, 11G, Kechuang Building, Quanzhi Science and Innovation Park,
Maozhoushan Industrial Park, Shajing Community, BaoAn District,
Shenzhen, 518104, P.R.China

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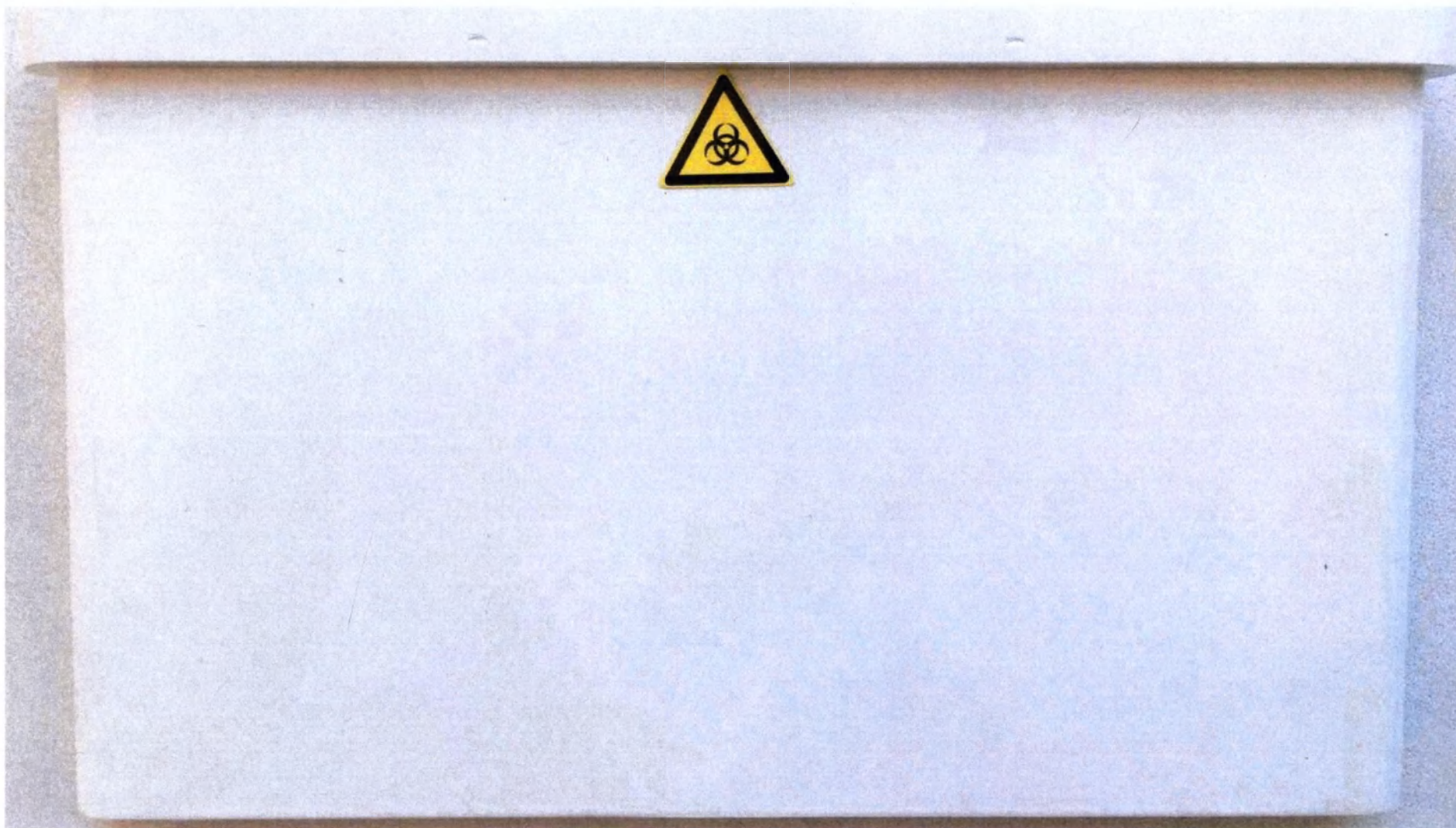
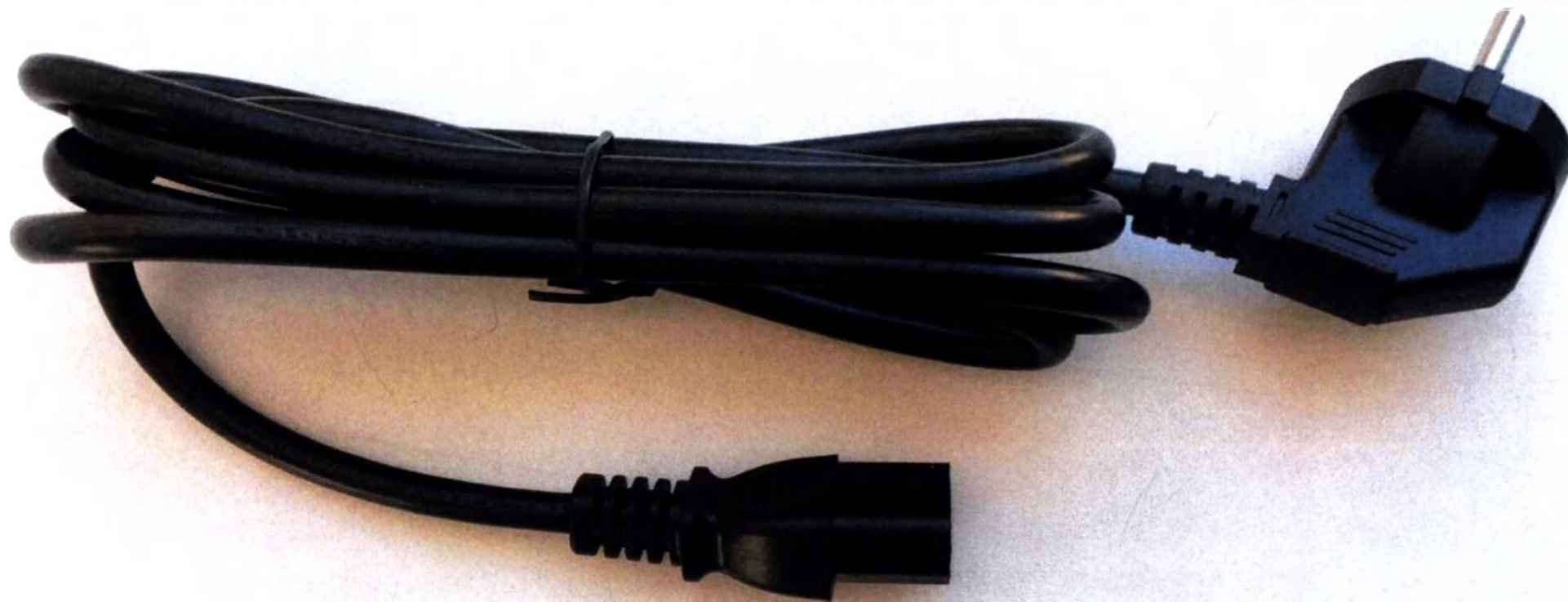
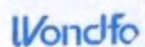
Immunoassay System

General Information

Product Name	
Model	
Serial Number	
Address	
Of installation	
Service Engineer	

Please Note that the service engineer responsible for installation should fill above information

Service/ Maintenance Record

Date	Service Type	Information & Note	Engineer
	☐ Repair ☐ Maintenance		
	☐ Repair ☐ Maintenance		
	☐ Repair ☐ Maintenance		
	☐ Repair ☐ Maintenance		
	☐ Repair ☐ Maintenance		

Shenzhen Tisenc Medical Devices Co., Ltd.

E-mail: tisenc@tisenc.com

<http://www.tisenc.com>

Tel: +86-0755-23225549

Address: 11F, 11G, Keckuang Building, Qianzhi Science and Innovation Park, Maozhoushan Industrial Park,
Shajing Community, Bao'an District, Shenzhen, 518104, P.R. China

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Automatic Chemiluminescence Immunoassay Analyzer Installation Sheet

1.General Information

User Name:			
User Address:			
Contact:		Phone:	
Model:		Serial Number:	
Engineer:		Date:	

2.Installation Check List

Item	Description	Record
1	Repacker the bags complete and no damage	
2	Analyzer Appearance complete and no damage	
3	The Accessories are complete	
4	The transport Return has been returned	
5	Screen of analyzer can light up after connect to power	
6	Touch screen can operate	
7	Incubator can operate and the temperature is in range	
8	USB connection is successfully setup	
9	Setup test procedures are successfully performed	
10	Provide the operation training to User	

3.Client Signature for Conformation

User's Name:	
Date:	
Contact Person:	
Signature:	

Guangzhou Wondfo Biotech Co., Ltd.

Automatic Chemiluminescence Immunoassay Analyzer

QUICK SOP of ACCRE 90

1. Turn on the **POWER SWITCH** from back of the analyzer, and then press **START** button in the front of instrument to open it.
2. To Log into the software by given **USERNAME** and **PASSWORD**.
3. Open the Reagent Box, put the Reagent Cartridge into the Reagent Rack.
4. Using pipettor manually to load sample into sample hole of Reagent Cartridge.
5. Click  switch icon, open the door of Reagent Chamber.
6. Put the Reagent Rack into the Reagent Chamber of analyzer.
7. Put the Assay Tips into the Reagent Chamber.
8. Enter the sample ID in the software.
 - 8-1 Click on the **TEST** icon.
 - 8-2 Choose one channel.
 - 8-3 Click on **SAMPLE ID** column, type the sample ID, and then click **OK**.
 - 8-4 Repeat 8-2 to 8-3 to load other samples
9. After all samples ID done, click  switch icon, to close the door of Reagent Chamber.
10. Click **START** icon to start test
11. The analyzer running test automatically.

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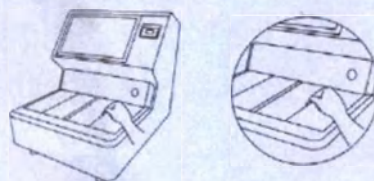
Wondfo


ACCRES 90 Pre-operation Guide

After you open the package of ACCRES 90, you must act as following steps first before POWER ON.


- ① To open two Automatic Doors one by one, by hand-holding the adhesive tape of each door.

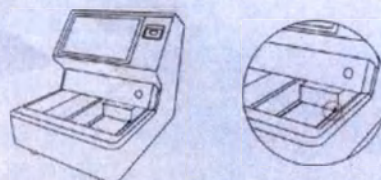
- ② Use the 2.5mm inner hexagon wrench to unscrew the screws on metal plate.

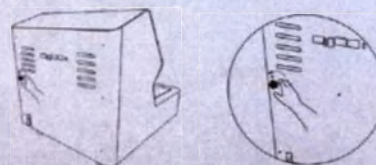- ③ Turn the TRANSPORT FIXED HANDLE as counterclockwise direction until pull it out.

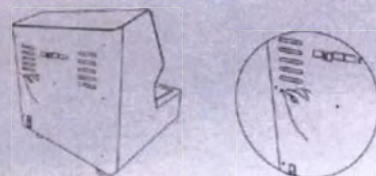- ④ Push the TRANSPORT FIXED PLUG into the above handle hole position.

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