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CERTIFICATE

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD. on the annexed DOCUMENT is genuine.

China Council for the Promotion
of International Trade

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Authorized

Signature:

Liu Yongcui

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(Date: May. 21, 2019)

证 查询网址 Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

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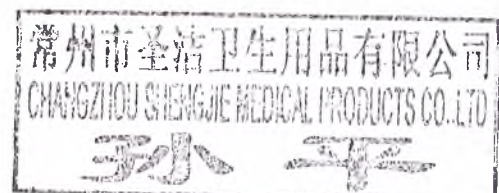
Disposable non-sterile dental cotton rolls Clean + Safe®

Hardware versions:

- No.1 – 8 mm in diameter
- No.2 – 10 mm in diameter
- No.3 – 12 mm in diameter

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The cotton rolls Clean + Safe® are made of 100% natural absorbent cotton and safe adhesive, have no odour, do not contain cancerigenic, mutagenic, toxic or chemically-active materials or chloride, are grouped into package with a help of polyethylene film.

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Application conditions:

Air temperature in the healthcare facility: $+15^{\circ}\text{C}$ – $+25^{\circ}\text{C}$;
Relative humidity: max. $80\pm 2\%$;
Atmospheric pressure: 755 ± 20 mm Hg.

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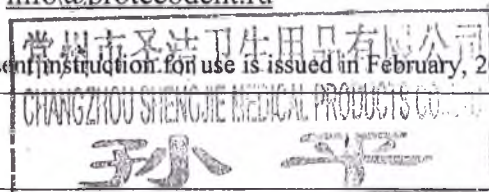
Information on undesired events connected with use of the medical device " Disposable non-sterile dental cotton rolls Clean + Safe[®]" which have signs of an adverse event (incident), should be forwarded to the authorized representative of the manufacturer: "PROTECO", JSC, legal address: 192102, Saint Petersburg, Salova street 57, Building 1, Letter "I", tel.: / fax: +7 (812) 635-88-90, website: www.protecodent.ru, e-mail: info@protecodent.ru

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Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe[®]

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Approved
«20» February 2019

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President "Changzhou Shengjie Medical Products Co., Ltd"

Mr. Sun Ping

Address: No.216 Liangchang Road, Jintan District,
Changzhou city Jiangsu - China

INSTRUCTION FOR USE

Disposable non-sterile dental cotton rolls Clean + Safe®

2nd revision 2019

1. Name of the medical device

Disposable non-sterile dental cotton rolls Clean + Safe®

Hardware versions:

No.1 – 8 mm in diameter

No.2 – 10 mm in diameter

No.3 – 12 mm in diameter

Hereinafter referred to as rolls, devices, cotton rolls Clean + Safe®.

2. Manufacturer of the medical device

"Changzhou Shengjie Medical Products Co., Ltd"

Address: No.216 Liangchang Road, Jintan District, Changzhou city Jiangsu - China

Tel: 0086-519-82893289 fax: 0086-519-82893239

3. Purpose

The cotton rolls Clean + Safe® are intended for absorption of saliva and blood during various dental procedures to provide a dry field around the teeth to be treated, for isolation of excretory ducts of salivary glands, for prevention of exposure of mucous membrane to medicines that can injure it.

The cotton rolls Clean + Safe® are recommended for the following users:

- for medical personnel in order to perform different dental procedures.
- for physical persons using the medical device as intended by the manufacturer.

4. Application.

Dentistry. Dental rooms in various healthcare organizations, private dental offices, dental clinics including private clinics.

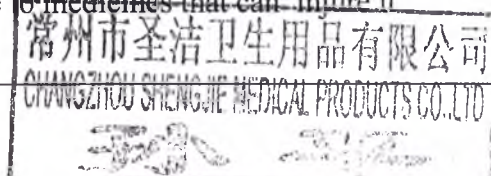
5. Indications, contraindications, precautions and side effects, for use of the medical device according to the intended purpose

Indications:

During different dental procedures:

- to provide a dry field around the teeth to be treated;
- for isolation of excretory ducts of salivary glands;
- for prevention of exposure of mucous membrane to medicines that can injure it

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

Contraindications:

Contraindications for use of rolls is an individual intolerance of raw material components.

Precautions:

- Read instruction before use. Do not use the rolls with defects or expired shelf life.
- Do not use if the package is damaged!
- Do no reuse.

Side effects related to use of the medical device according to its intended purpose: irritant effect or allergic reaction caused by an individual incompatibility with materials of the medical device.

6. General information, operating principle

The cotton rolls Clean + Safe® are made of 100% natural absorbent cotton and safe adhesive, have no odour, do not contain cancerogenic, mutagenic, toxic or chemically-active materials or chloride, are grouped into package with a help of polyethylene film.

The device is non-sterile, single-use and represents a roll made of fluffy mass of fibres slightly interwoven together in different directions. It has high rates of whiteness (more than 72%), capillarity action (85 mm), absorbability (not less than 19 g), it does not contain any foreign impurities, neps, it has a high breathability and filterability. High flexibility enables the medical device to take shape of the oral cavity and to provide a high absorption retaining the initial shape.

The cotton rolls have already been used for 100 years in the clinical practice. The medical devices are safe for users and patients. CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD has been manufacturing the cotton rolls for more than 10 years; the medical devices have been sold in many countries. No incidents or critical complaints have been registered. Only one complaint of a patient with a burning sensation has been registered for all this time. However, the check showed that the medical device is safe and that case happened as a result of patient's individual intolerance.

The rolls are manufactured of different sizes (diameters), which provides a good opportunity to choose the best size for a particular clinical case.

7. Materials, sizes.

The cotton rolls are made of 100% absorbent cotton and adhesive FVH6 (CMC – carboxymethylcellulose).

The main dimensions of the medical device are given in Table 1.

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

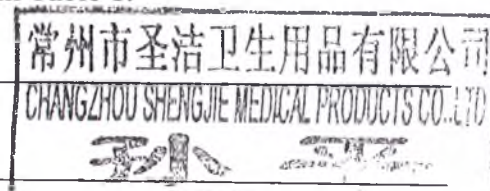
Table 1

Version	Weight , g	Diameter , mm	Length , cm
No.1 – 8 mm in diameter	0.35±10%	8±5%	3.8±0.1
No.2 – 10 mm in diameter	0.5±10%	10±5%	3.8±0.1
No.3 – 12 mm in diameter	0.7±10%	12±5%	3.8±0.1

8. Information about the sterile state of the medical device, information about the procedure for processing the medical device for reuse.

The medical device Disposable non-sterile dental cotton rolls Clean + Safe® is produced non-sterile and is intended for single use only. This single use device after dental procedures is not a subject to disinfection and its reuse is prohibited.

For invasive procedures, sterilization of the medical device Disposable non-sterile dental cotton rolls Clean + Safe® is carried out. Sterilization of the rolls is carried out by autoclaving at a temperature of 135 ° C for 30 minutes.

9. Instruction for use

1. Take a package of cotton rolls Clean + Safe® of the desired size depending on the clinical case.
2. Take a roll out of the package.
3. Check the shelf life, inspect visually integrity of the package and the device.
4. It is recommended to sterilize the rolls by autoclaving in at a temperature of 135 ° C for 30 minutes.
5. Place the roll into the opened mouth of the patient using dental tweezers or a special roll holder.
6. Carry or remove used cotton rolls using dental tweezers or a special roll holder.
7. After use dispose the medical device as appropriate.

10. Data on disposal or destruction of the medical device

Medical devices must be disposed in accordance with medical procedures adopted in healthcare institutions and organizations, and local, state or federal norms and regulations. The choice of safe disposal and destruction methods depends on capacity and field of medical

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

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organization, availability of waste decontamination/detoxification facilities, waste detoxification/disposal method used in the administrative territory (incineration, collection to waste dump, recycling).

Depending on degree of epidemiological, toxicological and radiation hazard the medical device "Disposable non-sterile dental cotton rolls Clean + Safe®" fall to category of epidemiologically hazardous medical waste: materials and instruments, objects contaminated with blood and/or other biological fluids.

After use, medical devices "Disposable non-sterile dental cotton rolls Clean + Safe®" are placed in special packages for medical waste differing in color depending on the hazard class of the waste (yellow). After filling no more than $\frac{3}{4}$ of the volume, the packages are tied up using screeds and transported to a temporary storage area. Transportation of garbage from institutions to the place of disposal is carried out by specialized companies.

Consumer package, defective devices, packages with violation of integrity and devices with expired shelf life refer to epidemiologically safe waste, similar in composition to household waste.

11. Data on labeling, packaging and scope of supply

11.1. Packaging

Multiple (consumer) package

The medical devices "Disposable non-sterile dental cotton rolls Clean + Safe®", Nos 1, 2 and 3 in quantity of 840, 600 and 450 pcs, respectively, are packed in a multiple (consumer) package – polyethylene bag. Materials and design of the package do not have a negative impact on its contents and preserve the devices when stored in dry, clean and well-ventilated rooms, provide the minimal risk of contamination when the package is opened, as well as protect the contents under normal conditions of transportation and storage.

Shipping package.

Cardboard box. Quantity of multiple (consumer) packages in a box: No.1 – 30 pcs, No.2 – 60 pcs, No.3 – 30 pcs.

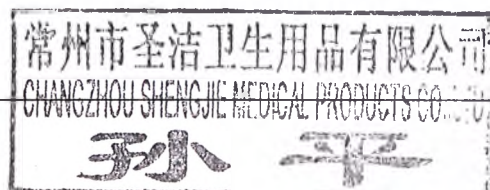
11.2. Labeling.

Multiple (consumer) package

The package must contain the following information:

- Name of the medical device.
- Manufacturer, address of production site.
- Importer (manufacturer's authorized representative), address, phone number.
- Sizes: No. (diameterXlength).
- Composition.
- Quantity of pieces in the package.
- Shelf life.

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®


- Number and date of Registration Certificate.
- Date of manufacturing (symbol)
- Batch or Lot No.(symbol)
- For single use only (symbol)
- CE Directives conformity (symbol)
- Non-sterile (symbol)
- Do not use if package is damaged (symbol)
- Sterilization mode
- Storage conditions "Temperature range", "Avoid direct exposure to sunlight", "Protect from moisture"(symbols)
- Disposal "Not allowed for normal disposal" (symbol)
- Declaration of conformity to GOST R sign. (symbol)

Shipping package

The package must contain the following information:

- Name of the medical device.
- Manufacturer, address of production site.
- Importer (manufacturer's authorized representative), address, phone number.
- Sizes: No. (diameterXlength), batch.
- Weight, g.
- Quantity of pieces in the package.
- Shipping conditions "Temperature range", "Avoid direct exposure to sunlight", "Protect from moisture"(symbols)
- Date of manufacturing (symbol)
- Shelf life.

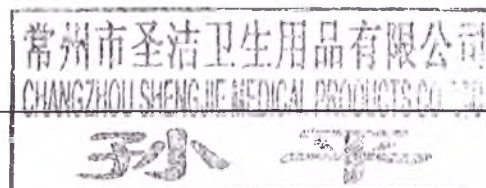Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

Description of symbols used in multiple (consumer), shipping package labeling:

Symbol	Description	Symbol	Description
	Date of manufacturing		Temperature range
	Lot number		Avoid exposure to sunlight
	CE marking		Not allowed for normal disposal
	For single use only		Protect from moisture
	Non-sterile		Declaration of conformity GOST R
	Do not use if package is damaged		

11.3. Scope of supply

- The medical devices "Disposable non-sterile dental cotton rolls Clean + Safe®" (choose the needed hardware version within the range of presented variants according to para.1 "Multiple (consumer) package").
- Instruction for use (part of accompanying documentation).

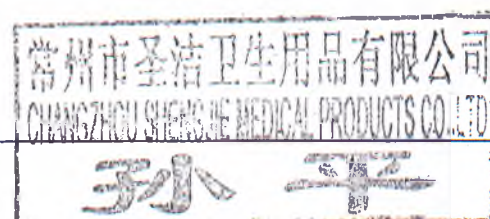
12. Transportation and storage conditions

The rolls are to be transported by all kinds of transport in covered vehicles, in accordance with the shipping rules applicable for such transport and precautions indicated on the shipping tare.

Transportation conditions:

- Air temperature: -20°C to +40°C
- Relative humidity: max. 80±2%;
- Atmospheric pressure: 755±20 mm Hg.

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

The devices must be stored in clean dry rooms at a distance of min.1 m from heating devices and must be protected against atmospheric actions and soil moisture. Packages are to be protected from contamination, dust, soaking and breakage. Keep away from children.

Shelf life of the medical device provided the storage and transportation conditions are observed is 5 years from the date of manufacture.

Storage conditions:

- Air temperature: -10°C to +40 °C;
- Relative humidity: max. 80±2%;
- Atmospheric pressure: 755±20 mm Hg.

Application conditions:

Air temperature in the healthcare facility: +15 °C – +25 °C;
Relative humidity: max. 80±2%;
Atmospheric pressure: 755±20 mm Hg.

13. Warranties

CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD, China, guarantees that the present medical device has been manufactured and packed with proper accurateness specified in Directive 93/42/EEC. CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD, China, has EC certificates that show that the quality management system implemented in CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD, China, meets requirements of Directive 93/42/EEC and ISO 13485.

CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD, China, will replace any package in case the defects have occurred in the process of manufacturing and packaging (evidence can be provided), and CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD", China, or its distributors have been informed of these defects.

The user bears responsibility for definition of device's fitness for use. CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD", China, and its distributors bear no responsibility for harm or damage of any nature directly or indirectly related to use of the device.

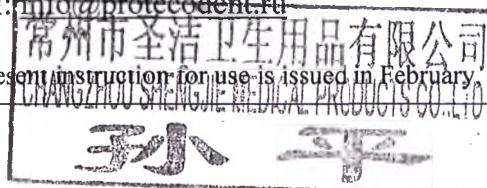
If the present device became defective within the warranty period, the only option and the only responsibility of CHANGZHOU SHENGJIE MEDICAL PRODUCTS CO., LTD", China, is to be troubleshooting and replacement of the device. The responsibility with respect to the device comes to an end, if any defect caused by transportation, storage or improper use of the user, as well as other factors beyond control of the manufacturer, have occurred.

Shelf life of the medical device provided the storage and transportation conditions are observed is 5 years from the date of manufacture.

Information on undesired events connected with use of the medical device " Disposable non-sterile dental cotton rolls Clean + Safe®" which have signs of an adverse event (incident), should be forwarded to the authorized representative of the manufacturer: "PROTECO", JSC, legal address: 192102, Saint Petersburg, Salova street 57, Building 1, Letter "I", tel.: / fax: +7 (812) 635-88-90, website: www.protecodent.ru, e-mail: info@protecodent.ru

Instruction for use of the medical device (2nd revision 2019):
Disposable non-sterile dental cotton rolls Clean + Safe®

The present instruction for use is issued in February, 2019.

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(Translation)

(2019) CCZ JW Zi, No. 268

Applicant: Changzhou Shengjie Medical Products Co., Ltd., Domicile: No. 216, Liangchang Road, Jintan District, Changzhou City, Jiangsu Province.

Legal representative: Sun Ping, male, born on November 12, 1970, ID No. 320422197011123855.

Agent: Sheng Bin, male, born on November 24, 1989, ID No. 320482198911243810.

Issue under notarization: Seal

This is to certify that the seal of "Changzhou Shengjie Medical Products Co., Ltd. Sun Ping" affixed on the English copy attached hereto presented to the notary by Sheng Bin, the agent of Changzhou Shengjie Medical Products Co., Ltd., is found to be authentic.

Notary: Guo Wenwei (Signature)
Changzhou Notary Public Office (Seal)
Changzhou City, Jiangsu Province
The People's Republic of China
May 21, 2019

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(Translation)

(2019) CCZ JW Zi, No. 269

Applicant: Changzhou Shengjie Medical Products Co., Ltd., Domicile: No. 216, Liangchang Road, Jintan District, Changzhou City, Jiangsu Province.

Legal representative: Sun Ping, male, born on November 12, 1970, ID No. 320422197011123855.

Agent: Sheng Bin, male, born on November 24, 1989, ID No. 320482198911243810.

Issue under notarization: The translated copy in conformity with the original copy

This is to certify that the English translated copy of NOTARIAL CERTIFICATE (2019) CCZ JW Zi, No. 268 attached hereto is in conformity with the original Chinese copy.

Notary: Guo Wenwei (Signature)
Changzhou Notary Public Office (Seal)
Changzhou City, Jiangsu Province
The People's Republic of China
May 21, 2019

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