Instructions for Use

For dental use only

CELTRA DUO Starter Kit for preparation of glass-ceramic restorations

Manufactured by

**DeguDent GmbH, Rodenbacher Chaussee 4,
63457 Hanau-Wolfgang, Germany**

1	Description	2
2	Photos	4
3	Composition and materials	7
4	Size and weight	9
5	Technical characteristics	10
6	Compatible stains and glazes	12
7	Compatible luting cements	12
8	General safety notes	12
9	STEP-BY-STEP INSTRUCTIONS	13
10.	Cleaning, disinfection and sterilization	18
11.	Packaging	18
12.	Storage and shelf-life	19
13.	Transportation	19
14.	Environmental requirements	19
15.	Conditions of use	19
16.	List of applied International Standards	19
17.	Maintenance and repair	19
18.	Disposal	19
19.	Symbols used in labelling:	20
20.	Manufacturer's warranties	20
21.	Authorized Representative	20

DeguDent
DeguDent GmbH
Rodenbacher Chaussee 4
63457 Hanau-Wolfgang
Germany

1 Description

1.1 Name of the medical device:

CELTRA DUO Starter Kit for preparation of glass-ceramic restorations

I. CELTRA DUO glass-ceramic material in blocks in Celtra Duo Starter Kit, including

- CELTRA DUO glass-ceramic material in blocks: low translucency (LT) A1- 2 pieces, A2- 3 pieces, A3- 3 pieces, A3.5- 2 pieces, B2- 1 piece; high translucency (HT) A1-2 pieces, A2- 3 pieces, A3 - 3 pieces,
- Celtra Universal Overglaze in 5 g jar or Dentsply Sirona Universal Overglaze in 5 g jar – 1 piece,
- MagicStain Brush – 1 piece,
- Firing Pad – 3 pieces per 1 pack,
- Illustrated technical guide – 1 piece.

1.2 Information about medical device manufacturer/developer:

«DeguDent GmbH», Germany

Rodenbacher Chaussee 4, 63457 Hanau-Wolfgang, Germany

Tel. +49 6181 59-50

1.3 Addresses of medical device manufacturing sites:

- 1) DeguDent GmbH, Rodenbacher Chaussee 4, 63457 Hanau-Wolfgang, Germany
- 2) DENTSPLY International, Inc., DENTSPLY Ceramco, 570 West College Avenue, York, Pennsylvania (PA), 17401, USA

1.4 Intended use

This product is designed for processing with CAD-CAM systems, for the manufacture of dental restorations, for example, partial and all-ceramic crowns, inlays, onlays and veneers, and also may be used as a material for fully contoured applications.

1.5 Scope

Dentistry

1.6 Indications

CELTRA DUO is a system of fully crystalized zirconia-reinforced lithium silicate glass ceramics processed through CAD-CAM technique for the fabrication of single unit dental restorations, for example all-ceramic crowns, inlays, onlays and veneers.

CELTRA DUO can be used as a material for fully contoured applications.

1.7 Contraindications

CELTRA DUO dental material is contraindicated in the following cases:

- Installation of fully veneered crowns on molars
- Very deep subgingival preparation
- Patients with a significantly reduced number of remaining teeth
- Patients with bruxism

1.8 Medical device classification

Risk class 2a

GMDN code: 16187 Dental Ceramic

1.9 Description:

CELTRA DUO is a zirconia-reinforced lithium silicate (ZLS) delivered at final strength. It can be delivered directly after finishing and polishing. The milled restorations have a flexural strength of 210 MPa. Alternatively, an additional stain and glaze firing will increase the material's flexural strength to 370 MPa.

1.10 Delivery forms

I. CELTRA DUO glass-ceramic material in blocks in Celtra Duo Starter Kit, including

- CELTRA DUO glass-ceramic material in blocks: low translucency (LT) A1- 2 pieces, A2- 3 pieces, A3- 3 pieces, A3.5- 2 pieces, B2- 1 piece; high translucency (HT) A1-2 pieces, A2- 3 pieces, A3 - 3 pieces,
- Celtra Universal Overglaze / Dentsply Sirona Universal Overglaze in 5 g jar – 1 piece,
- MagicStain Brush – 1 piece,
- Firing Pad – 3 pieces per 1 pack,
- Illustrated technical guide – 1 piece.

Contents of delivery

The product is supplied together with the Instructions for use.


2 Photos

2.1 Zirconia-Reinforced Lithium Silicate (ZLS) Dental material CELTRA DUO in blocks for preparation of glass-ceramic restorations in Celtra Duo Starter Kit

Pic. 1.

Pic. 2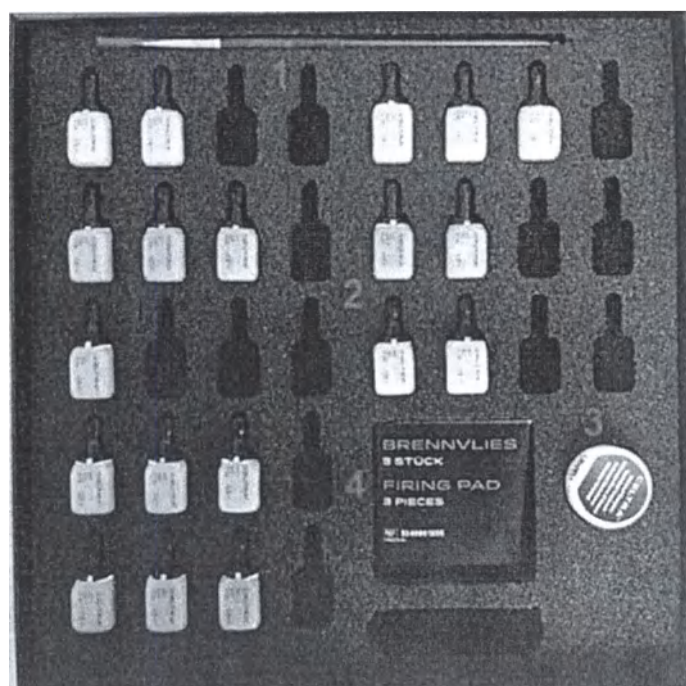

Celtra Duo Starter Kit:

1 – MagicStain brush


2 – CELTRA DUO glass-ceramic material in blocks, shades: low translucency (LT) A1 – 2 pieces, A2 – 3 pieces, A3 – 3 pieces, A3.5 – 2 pieces, B2 – 1 piece; high translucency (HT) A1 – 2 pieces, A2 – 3 pieces, A3 – 3 pieces (see paragraph 2.3)

3 – Celtra Universal Overglaze in 5 g jar or Dentsply Sirona Universal Overglaze in 5 g jar – 1 piece


4 – Firing Pad – 3 pieces per pack.

2.2 CELTRA DUO glass-ceramic material in blocks: comparison between two holders*.

ocks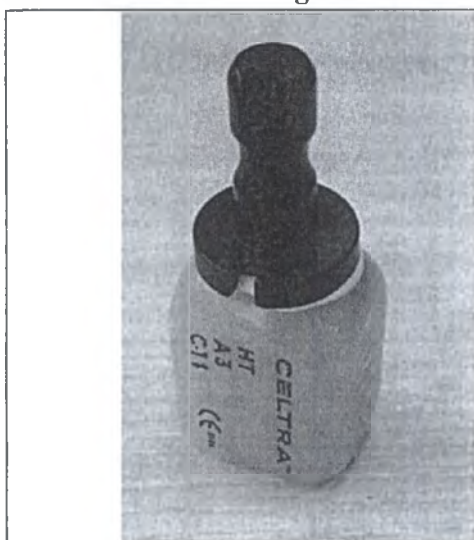

Pic.1 Holder with black coating

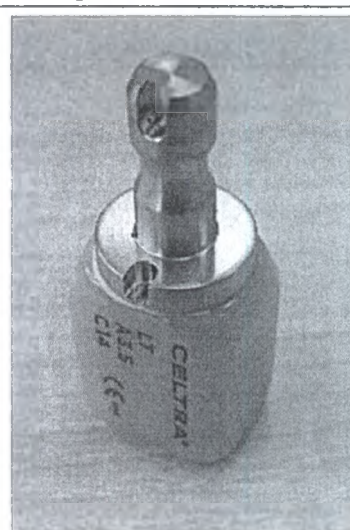
Pic.2 Stainless steel holder

* - difference only in the holder material

2.3 Glaze: comparison of glaze jars **



Pic.5 CELTRA Universal Overglaze in 5 g jar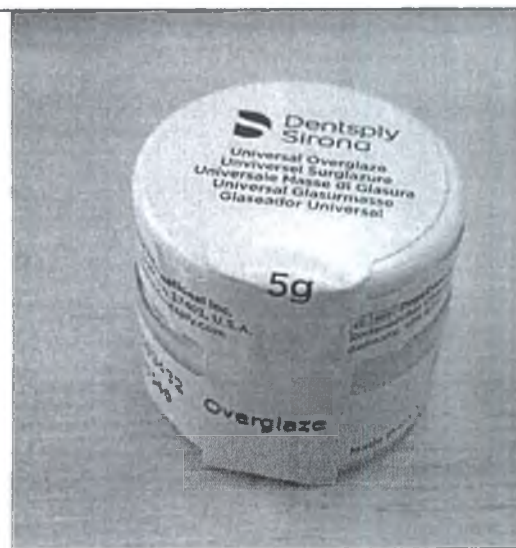

Pic.6 Dentsply Sirona Universal Overglaze in 5 g jar


** - Difference in the name of the glaze, appearance of the package and composition of the glaze

2.4 Firing Pad – 3 pieces per pack ***.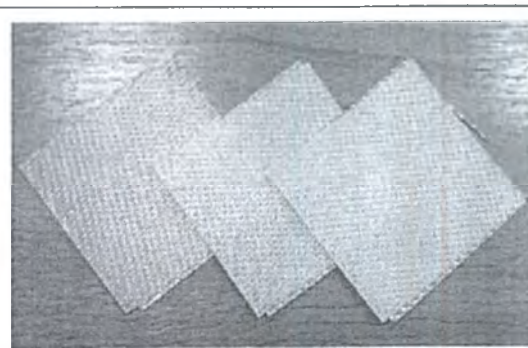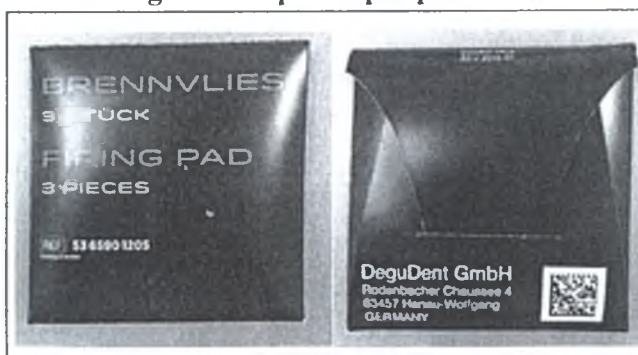

A1

1

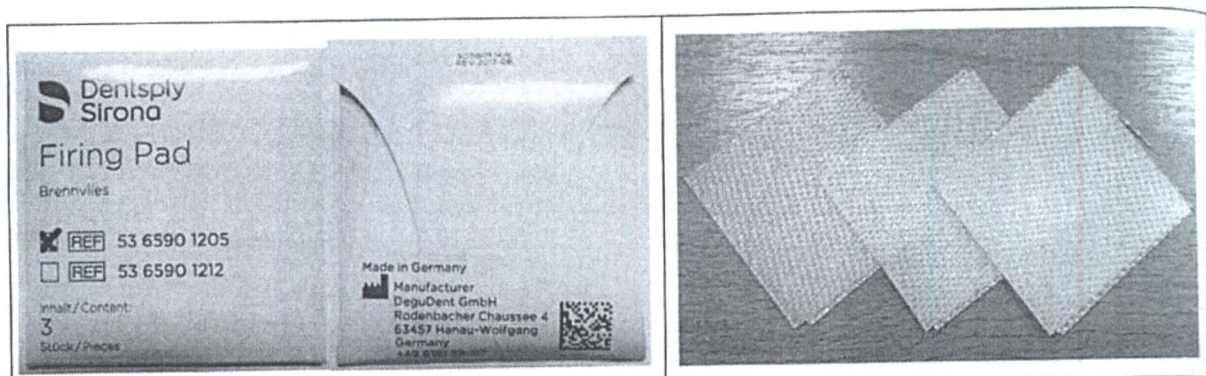*** - difference in the package layout only

3 Composition and materials

3.1 CELTRA DUO glass-ceramic material in blocks

3.1.1 The overall composition of CELTRA DUO material in all shades in the range format.

Component
SiO ₂
P ₂ O ₅
Al ₂ O ₃
Li ₂ O
K ₂ O
ZrO ₂
CeO ₂
Tb ₄ O ₇
Na ₂ O
V ₂ O ₅
Pr ₆ O ₁₁
MnO ₂
Nd ₂ O ₃
Fe ₂ O ₃
Er ₂ O ₃

3.1.2 Holder

1) Silver holder: stainless steel

or

2) Black holder: Lead-free alloy with black coating. Black coating is achieved by means of anodic oxidation.

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3.2.1 Celtra Universal Overglaze

Overglaze appearance: paste. Paste consists of 70% powder and 30% gel.

Powder composition is shown below:

Component
SiO ₂
Al ₂ O ₃
K ₂ O
Na ₂ O
Li ₂ O
CaO
BaO
MgO
CeO ₂
B ₂ O ₃
SnO ₂
Sb ₂ O ₃
TiO ₂
F ⁺

The composition of the gel used to knead the paste: ammonium nitrate, butylene glycol, propylene glycol, silicon dioxide, colloid, tylose H.

3.2.2 Dentsply Sirona Universal Overglaze

Overglaze appearance: paste. Paste consists of 70% powder and 30% gel.

Powder composition is shown below:

Component
SiO ₂
Al ₂ O ₃
Na ₂ O
K ₂ O
Li ₂ O
BaO
CaO
B ₂ O ₃
CeO ₂
Tb ₂ O ₃

TiO ₂
Sb ₂ O ₃
SnO ₂

of

The composition of the gel used to knead the paste: ammonium nitrate, butylene glycol, propylene glycol, silicon dioxide, colloid, tylose H.

3.2.3 Jar with the lid

Jar	Glass
Lid	Low density polyethylene

Parameters: height of the jar 22.0 ± 0.8 mm; external diameter: 30.0 ± 0.5 mm

3.3 Brush


Brush	Bristle of the brush	Siberian Weasel tail Finest Kolinsky Hair
	Tip	Nickel plated brass
	Handle	Wood (birch tree)
	Protective cap for bristle	Low density polyethylene

3.4 Firing pads



SiO₂

4 Size and weight

4.1 CELTRA DUO glass ceramic material in blocks

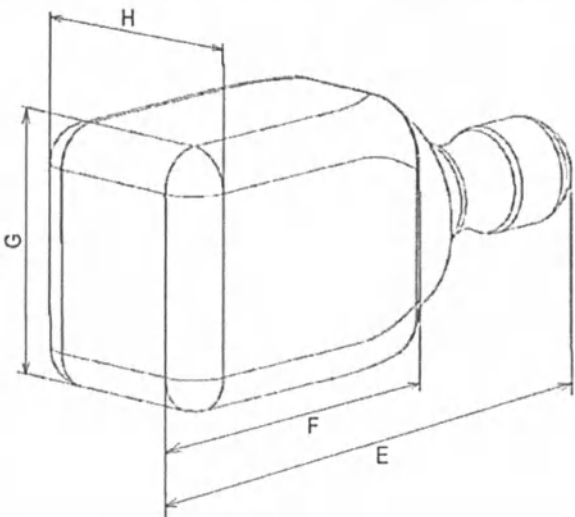	E: 33.5 mm (+2.4 mm; - 0.1 mm) F: 18 mm (+2.3 mm; - 0 mm) G: 14.7 mm (+0.5 mm; - 0 mm) H: 12.5 mm (+0.4 mm; - 0 mm)
	Weight of the blocks: (7,8 – 10 g)

4.2 Celtra Universal Overglaze / Dentsply Sirona Universal Overglaze jar

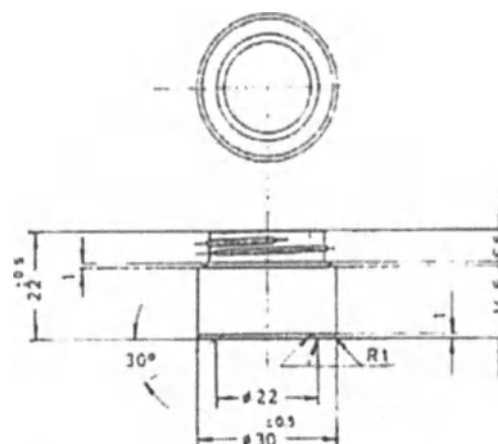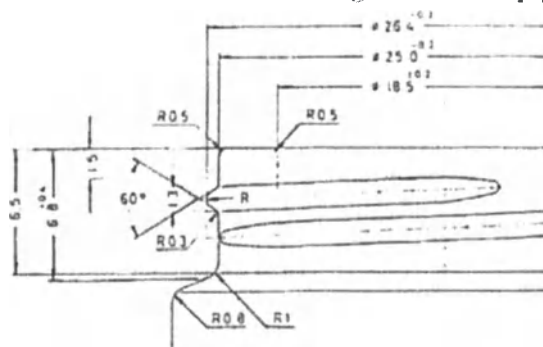Nominal volume: 6.5 ml $\pm$ 0.5 ml.

Overglaze weight $\geq$ 5 g

Jar weight 20.1 $\pm$ 0.3 g

Jar with Overglaze weight $\geq$ 25.1 g

4.3 Brush

Handle: 132 $\pm$ 1 mm; Tip: 29 $\pm$ 1 mm; Bristle: 5.5 $\pm$ 1 mm, Protective cap for bristle: 21 mm x 220 mm $\pm$ 1 mm

Brush weight 2.94 $\pm$ 0.3 r

Weight of protective cap for bristles: 0.16 $\pm$ 0.16 g

4.4 Firing pads

Size: 50 x 50 mm $\pm$ 2 mm

Weight of the pads including package 7.4 $\pm$ 0.74 r

Weight of a single pad is Pad 1.6 $\pm$ 0.16 r.

5 Technical characteristics

5.1 Zirconia-Reinforced Lithium Silicate (ZLS) dental material CELTRA DUO in Blocks, for preparation of glass-ceramic restorations with accessories.

Parameter	Measurement
Flexural strength	No less than 210 MPa**
Chemical solubility	<0.05%
Coefficient of thermal expansion (CTE)	$(11.6 \pm 0.5) \times 10^{-6} \text{K}^{-1}$
Voids rating / porosity	On a surface area of 1 mm ² , the number of pores with a diameter greater than 30 μm should not exceed 16 pieces. and the number of pores with a diameter of 40 to 150 μm should not exceed 6 pieces. There should be no pores larger than 150 μm in diameter.
Linear shrinkage	Less than 16%
Resistance to coloration, the number of spots on the samples when aged in dyeing solution	0

Density of the prepared material	Not more than 2.6 g/cm ³
----------------------------------	-------------------------------------

** The flexural strength can be increased up to > 300 MPa with the help of an additional glaze firing process.

5.2 Celtra Universal Overglaze in 5 g jar.

Indicator	Measurement*
Appearance	Opaque paste
Flexural strength	≥100 MPa
Chemical solubility	<0.05%
Coefficient of thermal expansion (CTE)**	$(7.9 \pm 0.1) \times 10^{-6} \text{K}^{-1}$
Glass transition temperature, °C**	517±0.7
Softening temperature, °C**	576±0.1
Voids rating / porosity	On a surface area of 1 mm ² , the number of pores with a diameter greater than 30 μm should not exceed 16 pieces, and the number of pores with a diameter of 40 to 150 μm should not exceed 6 pieces. There should be no pores larger than 150 μm in diameter.
Linear shrinkage	Less than 16%
Resistance to coloration, the number of spots on the samples when aged in dyeing solution	0

* In accordance with the requirements of ISO 6872, Dentistry - ceramic materials.

** The result was calculated as the average of the results obtained for 2X and 4X firing.

5.3 Dentsply Sirona Universal Overglaze in 5 g jar

Indicator	Measurement*
Appearance	Opaque paste
Flexural strength	≥100 MPa
Chemical solubility	<0.05%
Coefficient of thermal expansion (CTE)**	$(7.9 \pm 0.1) \times 10^{-6} \text{K}^{-1}$
Glass transition temperature, °C**	517±0.7
Softening temperature, °C**	576±0.1
Voids rating / porosity	On a surface area of 1 mm ² , the number of pores with a diameter greater than 30 μm should not exceed 16 pieces, and the number of pores with a diameter of 40 to 150 μm should not exceed 6 pieces. There should be no pores larger than 150 μm in diameter.
Linear shrinkage	Less than 16%
Resistance to coloration, the number of spots on the samples when aged in dyeing solution	0

* In accordance with the requirements of ISO 6872, Dentistry - ceramic materials.

** The result was calculated as the average of the results obtained for 2X and 4X firing.

5.4 Brush

Not applicable due to the specific nature of the product.

5.5 Firing pads

Heat resistance up to 1200 °C.

6 Compatible stains and glazes

CELTRA DUO restorations are compatible with CELTRA Universal Overglaze / Dentsply Sirona Universal Overglaze for customizing and glazing (Overglaze may be supplied with blocks; dyes are purchased separately). The use of other stain and glaze systems is not recommended (see Warnings). Use of other correction porcelain with CELTRA DUO is at the discretion and sole responsibility of the dental practitioner.

7 Compatible luting cements

CELTRA DUO restorations are compatible with conventional adhesive bonding resin cement systems, including all DENTSPLY adhesive resin cement systems (available separately, see complete Directions for Use). Full coverage crown restorations are compatible with selfadhesive resin cements, including all DENTSPLY selfadhesive resin cements (available separately, see complete Directions for Use). Inlays and onlays may be cemented with selfadhesive resin cements, however, adhesive bonding is recommended. Use of other cements or cement systems with CELTRA DUO is at the discretion and sole responsibility of the dental practitioner.

8 General safety notes

Be aware of the following general safety notes and the special safety notes in other sections of these directions for use.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

8.1 Warnings

- If properly processed and used, adverse effects of these medical products are highly unlikely. However, reactions of the immune system (such as allergies) or localized paresthesia (such as an irritating taste or irritation of the oral mucosa) cannot be completely ruled out as a matter of principle. In case of skin sensitization or rash, discontinue use and seek medical attention.
- CELTRA DUO restorations are not suitable for patients with clinical symptoms of parafunctional habits or bruxism (see Contraindications).
- Do not inhale dust particles during grinding. Wear suitable protective mask.
- Avoid use of thick ceramic pins or third-party auxiliary firing pastes, correction porcelains or stains and glazes. Such use may result in cracks in CELTRA DUO restorations.
- In patients with hypersensitivity to any of the ingredients, this medical device may not be used at all or only under the particular scrutiny of the dentist or physician in charge.
- Other ceramic masses, other than CELTRA corrective mass and Celtra Universal Overglaze/Dentsply Sirona Universal Overglaze dyes and glazes, may not correspond to

CELTRA DUO block by thermal characteristics and have an improper CTE value. Therefore, they are not suitable for application.

- CELTRA DUO is compatible with and thermally matched to CELTRA Correction Porcelain for desired contour adjustment (if necessary).

8.2 Precautions

- This product is intended to be used only as specifically outlined in these Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and is the sole responsibility of the practitioner.
- Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
- Contact with saliva, blood and/or some astringent solutions during adhesive procedures may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- CELTRA DUO restorations require adequate preparation reduction and restoration thickness. Insufficient wall thickness may lead to premature failure.
- CELTRA DUO restorations are designed to be polished and/or glazed before insertion. Direct insertion without polishing or glazing may lead to excessive wear.
- Use only in well ventilated areas.

Interactions

- CELTRA DUO blocks are designed to be fabricated using a Sirona CAD/CAM system. Cutting blocks using noncompatible CAD/CAM system may lead to inadequate or unacceptable restorations.
- Adjustments should be made with water-cooled diamond instruments. Avoid use of tungsten carbide cutters.

8.3 Adverse reactions





No adverse reactions have been reported for CELTRA DUO. Should you hear or receive information about any adverse effects – even when doubtful – please notify the manufacturer.

9 STEP-BY-STEP INSTRUCTIONS

9.1 Preparation:

Proper reduction of the hard tissue of the tooth during preparation is essential for maximizing the strength, shade and retention of the finished restoration. When preparing anterior or posterior teeth, the anatomical form has to be reduced as shown.

Minimum wall thickness: The following diagram shows the specified minimum wall thickness for each indication. The minimum wall thickness must still be ensured after all manual adjustments have been made.

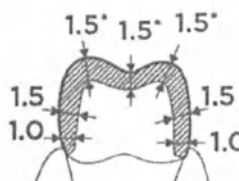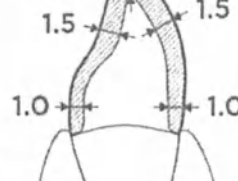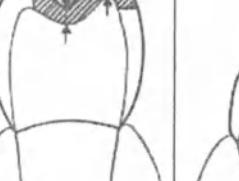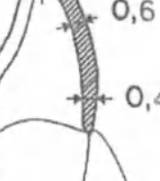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* 1.5-2mm for frameworks that are exclusively polished

9.1.1 Inlays and onlays

A conventional inlay/onlay design is recommended. Do not prepare undercuts. Ensure that the cavity walls form an angle of 5-6 degrees with the long axis of the tooth. Ensure that all sharp edges and angles are rounded. In centric and dynamic occlusion, reduce incisally/occlusally by 1.5 to 2 mm.

CELTRA DUO inlays and onlays are ideally delivered by fully adhesive cementing.

Alternatively, highly retentive inlay or onlay restorations may be cemented with selfadhesive resin cements.

9.1.2 Full crowns

■ Ensure that there is an axial reduction of 1.0 to 1.5mm with the walls form an angle of 5-6 degrees with the long axis of the tooth. In centric and dynamic occlusion, reduce incisally/occlusally by 1.5mm. The lingual shoulders must be extended at least 1.0mm into the proximal contacts surfaces. It is recommended to use a shoulder preparation without a bevel: All angles must be rounded, and the preparation surfaces must be smooth.

CELTRA DUO full crowns can be delivered by fully adhesive or selfadhesive cementing. See complete Directions for Use of the adhesive and cement products.

9.1.3 Veneers

- The standard reduction is 0.6 mm for the labial surface and 0.4mm in the gingival area (since the enamel is thinner in this region). Reduce the labiolingual incisal angle by 1.0 to 1.5mm. The preparation margins should be located in enamel. A chamfer or rounded-shoulder preparation is recommended for all veneer margins. Proximal extensions must be located far enough proximally to conceal preparation margins from site and to avoid proximal gingival undercuts.
- CELTRA DUO veneers are delivered by fully adhesive cementing. See complete Directions for Use of the adhesive and cement products. Selfadhesive cementing is not recommended for veneer restorations.

9.2 CAD/CAM processing

CELTRA DUO restorations are produced with CEREC or inLab CAD/CAM systems by Sirona. If you have any questions about these systems, please contact Sirona Dental Systems GmbH.

9.2.1 Software requirements

- CELTRA DUO is supported by CEREC version 4.2 and above. When using previous software versions, use of the IPS e.max® system standard grinding program is recommended (do not use the e.max® system speed program).
- In this case, the minimum wall thickness will have to be adapted to the CELTRA DUO recommendation; see Preparation

9.2.2 Grinder and block sizes

In the CEREC software, select the CELTRA DUO material. The Sirona CELTRA® CAM grinding unit will prompt you to insert a CELTRA DUO Block. For detailed processing steps, please consult the Instructions for Use and technical manuals of the appropriate CAD/CAM systems. Make sure to follow the manufacturers' recommendations.

9.2.3 Finishing the restoration (Without the optional staining and/or glaze firing)

Clean the restoration in an ultrasonic water bath or with a steam cleaner. Carefully blow-dry the restoration in an air stream.

2. Try in the final restoration to examine its fit. If necessary, adjust the proximal and occlusal contacts.
3. When making adjustments to the ceramic material, make sure to avoid overheating (e.g. by providing irrigation). Ideally, all adjustments should be made with water-cooled instruments.
4. Diamond polishing bodies (< 60 pm) are recommended for polishing the occlusal surfaces. The use of tungsten carbide burs is not recommended.
5. Smooth the remaining surfaces of the restoration with a rubber polisher at low speed and minimum pressure.
6. Thoroughly clean the restoration.

9.3 Staining, Glazing and Applying Corrections to milled CELTRA DUO restorations

- If milled restorations require no corrections, proceed to step 9.3.1 for optional staining and glazing instructions.
- If milled restorations require corrections, proceed to step 9.3.2 for Application of CELTRA Correction Porcelain instructions.

9.3.1 Instructions for optional staining and glaze-firing of CELTRA DUO CAD/CAM restorations

Custom staining of CELTRA DUO restorations can be performed using CELTRA Universal Stains and Glaze instructions below.

For best results, any sanding marks on the surface should be removed with a suitable ceramic-coated rubber polishing wheels (not too hard; e.g. DeguDent TwisTec® ceramic polishers white or yellow) or polished smooth at moderate speed (not exceeding 8,000 rpm). The setting of the furnace parameters depends on the type of furnace, so the programming may vary slightly.

Procedure: For CELTRA Universal / Dentsply Sirona Universal Stains and Glaze application the following steps must be observed:

- Before applying the stains and glaze, the restoration must be clean and free of grease. Clean the surface of the restoration with a steam cleaner or submerge for 10 minutes in an ultrasonic cleaner with distilled water. Any contamination after cleaning must be prevented.
- Place the cleaned crown on the prepared DENTSPLY Prosthetics die material. To achieve the correct shade, prepare the die material according to the shade recommendations in a Table 1 below.

Table 1. Recommended Die Material Selection

Shade	A1	A2	A3	A3.5	B2
Die material	F1	F12	F10	F9	F11

* Refer to CELTRA DUO Body & Enamel Stain Recommendations for CELTRA LT Blocks (available separately) for staining instructions to achieve all 16 shades from the five LT CELTRA DUO blocks (A1, A2, A3, A3.5 and B2) currently available.

- Mix the mass thoroughly with spatula. Do not use a metal spatula. Withdraw desired amount of Celtra glaze and place it on the mixing palette. For a thinner consistency dilute the material with CELTRA Stain and Glaze Liquid.
- Apply a thin layer of glaze to the entire crown surface using a brush. Apply a thin layer of stain in the gingival area and spread with a brush toward the incisal region (or as needed). Check the shade using the shade tab. Use the brush to increase or decrease the stain amount to achieve the desired shade.
- Again using the brush, apply a thin layer of incisal stain to the incisal edge. Check the shade using the shade ring. Use the brush to increase or decrease the stain amount to achieve the desired esthetic incisal effects.

Notes:

- A more intensive shade effect can be achieved by repeating cycles of applying and firing the material. However, the use of too many layers of stain may result in an unnatural appearance.
- The cusps and fissures can be individually characterized with stains.
- The basic shade is determined based on the Vita shade groups (A, B, C, D) (see Table 2).

Table 2. Assignment of Shade Groups

Vita Shade Group	Bleached	A	B	C	D
Universal Stain Shade	Stain 0	Stain 1	Stain 2	Stain 3	Stain 4

Table 3. Use of Incisal Stains

Stain	Tooth Shade
Incisal Stain i1	A1, A2, A3, B1, B2, B3, B4, D2, D3, Bleached
Incisal Stain i2	A3.5, A4, C1, C2, C3, C4, D4

9.3.2 Use of glazes and stains by third-party manufacturers

Only the native range of glaze and stains (Celtra Universal / Dentsply Sirona Universal Stains&Glaze) in conjunction with the appropriate liquid is recommended for customizing and glazing CELTRA DUO., as its compatibility with CELTRA DUO has been tested and is guaranteed require. Glazes and stains by third-party manufacturers may require other firing temperature and cycles than those recommended for CELTRA DUO.

9.3.3 Firing — CELTRA Universal / Dentsply Sirona Universal Stains and Glaze Only

General: Firing objects must be placed on a carrier within the ceramic furnace. Placing the objects on plain firing pad with firing tray or on firing pad with a platinum foil is recommended.

9.3.4 General firing recommendations

- The optional glaze firing will increase the flexural strength of CELTRA DUO to 370 MPa. To increase the flexural strength it is only necessary to fire the restoration up to 820°C. The application of glaze is an option but not necessary. In case of firing without glaze skip the Predrying and drying steps of the firing program. Make sure to polish the restoration first before firing.
- Starting temperature 500°C
- Place the object on the firing tray/firing table of the furnace.

Table 4	Glazing	
	1 st glaze firing	2 nd & subsequent glaze firing
Pre-drying	2:00 min	2:00 min

Drying¹	2:00 min	2:00 min
Preheating	2:00 min	2:00 min
Start Temp	500 °C	500 °C
Heating rate	55°C/min	55°C/min
Final Temp	820°C	770°C
Vacuum	off	off
Hold time	1:30 min	1:30 min

¹ *Drying time depends on the type of furnace.*

- Additional glaze firings may be performed at 770°C in order to accentuate the shade or to correct it with glaze, or increase the gloss. It is necessary to coat the entire surface with CELTRA Universal / Dentsply Sirona Universal Glaze (available separately) to obtain a uniform glossy finish.
- Finally, check the shade against a shade tab and adjust if necessary.
- Note: After the first firing, the restoration may not appear sufficiently glazed. In this case, apply a thin layer of CELTRA Universal / Dentsply Sirona Universal Glaze to the entire surface with a brush. After applying the glaze, perform a second stain/ glaze firing as defined in the firing recommendations. Additional stain/glaze firing cycles can be performed using the same parameters.

9.3.5 Application of CELTRA Correction Porcelain to CELTRA DUO restorations

Procedure: Mix correction porcelain with distilled water or Ceramco Modeling Liquid U to desired consistency. Apply porcelain where applicable. Fire to complete the restoration following the procedures and firing programs listed below:

9.3.5.1 First Glaze & Correction Porcelain on a Crown - Apply CELTRA Universal Glaze over the entire CELTRA crown then apply Celtra Correction porcelain to the required areas and fire at 820°C according to 1st correction firing program a) in Table 5. Polish to a high gloss, if needed.

9.3.5.1.a Second Glaze: If after first firing the required glaze appearance is not achieved, re-apply glaze, and fire according to 2nd & subsequent glaze firing program in Table 5.

Table 5: Correction

Table 5	Correction	
	a) 1 st correction firing/ b) 2 nd correction firing	2 nd & subsequent glaze firing
Pre-drying	2:00 min	2:00 min
Drying¹	2:00 min	2:00 min
Preheating	2:00 min	2:00 min
Start Temp	500 °C	500 °C
Heating rate	55°C/min	55°C/min
Final Temp	a) 820 °C /b) 810°C	770 °C
Vacuum	on 1:00 min	off
Hold time	0:30 min	1:30 min

¹ *Drying time depends on the type of furnace.*

9.3.5.2 Second Application of Glaze & Correction Porcelain (or application of Correction Porcelain after previous glazing): If a second application is needed: apply CELTRA Universal / Dentsply Sirona Universal over the entire CELTRA DUO crown, then apply small amount of Correction Porcelain to the desired areas. Fire at 810°C according to 2nd correction firing program b) in Table 5.

Note: Never fire the 2nd correction with 820°C! Contour the areas where correction porcelain was applied, if needed, and polish to a glaze finish.

9.3.5.3 Third Glaze (optional): If after firing the required glaze appearance is not achieved, re-apply glaze over the entire unit, and fire following 2nd & subsequent glaze firing program in Table 5.

9.4 Cementing

9.4.1 Preparation of the CELTRA restoration

- Clean the restoration with an ultrasonic or steam cleaner or with alcohol.
- Apply 5% hydrofluoric acid etching gel (Available separately, see manufacturer's complete Directions for Use) to the interior of the restoration only and allow to soak for 30 seconds.
- CAUTION: Follow manufacturer's precautions. Do not allow tissue or eyes to come into contact with the acid! Remove the hydrofluoric acid as per the manufacturer's instructions.
- Dry the restoration in an air stream.
- It is recommended to silanize the etched surfaces immediately.
- At chairside, apply silane only to those surfaces required for adhesive cementing.
- Allow to soak for 60 seconds. If the silane layer is no longer liquid, add more silane. Blow-dry in a powerful air stream. (Recommended material: Calibra® Silane, Available separately, see complete Directions for Use).

9.4.2 Cementation

Depending on the indication, CELTRA DUO restorations can be selfadhesively or fully adhesively cemented. Compatible time-proven adhesive cementing materials are available as part of the Dentsply Sirona range of products. Cements are available separately.

	Selfadhesive	Fully adhesive
Inlays	R	HR
Onlays	R	HR
Crowns	HR	HR
Veneers		HR

R = recommended HR = highly recommended

CEREC and inLab are not trademarks of DENTSPLY International, Inc.

Directions for use in other languages: www.dentsply-degudent.com

10. Cleaning, disinfection and sterilization

The products are supplied non-sterile. Sterilization of dental material and accessories is not required.

11. Packaging

- Consumer packaging

CELTRA Duo is a ceramic material that is not sensitive to moisture, heat or chemicals. During the milling of the crowns and bridges (the final products that come in contact with the patient), the surface of the workpiece is removed, so any surface contamination will not be critical to the patient's safety.

Packaging provides protection against mechanical wear or contamination during transport and storage.

The packaging is represented by a cardboard box.

The packaging of CELTRA DUO starter kit also includes polyurethane inserts.

Firing pads – cardboard packaging.

- Transportation packaging

Consumer packages with the product are packed in a large cardboard box (if necessary, a bubble film is used), which is then sealed with adhesive tape.

12.Storage and shelf-life

Store in a dry place between +10 °C - +35 °C and relative humidity <80%.

Protect from moisture.

Do not use after expiration date.

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

Shelf-life of Celtra Duo blocks: 15 years since the manufacturing date.

Shelf-life of Celtra Universal Overglaze / Dentsply Sirona Universal Overglaze: 4 years since the manufacturing date.

13.Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 10 °C and 35 °C and relative humidity <80%.

14.Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

15.Conditions of use

The product is properly maintained in the mouth, in the presence of the body fluids at nominal temperatures from 32° to 42° C. This product should be used only by qualified dental personnel in clinical environment.

16.List of applied International Standards

ISO 6872:2015-06	Dentistry - Ceramic materials
EN ISO 13485:2012	Medical devices. Quality management systems. Requirements for regulatory purposes (ISO 13485:2003)
EN ISO 13485:2012/AC2012	Medical devices - Quality management systems - Requirements for regulatory purposes - Technical Corrigendum 1 (ISO 13485:2003+Cor 1:2009)
EN ISO 14971:2012	Medical devices. Application of risk management to medical devices (ISO 14971:2007, corrected version 2007-10-01)
EN 1641:2009	Dentistry. Medical devices for dentistry. Materials
EN ISO 7405:2008 + A1:2013	Dentistry. Evaluation of biocompatibility of medical devices used in dentistry
EN ISO 10993-1:2009 + AC:2010	Biological evaluation of medical devices -- Part 1: Evaluation and testing within a risk management process

17.Maintenance and repair

The products are not subject to repair. Special maintenance is not required.

18.Disposal

Disposal is carried out in accordance with local legislation and protocols of a specific medical facility.

Dispose in accordance with the sanitary rules and regulations of the Russian Federation, 2.1.7.2790-10 SanPiN "Sanitary requirements for handling of medical waste." Hazard category of medical waste - B.

This product should be used only by qualified dental personnel in clinical environment.

19. Symbols used in labelling:

Symbol	Description
	Reference number
	Serial number (batch/lot number)
	Expiration date
	for Dental use only (Rx);
	Name and address of manufacturer
	Store in dry place
	Consult Instruction for Use
	CE mark
	Caution: consult the Instructions for Use

20. Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Failure to follow the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

21. Authorized Representative

Dentsply Sirona LLC, 115432, Moscow, Andropov prospect 18, building 6, offices 1 39 40.
Tel.: +7 (495) 725-10-87

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Eberhard Uhlig
Notar

Document Register No.: 1050/2018 U

I herewith certify the signature enforced in front of me of

Mr. Volker Wagner, born on May 25, 1969,
with registered office at Rodenbacher Chaussee 4, 63457 Hanau.

Mr. Volker Wagner is acting as Prokurist (holder of a special statutory power of representation) to represent the company **DeguDent GmbH**, with registered office at Hanau.

I hereby certify on the basis of inspection on 29.10.2018 of the electronic commercial register at the local court in Hanau, HRB 7182, that **Mr. Volker Wagner** is entitled to represent the company **DeguDent GmbH**.

Mr. Volker Wagner is personally known to the notary.

I do establish at the same time, that the interested parties denied the question of a prior involvement according to § 3 Sec. No. 7 BeurkG.

Hanau, October 29th, 2018
(Rodenbacher Chaussee 4)

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