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Auto Refract / Keratometer

Operation Manual

Please read and understand this operation manual prior to use of the device.

MEDIZS Inc.

<http://www.medizs.com>

5-5 Check setting

- Please check the information such as VI, CYL format, SPH unit, form, KER unit, KER index, date, etc on the measure screen & mode.

5-6 To connect with other equipment

- This equipment is able to transfer test results other equipment with the dealer for details.

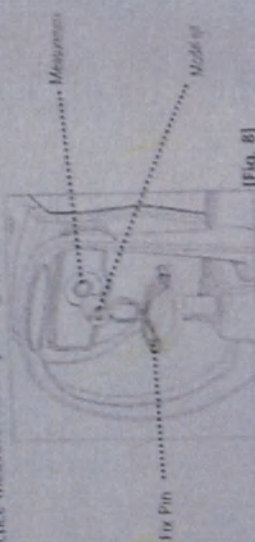
WARNING

Be sure to turn OFF the power switch before connecting or disconnecting the cables. Also, do not handle them with wet hands. Otherwise, get an electric shock that may result in death or serious injury.

6. Practicing by model eye

Practice measurement by using the model eye offered with kit.

6-1 Turn on the power

6-2 Set the model eye on

- Align the holes on the base of the model eye with the total rest and then insert pins.

6-3 Release stage lock

- Rotate the stage holding lever to 'UNFOLDING' direction (left).

6-4 Mode selection

- Choose REF or R/K mode by rotating the Operation Buttons front body.

Refer to 4-4 Operation Buttons & Wheel

6-5 Adjust the height and focus on the model eye

1. Adjust the height of the model eye. (Support the height auto tracking)
 - Adjust the height of the model eye to match with measurement window by using chinrest up/down button.
 - By looking at focusing circle, turn the Body Wheel up or down aims to the center, REF11 does height auto tracking.

※Operation the Body wheel ※

«Turn the Body wheel»

«Turn by pressing the Body Wheel»

- Turning up & down the Body Wheel : the fine-tuning height of main body.

- Turning up & down by pressing the Body Wheel : Lifting up & down the height of the main body quickly.

«The limited point of the main body movement»

- Red 'X' mark & Limit sign. Using Body Wheel cannot raise or lower the main body.

2. Adjust the Focus on ring of the model eye

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