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Hematology Analyzer Lyse

LYE-1 Lyse is applicable to the DH3X (where X represents the digit and/or letter) series hematology analyzers manufactured by Dymind. It is used to lyse (destroy) red blood cells for hemoglobin determinations or aid in the counting of white blood cells. Use the LYE-1 Lyse as instructed by the operation manual of the analyzer.

STORAGE AND STABILITY:

The product will be stable for 2 years when stored at 2°C to 30°C (35°F to 86°F).

If the operating environment is 15°C to 30°C (59°F to 86°F), discard opened container after 60 days.

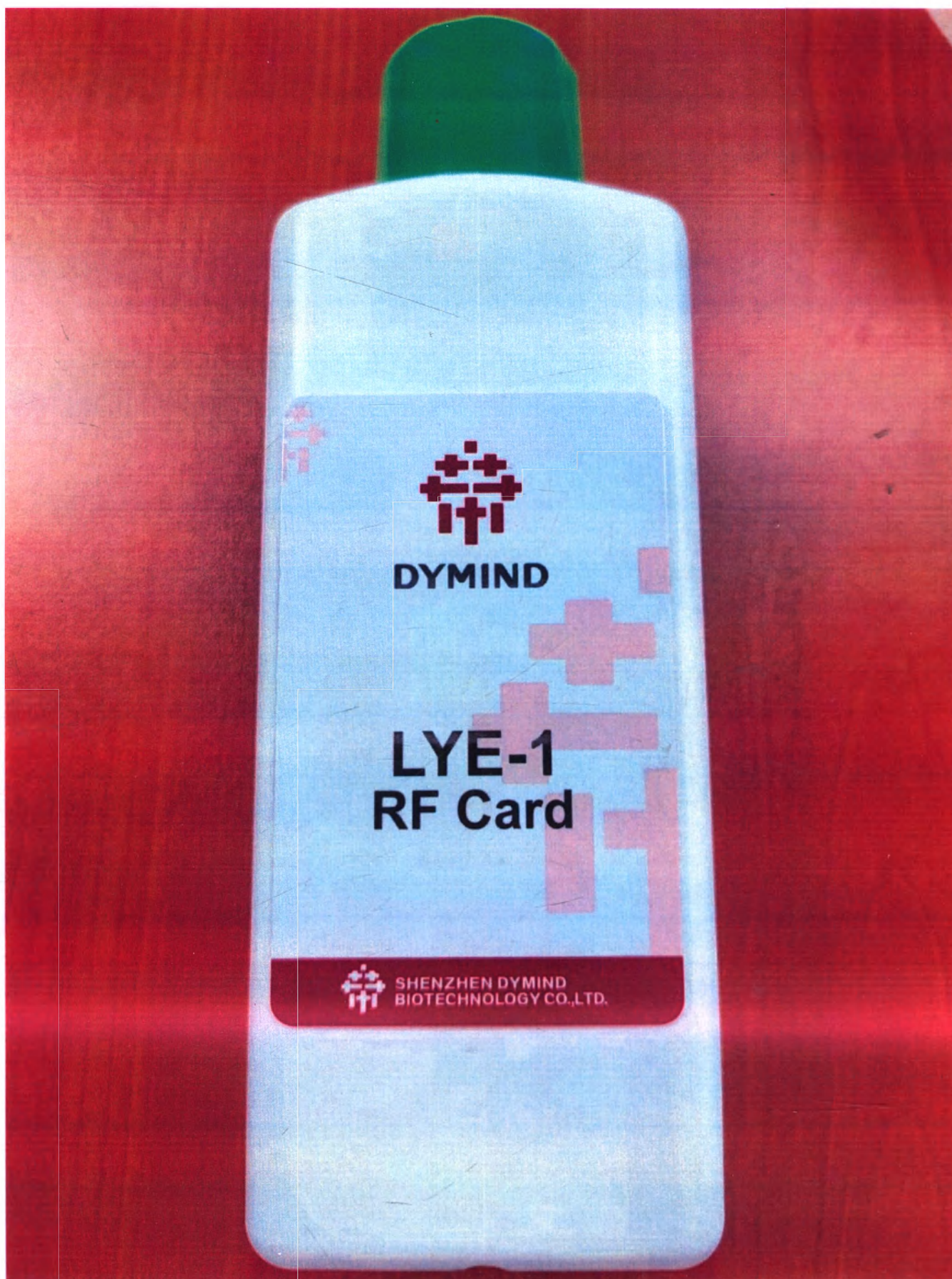
Eunitor GmbH
Kennedydamm 5, 40476 Duesseldorf, Germany


Shenzhen Dymind Biotechnology Co., Ltd.
10th Floor, Building B, High-tech Park, Guangqiao Road,
Yutang Street, Guangming District, Shenzhen 518107, P.R.

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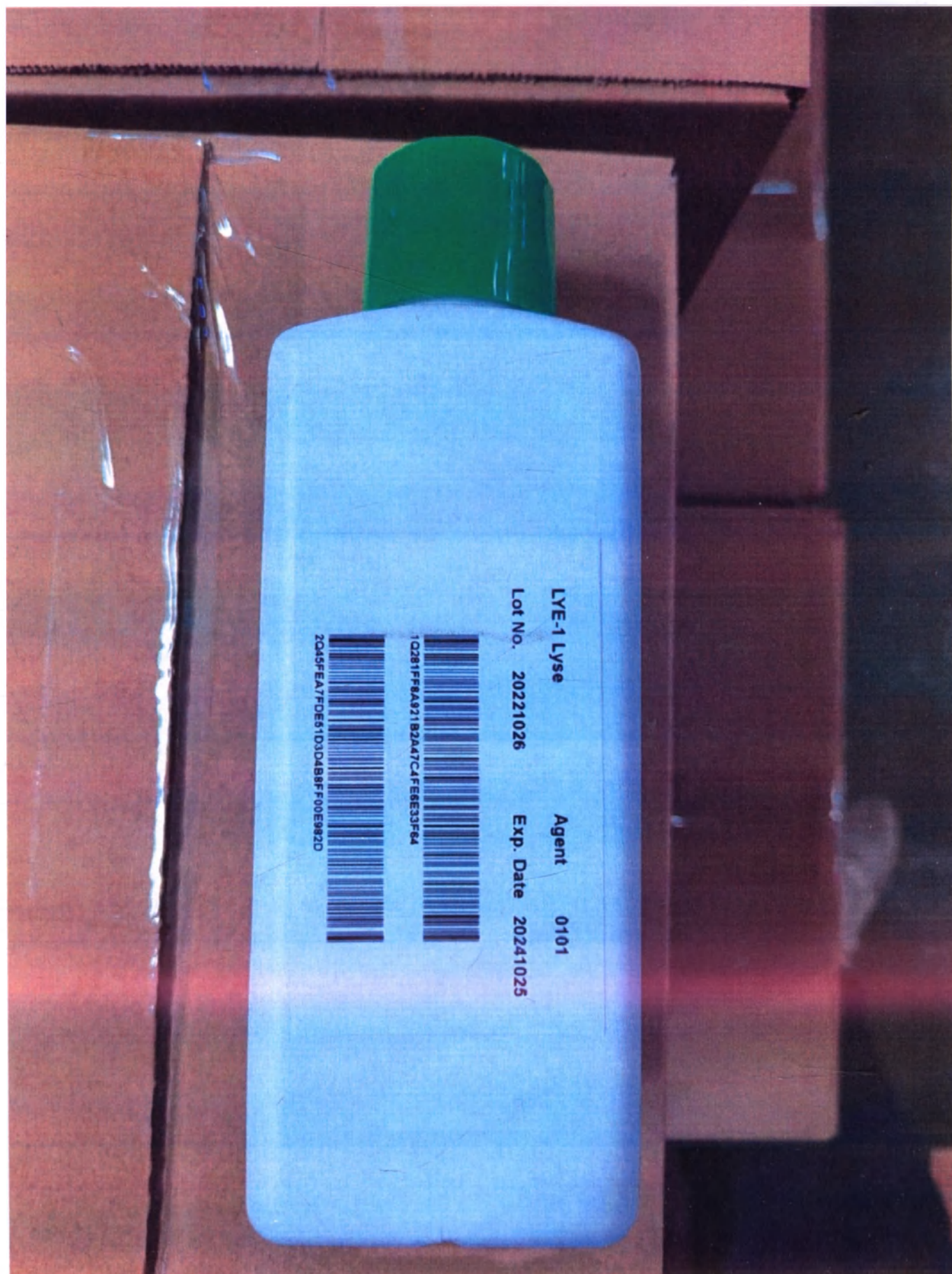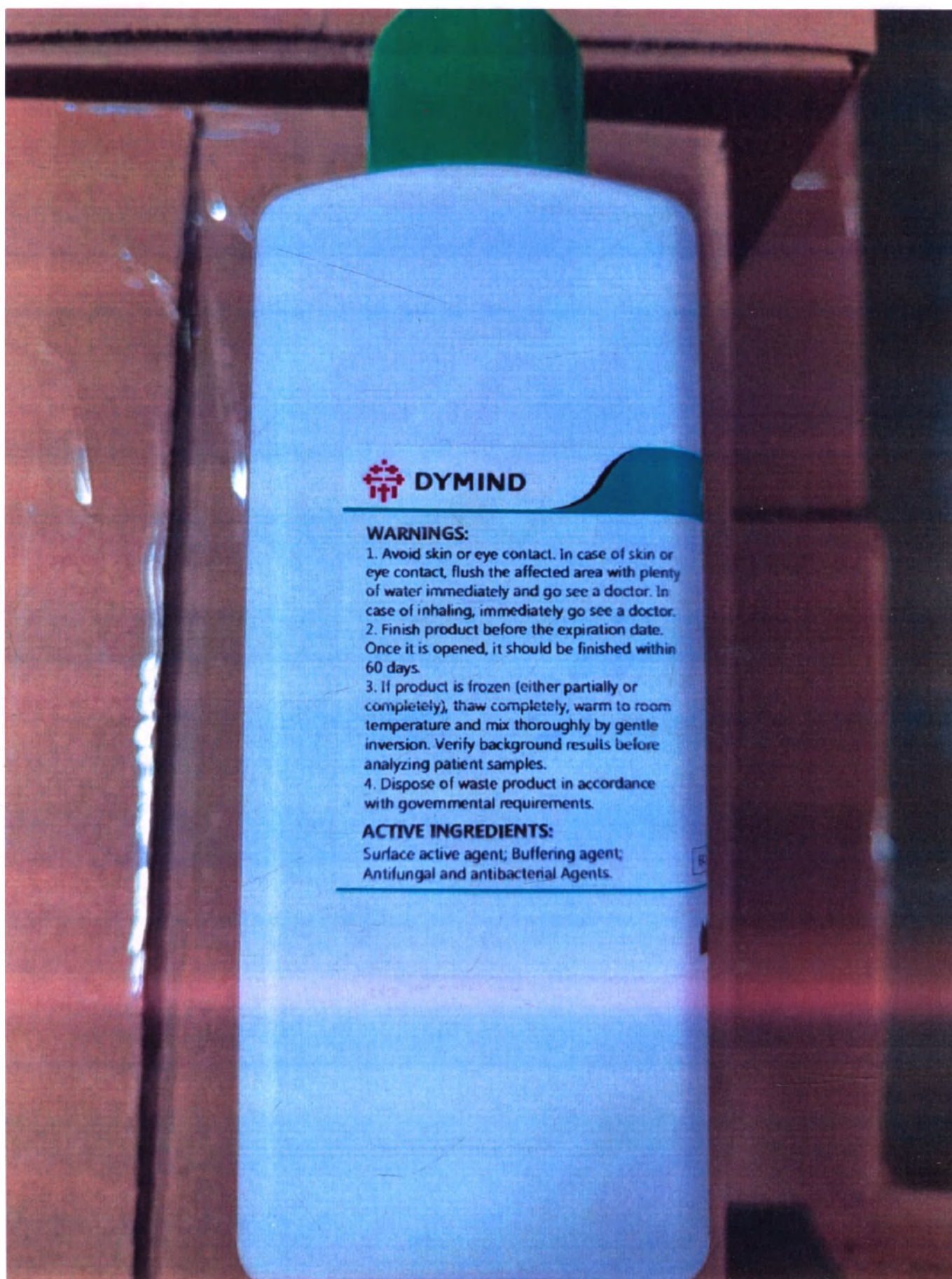
DYMIND

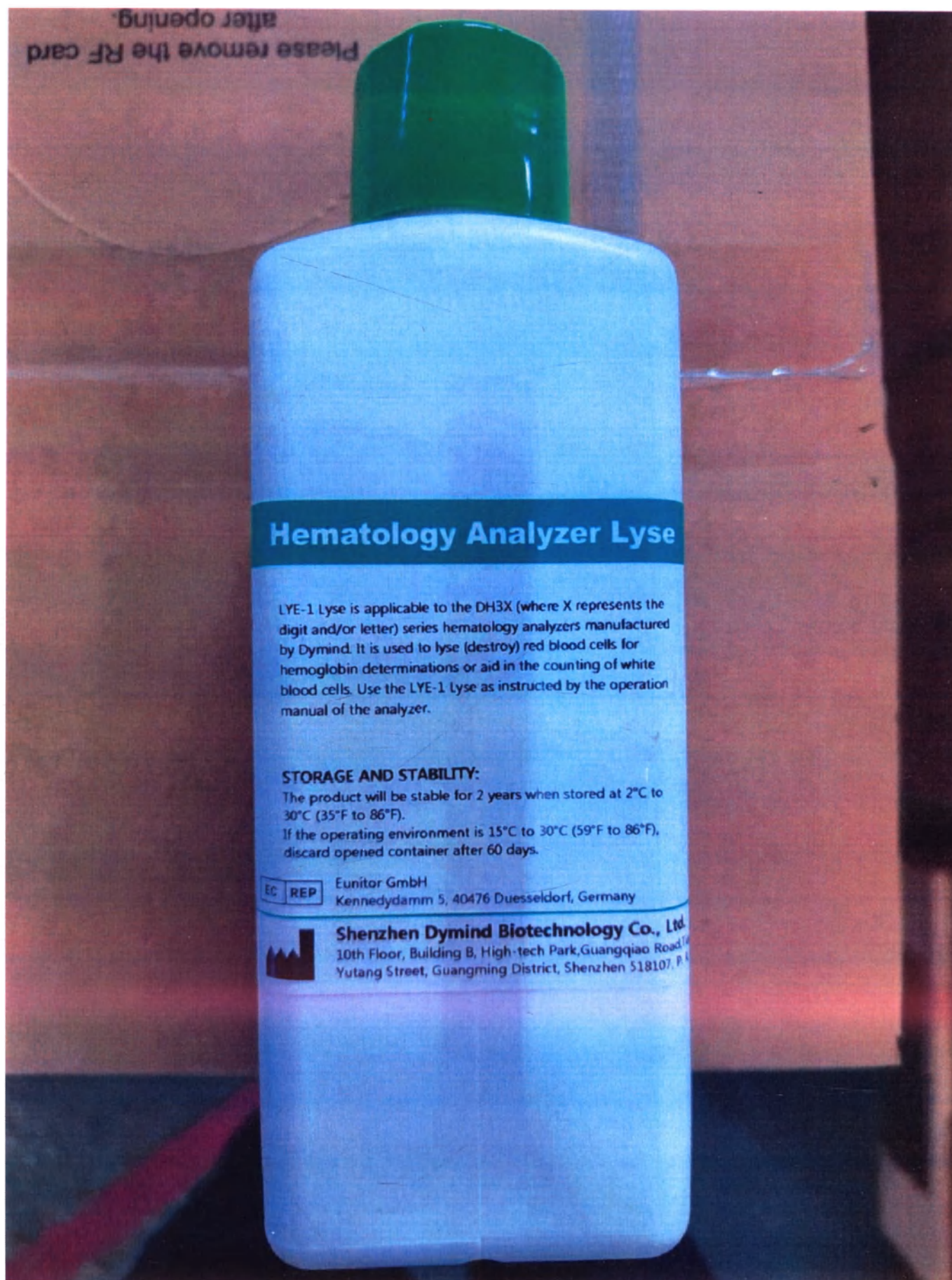
WARNINGS:

1. Avoid skin or eye contact. In case of skin or eye contact, flush the affected area with plenty of water immediately and go see a doctor. In case of inhaling, immediately go see a doctor.
2. Finish product before the expiration date. Once it is opened, it should be finished within 60 days.
3. If product is frozen (either partially or completely), thaw completely, warm to room temperature and mix thoroughly by gentle inversion. Verify background results before analyzing patient samples.
4. Dispose of waste product in accordance with governmental requirements.


ACTIVE INGREDIENTS:

Surface active agent; Buffering agent;
Antifungal and antibacterial Agents.

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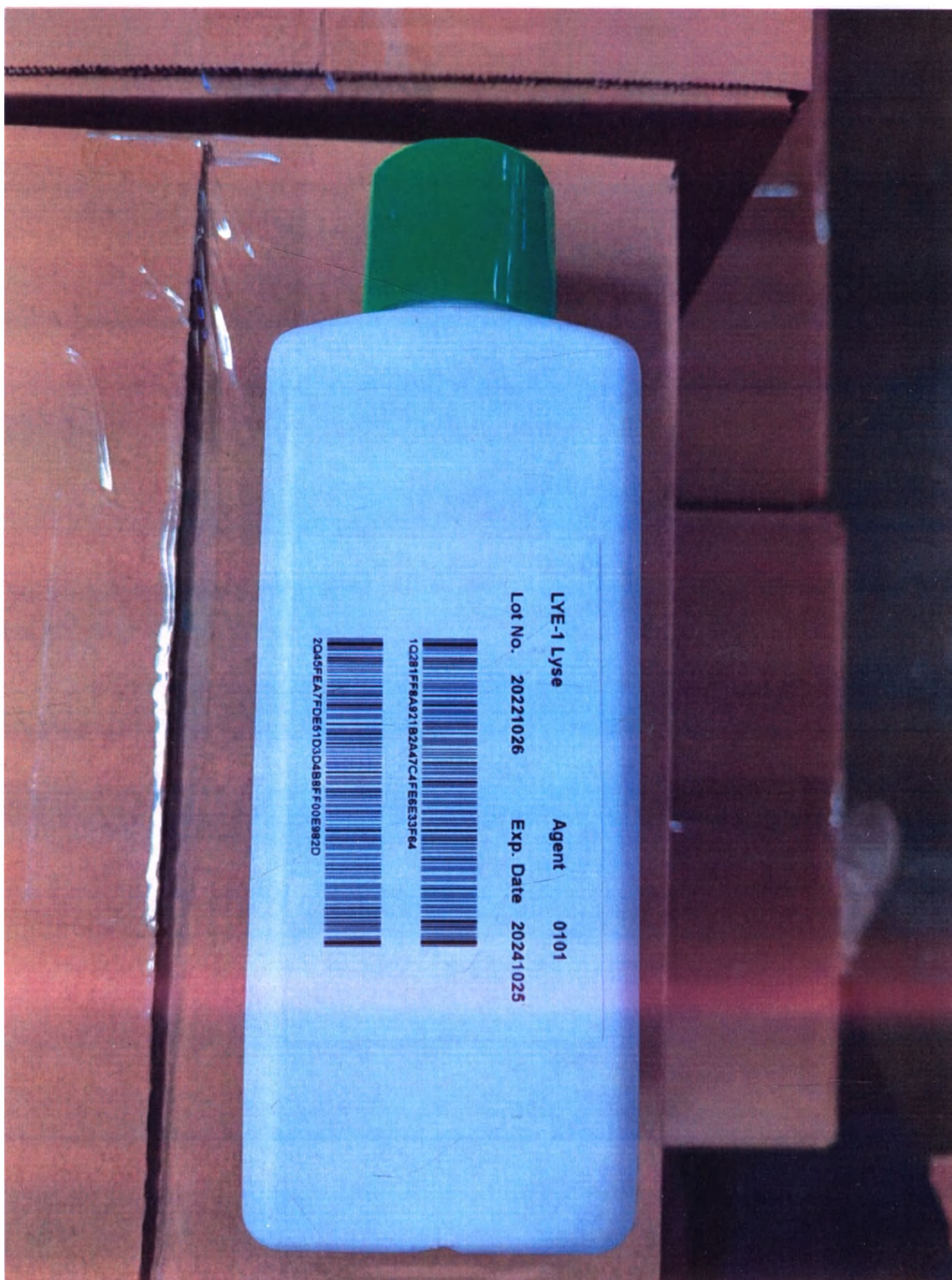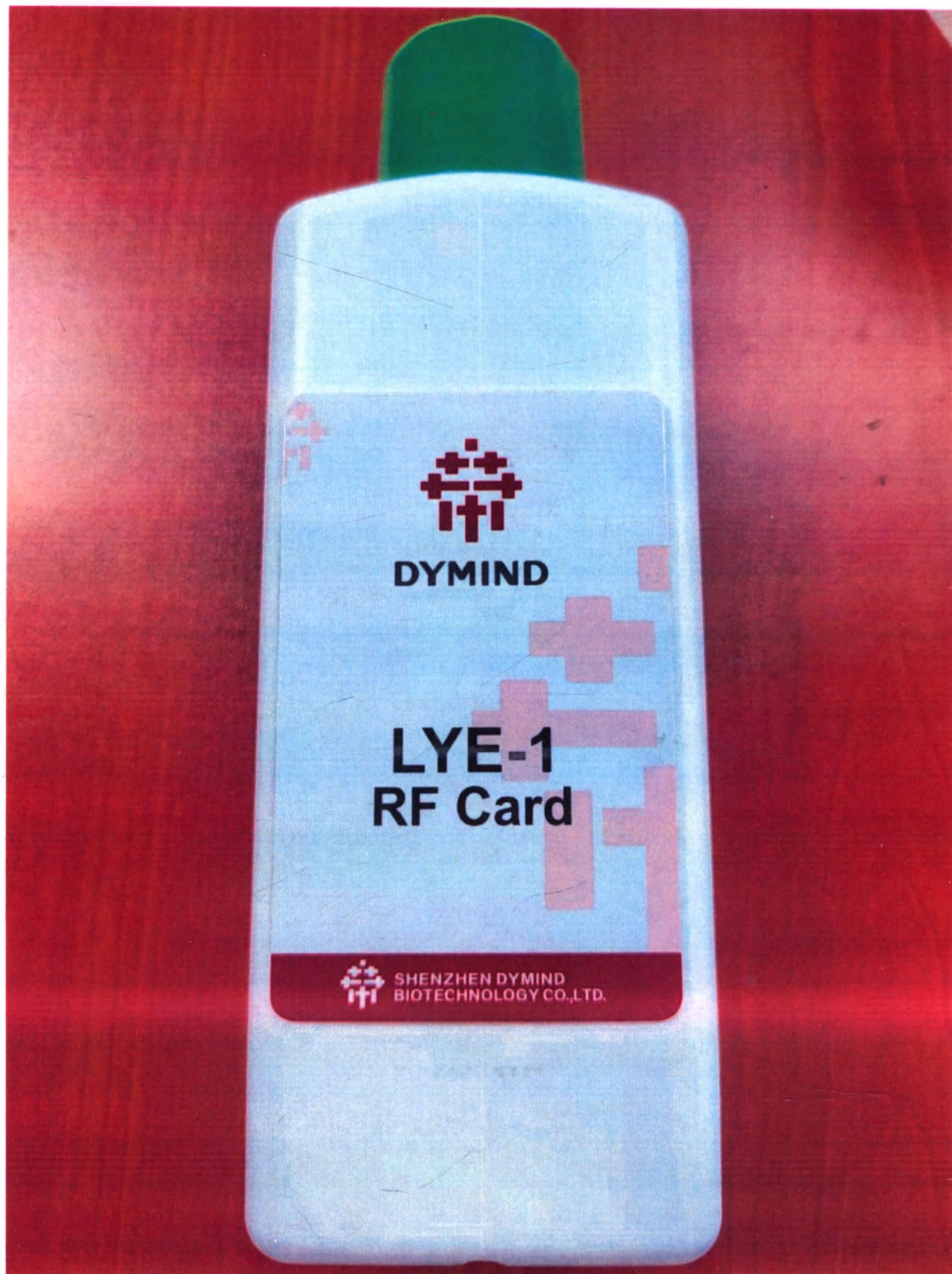
Hematology Analyzer Lyse

LYG-1 Lyse is applicable to the UN7X (where X represents the digit and/or letter) series hematology analyzers manufactured by Dymind. It is used to lyse (destroy) red blood cells for hemoglobin determinations or aid in the counting of white blood cells. Use the LYG-1 Lyse as instructed by the operation manual of the analyzer.

STORAGE AND STABILITY:

The product will be stable for 2 years when stored at 2°C to 30°C (35°F to 86°F).

If the operating environment is 15°C to 30°C (59°F to 86°F), discard opened container after 60 days.

EC REP

Eunitor GmbH
Kennedydamm 5, 40476 Duesseldorf, Germany

Shenzhen Dymind Biotechnology Co., Ltd.
10th Floor, Building B, High-tech Park, Guangqiao Road,
Yutang Street, Guangming District, Shenzhen 518107, P.R.

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DYMIND

WARNINGS:

1. Avoid skin or eye contact. In case of skin or eye contact, flush the affected area with plenty of water immediately and go see a doctor. In case of inhaling, immediately go see a doctor.
2. Finish product before the expiration date. Once it is opened, it should be finished within 60 days.
3. If product is frozen (either partially or completely), thaw completely, warm to room temperature and mix thoroughly by gentle inversion. Verify background results before analyzing patient samples.
4. Dispose of waste product in accordance with governmental requirements.

ACTIVE INGREDIENTS:

Surface active agent; Buffering agent;
Antifungal and antibacterial Agents

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Country : Russia-Company BW, LLC