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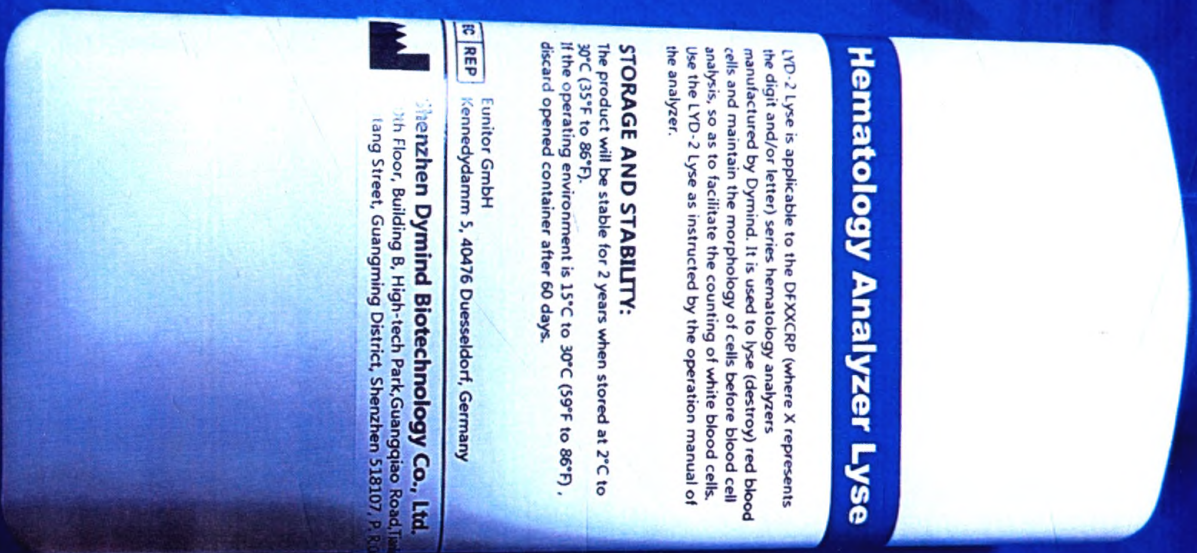
Hematology Analyzer Lyse

LYD-2 Lyse is applicable to the DEXXCRP (where X represents the digit and/or letter) series hematology analyzers manufactured by Dymind. It is used to lyse (destroy) red blood cells and maintain the morphology of cells before blood cell analysis, so as to facilitate the counting of white blood cells. Use the LYD-2 Lyse as instructed by the operation manual of the analyzer.

STORAGE AND STABILITY:

The product will be stable for 2 years when stored at 2°C to 30°C (35°F to 86°F).
If the operating environment is 15°C to 30°C (59°F to 86°F), discard opened container after 60 days.

REP

Eunitor GmbH
Kennedydamm 5, 40476 Dueseldorf, Germany

Shenzhen Dymind Biotechnology Co., Ltd.
3th Floor, Building B, High-tech Park, Guangdong Road, Jia
Lang Street, Guangming District, Shenzhen 518107, P.R. China

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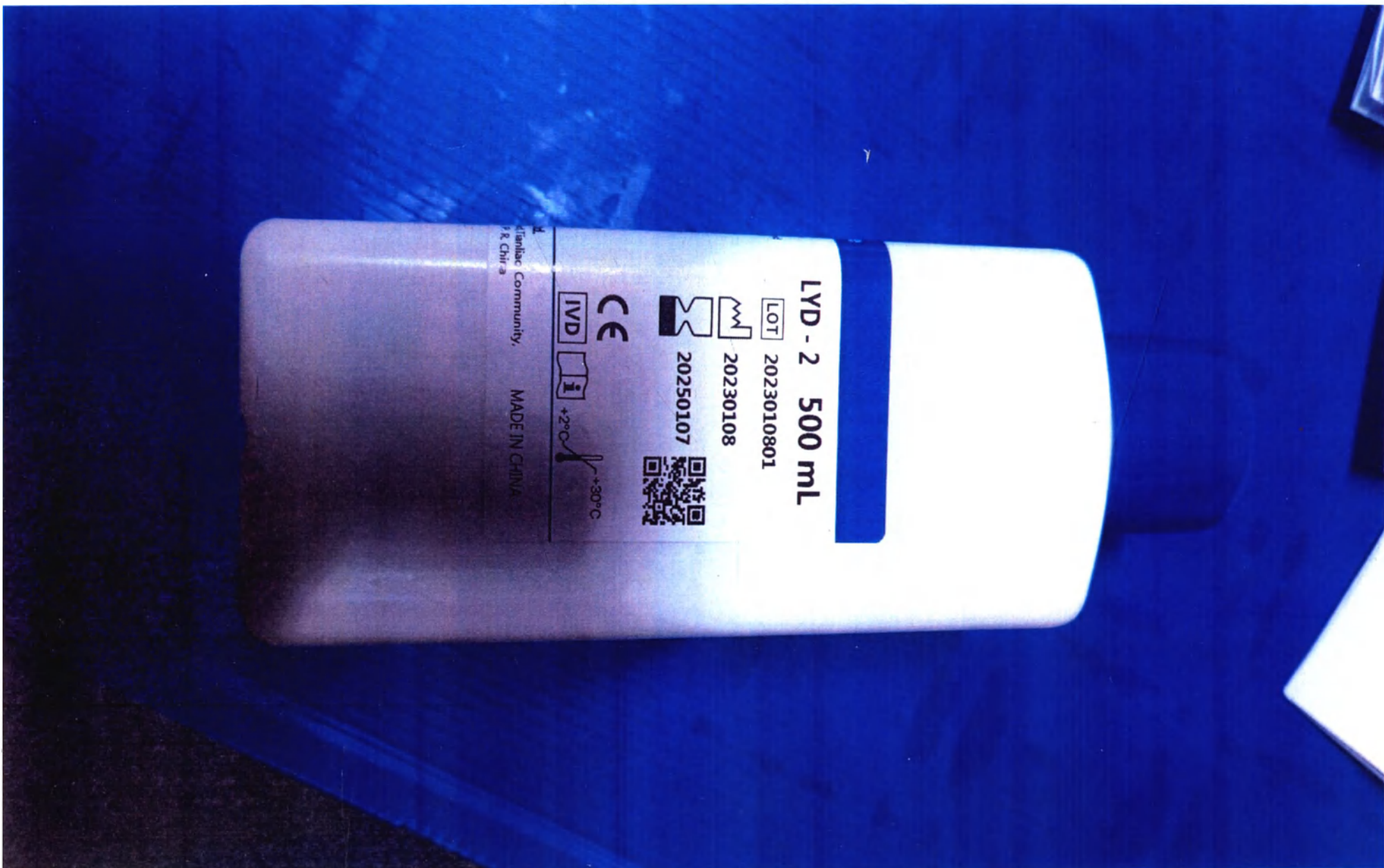
WARNINGS:

1. Avoid skin or eye contact. In case of skin or eye contact, flush the affected area with plenty of water immediately and go see a doctor. In case of inhaling, immediately go see a doctor.
2. Finish product before the expiration date. Once it is opened, it should be finished within 60 days.
3. If product is frozen (either partially or completely), thaw completely, warm to room temperature and mix thoroughly by gentle inversion. Verify background results before analyzing patient samples.
4. Dispose of waste product in accordance with governmental requirements.

ACTIVE INGREDIENTS:

Surface active agent; Buffering agent; Antifungal and antibacterial Agents.

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DYMIND

WARNINGS:

1. Avoid skin or eye contact. In case of skin or eye contact, flush the affected area with plenty of water immediately and go see a doctor. In case of inhaling, immediately go see a doctor.
2. Finish product before the expiration date. Once it is opened, it should be finished within 60 days.
3. If product is frozen (either partially or completely), thaw completely, warm to room temperature and mix thoroughly by gentle inversion. Verify background results before analyzing patient samples.
4. Dispose of waste product in accordance with governmental requirements.

ACTIVE INGREDIENTS:

Surface active agent; Buffering agent;
Antifungal and antibacterial Agents.

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Hematology Analyzer Lyse

LYC-2 Lyse is applicable to the DFSX (where X represents the digit and/or letter) series hematology analyzers manufactured by Dymind. It is used to lyse (destroy) red blood cells and maintain the morphology of cells before blood cell analysis, so as to facilitate the counting of white blood cells. Use the LYC-2 Lyse as instructed by the operation manual of the analyzer.

STORAGE AND STABILITY:

The product will be stable for 2 years when stored at 2°C to 30°C (35°F to 86°F).
If the operating environment is 15°C to 30°C (59°F to 86°F), discard opened container after 60 days.

Eunitor GmbH
Kennedydamm 5, 40476 Duesseldorf, Germany

Shenzhen Dymind Biotechnology Co., Ltd.
10th Floor, Building B, High-tech Park, Guangqiao Road,
Yutang Street, Gangning District, Shenzhen 518107, P.R.

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Hematology Analyzer Lyse

LYC-2 Lyse is applicable to the DF5X (where X represents the digit and/or letter) series hematology analyzers manufactured by Dymind. It is used to lyse (destroy) red blood cells and maintain the morphology of cells before blood cell analysis, so as to facilitate the counting of white blood cells. Use the LYC-2 Lyse as instructed by the operation manual of the analyzer.

STORAGE AND STABILITY:

The product will be stable for 2 years when stored at 2°C to 30°C (35°F to 86°F).

If the operating environment is 15°C to 30°C (59°F to 86°F), discard opened container after 60 days.

EC REP

Eunitor GmbH
Kennedydamm 5, 40476 Duesseldorf, Germany

Shenzhen Dymind Biotechnology Co., Ltd.

10th Floor, Building B, High-tech Park, Guangqiao Road, Tianshan
Yutang Street, Guangming District, Shenzhen 518107, P. R. C.

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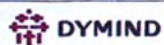
Made in China

Eunitor GmbH
Kennedydamm 5, 40476 Duesseldorf, Germany

Shenzhen Dymind Biotechnology Co., Ltd.
10th Floor, Building B, High-tech Park, Guangqiao Road, Tianliao Community, Yutang Street,
Guangming District, Shenzhen 518107, P.R.China

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Country : Russia-Company BW, LLC