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Self-adhesive electrodes

APPLICATION

1. Clean the skin thoroughly at the prescribed location and dry carefully to remove lotion, oil and make-up. Fig. 1
2. Connect the electrodes to the cables. Fig. 2
3. Remove the electrode by lifting at the edge. Fig. 3, 4
4. Place the electrode on the surface to treat.
5. Turn the stimulator on and follow the instruction of use from the manufacturer and/or the practitioners.
6. After use, turn the stimulator off and replace the electrodes on the siliconed plastic film.

MAINTENANCE

1. For an optimal adhesion, clean the gel surface after each use. Apply a drop of water and keep drying for a few seconds. Replace the electrodes on the plastic film. Fig. 5
2. The life time of the electrode depends on the maintenance, the skin conditions and the type of treatment. Keep at ambient temperature in original packing. Fig. 6

IMPORTANT

1. Do not connect the electrodes while driving a vehicle, or operating machinery.
2. Do not apply on wounds.
3. If skin irritation occurs, discontinue use.
4. Remove the electrodes before bathing.
5. The electrodes are intended for single use.

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Self-adhesive electrodes

APPLICATION

- 1 Clean the skin thoroughly at the prescribed location and dry carefully to remove lotion, oil and make-up. Fig. 1
- 2 Connect the electrodes to the cables. Fig. 2
- 3 Remove the electrode by lifting at the edge. Fig. 3, 4
- 4 Place the electrode on the surface to treat.
- 5 Turn the stimulator on and follow the instruction of use from the manufacturer and/or the practitioners.
- 6 After use, turn the stimulator off and replace the electrodes on the siliconed plastic film.

MAINTENANCE

- 1 For an optimal adhesion, clean the gel surface after each use. Apply a drop of water and keep drying for a few seconds. Replace the electrodes on the plastic film. Fig. 5
- 2 The life time of the electrode depends on the maintenance, the skin conditions and the type of treatment. Keep at ambient temperature in original packing. Fig. 6

IMPORTANT

- 1 Do not connect the electrodes while driving a motorised vehicle, or operating machinery.
- 2 Do not apply on wounds.
- 3 If skin irritation occurs, discontinue use and contact your physician.
- 4 Remove the electrodes before bathing and swimming.
- 5 The electrodes are intended for single patient use only.

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