JD

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BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

Soft Contact Lens BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology

⚠ INSTRUCTIONS FOR USE

This leaflet is intended to inform users of Bausch + Lomb hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

PRODUCT NAME	MATERIAL	WATER CONTENT	PACKAGING SOLUTION
Soft Contact Lens BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism	Samfilcon A	46% by weight	Borate buffered saline with Poloxamine
Soft Contact Lens BAUSCH + LOMB ULTRA® (samfilcon A) for Presbyopia	Samfilcon A	46% by weight	Borate buffered saline with Poloxamine

Description and wear application

These lenses are designed for daily wear with a replacement period as prescribed by the eye care professional. They may also be used as disposable lenses to be discarded after each removal. When reusing lenses they must be cleaned, rinsed and disinfected between removal and reinsertion.

Recommended wearing period:

Daily wear - (with removal for the night) - 30 days.

Extended wear (without removing for the night) 7 days and 6 nights. The term of replacement of lenses for prolonged wear is up to 30 days in accordance with the recommendations of a ophthalmologist.

DISPOSABLE WEAR

When prescribed for Disposable Wear, the Bausch + Lomb (samfilcon A) Contact Lens for Astigmatism and for Presbyopia, is to be discarded after each removal.

Always follow your eye care practitioner's recommended wearing schedule.

BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam. **STERILE**

If the packaging is damaged or opened **DO NOT USE**.

Insertion and removal instructions

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens down to the white part of the eye and remove it. Use a rewetting drop if the lens is difficult to remove.

Caring for your lenses

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

Warnings and precautions

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Do not wear daily disposable lenses for longer than a day or while sleeping.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes.

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication.

If you experience any of the following problems, remove your lenses immediately:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal and insertion of a new lens, then remove the lens and contact your eye care practitioner. These symptoms can lead to more serious conditions if ignored.

BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

Transportation, storage and operation conditions

Storage: At a temperature of 1 ° C to 30 ° C. Do not freeze.

Samfilcon A (46% water content) Contact Lens	
< - 20°C	Avoid
- 20°C to 1°C	1 month
1°C to 30°C	Until Expiry
30°C to 60°C	3 month to 60°C
> 60°C	Avoid

Technical characteristics

Diameter	13.5mm to 15.0mm
Center Thickness	0.05mm to 0.75mm (varies with power)
Base Curve	7.8mm to 9.5mm
Power Range	+20.00D to -20.00D
Cylinder Power (Toric)	-0.75D to -5.00D
Cylinder Axis (Toric)	10° to 180°
Add Power (Multi-Focal)	+0.75D to +5.00D

The chemical composition of the borate buffered

Composition	%
Sodium chloride	< 1%
Sodium borate	< 1%
Boric acid	< 1%
Poloxamine	0.50
Purified water	Balance

Characteristics of the packaging solution (Stability Interval Specification)

pH	6.5 – 8.0
Osmolality	210 - 420 mOsm/kg
Color	≤ 50 units of chromaticity on the APHA scale
Clarity	Clear

BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

Odor **No odor**

Information about packaging

The packaging materials used for Bausch + Lomb (samfilcon A) ULTRA® contact lens consists of:

- polypropylene blister
- plastic coated aluminium

overall Dimensions of the Blister:

- Length – 47.5 mm
- Width – 29.5 mm
- Height – 8 mm

Expiry date

5 years.

Recycling

Soft contact lenses are non-toxic, do not create an excessive load on the person or the environment during operation, storage or transportation. To prevent damage to the environment, it is necessary to comply with safety requirements, storage conditions, transportation and operation.

Soft contact lenses are sent for disposal in accordance with the hygienic requirements of SanPin 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of the territories of urban and rural settlements, for water bodies, drinking water and drinking water supply, atmospheric air, soils, residential premises, operation of industrial and public premises, organization and conduct of sanitary and anti-epidemic (preventive) measures" adopted in the Russian Federation.

Soft contact lenses to be disposed of after use, as well as in connection with the expiration of the shelf life or in case of violation of the integrity of the package, belong to the hazard class A. Disposal into containers with household garbage is allowed.

Warranty

Warranty contact soft lenses are not provided.

Authorized representative of the manufacturer in the Russian Federation

BAUSCH + LOMB ULTRA® (samfilcon A) for Astigmatism and for Presbyopia
with MoistureSeal Technology
Instructions For Use
Manufacturer: Rochester
Market: Russian Federation

«Bausch Health» LLC, Shabolovka St. 31, Bld. 5, Moscow, 115162, Russia
Tel/fax+7-495-510-28-79

BAUSCH & LOMB INCORPORATED

By: Marc Halavi
Name: Marc Halavi
Title: Attorney-in-Fact

18 November 2021
Date

State of New Jersey

County of Union

Subscribed and sworn to (or affirmed) before me on this 18th day of November, 2021, by

Marc Halavi, proved to me on the basis of satisfactory evidence to be the person(s) who appeared before me.

Rita E. Moore
Signature of Notary Public

RITA E MOORE
Notary Public, State of New Jersey
My Commission Expires
September 18, 2024

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