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
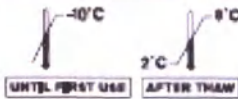
PRODUCED FOR ABBOTT BY
Abbott Diagnostics
A Div. of Abbott Scientific Company LLC
A Part of Abbott Scientific, Inc.
1385 Valley Pike
Burlington, MA 01803, USA
PRODUCT OF USA

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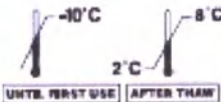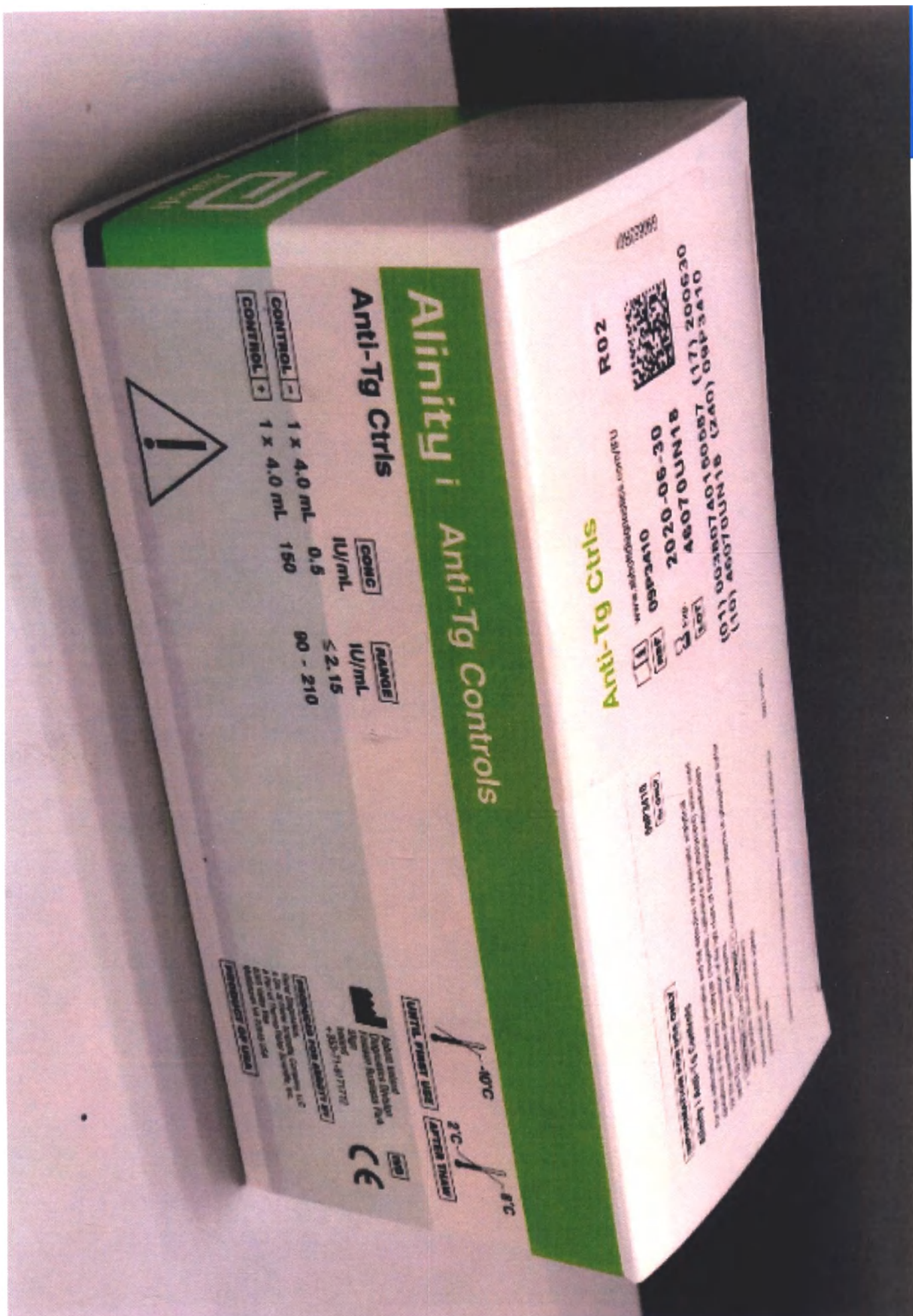
Alinity i Anti-Tg Controls

Read Highlighted Changes: Revised March 2018.

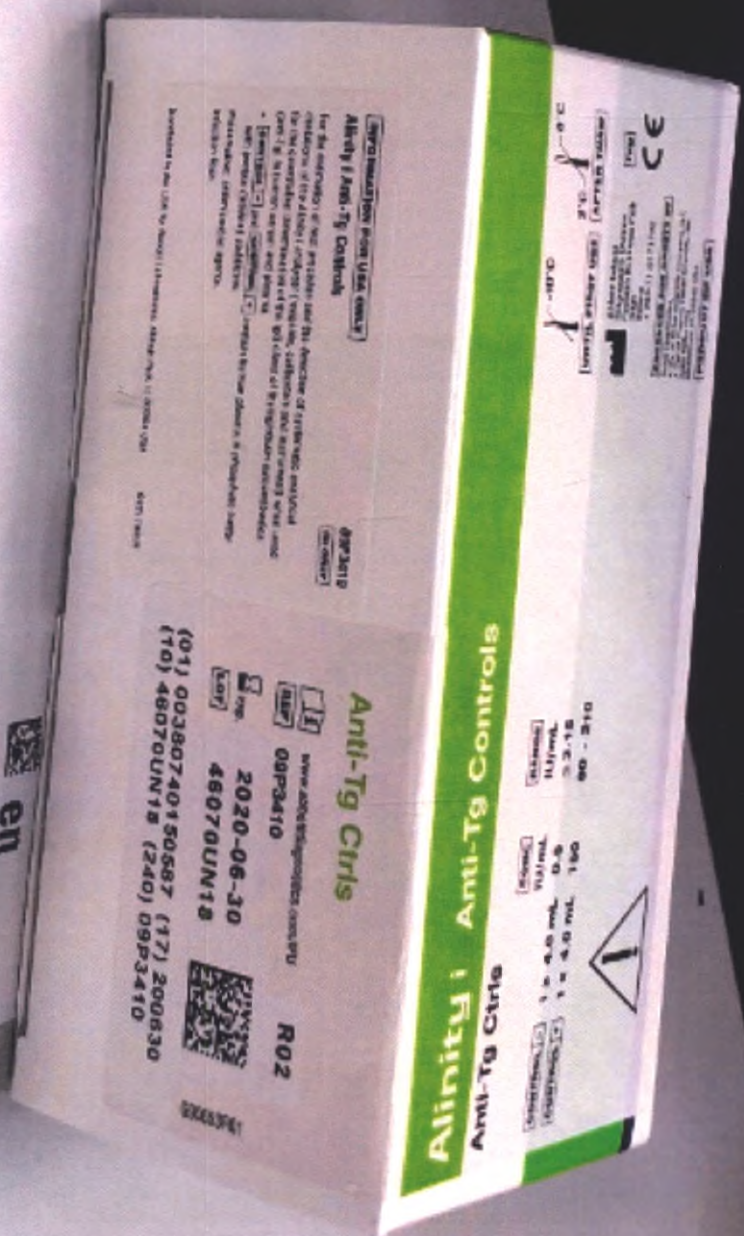
NAME

Alinity i Anti-Tg Controls (also referred to as Anti-Tg Ctrls)

INTENDED USE

The Alinity i Anti-Tg Controls are for the estimation of test precision and the detection of systematic analytical deviations of the Alinity i analyzer (reagents, calibrators and instrument) when used for the quantitative determination of the IgG class of thyroglobulin autoantibodies (anti-Tg) in human serum and plasma.

For additional information, refer to the Alinity i Anti-Tg reagent package insert and the Alinity i-series Operations Manual.

Anti-Tg Ctrls

Lot: 00380740150587 (17) 200630
(10) 48070UN18 (240) 09P3410

en
Anti-Tg
G80652R02
C9P340

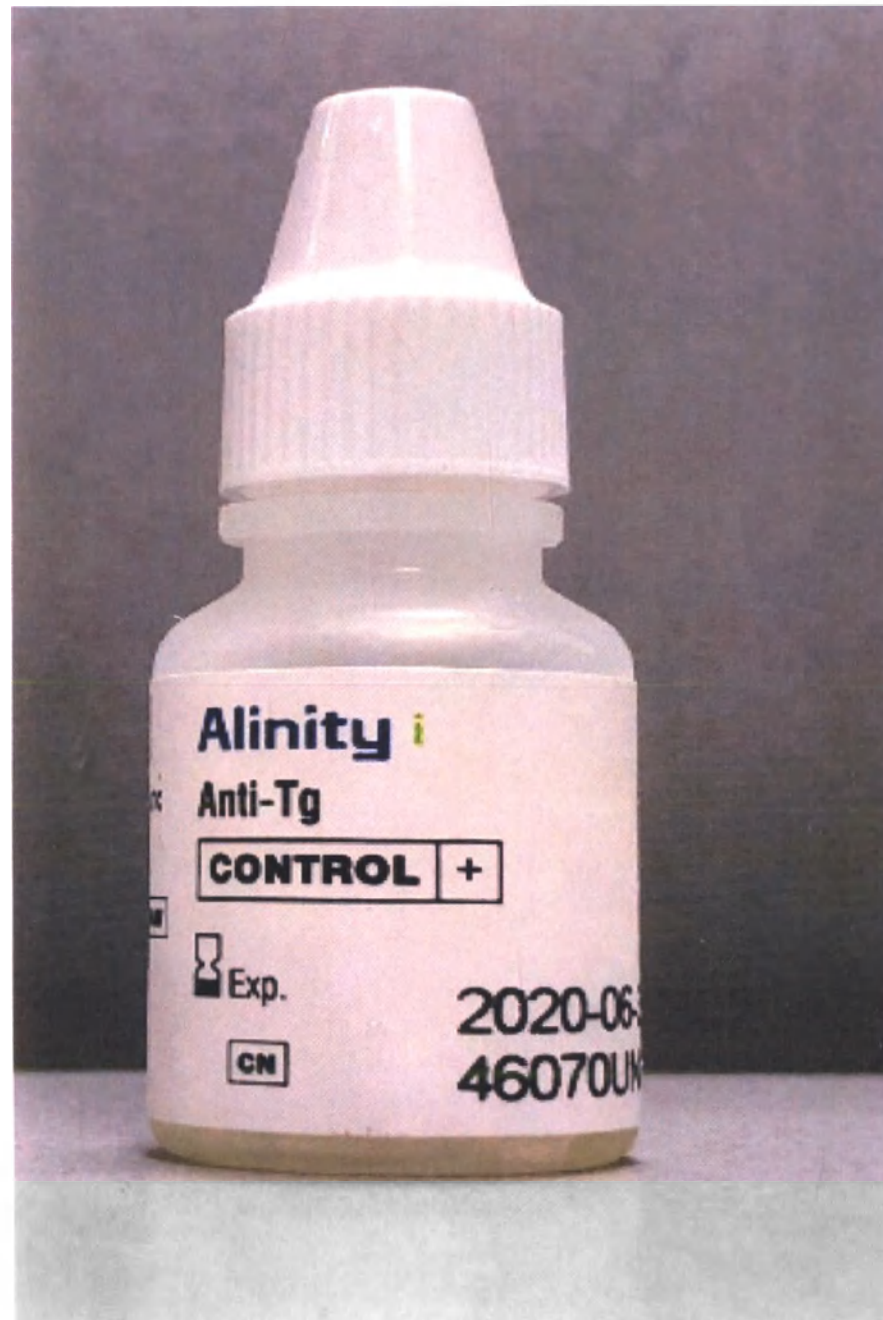
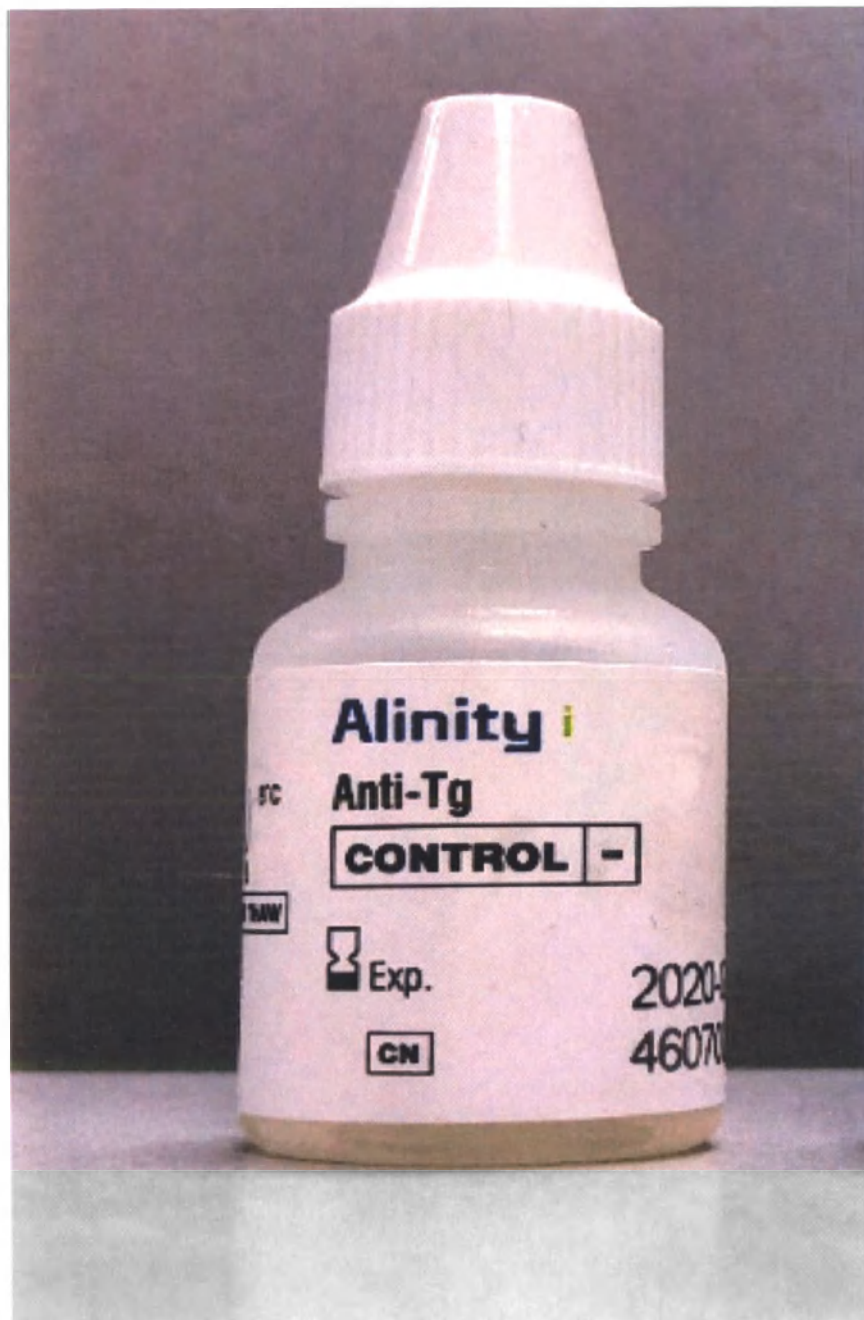
For a detailed discussion of safety precautions during system operation, refer to the Alinity i-series Operations Manual, Section B.

PREPARATION FOR USE

- Thaw completely at room temperature (15 to 30°C) for 45 to 60 minutes.

STORAGE

- Prior to use, mix thoroughly by gentle inversion (5 to 10 times).
- This product is shipped on dry ice.
- Self-detecting freezers are recommended.

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