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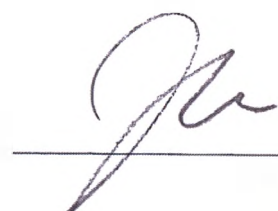
16.12.2024
«Approved»

(signature)
Kim Beom Jun,
CEO
Dental Studio Co. Ltd.

**INSTRUCTIONS FOR USE
FOR THE MEDICAL DEVICE**

Karl Fritz® dental clamping instrument

5. Scope of application, contraindications and possible side effects

Scope of application: dentistry, maxillofacial surgery, dental surgery.

Contraindications: possible allergic reaction to materials.

Side effects: Not known.

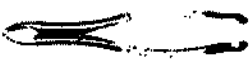
6. Precautions.

1. The instruments are supplied in a non-sterile package. It is necessary to sterilize the instruments before using them.
2. Check the instruments for mechanical damage. Before using each instrument, the specialist must make sure that the instruments are in working condition; their surfaces shall be free of any contaminations.
3. The instruments must be used in accordance with their intended purpose and instructions for use.
4. This device can be used only by dentists who received qualifications in the application of appropriate treatment methods and completed training courses.
5. Surgical interventions should be carried out in sterile conditions, medical staff should use masks, gloves, sterile clothes.

7. Description, design of devices and application procedure.

Designed for tooth and/or root fragments extraction, alignment of the alveolar ridge after tooth extraction, as well as for manipulation of the dental implant/denture during its insertion. These are reusable products.

Forceps

Product name	Description	The procedure of using	Appearance
Wynman forceps with inserts for 16 cm crowns, P/N 1110	The forceps are intended to hold restorations and their components (abutments, fixing screws, crowns, bridges) during installation in the patient's oral cavity.	1. Clean, disinfect and sterilize the device. 2. Bring the handles together, clamp the crown or a restoration component, then remove or install it.	
Crown opening forceps, P/N 1111	The forceps are intended for expansion of cuts in prosthetic crowns for the purpose of their removal.	1. Clean, disinfect and sterilize the device. 2. Bring the handles together, insert the applied parts into the cut in the crown and open it.	
Forceps are intended to hold tooth crowns, roots, root fragments and dental implants during their removal.			
Stieglitz forceps for root fragments, straight, P/N 1205	The beaks are straight and converging. Handles have a ratchet (multi-position).	1. Clean, disinfect and sterilize the device 2. Select an instrument that is suitable in size and purpose; 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Stieglitz forceps for root fragments, curved, P/N 1206	The beaks are curved and converging. Handles have a ratchet (multi-position).	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
150S universal forceps for extraction of upper jaw teeth, P/N 3201	The beaks are straight and converging. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
151S universal forceps for extraction of lower jaw teeth, P/N 3202	The beaks are straight and converging. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 53R for extraction of upper right molars, P/N 3206	The beaks are straight, not converging, with a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose.	

Product name	Description	The procedure of using	Appearance
Forceps 53S for extraction of upper left molars, P/N 3207	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 67A for extraction of upper third molars, P/N 3208	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 10S for extraction of upper molars, P/N 3209	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 150 for extraction of upper anteriors and premolars, P/N 3210	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 17 for extraction of lower molars, P/N 3211	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 222 for extraction of lower third molars, P/N 3212	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 151 for extraction of lower anteriors and premolars, P/N 3213	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps I for extraction of upper incisors, P/N 3214	The jaws are straight, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Universal forceps 300 for extraction of upper roots, P/N 3215	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Universal forceps 301 for extraction of lower roots, P/N 3216	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps EASY for implant removal, P/N 3220	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 150 EASY for extraction of upper anteriors and premolars, P/N 3251	The jaws are curved, not converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps EASY for extraction of upper labial teeth, P/N 3251S	The jaws are curved, not converging, without a spike. The handles of the forceps are spring-loaded (self-expanding).	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps 151 EASY for extraction of lower anteriors and premolars, P/N 3252	The jaws are curved, not converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps EASY for extraction of lower labial teeth, P/N 3252S	The jaws are curved, not converging, without a spike. The handles of the forceps are spring-loaded (self-expanding).	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Universal forceps 222 EASY for extraction of lower molars, P/N 3253	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Universal forceps EASY for root extraction, P/N 3254	The jaws are curved, converging, without a spike. The handles of the forceps are simple.	3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	

Product name	Description	The procedure of using	Appearance
Forceps EASY for extraction of upper molars, P/N 3255S	The jaws are curved, not converging, without a spike. The handles of the forceps are spring-loaded (self-expanding).	1. Clean, disinfect and sterilize the device. 2. Select an instrument that is suitable in size and purpose. 3. Bring the handles together, clamp the applied parts on the tooth, tooth root or implant, and then remove them.	
Forceps EASY for extraction of lower molars, P/N 3259S	The jaws are curved, not converging, without a spike. The handles of the forceps are spring-loaded (self-expanding).		

Technical specifications

Product name	Total length, mm	Applied part length, mm	Angle, °	Material
Wyman forceps with inserts for 16 cm crowns, P/N 1110	162.0	38.0	25	Stainless steel
Crown opening forceps, P/N 1111	150.0	24.0	No	Stainless steel
Stieglitz forceps for root fragments, straight, P/N 1205	145.0	33.0	No	Stainless steel
Stieglitz forceps for root fragments, curved, P/N 1206	140.0	28.0	45	Stainless steel
150S universal forceps for extraction of upper jaw teeth, P/N 3201	155.0	20.0	155	Stainless steel
151S universal forceps for extraction of lower jaw teeth, P/N 3202	150.0	20.0	140	Stainless steel
Forceps 53R for extraction of upper right molars, P/N 3206	188.0	30.0	135 / 135	Stainless steel
Forceps 53S for extraction of upper left molars, P/N 3207	188.0	30.0	130 / 140	Stainless steel
Forceps 67A for extraction of upper third molars, P/N 3208	190.0	40.0	130 / 145	Stainless steel
Forceps 10S for extraction of upper molars, P/N 3209	185.0	30.0	125 / 140	Stainless steel
Forceps 150 for extraction of upper anteriors and premolars, P/N 3210	178.0	25.0	155	Stainless steel
Forceps 17 for extraction of lower molars, P/N 3211	175.0	22.0	135	Stainless steel
Forceps 222 for extraction of lower third molars, P/N 3212	182.0	30.0	110	Stainless steel
Forceps 151 for extraction of lower anteriors and premolars, P/N 3213	170.0	20.0	120	Stainless steel
Forceps 1 for extraction of upper incisors, P/N 3214	172.0	26.0	No	Stainless steel
Universal forceps 300 for extraction of upper roots, P/N 3215	155.0	30.0	145	Stainless steel
Universal forceps 301 for extraction of lower roots, P/N 3216	155.0	20.0	120.0	Stainless steel
Forceps EASY for implant removal, P/N 3220	172.0	28.0	105	Stainless steel
Forceps 150 EASY for extraction of upper anteriors and premolars, P/N 3251	172.0	25.0	150	Stainless steel
Forceps EASY for extraction of upper labial teeth, P/N 3251S	110.0	20.0	155	Stainless steel
Forceps 151 EASY for extraction of lower anteriors and premolars, P/N 3252	172.0	22.0	120	Stainless steel
Forceps EASY for extraction of lower labial teeth, P/N 3252S	110.0	20.0	No	Stainless steel
Universal forceps 222 EASY for extraction of lower molars, P/N 3253	175.0	25.0	110	Stainless steel
Universal forceps EASY for root extraction, P/N 3254	180.0	38.0	115	Stainless steel
Forceps EASY for extraction of upper molars, P/N 3255S	120.0	27.0	No	Stainless steel
Forceps EASY for extraction of lower molars, P/N 3259S	105.0	20.0	No	Stainless steel

Bone cutters

Description: Bone cutters are intended to remove sharp edges of the alveolar bone after tooth extraction.

Product name	Description	Appearance
Beyer double bone cutters, P/N 1301	Beyer bone cutter with a hinged joint (lock) with double transmission of the applied parts compression from the handles.	
Friedman bone cutters, P/N 1303	Friedman bone cutters with a hinged joint (lock) with single transmission of the applied parts compression from the handles.	

Technical specifications

Product name	Total length, mm	Jaws length, mm	Applied part width, mm	Material
Beyer double bone cutters, P/N 1301	180.0	18.0	18.0	Stainless steel
Friedman bone cutters, P/N 1303	142.0	20.0	15.0	Stainless steel

The procedure for using bone cutters:

1. Clean, disinfect and sterilize the device;
2. Select an instrument that is suitable in size and purpose;
3. Bring the handles together and remove the sharp edges of the alveolar bone after tooth extraction.

8. Package contents

Karl Fritz® dental clamping instrument

I. Design options, 1 pc/package

1. Forceps to remove teeth and root fragments

- 1.24. Stieglitz forceps for root fragments, straight, P/N 1205;
 - 1.25. Stieglitz forceps for root fragments, curved, P/N 1206;
 - 1.26. Universal forceps 150S for extraction of upper jaw teeth, P/N 3201;
 - 1.27. Universal forceps 151S for extraction of lower jaw teeth, P/N 3202;
 - 1.28. Forceps 53R for extraction of upper right molars, P/N 3206;
 - 1.29. Forceps 53S for extraction of upper left molars, P/N 3207;
 - 1.30. Forceps 67A for extraction of upper third molars, P/N 3208;
 - 1.31. Forceps 10S for extraction of upper molars, P/N 3209;
 - 1.32. Forceps 150 for extraction of upper anteriors and premolars, P/N 3210;
 - 1.33. Forceps 17 for extraction of lower molars, P/N 3211;
 - 1.34. Forceps 222 for extraction of lower third molars, P/N 3212;
 - 1.35. Forceps 151 for extraction of lower anteriors and premolars, P/N 3213;
 - 1.36. Forceps 1 for extraction of upper incisors, P/N 3214;
 - 1.37. Universal forceps 300 for extraction of upper roots, P/N 3215;
 - 1.38. Universal forceps 301 for extraction of lower roots, P/N 3216;
 - 1.39. Forceps 150 EASY for extraction of upper anteriors and premolars, P/N 3251;
 - 1.40. Forceps EASY for extraction of upper labial teeth, P/N 3251S;
 - 1.41. Forceps 151 EASY for extraction of lower anteriors and premolars, P/N 3252;
 - 1.42. Forceps EASY for extraction of lower labial teeth, P/N 3252S;
 - 1.43. Universal forceps 222 EASY for extraction of lower molars, P/N 3253;
 - 1.44. Universal forceps EASY for root extraction, P/N 3254;
 - 1.45. Forceps EASY for extraction of upper molars, P/N 3255S;
 - 1.46. Forceps EASY for extraction of lower labial teeth and molars, P/N 3259S.
2. Forceps for manipulating a dental implant/prosthesis
 - 2.4. Forceps EASY for implant removal, P/N 3220;
 - 2.5. Wynman forceps with inserts for 16 cm crowns, P/N 1110;
 - 2.6. Crown opening forceps, P/N 1111;
 3. Bone cutters
 - 3.1. Beyer double bone cutters, art. 1301;
 - 3.2. Friedman bone cutters, art. 1303.

II. Instructions for use – 1 pc./pack. (in paper and electronic form)

9. Decoding of symbols used in packaging labeling

Symbol	Meaning	Symbol	Meaning
	Catalogue number		Caution
	Lot code		Read instructions for use or see electronic instructions for use
	Date of manufacture		
	Use before		QR-code
	Manufacturer		Non-sterile

10. Conditions of storage, transportation and operation

Storage conditions:

- store in a dry place protected from direct sunlight;
- temperature: +1 to +40 °C;
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

Transportation conditions:

Transportation of devices packaged into a transport container shall be carried out by all types of covered vehicles in compliance with the precautions indicated on the transport container and in accordance with the rules of cargo transportation applicable to that type of transport.

- temperature: -50 to +50 °C;
- relative humidity: 0-90 %;
- atmospheric pressure: 800-1 060 hPa.

Indoor conditions during medical device use

- temperature: +1 to +40 °C
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

11. Cleaning, disinfection and sterilization of devices

Used instruments shall be cleaned, disinfected and sterilized before being sent for storage.

1. Pre-sterilization cleaning

Instruments shall be cleaned and disinfected before sterilization.

1) Immerse the instruments with the remaining impurities (blood, soft tissues, bone fragments, etc.) in water. The instruments shall be kept in water before the following cleaning steps are started manually or in an ultrasonic bath.

2-1) Manual cleaning and disinfection of the instruments

a. Immerse the instruments with the remaining impurities in an enzyme disinfectant detergent diluted in water (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. Cleaning of all visible contaminations from the surface of surgical instruments is performed using a soft nylon brush with a short pile (cleaning time of each instrument is at least 4 minutes). Cleaning of the delicate applied parts of the instruments is carried out using cleaning needles.

c. The instruments are rinsed under running water to remove any remaining disinfectant.

2-2) Cleaning and disinfection of instruments in an ultrasonic bath (recommended by the manufacturer as the preferred method)

a. The instruments shall be completely immersed in the cleaning solution. It is recommended to use an enzyme disinfectant cleaner (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. The instruments are to be cleaned in an ultrasonic bath for 10–20 minutes, then rinsed with water.

c. The instruments are to be rinsed under running water to remove any remaining cleaning disinfectant.

3) Repeat steps a) and b) until the instruments are completely cleaned, then perform visual inspection of the instruments.

4) The instruments are thoroughly dried naturally or using drying cabinets for surgical instruments. If necessary, manual drying using napkins made of non-abrasive lint-free fabric is possible.

5) For more information and data on using the available enzyme and non-enzyme disinfectants, refer to the instructions of the manufacturers of these products.

2. Steam sterilization

After cleaning and disinfecting the instruments, it is necessary to carry out sterilization.

The instruments shall be sterilized by steam under pressure in an autoclave at a temperature from 132 °C to 134 °C for 15 minutes, followed by drying for 30 minutes.

12. Disposal

Medical devices shall be disposed of in accordance with the medical procedures adopted by the medical institutions, and the local, state, or federal regulations. The choice of methods for the safe disposal and destruction of devices depends on the capacity and profile of the medical organization, the availability of waste disinfection/neutralization facilities, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

Instruments are considered epidemiologically hazardous waste (devices contaminated with blood and/or other biological fluids of patients, as well as devices in contact with patients with infectious diseases). In this case, a medical organization only collects and temporarily stores the above-specified waste. Collection is carried out into single-use bags or containers with color labeling adopted for hazardous medical waste in a room or in places for temporary storage of medical waste before subsequent transportation to the place of destruction or disposal. Instruments in contact with the patient shall be pre-cleaned and disinfected in accordance with the established procedure before being disposed of. Transportation of medical waste from medical institutions, and their disposal is carried out by specialized organizations having a license to handle epidemiologically hazardous medical waste.

Consumer package and unused defective devices belong to epidemiologically safe waste, similar in composition to solid household waste (SHW).

13. Shelf life

The shelf life of the device is unlimited.

14. Service life of devices

With proper use, the service life of the instruments is unlimited, except for the effects of natural wear.

15. Maintenance and repair

Maintenance and repairs are not provided.

16. List of international standards that the medical device complies with

EN ISO 13485; EN ISO 14971; EN ISO 14155-1; EN ISO 14155; EN ISO 10993-5; EN ISO 15223-1; EN 1041; MEDDEV 2.7/1 Rev. 4; MEDDEV 2.12/1 Rev. 8; EN ISO 17665-1; EN ISO 11138-1; EN ISO 11737-1; ISO 17665-1; ISO 11135-1; ISO 13397-2.

Note: The latest editions of the standards are used: the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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LEE, JUNGYONG -----

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attorney-in-fact of
DENTAL STUDIO CO., LTD.

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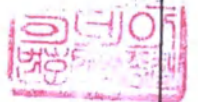
appeared before me and admitted
said principal`s subscription to
the attached

INSTRUCTIONS FOR USE FOR
THE MEDICAL DEVICE. -----

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This is hereby attested on this
20th day of Jan. 2025 at this office.

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(Cheongjin-dong, Samsong Bldg.) 701, 702,

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34, Jong-ro 3-gil, Jongno-gu, Seoul,
Republic of Korea

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This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, Feb. 2020 Under Law No.15150.

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