




ФОТОГРАФИЧЕСКИЕ ИЗОБРАЖЕНИЯ

Трубки эндотрахеальные ETVView с монитором,
производство
ETView Ltd., (ЭТВьюв Лтд, Израиль), Israel.

1. Трубка эндотрахеальная VivaSight-SL (TVT).

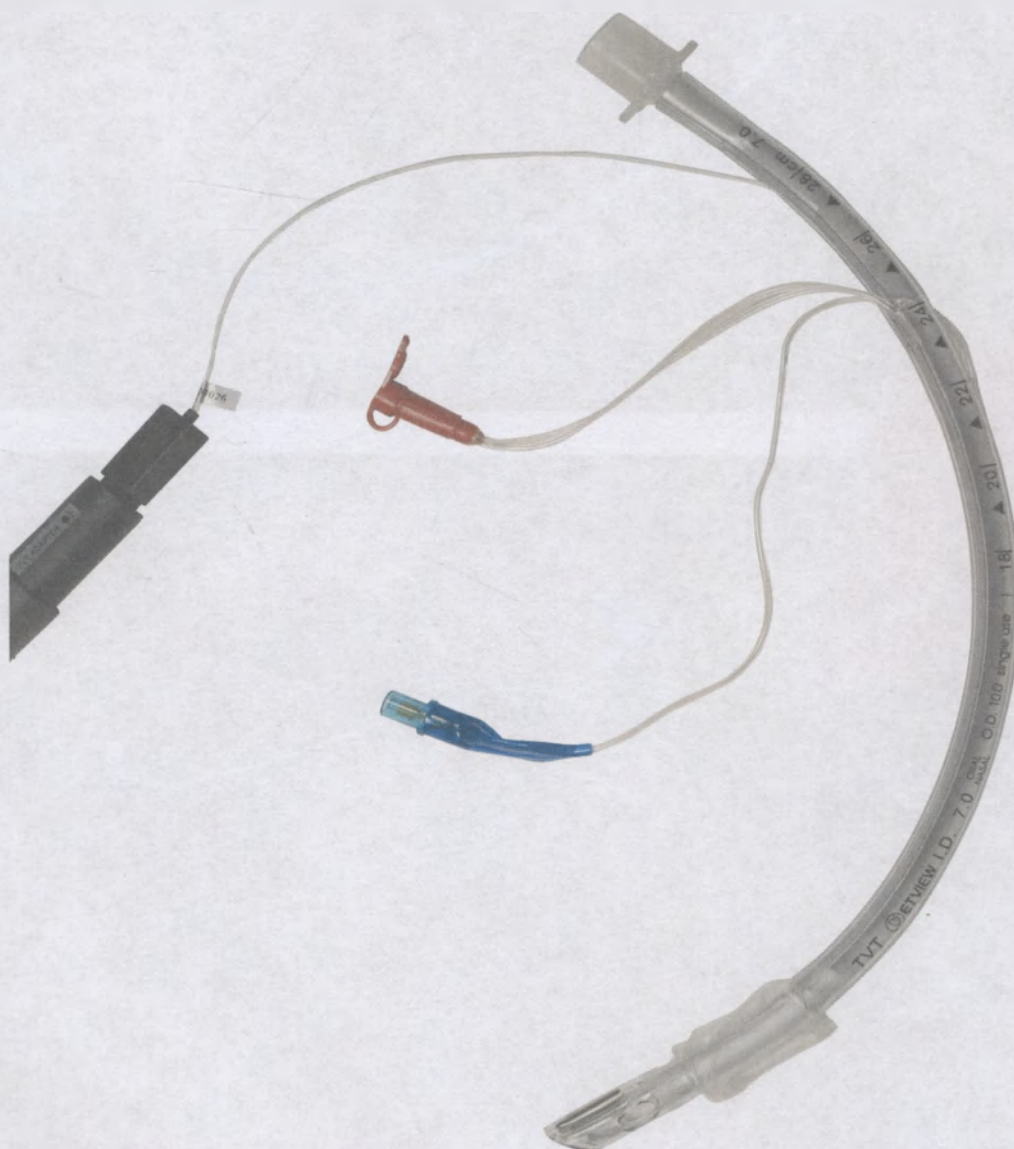

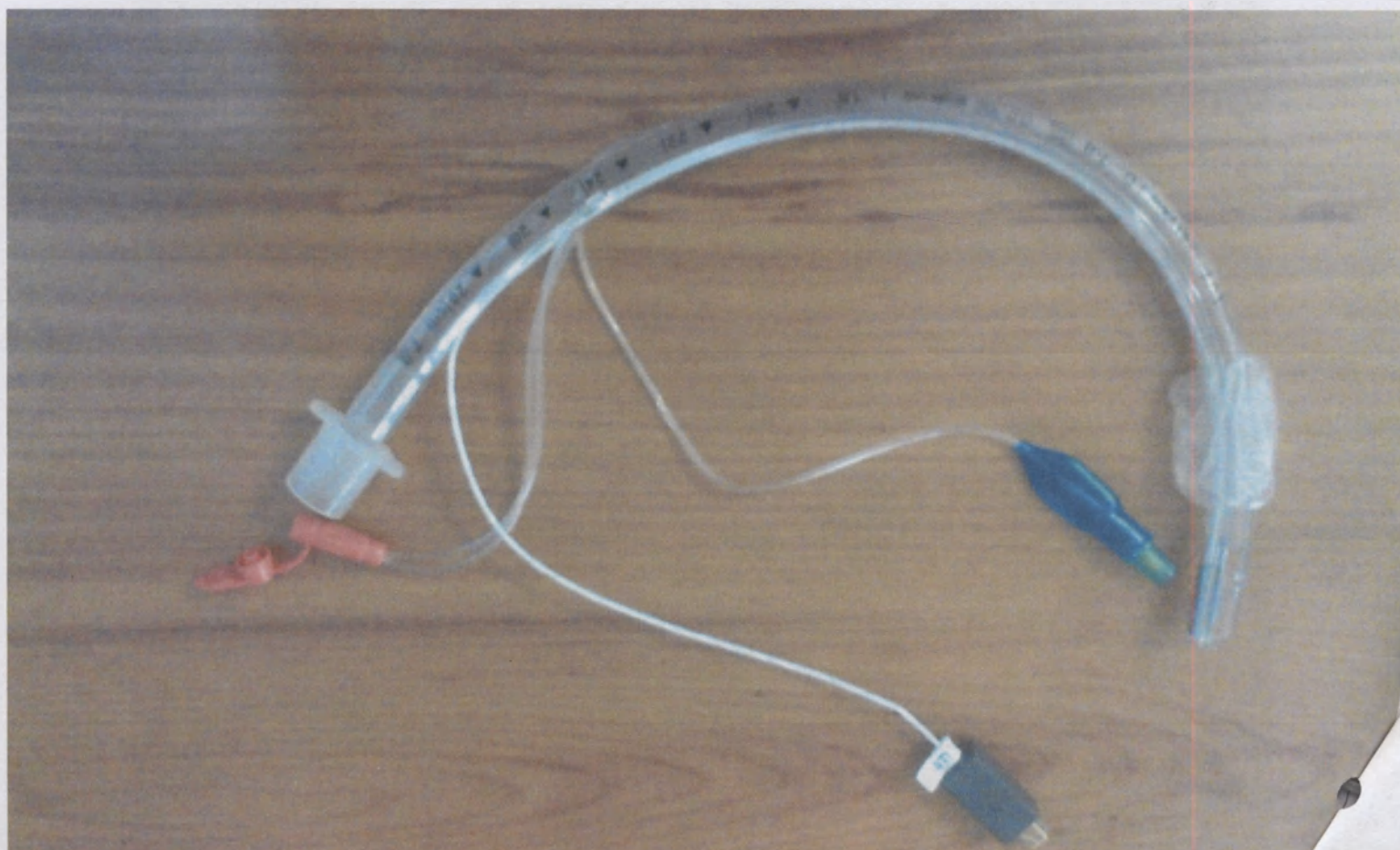

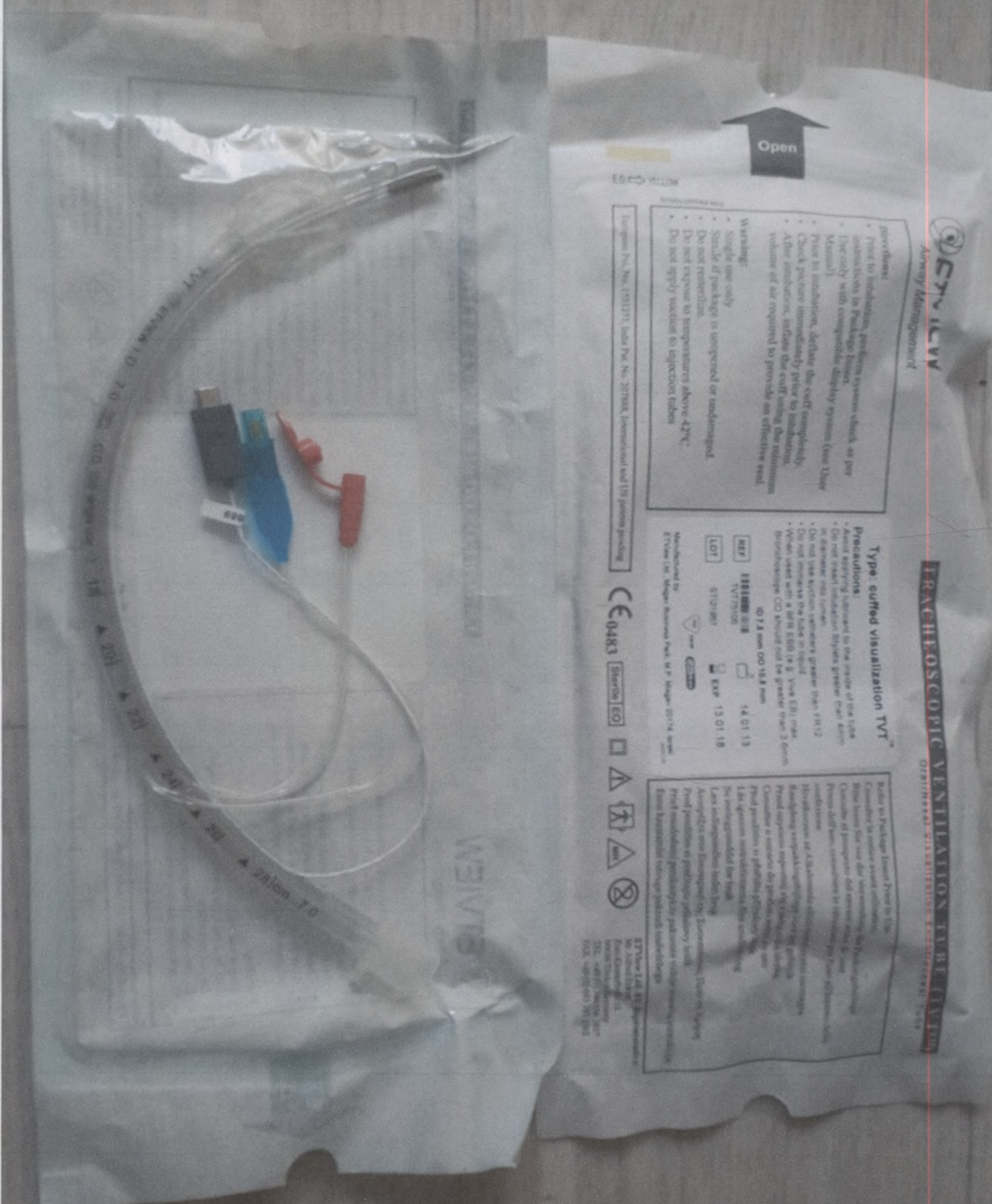

CVV
agement

TRACHEOSCOPIC VENTILATION

Oral/Nasal Visuali

ation, perform system check as per
in Package Insert.
with compatible display system (see User
ation, deflate the cuff completely.
are immediately prior to intubation.
ation, inflate the cuff using the minimum
ir required to provide an effective seal.

ly
package is unopened or undamaged.
ilize.
e to temperatures above 42°C.
suction to injection tubes

Type: cuffed visualization TVT™

Precautions:

- Avoid applying lubricant to the inside of the tube
- Do not insert Intubation Stylets greater than 4mm in diameter into lumen.
- Do not use suction catheters greater than FR12
- Do not immerse the tube in liquid.
- When used with a 9FR EBB (e.g. Viva EB) max. Bronchoscope OD should not be greater than 3.6mm

ID 7.5 mm OD 10.5 mm

REF

TVT75105



14.01.13

LOT

STI21957



EXP 13.01.18



DEHP


Latex free

Manufactured by:
ETView Ltd., Misgav Business Park, M.P. Misgav 20174, Israel



Refer to Package Insert
Consulter la notice au
Bitte lesen Sie vor de
Consulte el prospect
Prima dell'uso, cons
confezione
Hivatkozzon az All
Raadpleeg verpak
Przed użyciem zap
Consultar o encar
Před použitím si
Läs igenom instr
Se innleggssedde
Læs indlægssedle
Avatpéte oto B
Pred použitím si
Prieš naudodami
Enne kasutamist

551277, India Pat. No. 207888, International and US patents pending



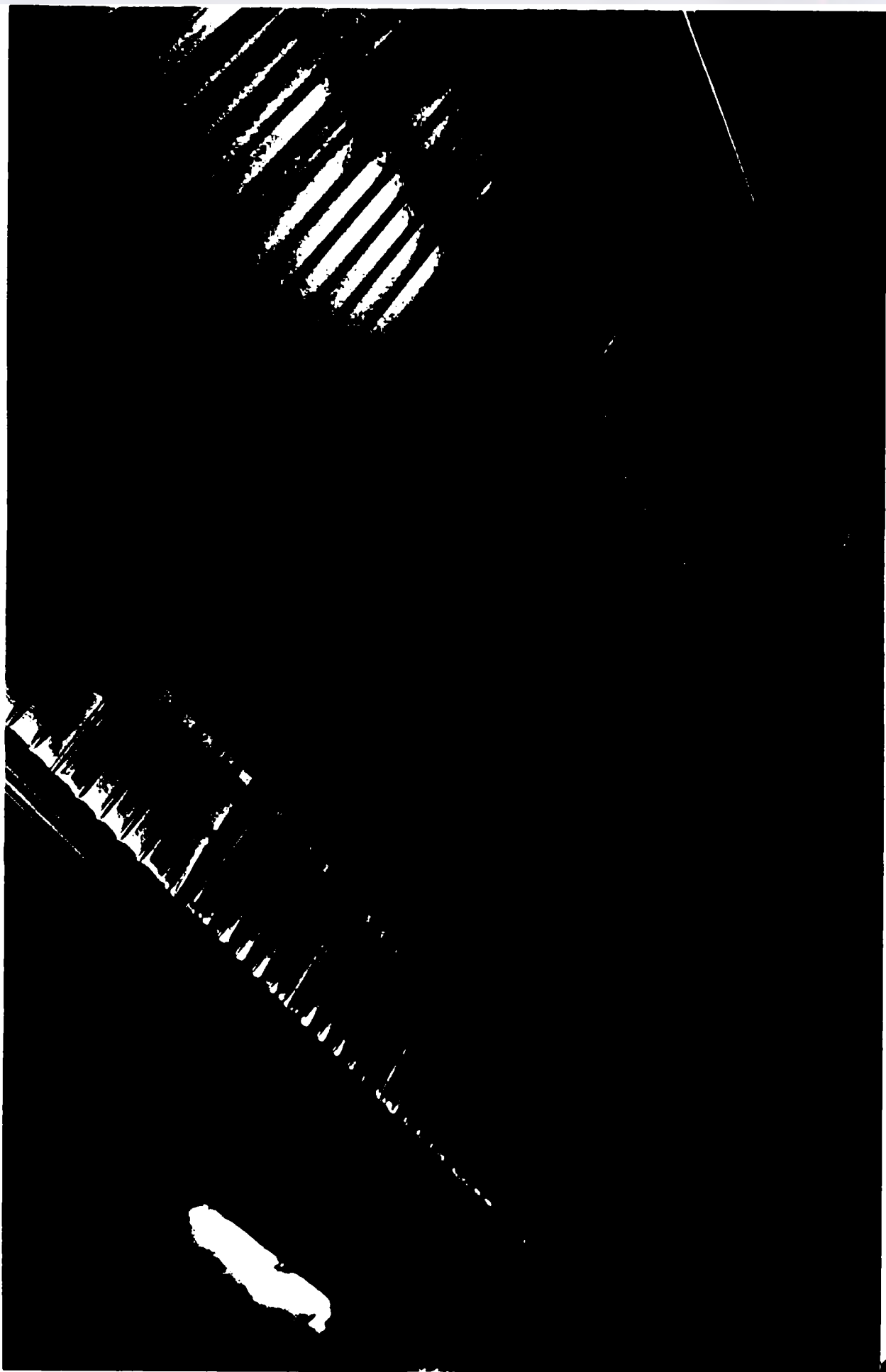

2. Трубка эндотрахеальная VivaSight-DL.

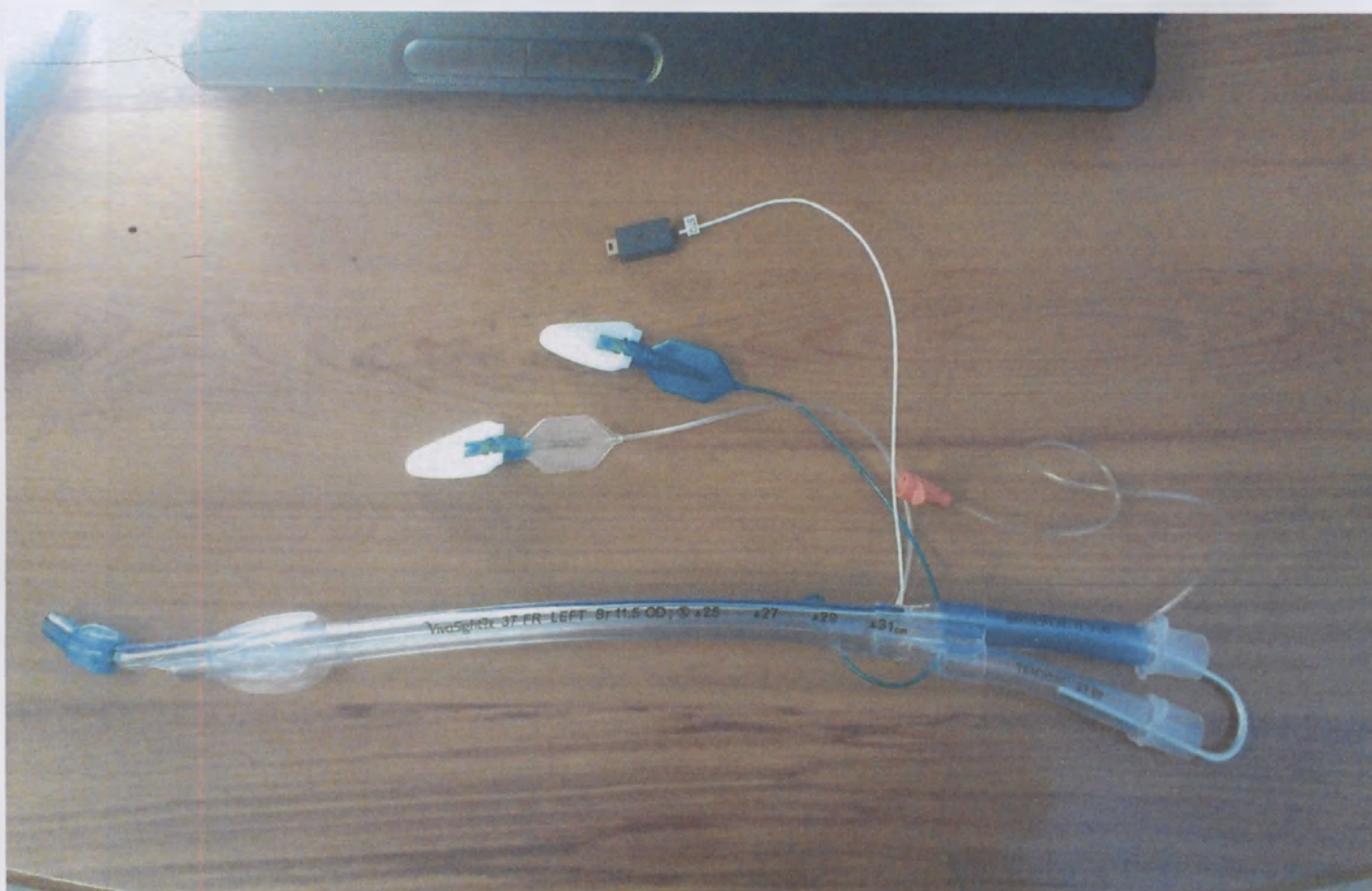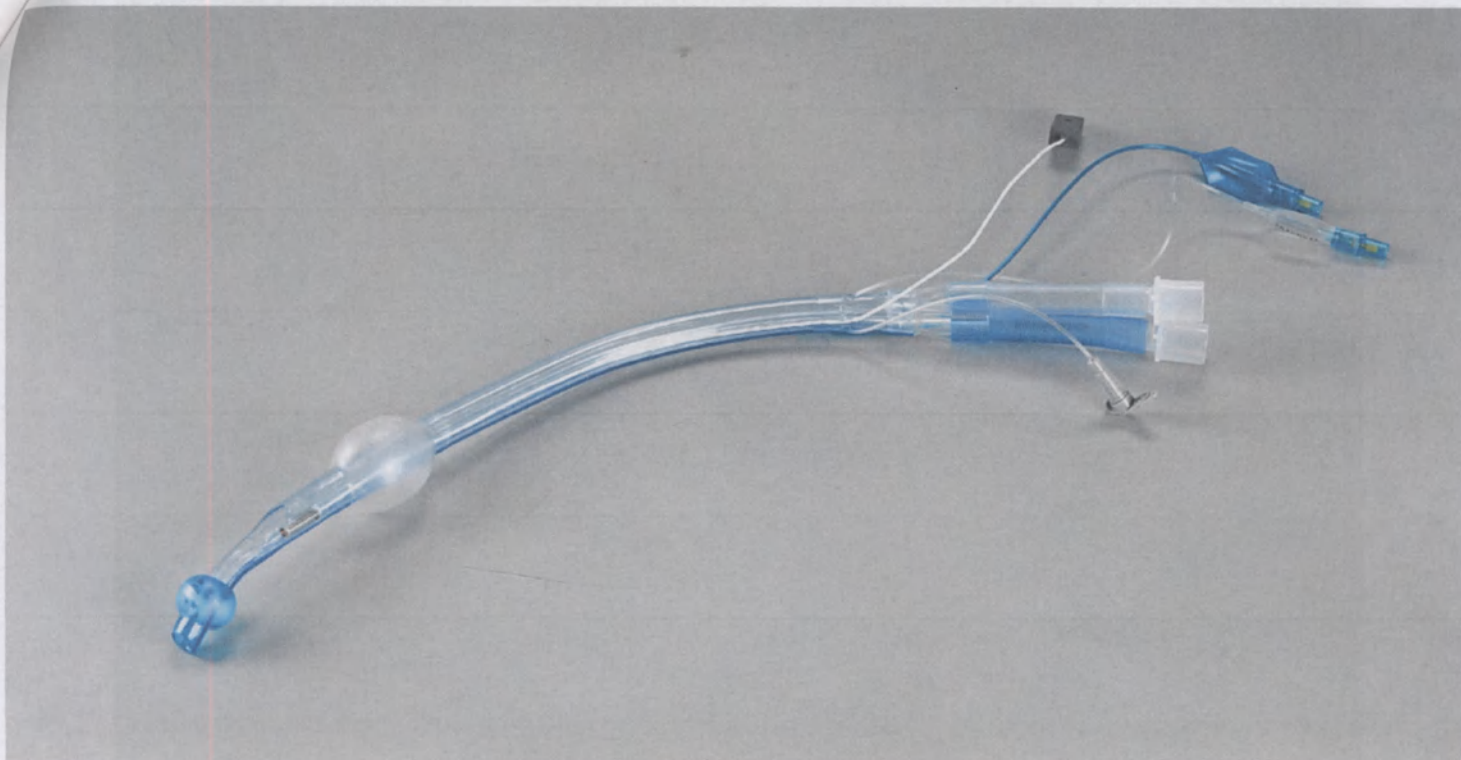

Management

Visualized dou

REF




DLVT37L

EAN-13



7 290013 745606

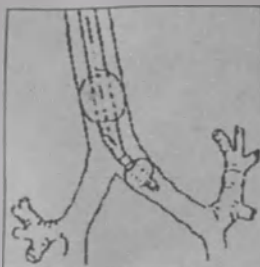

Precautions:

- This device is only for use in non- MRI environments.
- Avoid applying lubricant to the inside of the tube.
- Do not use instruments greater than 4.2mm in diameter within lumen.
- Do not use suction catheters greater than 12FR.
- Do not immerse the tube in liquid.

Size LEFT 37FR

LOT STI22032

QTY 1

EXP 10.03.13

Eff. ID 4.2mm

EXP 09.03.15


BR 11.5mm OD

Manufactured by:
ETView Ltd., Misgav Business Park
M. P. Misgav 20174, Israel
www.etview.com

Made in Israel

LB0010100


PATENT PENDING

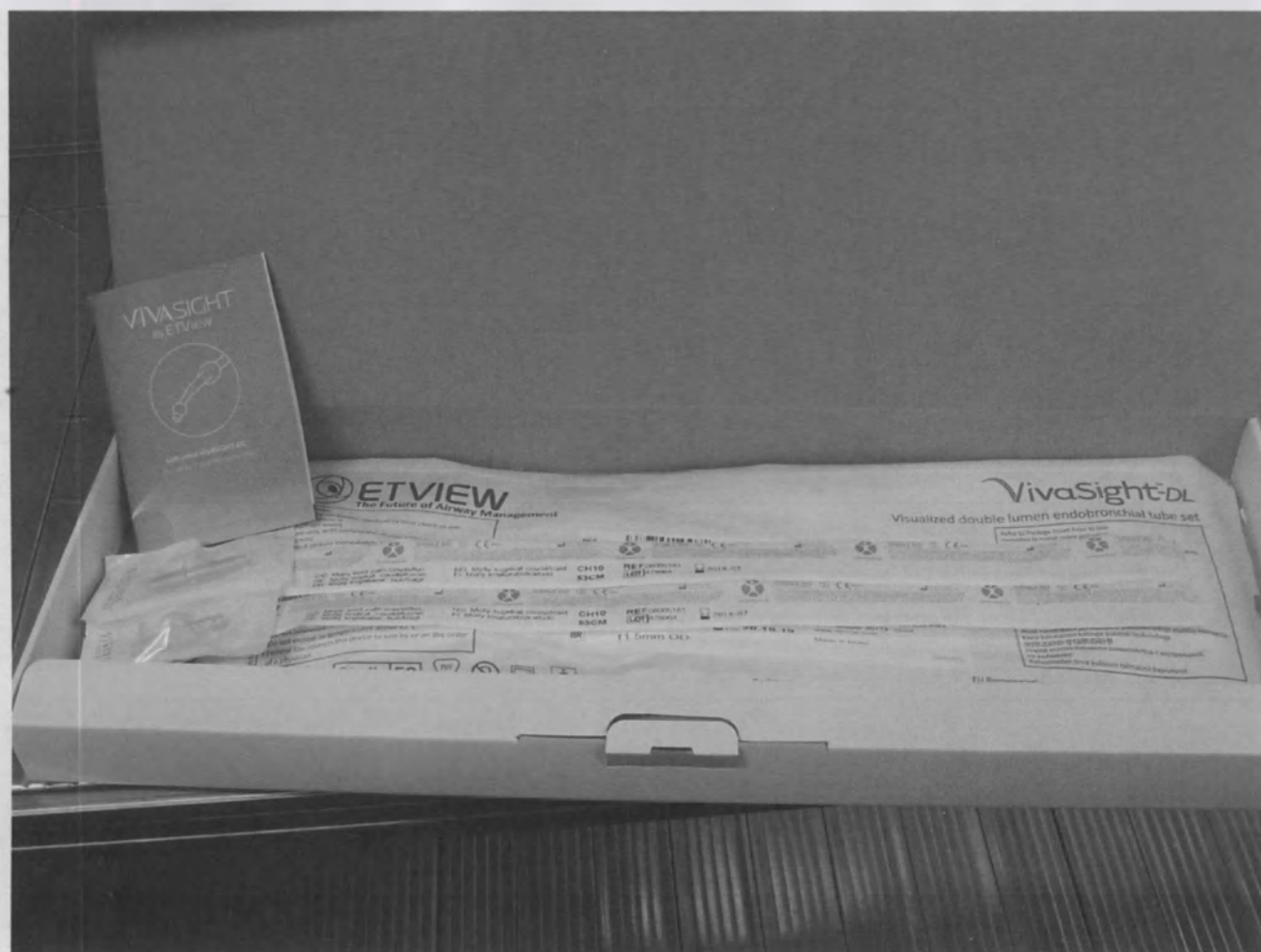

[illegible]

3. Эндобронхиальное блокирующее устройство VivaSight EB.



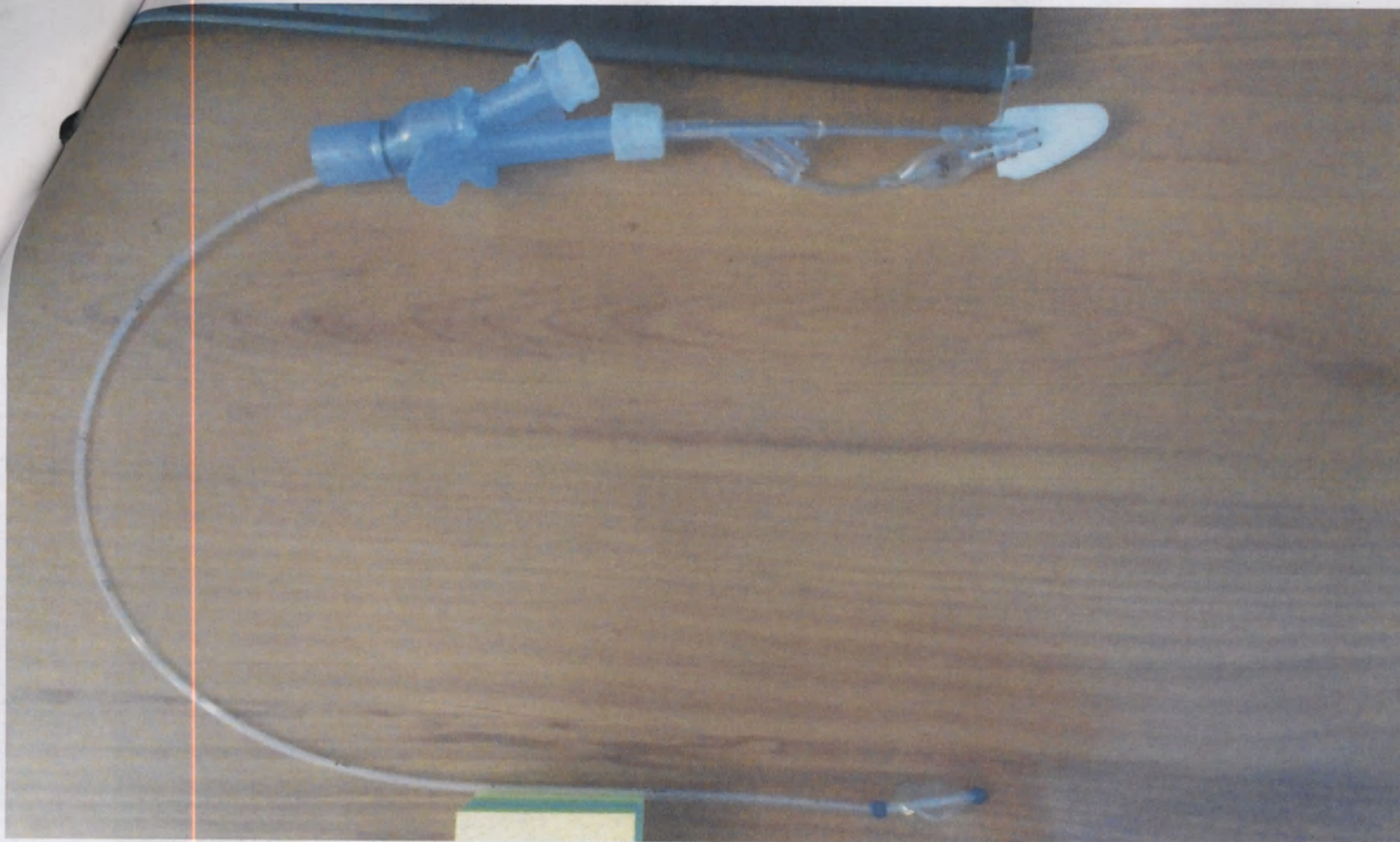

ETVIEW
The Future of Airway Management

VivaSight EB Clear Endo-Bronchial Blocker

Directions:

- Before use make sure that abnormality is not seen in the package or device.
- Test the cuff for integrity by inflating and deflating it completely prior to use.
- Prepare the patient. And lubricate the entire surface of the cuff.
- Keep the cuff inside the multi-joint connector and connect the multi-joint connector to the VivaSight-SL's ventilation connector after the VivaSight-SL is placed.
- Connect the anesthesia circuit to the multi-joint connector.
- Push VivaSight-EB into the VivaSight-SL. If you feel a resistance while inserting, twist VivaSight-EB slightly.
- Guide the VivaSight-EB cuff to the target bronchium. The cuff's direction can be changed by twisting the VivaSight-EB tube.
- After confirming the position of the cuff, by the visualization means of the VivaSight-SL, inflate the cuff to block the target bronchium and fix VivaSight-EB tube with the twist lock.
- Auscultate the lungs carefully and confirm that VivaSight-EB is working properly.
- The evacuation port can be opened following placement to deflate the lung or administer oxygen.
- Deflation of the cuff and removal the VivaSight-EB as per protocol.

Warning:

- Single use only.
- Sterile if package is unopened or undamaged.
- Do not re-sterilize.
- Store in a cool and dry place.

Refer to Package Insert Prior to Use

consultez le notice avant utilisation
Bitte lesen Sie vor der Verwendung die Packungsbeilage
consulte el prospecto del envase antes de usar
Prima dell'uso, consultare le istruzioni per l'uso all'interno della confezione
Hivalkozzon az Alkalmazás előtt be kell olvasni a csomagolási használati utasítását
Hreadstreeg verpakingsbeilage voor het gebruik
Przed użyciem zapoznaj się z załączoną ulotką
Consultar o encarte do produto antes do uso
Phad použitím si přečtěte příbalový leták
Las ignom instruktionerna före användning
Se innleggseddelen för bruk
Läs inläggseddelen inden brug

Ανατρέξτε στο Εγχειρίδιο της Συσκευασίας Πριν τη Χρήση
Pred použitím si prečítajte príbalový leták
Prieš naudotomą perskaitykite pakuočių viduje esančią instrukciją
Enne kasutamist tutvuge pakendi taabelohega
使用之前请务必阅读
Перед использованием ознакомьтесь с инструкцией на вкладыше
Kullarmaden once kullanim taimeatna başvurunuz

Precautions

- Prior to insertion or removal of VivaSight-EB make sure that the cuff is completely deflated.
- To guarantee complete deflation of the cuff use a 20cc syringe. For deflation pull the plunger all the way back to the 20cc mark and disconnect it from the inflation port without releasing the plunger.
- Use only water soluble lubricant.
- Use the same size VivaSight-SL as standard tube size required in combination with VivaSight-EB. Expert clinical judgment should be used in choosing the suitable tracheal tube size for each patient.

Cuff:	PU
REF	
EAN-13	
Size	9FR
	2013-04
 EXP	2018-03
LOT	1304020557

EU Representative:
Dr. Wim van den Brink
Osse EU
Geme van der Veestraat 44HS
1077 EL Amsterdam
The Netherlands
Tel: +31 (0) 208110550
Fax: +31 (0) 842213142
info@wimvandenberg.nl

Manufactured in China for:
ETView Ltd.
Nesgav Business Park
M.P. Migdal 20174, Israel
www.etview.com

LB0012500





4. Монитор EView VivaSight Max.







ETVIEW
CONTINUOUS AIRWAY CONTROL

7" Med. Monitor Kit

011 001 11 0 10010000100010 010

SA00014

11 010001011 0 000100001000011011 0 010

S/069A728990010

 **Manufacturer:** ETView Ltd., Misgav
Business Park, M. P. Misgav 20174 Israel,
www.etview.com
Patent Pending

VIVA SIGHT MAX

7" ETview Monitor Kit



0483






 Consult
instruction
for use


 10°C. 42°C.
 Limit of
Temperature



EC REP

Dr. Wim van den Brink
Gerrit van der Veenstraat 84HS, 1077 EL
Amsterdam, The Netherlands
Tel: +31 (0) 208110550,
Fax: +31 (0) 842213142
Mail: qsiteeu@qsitemed.com

180003303





5. Монитор портативный ETViva VivaSight Mini.








6. Кабели соединительные ETVivaSight Direct:

а) блок питания 5В



б) кабель питания

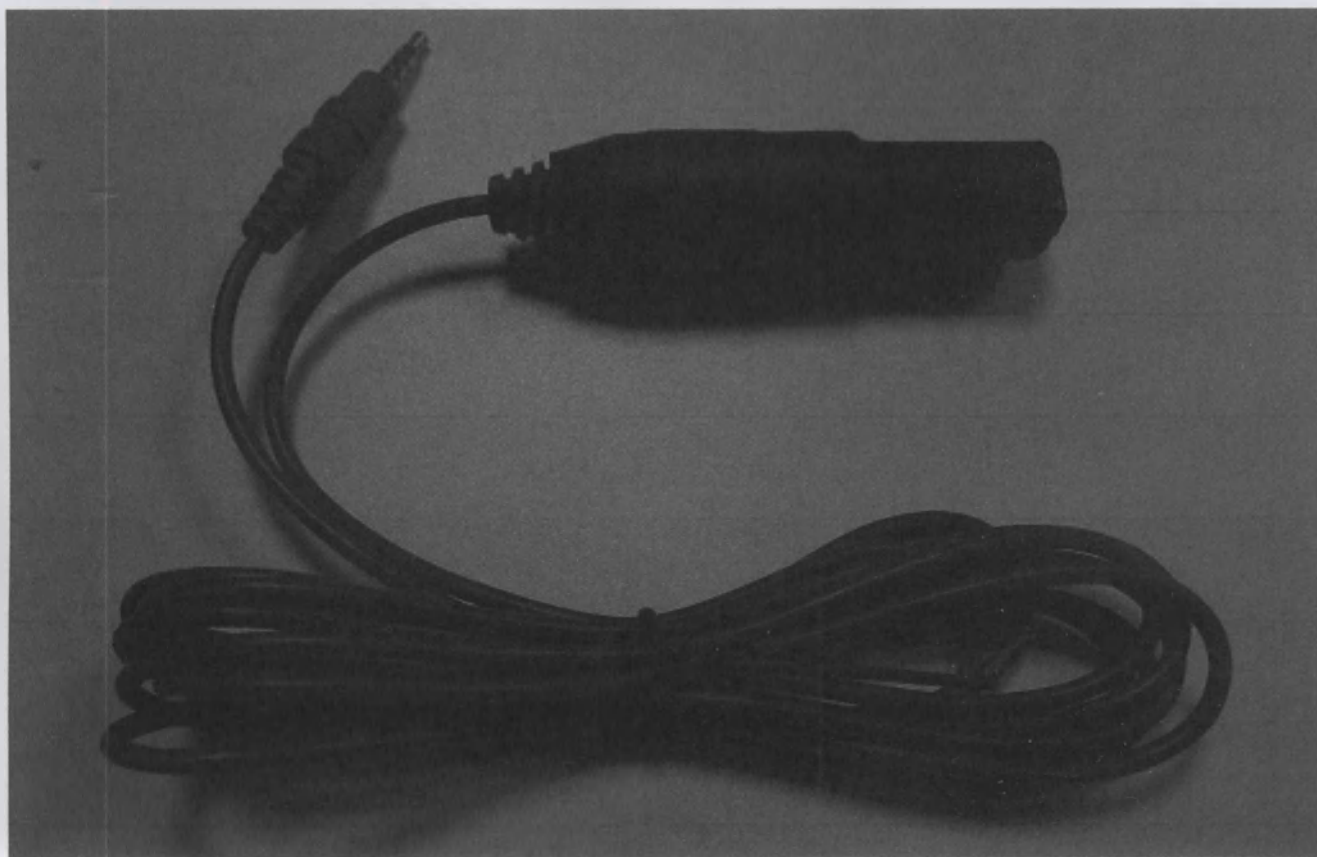


в) конвертер HD




г) штекер BNC



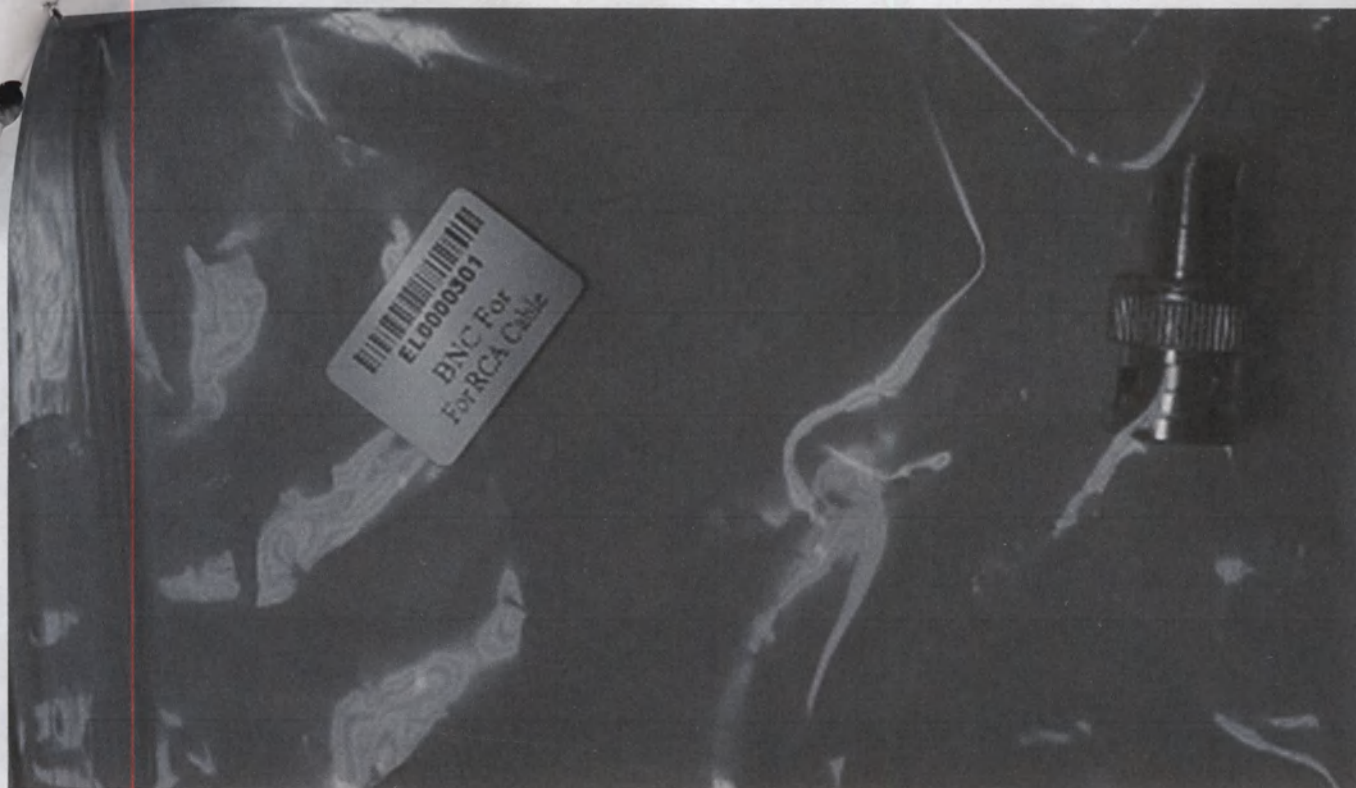

д) разъем-насадки

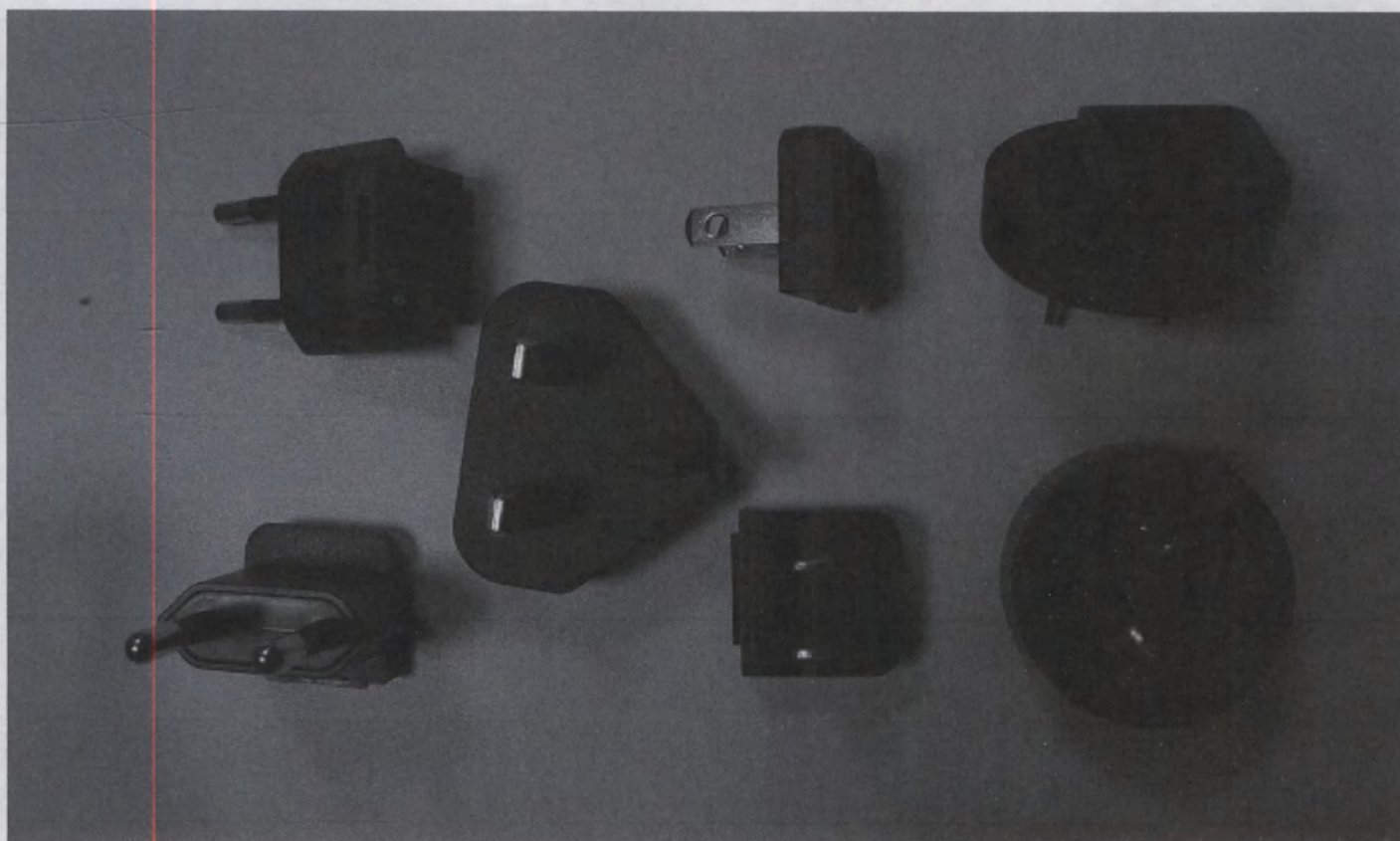

Генеральный директор
ООО «НевроМед»



С.А. Чернышев