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INSTRUCTIONS FOR USE
OF A MEDICAL PRODUCT

**Cure for removing of cerumen impaction
« NATUMA Aquaunplugged 50 ml »
for children and adults,**

manufactured by YS LAB, France

Handwritten signature or mark in blue ink.

1. Main Characteristics

Description: Cure for removing of cerumen impaction «NATUMA Aquaunplugged 50 ml» for children and adults is an aerosol product in pressurized metal container with internal antiseptic coating and specially designed unique ear-sprayer and plastic cap. Specially designed unique sprayer allows to spraying cure in the ear canal. Cure delicately separates cerumen impaction from the ear canal, thus facilitating its easy removal. It contains a clear colorless and odorless with slightly salty taste, which is a sterile seawater solution (100 ml contain 50-55 ml of natural sea water) with hypertonic NaCl concentration of 13-17 g/l, purified water, glycerin, castor oil, nonionic surfactants, essential oil of tea tree.

The product doesn't contain any conservants. It is not a drug.

Intended for children from 3 years and adults.

2. Technical Specifications

Technical and functional characteristics of the product are provided in the table below:

Name of characteristic	Description
Product description and design	Aerosol in pressurized metal container with anticeptic inner coating, specially designed earpiece and plastic cap. A four-layer bag with content and a special nonreturn valve maintaining sterility of the content during the whole term of use by preventing backflow of content and penetration of ambient air into the bag is placed inside the container..
Package materials	Container – aluminum (ALU 8); bag with nonreturn valve: four-layer bag – polypropylene (PP 75)-polyamide (OPA 15)-aluminum foil (ALU 8) –polyester (PETP 12); valve body – polypropylene (HJ 325MO); valve: piston – polyacetal resin (POMKepitalF2503); spring – stainless steel (AISI 316); inner liner – natural rubber (DB 219); external liner – butyl (U 133); valve cup – aluminum (ALU 8); earpiece – polyethylene (HDPE Purell GA 7760); cap – polypropylene (RJ370MO).
Product form	Spray with earpiece of special design
Dimensions	Filling volume 50 ml,
Mass with cap (g)	81±5
Size (mm) h-high; d-diameter	Balloon (h-110±1; d-35±1), cap (h-42±1; d-35±1), diffuser (h-38±1; d-23±1)
Balloon volume (ml)	50
Description of contents	Clear colorless and odorless liquid with a slight salty taste, being a sterile sea water solution (100 ml contain 50-55 ml of natural sea water) with hypertonic NaCl concentration 13-17 g/l, purified water, glycerol, castor oil, nonionic surfactants, manuka oil.
Contents composition	Natural sea water with hypertonic NaCl concentration of 13-17 g/l, purified water, glycerol, castor oil, nonionic sufactants, manuka oil, permitting to dissolve a cerumen impaction.
Percentage ratio	Natural sea water 45%, glycerol 6%, castor oil 2%, Polysorbate80 1%, manuka oil 0,01%, purified water 45,99%
pH	6.0 – 8.5
Storage conditions	Keep in a dry place protected from direct sunlight at +5 °C to +25 °C. Keep out of the reach of children.
Sterility	Sterile. Sterilization method: radiation. Maintains sterility during the

	whole period of use.
Maximum pressure for unclosing of aerosol valve	should not exceed 40 N (4 kp)
Allowable steam leakage through seal of nipple	should not exceed $0,1 \pm 0,05$ cm ³ during 6 hours under pressure $(1,20 \pm 0,05)$ MPa
Stability to axial force of compression (opening/closing)	Aerosol valve should withstand axial force of compression not less 150 N (15 kp) during 500 work cycles (opening/closing)
Allowable case deformity under overpressure	Allowable deformity of valve (case) under overpressure no more $0,3 \pm 0,05$ mm under overpressure of vapor: $(1,00 \pm 0,05)$ MPa – for aluminum case
Force of tube removal off pocket shank	Not less 60 N (6 kp). Allowable gap between tube's back and pocket - no more 0,8 mm.
Spray head's nozzle pressure	Spray head's nozzle should withstand pressure without falling out: Plastic nozzle - $(1,00 \pm 0,05)$ MPa

3. Intended use

Cure for removing of cerumen impaction «NATUMA Aquaunplugged 50 ml» for children and adults is intended for softening and removing of cerumen impaction from external ear canal.

The cure removes cerumen impaction gently and effectively.

Specially selected components allow to soften, separate and remove the cerumen impaction from the ear canal gently.

Unique ear-sprayer allow to spraying cure in the ear canal to influence on the cerumen impaction directly, don't damage the surface of the ear canal and the eardrum.

Conditions of use: for individual use and in health care institutions.

4. Indications and contraindications:

Indications for use:

Blockage of the ear canal by cerumen impaction.

Contraindications:

Damage to the eardrum and mastoidectomy.

Children up to 3 years of age.

Allergic reactions to any of the components.

Special instructions:

Pregnancy and breastfeeding are not contraindications.

Avoid contact with open flame, hot surfaces and direct sunlight. Do not freeze.

5. Preparation for work and use of the product

- 1) Shake before use.
- 2) Remove a cap without damaging a green nozzle, that can lead to trauma of the ear canal in future.
- 3) Gently introduce the tip of the nozzle into the ear canal (Fig.1).
- 4) Gently press the valve and hold for 1-2 seconds. Cure goes directly into the ear canal (Fig.2).
- 5) The cure soften cerumen impaction in 5-10 seconds. Repeat the procedure again to rinse away residue of earwax from the ear canal (Fig.3).

Fig.1

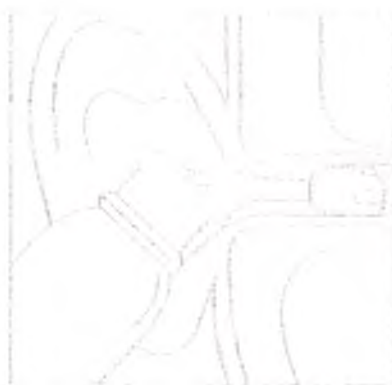
Fig.2

Fig.3

- 6) Wash the ear - sprayer in warm, soapy water.
In case of blockage the ear canal by earwax, use the cure 3 times a day for 5 days.

6. Sanitization indications: process the ear-sprayer by solution of 70% ethyl alcohol or boiling water before application of the medical device.

7. Packaging and labelling

Metal container with specially designed unique ear-sprayer and plastic cap in carton box with instruction for use.

The following information is specified on the container:

- The name, address and country of the manufacturer;
- The name of the medical device on Russian and English;
- Intended use;
- Application;
- Rules of storage;
- Indication of sterility;
- Serial number;
- Date of manufacture;
- Shelf life;
- Conditions of supply from pharmacies;

The following information is specified on the package:

- The name, address and country of the manufacturer;
- The name of the medical device on Russian and English;
- Intended use;
- Application;
- Rules of storage;
- Indication of sterility;
- Serial number;
- Date of manufacture;

Shelf life;

Conditions of supply from pharmacies;

Address for submission of claims.

8. Rules of storage, transportation, waste disposal.

Shelf life of the product – 3 years. Use before the date labeled on the package.

Store the product at the temperature from +5°C to +25°C, in a dry place protected from direct sunlight.

Keep out of the reach of children.

Cure for removing of cerumen impaction «NATUMA Aquaunplugged 50 ml» for children and adults can delivered by any kind of transport in covered vehicles in accordance with the current rules of this mode of transport at a temperature within the range of from +1 °C to +40 °C.

Cure for removing of cerumen impaction «NATUMA Aquaunplugged 50 ml» at the end of shelf life is disposed with other industrial waste in accordance with applicable national legislation.

9. Manufacturing organization, warranty

The product is manufactured by YS LAB, Le Forum, 2 rue Felix Le Dantec, QUIMPER 29000, France.

Cure for removing of cerumen impaction «NATUMA Aquaunplugged 50 ml» for children and adults and production facilities of YS LAB, France comply with all international quality standards, which is confirmed by the ISO 13485:2012 certificate №12656 Revision 5 and declaration of compliance for the product.

The manufacturer guarantees the quality of the product during the shelf life.

On the territory of the Russian Federation all the claims to quality must be send to FarmaStar Co., Ltd., 104005, Moscow, 2-ya Baumanskaya street, building 7, str.1A, tel. 8 495 969 21 84.

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