All-Russian Classification of
Products code 93 9890

Approved by
Signature/Stamp  .

Micro-Tech (Nanjing) Co., Ltd.
Chief executive officer
Mr. Derong Leng

18 "11" 2014

Operating Manual

INDIGO CARMINE DYE FOR ENDOSCOPY

Produced by: Micro-Tech (Nanjing) Co., Ltd

1. APPLICATION

Indigo Carmine is utilised to colour the digestive tract mucosa, highlight the contrasts in suspicious tissues within the digestive system and aid the detection of mucosal irregularities and detection of subtle mucosal abnormalities.

1.1. Supply package:

Each product is packed in an airtight vial, and sealed in an Aluminum-plastic composite film pouch:

- individual packaging: 1 airtight vial
- overall packaging: 10 airtight vial.

2. BASIC SPECIFICATIONS

Specifications:

Airtight vial volume: 15 ml.

Every airtight vial contains 15ml of sterile non pyrogenic suspension for staining.

Weight of airtight vial (with sterile suspension) is 29 gm

Sterile, single use.

Sterile: yes.

The suspension is sterilised by steam sterilization.

PH from 3,0 to 7,0

Non pyrogenic.

Do not contain any medications.

Non toxic.

Indigo Carmine suspension is sterilised by filtration and aseptically filled in a ready to use, disposable.

3. INDICATIONS AND CONTRAINDICATIONS

Indigo Carmine is a suspension containing disulfonate and pure water. The particles remain in the digestive tract site staining and observing it. Each steam sterilized, non-pyrogenic airtight vial contains 15ml of Indigo Carmine. The composition of Indigo Carmine is as follows:

- Pure water 99,6%
- Disulfonate 5,5' 0,4%

INDICATION FOR USE

Indigo Carmine is utilised to colour the digestive tract mucosa, highlight the contrasts in suspicious tissues within the digestive system and aid the detection of mucosal irregularities and detection of subtle mucosal abnormalities.

CONTRAINDICATIONS

1. Serious hepatic insufficiency or renal inadequacy;
2. Gastrointestinal obstruction or perforation;
3. Allergic to staining of mucosa.
4. Any other contradictions associated with endoscopy procedure.

WARNING AND PRECAUTIONS

Indigo Carmine is aseptically processed single-use only product. Leakage of product may be evidence of damage. Discard any unused product after the airtight vial has been opened. Although the material is non-toxic, Indigo Carmine may stain clothing or porous materials.

COMPLICATIONS

- Allergic reaction

4. SETTING-UP PROCEDURES

Insert an airtight vial of Indigo Carmine to a syringe with Luer-Lock connector, and then connect a syringe with Luer-Lock connector to a spray catheter.

5. OPERATIONAL PROCEDURE

Under endoscopic monitoring, spray Indigo Carmine by spray catheter which was provided through endoscopic working channel. After suspicious sites of gastric mucosa were stained, use a water pump, which should be connected to endoscope, or syringe with Luer-Lock connector to wash off Indigo Carmine using sterile water. Due to Indigo Carmine permeates and absorbs on the cover of damaged (changed) epithelium longer than on the healthy tissues, sites of intestinal metaplasia, benign and malignant tumors became visible under endoscopic monitoring.

IMPORTANT: making of target biopsy and furthering morphological studies are necessary for clarifying the diagnosis.

The dosage shall be determined by a doctor according to the actual situation of a patient, and the maximum dosage shall not exceed 50ml.

6. DISPOSAL INFORMATION

According to the Sanitary and Epidemiological Requirements for Disposal of Medical Waste: Sanitary and Epidemiological Regulations and Norms 2.1.7.2790-10. Class B.

7. PACKAGING

Packaging: Indigo Carmine in an individual glass airtight vial.

Overall packaging: Aluminum-plastic composite film pouch.

8. MARKING

- Marked:
- Each individual package contains:
- Sterile A
- Temperature of storing
- Single use
- Avoid direct sun
- Manufacturer
- Distributor
- Date of manufacturing
- Period of storing
- Fragile

9. TRANSPORTATION

Indigo Carmine can be shipped by any mode of covered transport in accordance to the requirements of the Russian National Standard 50444 and shipping rules for every means of transport.

Shipping by sea should meet the Rules for the Safe Carriage of General Cargoes by Sea. Dispatch type: containers and small shipment.

Store the product at room temperature.

10. WARRANTY

The product warranty period is 2 years from the day of manufacture.

The storage period is 2 years.

11. RULES OF STORAGE AND USAGE

Storage:

Store the product at room temperature in a dark place, avoiding temperature extremes.

Period of validity: 2 years.

Do not use if packaging has been damaged.

Do not use after the expiration time.

Single use: yes.

Mentioned safety:

Do not use if packaging has been damaged.

Do not use after the expiration time.

List of international and Russian requirements

GOST RF 2.102-68, GOST RF 2.114-95, GOST RF 2.601-2006, GOST R RF 15.013-94, GOST R 50444-92, GOST 15150-69, GOST R ISO 10993-2-2009, ISO 10993-1-2011, 10993-4-2011, 10993-10-2011, 10993-11-2011, 10993-12-2011, 10993-13-2011, GOST R 52770-2007, GOST R 51148-98, MU 1.1.037-95, Directive 93/42/EEC, ISO 13485:2003, Regulation (EC) No. 1272/2008 of the European Parliament (on substances and mixtures) of 16.12.2008, Standard EN 374 (hands protection), Standard EN 166 (eyes protection), Directive 67/548/EEC, Regulation (EC) No. 1907/2006.

The product was produced according to Directive 93/42/EEC from 14.06.1993, appendix 2, class 2a.

The condition of safety using:

Do not use if packaging has been damaged.

Do not use after the expiration time.

Single use: yes

12. LIST OF INTERNATIONAL CERTIFICATES AND A DECLARATION OF CONFORMITY

- Certificate ISO 13485:2003 (EN ISO 13485:2012) from 21.08.2014
- EC certificate from 20.08.2014
- Declaration of conformity from 17.04.2014

WARRANTY CERTIFICATE

Note of Sale

Product category _____

Manufacturer _____

Model _____

Serial/identification number _____

Date of sale _____

Seller organization name _____

Seller's signature _____

The product is tested, no defects found.

Operating rules were read and accepted by:

Buyer's signature _____

Warranty period – 2 years.

For warranty questions, please contact:

Official distributor in Russian Federation – Endo Stars LLC, legal address:
197101, Russia, Saint-Petersburg, Bolshaya Monetnay, 27, lit. A, 12H; tel./fax: +7-800-555-56-57, e-mail: endo@endo-stars.com.

ATTENTION! Make sure that the Warranty Certificate is filled in correctly. The serial/identification number must correspond exactly with the one in the Certificate. All fields in the Certificate (except the fields for the warranty center) must be filled in.

In case of inaccurate or incomplete filling of the Certificate, the warranty for the product is regarded as invalid.

The product should not bear the marks of mechanical or thermal action, as well as of various chemical agents. The product use is allowed only in accordance with the Operators Manual.

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